

MEDLIS RAMPS

Medlis Engelli Yaşam Çöz İç ve Dış Tic San Ltd Şti

ASO 1. Org San, Özbekhanlığı Sk ,No 33, Sincan, Ankara, Turkey

Tel : +90 312 315 8795 // e-mail: office@medlisramps.com



Support Feet Installation Chart



ROLLING RAMP

Portable Wheelchair Ramp

Art No	Total Length		Includes			Recommended ramp sections where support legs be mounted (Support feet are installed to each of the 6th or 7th ramp sections.)						
	cm	inch	Section Qty	Access Profiles Qty	Support Feet Qty	1st feet	2nd feet	3rd feet	4th feet	5th feet	6th feet	7th feet
RR7580	80	31,5	3	2	-	-						
RR75100	100	39,4	4	2	-	-						
RR75120	120	47,2	5	2	-	-						
RR75140	140	55,1	6	2	-	-						
RR75160	160	63,0	7	2	-	-						
RR75180	180	70,9	8	2	-	-						
RR75200	200	78,7	9	2	-	-						
RR75220	220	86,6	10	2	-	-						
RR75240	240	94,5	11	2	1	5-6						
RR75260	260	102,4	12	2	1	5-6						
RR75280	280	110,2	13	2	1	6-7						
RR75300	300	118,1	14	2	1	6-7						
RR75320	320	126,0	15	2	1	7-8						
RR75340	340	133,9	16	2	1	7-8						
RR75360	360	141,7	17	2	2	5-6	11-12					
RR75380	380	149,6	18	2	2	6-7	11-12					
RR75400	400	157,5	19	2	2	6-7	12-13					
RR75420	420	165,4	20	2	2	6-7	12-13					
RR75440	440	173,2	21	2	2	6-7	12-13-14					
RR75460	460	181,1	22	2	2	7-8	13-14					
RR75480	480	189,0	23	2	3	6-7	12-13	18-19				
RR75500	500	196,9	24	2	3	6-7	12-13	18-19				
RR75520	520	204,7	25	2	3	6-7	12-13	18-19				
RR75540	540	212,6	26	2	3	6-7	12-13	18-19				
RR75560	560	220,5	27	2	4	5-6	11-12	16-17	22-23			
RR75580	580	228,3	28	2	4	5-6	11-12	16-17	22-23			
RR75600	600	236,2	29	2	4	5-6	11-12	16-17	22-23			
RR75620	620	244,1	30	2	4	6-7	12-13	18-19	23-24			
RR75640	640	252,0	31	2	3	6-7	12-13	18-19	24-25			
RR75660	660	259,8	32	2	4	6-7	12-13	18-19	24-25			
RR75680	680	267,7	33	2	5	5-6	11-12	17-18	22-23	27-28		
RR75700	700	275,6	34	2	5	5-6	11-12	17-18	22-23	27-28		
RR75720	720	283,5	35	2	5	6-7	12-13	18-19	23-24	28-29		
RR75740	740	291,3	36	2	5	6-7	12-13	18-19	24-25	29-30		
RR75760	760	299,2	37	2	5	6-7	12-13	18-19	24-25	30-31		
RR75780	780	307,1	38	2	5	6-7	12-13	18-19	24-25	30-31		
RR75800	800	315,0	39	2	6	5-6	11-12	17-18	22-23	27-28	33-34	
RR75820	820	322,8	40	2	6	5-6	11-12	17-18	22-23	27-28	33-34	
RR75840	840	330,7	41	2	6	6-7	12-13	18-19	24-25	30-31	34-35	
RR75860	860	338,6	42	2	6	6-7	12-13	18-19	24-25	30-31	35-36	
RR75880	880	346,5	43	2	7	5-6	11-12	16-17	21-22	26-27	31-32	37-38
RR75900	900	354,3	44	2	7	5-6	11-12	16-17	21-22	26-27	31-32	37-38
RR75920	920	362,2	45	2	7	6-7	12-13	17-18	23-24	28-29	33-34	38-39
RR75940	940	370,1	46	2	7	6-7	12-13	18-19	24-25	29-30	34-35	39-40
RR75960	960	378,0	47	2	7	6-7	12-13	18-19	24-25	30-31	36-37	41-42
RR75980	980	385,8	48	2	7	6-7	12-13	18-19	24-25	30-31	36-37	42-43
RR751000	1000	393,7	49	2	7	6-7	12-13	18-19	24-25	30-31	36-37	42-43