## ENGLISH

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

### INTENDED USER PROFILE

The intended user should be a licensed medical professional, the patient or the patient's caregiver. The user should be able to read, understand and be physically capable of performing the directions, warnings and cautions in the information for use.

### INTENDED USE/INDICATIONS

- Immobilization of the thumb or finger due to sprain/strains, fractures or dislocations.

### PERFORMANCE CHARACTERISTICS

Providing immobilization or controlled movement of the limb or body segment.

### CONTRAINDICATIONS

- None

### WARNINGS AND PRECAUTIONS

- If you have skin disorders/injuries in the relevant part of the body, the product should be fitted and worn only after consultation with a medical professional.
- Do not use over open wounds.
- If pain, swelling, changes in sensation or other unusual reactions occur while using this product, you should contact your doctor immediately.
- Do not use this device if it was damaged and/or packaging has been opened.
- NOTE:** Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device.

### APPLICATION INSTRUCTIONS

- Position splint on desired finger or thumb. Stay is malleable and may be contoured for individual patient needs.
- Secure finger tip strap and strap around the palm. Palm strap may be repositioned if necessary. If using splint for the thumb, the palm strap will wrap around the wrist.
- Secure strap around the poignet and the fixer.
- All straps should be secured to a snug but comfortable fit.

### CARE

To clean, Splint may be wiped down if necessary.

**NOT MADE WITH NATURAL RUBBER LATEX.**

**FOR SINGLE PATIENT USE ONLY**

**WARRANTY** DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale.

NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

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©2021 DJO, LLC 13-1326-0-00000 REV G - 2021-11-09