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PASTILLERO CON ALARMA

K40005

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PILL REMINDER ALARM

K40005

The pill reminder with alarm is a countdown timer up to 99 hours. It can remind user to take medication by beeping on scheduled time. There are three large compartment drawers for multiple medications.

INSTRUCTIONS FOR USE

- Battery is included, but if you need to replace the battery, unscrew the cover on the backside of the pill reminder. Insert a fresh LR44 battery (positive side up) into battery holder and screw the lid back.
- Using "H" and "M" keys, you can set how long it is before you need to take your next tablets.
- When the time is correct, press the red key to start the timer and the time shown on the display will flash.
- When the time is up, the pill reminder with alarm will beep. Press the red key once to turn it off. Reset the timer for the next occasion when you need to take your tablets. You don't need to open the cover to set next interval to take pill if the time interval is the same as the previous one. Just depress the red key again to start another cycle.
- To pause the timer, press the red key.
- To resume countdown, press the red key again.
- To reset the timer to zero, you need to pause the timer first and then press both "H" and "M" keys simultaneously (RESET).

MAINTENANCE

- This product is designed for minimum maintenance.
- Can be easily cleaned with a cloth.
- Do not clean it with water.

WARNINGS

- DO NOT use UV light to disinfect the product.
- Periodically inspect for wear and damage.

Any serious incident related to the product must be reported to the manufacturer and to the competent authority of the Member State in which the user and patient are established.

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