

## 1) COMPONENTES / COMPONENTS / KOMPONENTEN / COMPOSANTS / COMPONENTI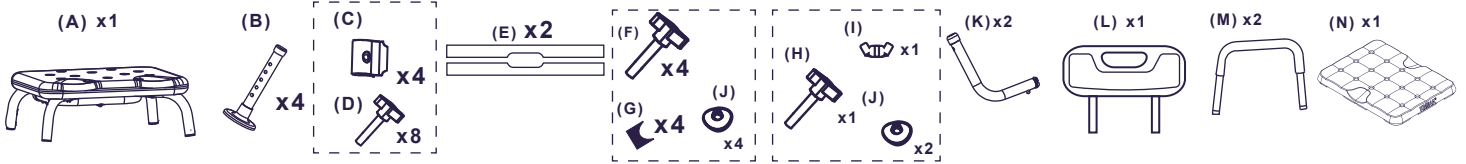

## 2) MONTAJE / ASSEMBLY / VERSAMMLUNG / MONTAGE / MONTAGGIO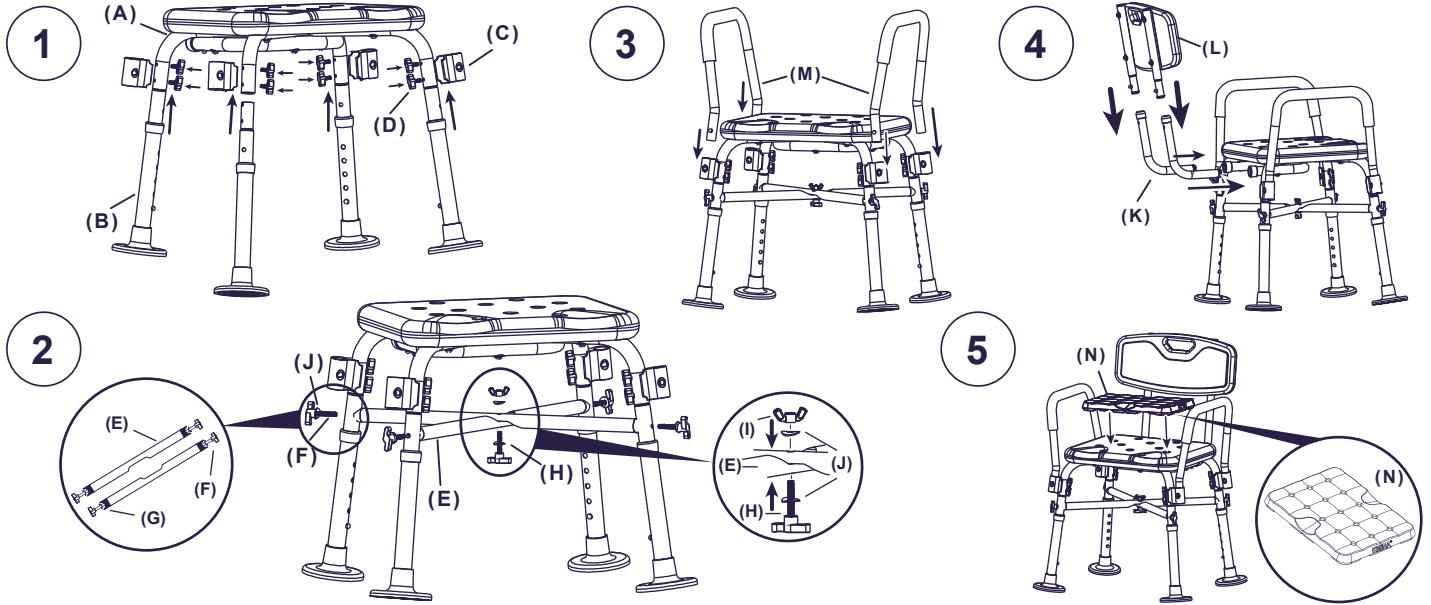

EN

	REFERENCIA
KMINA - COMFORT CHAIR WITH HANDLES (BLACK)	K30016
KMINA - COMFORT CHAIR WITH HANDLES (WHITE)	K30017

The **KMINA COMFORT CHAIR WITH HANDLES** with cushion should be used as a sitting aid in the bathroom or shower by elderly persons, persons with impaired mobility or stability, persons who are unable to stand during bathing, or persons with disabilities.

### CHARACTERISTICS

#### A) CHAIR:

- Chair weight: 10 lb.
- Maximum user weight: 330 lbs.
- Quick and easy adjustment from 18.3" to 22" height (including cushion) in 5 positions.
- Ergonomic seating surface 16" x 13" x 2". Perfect for attaching the KMINA cushion.
- Safe, comfortable and stable thanks to the wide tips.
- Corrosion-free thanks to its aluminium frame.

#### B) CUSHION :

- Cushion dimensions\*: 14"x11"x1.3"
- Cushion weight: 10 oz (approx).
- 100% waterproof and non-slip.

### ASSEMBLY

- Turn the seat (A) upside down and place the 4 legs (B) inside the frame tubes. Using the small screws (1.18") (D), adjust the brackets (C) to the tube so that the hollow part faces outwards. Make sure that all 4 legs are adjusted to the same height.
- Place the cross tubes (E) in an X-shape, with the flat part of both tubes together. Use the 4 big screws (1.77") (F) already assembled with their washers (J) and the black gaskets (G) to tighten them to the structure. Then tighten the medium screw (1.57") (H) in the centre of the cross tubes (E) using 1 washer (J) on each side of the tube and tightening it with the nut (I), as shown in picture 2.
- Insert the armrest (M) into the holes of the brackets (C) with the curved part of the tube facing outwards.
- Insert the L-shaped tubes (K) into the seat frame (A) by tightening them with the safety clip. Do the same with the backrest (L) into the L-shaped frame (K).
- The KMINA bath cushion (N) can now be placed on the chair.

### WARNINGS!

- Ensure that the spring-loaded buttons click into the holes in the legs.
- During the injection process, the dimensions may vary and some damage may occur.
- Do not stand on this product or tilt it. Do not use it as a climbing aid.
- Do not use for sideways transfers or partial loading. Do not use outdoors.
- Do not sit on it unless all four legs are in full contact with the ground.
- Use the bath seat cushion for its intended purpose.
- Check that the cushion fits correctly on the seat where it is to be used and does not slip.
- The dimensions of the cushion may vary during the injection process\*.
- Never use caustic products, paint strippers, solvents, sharp or cutting objects in direct contact with the cushion.
- Regularly inspect the product for normal wear and tear.
- Any serious incident involving the product must be reported to the manufacturer and to the competent authority of the Member State in which the user and the patient are established.

### CLEANING AND DISINFECTION

The product can be cleaned and disinfected with products available on the market. Always keep the product clean and wash it regularly.

- Clean before each use if you change users or have been in contact with body fluids.
- Never use drying presses or iron the cushion. It is not recommended to use: chlorine, bleach, alcohol disinfectants or abrasive products.

### CERTIFICATION

This product has the corresponding CE mark, in compliance with the Medical Device Directive 93/42 / EEC Class 1.

### WARRANTY

KMINA warrants this device to be free from defects. The warranty will remain in effect for 3 years from the date of original purchase by the consumer. The warranty does not extend to parts exposed to natural wear and tear that require periodic replacement.

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