

RISER ASSEMBLY / MONTAJE ELEVADOR / MONTAGGIO RIALZO / ASSEMBLÉ REHAUSSEUR / MONTAGE TOILETTENSTEIG

1

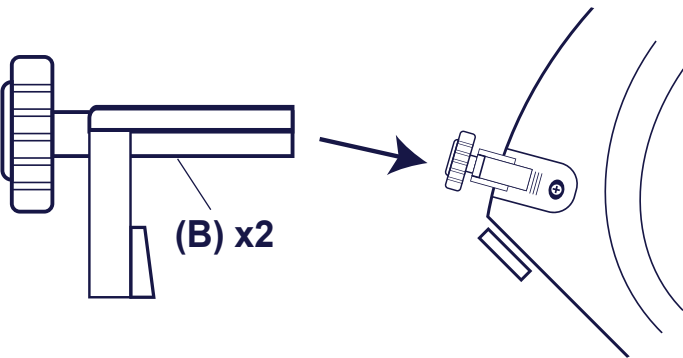2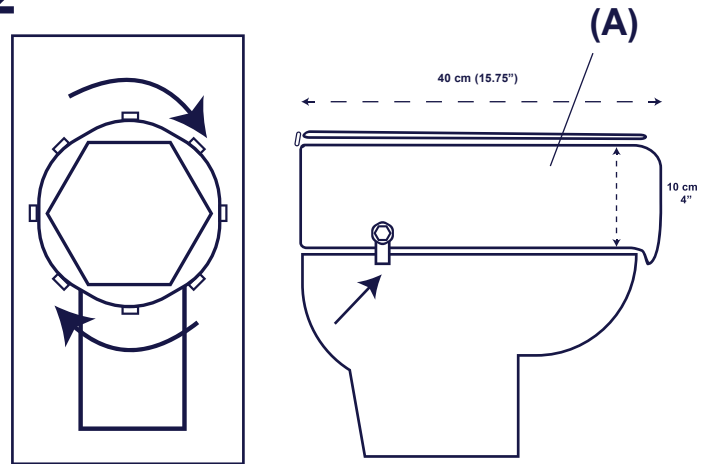

EN

REFERENCE

KMINA - TOILET RISER SOFT 4"

K30005

The KMINA - TOILET RISER SOFT 4" is perfect for people with difficulty getting up from the toilet, providing security and firmness, for the elderly, disabled or people with reduced mobility. Don't use this product without reading and understanding the instructions. If necessary, contact a healthcare professional, dealer or technical personnel first to avoid possible injury. Furthermore, it fits most classic oval toilets.

ASSEMBLY

The box contains: A. Toilet riser with cover and B. Screw-in brackets (2).

To install the toilet riser:

- Place the toilet riser face down on a flat surface and rotate the brackets into the appropriate side locations.
- Tighten and turn the screw handle clockwise to lock the brackets.
- Don't fully tighten the brackets, otherwise it will not be possible to position the seat.
- Lift the cover and place the toilet riser over the toilet. Make sure the inner edge is placed inside the toilet and the front edge is placed outside. The toilet riser should fit into the toilet in a perfect flat position.
- Rotate the head of the brackets clockwise until the toilet riser is secure and immovable.

CARE AND CLEANING

- All parts of the toilet riser can be cleaned with soap and hot water or disinfected with NON-ABRASIVE products.

GENERAL WARNINGS

- For a correct use of the device always consult your doctor or therapist.
- Make sure the product is delivered complete and keep it packed away from heat sources. Don't use the product if it is damaged.
- The raised toilet seat must not be used to support more than 300 lb.
- We recommend using only one user.
- Check the screw-in brackets and, if necessary, retighten.
- The device must be disinfected before reuse.
- DO NOT allow children to play or operate the device.
- DO NOT use the cover as a seat.
- Use the toilet lift only for the purpose for which it was designed. The company is not responsible in any way for any misuse of the toilet lift, or for injuries or damage caused as a result of it.
- Any serious incident related to the product must be reported to the manufacturer and to the competent authority of the Member State in which the user and patient are established.

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