## **KMINA PRO CANE USER MANUAL**

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## **KMINA COMFORT ERGONOMICAL CANE USER MANUAL**

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**KMINA PRO Walking Sticks** are designed for people who need extra support when walking. They are made of carbon fibre or lightweight aluminium, which makes them a light support. They have a soft grip finish and a lightweight tip.

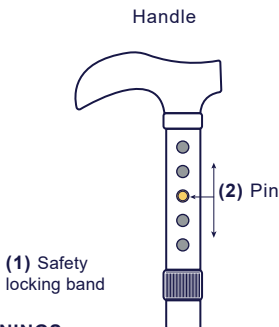
## HEIGHT ADJUSTMENT

Do not use this equipment without first reading and understanding these instructions. If needed, contact health care professional, dealer or technical personnel before attempting to use, otherwise injury may occur.


Adjust the height following the next instructions:

1. Stand on a flat surface and let your arm hang down naturally by your side. Set the cane height so the top part of the handle touches the inside part of the wrist.
2. Release the safety locking thread (1) by turning it clockwise once. Press the clip (2) and slide the tube up or down to the appropriate height in one of the 9 settings (6 in Ergonomic\*).
3. To set the height make sure that the pin is correctly placed in the hole and place the safety locking band again.

When using the cane, **grip the handle placing its long part backwards.**

### \*ERGONOMICAL Handle
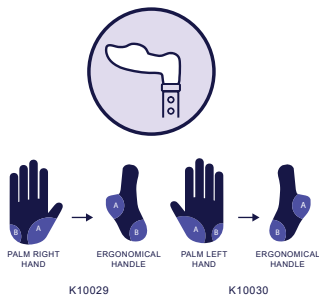
## WARNINGS

- Walking sticks must be regularly checked for any damage or wear. To avoid this, it is recommended to store the baton in the bag it comes with.
- **If the cane is foldable**, hold the whole stick when unfolding it, so it does not spring out and cause any damage or surprise to the user or others around.
- **If the cane is foldable**, keep fingers clear when folding it, so it do not get caught in any way.
- Take extra care when using on slippery surfaces, a walking stick cannot stop you from slipping.

- Do not apply excess weight to the stick and never exceed the indicated maximum weight.
- Walking sticks are solely for use as an aid to walk and they are not designed to bear the total body weight.
- Before using the stick, check if every part is securely connected and fixed. If there is any sign of cracking or bending, the stick should not be used.
- Always ensure the push-button height adjusting pin is correctly positioned and is securely locked in one of the holes before use.
- Ensure the safety locking band is tight when the stick is in use.
- Do not store this product in temperatures below freezing.
- Only use the walking stick for the intended purpose. KMINA cannot be held responsible for any misuse of the cane and consequent injury or damage.
- Any serious incident related to the product should be reported to the manufacturer and the competent authority of the Member State in which they are user and patient set.
- Keep the cane clean by wiping with a damp cloth on regular basis. Do not use abrasive cleaning materials.

## TECHNICAL SPECIFICATIONS

PRODUCT	CODE	MATERIAL	FOLDABLE?	HEIGHT	WEIGHT	FOLDING DIMENSION	MAX. USER WEIGHT
KMINA PRO Cane - Folding Fiber	K10018	CARBON FIBER	YES	32.3" - 37.4"	9 ± 1 oz	13"	220 lb
KMINA PRO Cane - Fiber	K10022	CARBON FIBER	NO	28" - 38.5"	8 ± 1 oz	NA	220 lb
KMINA PRO Cane - Folding Aluminium	K10019 K10020 K10021 K10031	ALLUMINIUM	YES	32.3" - 37.5"	11 ± 1 oz	13"	220 lb
KMINA PRO Cane - Aluminium	K10023 K10025 K10024 K10032	ALLUMINIUM	NO	30" - 36.5"	10 ± 1 oz	NA	220 lb
KMINA COMFORT Cane - Folding Ergonomical	K10029 K10030	ALLUMINIUM	YES	33" - 37.4"	16 ± 1 oz	11.4"	220 lb

CODE	
KMINA PRO CANE - Folding Carbon Fiber	K10018
KMINA PRO CANE - Folding Aluminium	K10019
KMINA PRO CANE - Fiber	K10022
KMINA PRO CANE - Folding Aluminium	K10023
KMINA PRO CANE - Blue Flowers	K10025
KMINA PRO CANE - Red Flowers	K10024
KMINA PRO CANE - Folding Blue Flowers	K10021
KMINA PRO CANE - Folding Red Flowers	K10020
KMINA PRO CANE - Wooden	K10031
KMINA PRO CANE - Folding Wooden	K10032
KMINA COMFORT CANE - Right Folding Ergonomical	K10029*
KMINA COMFORT CANE - Left Folding Ergonomical	K10030*

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