KMINA PRO OUTDOOR ROLLATOR USER MANUAL

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FOLDING SYSTEM

1. To unfold it, remove the hitch located in the rollator's later part and push down on the frame of the seat and handle so as to fix it. The rollator will be completely unfolded once you hear the click sound. In order to be sure about the completeness of the action, press the seat with your hand while grabbing one of the handles with the other.
2. To fold it, remove the front bag and push up the gray handle from the seat. Once folded, you would be able to grab it from the hitch located in the rollator's later part.
3. To remove the bag, unhook the tabs located at the front end of the outer seat frame.

HEIGHT ADJUSTMENT

The handlebar is adjustable to the user's height. To get the appropriate height:

1. Stand straight to one side of the rollator, with your shoulders relaxed and your arms at 15°.
2. Adjust the height at the same level of your wrists using the small nuts on the bottom of each of the handles. Once loosened, you will need to press the release buttons on both sides of the rollator. Release button as handle moves and retighten nut when desired height is achieved.

BRAKE SYSTEM

When the rollator is moving, the brakes work by pressing the brake-wire in direction to the handlebar.

1. To block the rollator brakes: Push down the brakes levers. When listened the click sound, the brakes will be correctly unblocked.
2. To unblock the rollator brakes: Push up the brakes levers. When listened the click sound, the brakes will be correctly unblocked.

The brake's pressure is adjusted by twisting the adjustment nut, located in the upper part from the brake pads, in the back wheel.

1. By twisting the nut to your left, the brake pressure will raise.
2. When turn clockwise, the pressure needed to stop the rollator movement will decrease.

TO SIT DOWN

1. Place the rollator on a solid, level surface. Block the brakes, sit up straight, and don't lean back.

OVERALL DEPTH	OVERALL WIDTH	OVERALL HEIGHT	FOLDING DIMENSION
27.5"	25.5"	40.5" - 35.2"	35.2"x27.5"x9.5"
			

SEAT HEIGHT	HANDLE DISTANCE	Ø FRONT WHEELS	Ø REAR WHEELS
23"	19.5"	10"	8"

HANDLE ADJUSTMENT RANGE					
MAX. - MIN. HEIGHT	40.5" - 35.2"	ADJUSTMENTS	5	ADJUSTMENTS DISTANCE	1.3"

BAG MAX. LOAD	MAX. USER WEIGHT	ROLLATOR WEIGHT
11 lb	300 lb	15.5 lb

WARNINGS

- This is a walking device for compensation and improvement respectively of restricted ability of walking. This rollator can be used indoors and outdoors.
 - Ensure the Rollator is fully open and secured in the correct position before use.
 - Ensure the handle is set to the correct height and fully secured. Test the Rollator for stability before applying full weight to the handles.
 - Test the brake mechanism before use. When the wheels are locked, the brake pad should prevent the wheels from turning. If the wheels turn while the brakes are locked, refer to adjusting the brake pressure section. When the brakes are released, the wheels should turn freely.
 - This walker is a walking aid only and should not be used as a device to be carried. Use the seat for sitting and resting.
 - Use only on even and solid surfaces, never on an incline or uneven ground.
 - Do not leave the Rollator unattended without locking the brakes.
 - Do not stop, turn or reverse on a ramp or incline. Do not make sharp turns.
 - Use bag only for transport of objects and do not exceed the maximum load: 11 lb.
 - Do not exceed the maximum user weight: 300 lb.
 - Do not expose the product to temperatures above 104°F for a long time, otherwise the aluminum tubes cover heat, which could lead to burns.
 - The Rollator should be inspected before each use to ensure it is free from damage and excessive wear. If you do find wear or damage to your Rollator, DO NOT USE.
 - Keep your Rollator clean. The aluminum tubes can be easily cleaned with a damp cloth. The fabric pieces must be washed by hand (maximum 86°F) and with neutral soap.
 - KMINA warrants that the Rollator is free from defects in material and workmanship. The warranty shall remain in effect for 2 years from the date of original consumer purchase. The warranty does not extend to parts that are exposed to natural wear and tear that will require periodic replacement such as brakes, wheels, handgrips and bags. This warranty does not cover the following:
 - Replacement parts supplied by anyone other than KMINA or the stockist.
 - Any failure of the unit during the warranty period if the failure is not caused by defect in material or workmanship or if the failure is caused by unreasonable use.
 - KMINA is not liable for incidental or consequential injury or damage property.
 - Should it be necessary to return the rollator, it is essential that it is adequately packed to protect it in transit, preferably in its original packaging, otherwise KMINA cannot accept any responsibility for transit damage to the unit.
- Any serious incident related to the product must be reported to the manufacturer and to the competent authority of the Member State in which the user and patient are established.
- This does not affect your statutory rights.
 - Please retain these instructions for future reference.

REFERENCIA	
KMINA PRO II ANDADOR	K10017

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