KMINA

**USER MANUAL
MANUAL DE USUARIO
BENUTZERHANDBUCH
MANUEL D'UTILISATION
MANUALE UTENTE**

**KMINA PRO - CANE WITH SEAT (BLACK)
KMINA PRO - BASTÓN CON ASIENTO (NEGRO)
KMINA PRO - GEHSTOCK MIT SITZ (SCHWARZ)
KMINA PRO - CANNE AVEC SIÈGE (NOIR)
KMINA PRO - BASTONE CON SEDILE (NERO)**

EN	REFERENCIA	
	KMINA PRO - CANE WITH SEAT (BLACK)	K10058

The **KMINA PRO - CANE WITH SEAT** is designed for people who need extra support to walk. The cane is adjustable in height and has a folding seat to sit down and rest. It is designed for the elderly, people with mobility problems, with knee pain or in rehabilitation. Materials: aluminum tubes, plastic handle and seat.

INSTRUCTIONS FOR USE

As a walking support, the cane with seat can be used like any other cane.
 1. *To unfold the chair:* Pull the lower part of the seat until the legs are completely open. The user should sit so that the handle of the cane is at the front. **IMPORTANT:** do not use the handle as a backrest.
 2. *To fold the chair:* pull the upper part of the seat and hook the clip at the bottom of the seat with the cane tube, leaving the seat completely upright.

WARNINGS

- Regularly check the cane with seat for damage or wear. Periodically check that all screws are tightened securely.
- Take care when using the cane on uneven or slippery surfaces to avoid possible falls.
- Do not apply excessive weight to the pole and never exceed the maximum weight rating.
- Do not exceed the maximum supported weight of the product, which is 100 kg.
- Before using the cane with seat, check that all parts are securely connected and fastened. In case of signs of breakage or damage, do not use the cane.
- Do not store the cane with seat at temperatures below freezing point.
- Use the cane with seat only for the purpose for which it was designed. The company is in no way responsible for any misuse of the cane, or for injury or damage resulting from its use.
- Do not bring the cane with seat close to abrasive materials.
- Any serious incident involving the product must be reported to the manufacturer and to the competent authority of the Member State in which the user and the patient are established.

CLEANING AND MAINTENANCE

- Keep your cane with seat clean using a damp cloth. Do not use abrasive cleaning materials.
- Do not keep in the sun for long periods of time.

WARRANTY

KMINA warrants this device to be free from defects. The warranty will remain in force for 3 years from the date of original purchase by the consumer. The warranty does not extend to parts exposed to natural wear and tear that require periodic replacement. This warranty does not cover:
 - Replacement parts supplied by anyone other than KMINA or the authorized dealer.
 - Any failure of the unit during the warranty period that is not caused by a defect in material or workmanship or if the failure is caused by misuse.
 - KMINA is not liable for accidental damage or damage to property.

HEIGHT	FOLDED DIMENSIONS	SEAT HEIGHT	SEAT WIDTH	COLOR	Ø TIP TUBE	USER WEIGHT
34" - 38"	33" x 12"	19.5" - 22.5"	9"	BLACK	7/8"	220 lb

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