



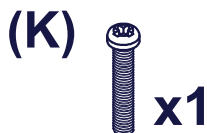
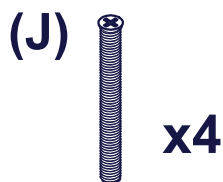
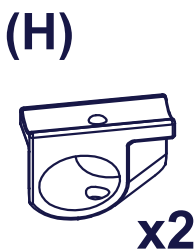
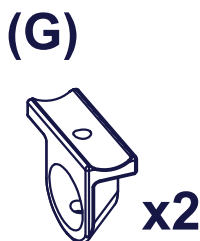
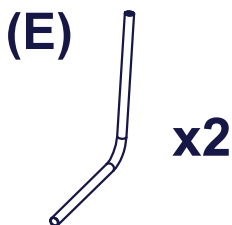
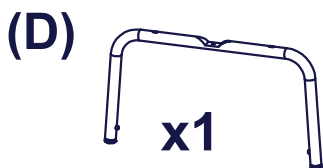
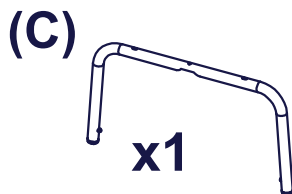
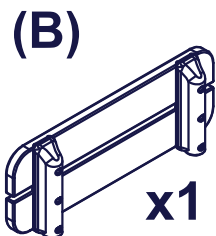
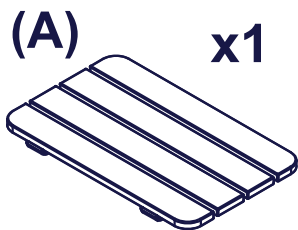
KMINA


**USER MANUAL
MANUAL DE USUARIO
BENUTZERHANDBUCH
MANUEL D'UTILISATION
MANUALE UTENTE**

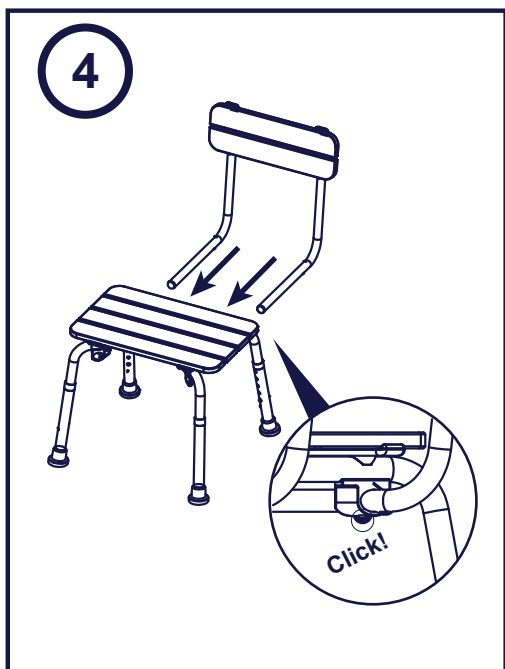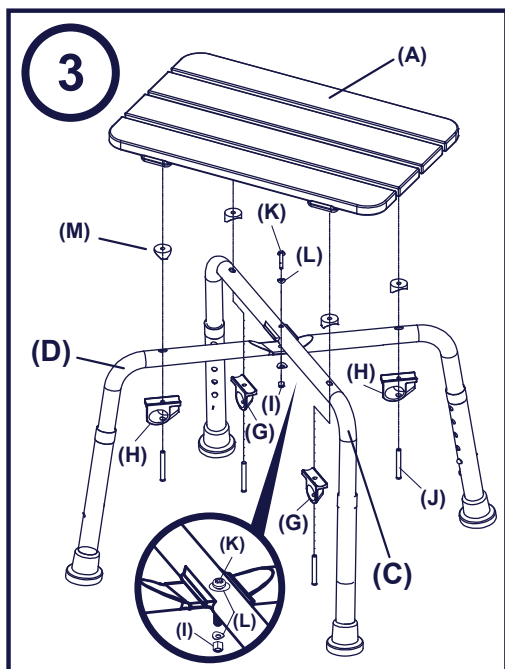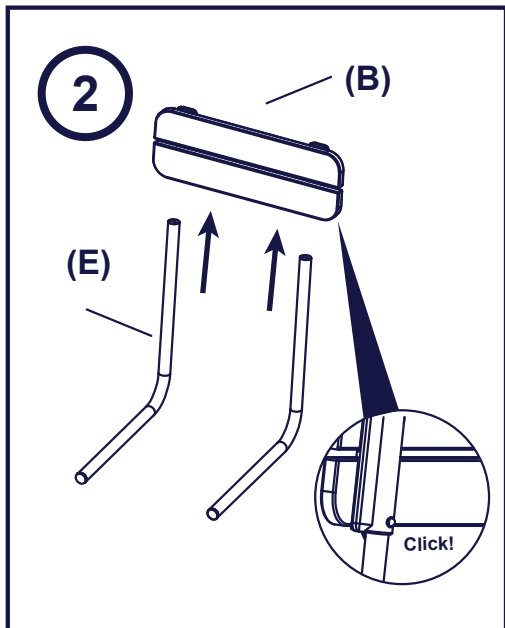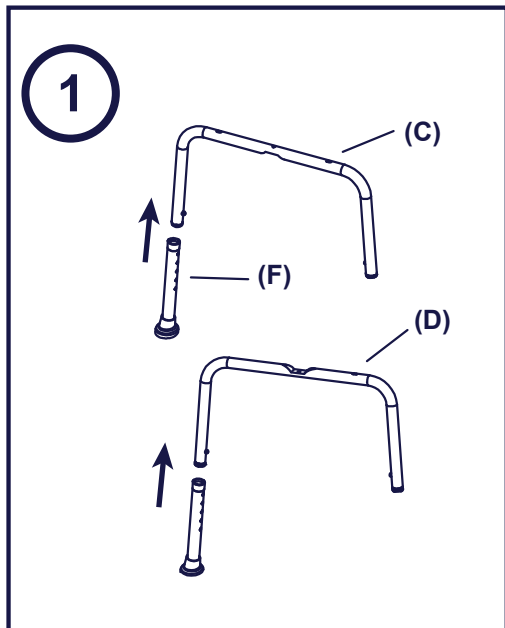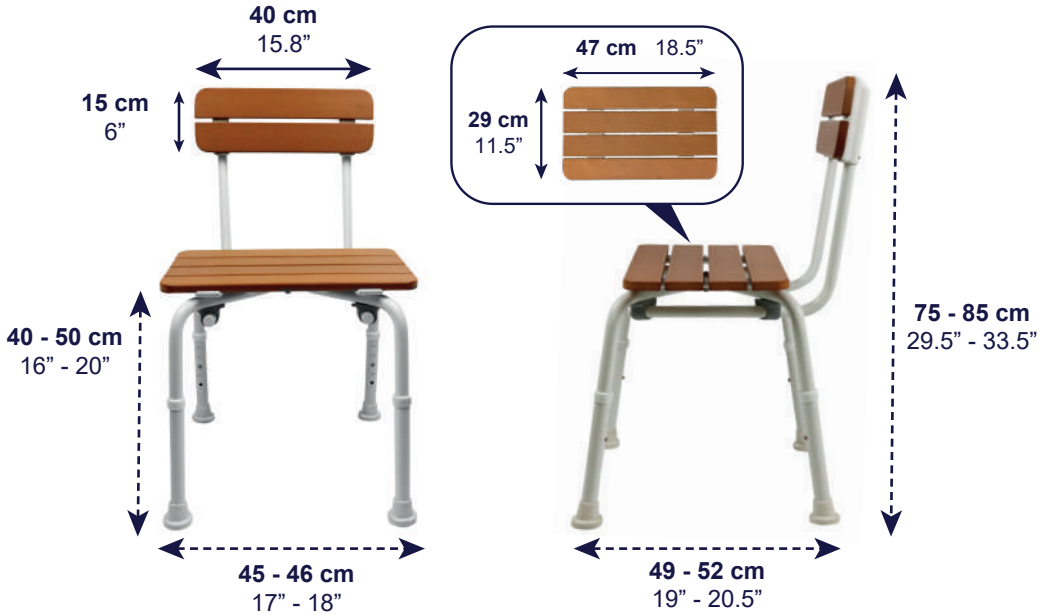**KMINA - SHOWER CHAIR WOOD EFFECT
KMINA - SILLA DE DUCHA EFECTO MADERA
KMINA - DUSCHSTUHL MIT HOLZOPTIK
KMINA - CHAISE DE DOUCHE EFFET BOIS
KMINA - SEDIA DA DOCCIA EFFETTO LEGNO**

EN

REFERENCE

KMINA - SHOWER CHAIR WOOD EFFECT

K30025

The **KMINA - SHOWER CHAIR WITH WOOD EFFECT** should be used as a sitting aid in the bathtub or shower. It is indicated for people who cannot or have problems standing, either due to lack of balance, lack of leg strength, or knee or hip problems. It should be used by users with sufficient torso stability and a maximum weight of 300 lbs.

FEATURES

- The seat height is easily and quickly adjustable in 5 positions from 16" - 20"
- Seat and backrest in High impact Polystyrene (HIPS), with natural faux timber.
- Safe and stable thanks to the sloping rubber ends.
- Corrosion-free thanks to aluminium frame.
- Weight: 9.4 lb.

ASSEMBLY

- 1. Adjusting the legs.** Insert each of the 4 legs (F) into the leg frames (C and D) and adjust the 4 legs to the same height, in one of their 5 settings.
- 2. Adjusting the backrest.** Place the two backrest tubes (E) into the holes on the back of the backrest (B), adjusting it until you hear the clamping click.
- 3. Seat adjustment.**
 - Place the leg structures, so that the C is above the D. Screw them with the help of the short screw (K) placing each washer (L) on each side of the leg structure and screwing the nut (I) until it is clamped.
 - Next, place each of the 4 fastening stops (M) in the upper holes of the leg structure, fit the seat (A), and adjust it by inserting the long screws (J) inside each of the supports of the backrest (G and H). *** Make sure you fit pieces G (x2) to frame for legs C and pieces H (x2) to frame D ***. The upper part of the pieces G and H must be completely engaged with the tubes of the leg structures. The tops of parts G and H must be fully engaged with the leg frame tubes. You can use the supplied tools, wrench (N) and Hex key (O) to help you adjust the seat correctly.
- 4. Assembling the chair.** Finally, insert the assembled backrest through the holes in the backrest supports (G and H) until you hear the clamping click, which indicates that the chair is fully engaged and ready for use.

WARNING!

- Make sure that all four legs are adjusted to the same height.
- Make sure that the spring-loaded buttons engage the holes in the legs with a click.
- Do not stand on this product or tilt it.
- Do not use as a climbing aid.
- Do not use for sideways transfers or partial loading.
- Do not sit on it unless all four legs are in full contact with the ground.
- Do not use outdoors.
- Any serious incident involving the product must be reported to the manufacturer and to the competent authority of the Member State in which the user and the patient are established.

CLEANING AND DISINFECTION

The product can be cleaned and disinfected with commercially available products. Always keep the product clean and wash it regularly.

1. Remove the legs.

2. Use water and a mild, non-abrasive detergent to clean the separate parts.
3. Disinfect by wiping all accessible surfaces with disinfectant.

WARRANTY INFORMATION

KMINA guarantees that this device is free from defects. The warranty shall remain valid for 3 years from the date of original purchase by the consumer. The warranty does not extend to parts exposed to natural wear and tear that require periodic replacement. This warranty does not cover:

- Replacement parts supplied by anyone other than KMINA or the authorised dealer.
- Any failure of the unit during the warranty period that is not caused by a defect in material or workmanship or if the failure is caused by misuse.
- KMINA is not liable for accidental damage or damage to property.

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