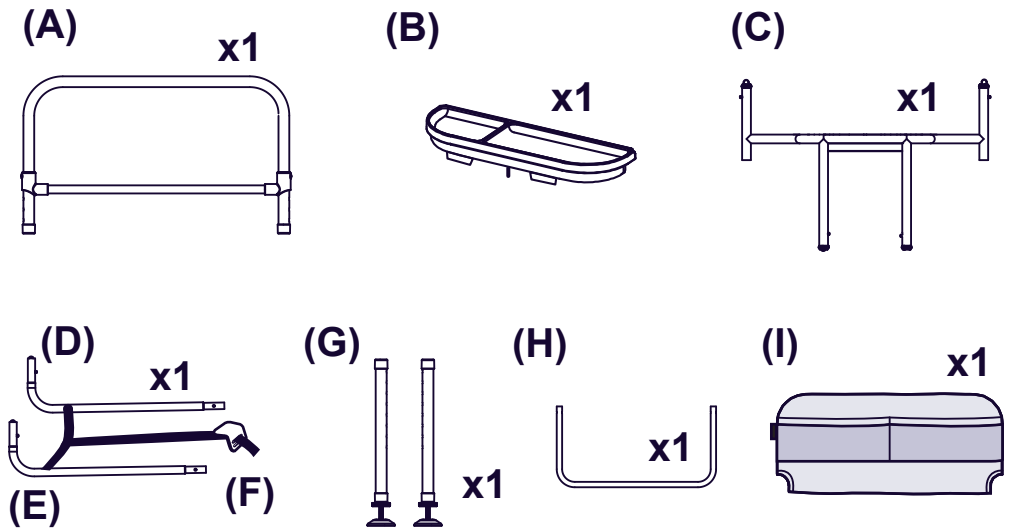
KMINA

USER MANUAL
MANUAL DE USUARIO
BENUTZERHANDBUCH
MANUEL D'UTILISATION
MANUALE UTENTE


KMINA - BED RAIL
KMINA - BARANDILLA PARA CAMA
KMINA - BETTGITTER
KMINA - BARRIÈRE DE LIT
KMINA - BARRIERA LETTO

1 COMPONENTS / COMPONENTES / KOMPONENTEN / COMPOSANTS / COMPONENTI

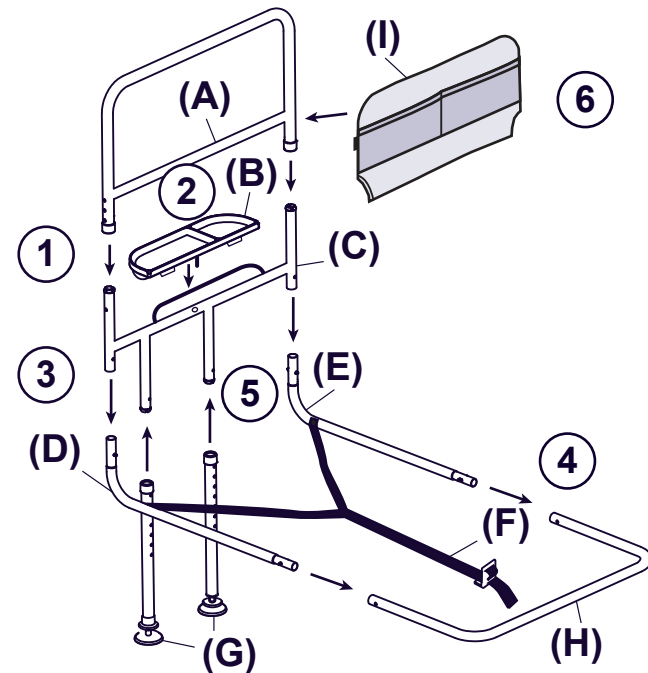2 ASSEMBLY / MONTAJE / VERSAMMLUNG / MONTAGE / MONTAGGIO

The **KMINA BED RAIL** is designed to prevent users from falling out of bed, as well as to provide additional support when getting in and out of bed. It is intended for the elderly, people with a disability or injury who need extra support in bed.

COMPONENTS

A. Top rail. B. Tray C. Bottom rail. D. Right L-shaped structure. E. Left L-shaped structure. F. Tape with metal stop. G. Legs. H. End frame. I. Fabric cover. **Materials:** steel, polyester nylon, PP, TPR.

ASSEMBLY

1. Insert the bottom rail (C) through the top rail (A) by means of the safety click in one of its three settings. Check that both tubes are in the same adjustment. The tray of the lower rail must be facing outwards.
2. Insert the tray (B) into the recess in the bottom rail (C) provided for this purpose by fitting it into the hole until it clicks into place.
3. Fit the shorter part of the right and left L-shaped structure (D and E) into the tubes of the bottom rail (C) by means of the safety clicks. Make sure that both clicks have exited through the correct hole and are securely fastened.
4. Attach the end frame (H) to the longer part of the L-shaped structure (D and E) using the locking clips. Make sure that both clips have come out of the correct hole and are securely fastened.
5. Insert both legs (G) through the lower holes in the bottom rail (C).
6. Attach the fabric cover (I) to the top rail (A).

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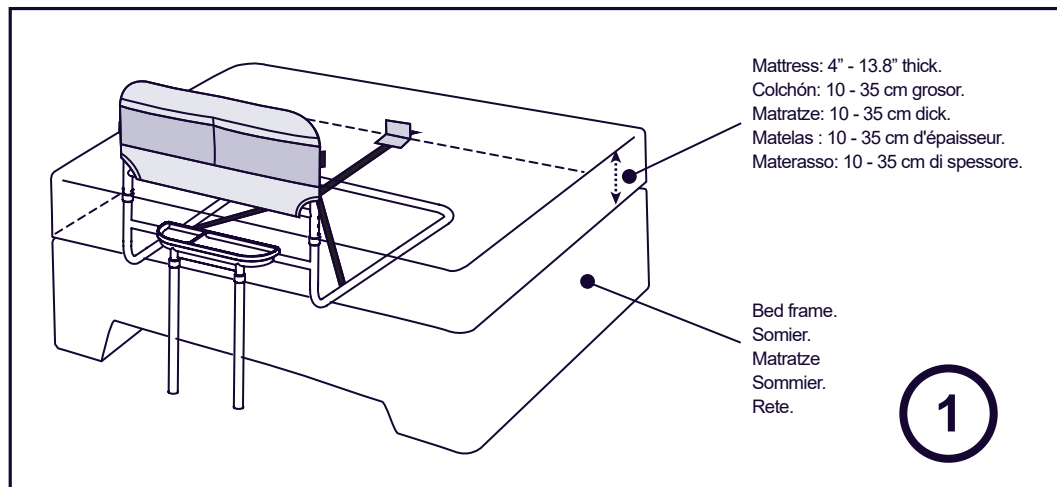
EN

Installation on the bed. Slide the bed rail under the mattress, with the legs at the side of the bed. It may be necessary to raise the mattress slightly to position the rail correctly. Push the rail all the way in until the outside of the rail is at the edge of the mattress. Make sure that the rail is completely slid under the mattress.

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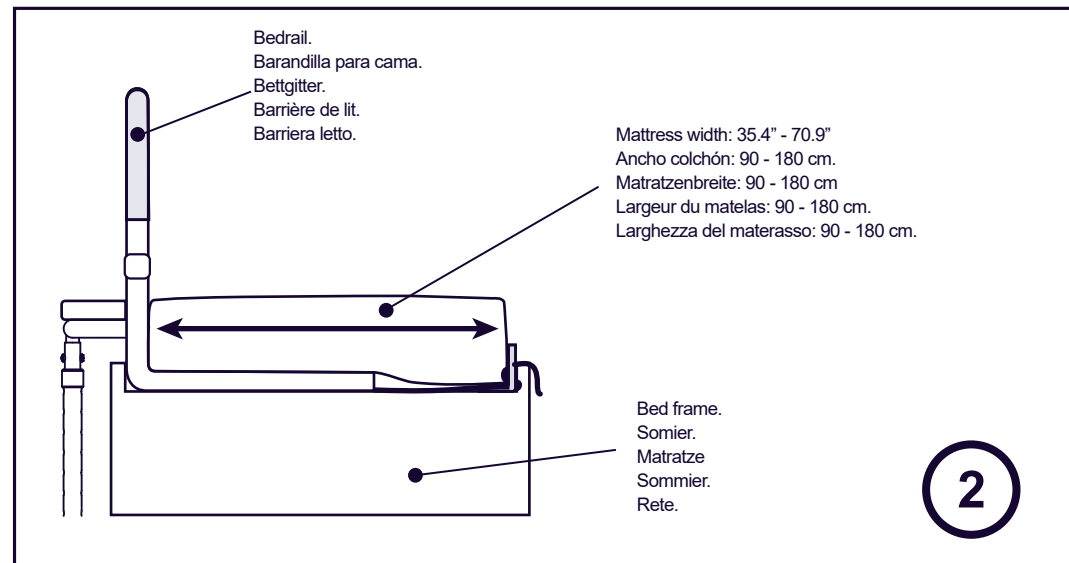
EN

Fitting the seat belt. Unfasten the safety belt and adjust the length with the pin so that the distance between the rail and the metal stop (E) is approximately equal to the width of the mattress.

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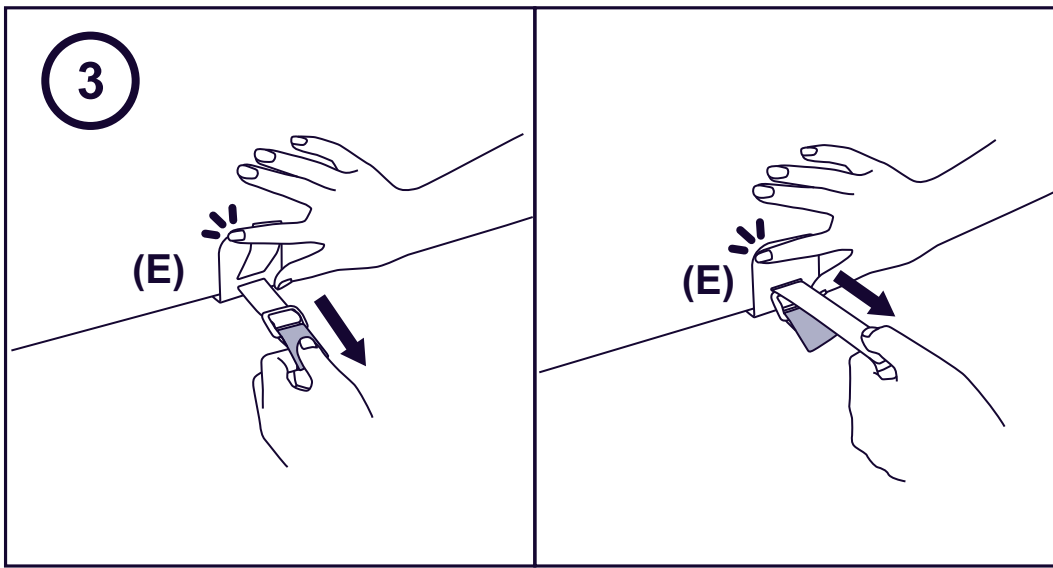
EN

Tighten the seat belt. Stretch the red strap until the metal stopper (E) is fully engaged with the mattress. Check that the rail is correctly slid under the mattress and tighten it securely with the safety belt.

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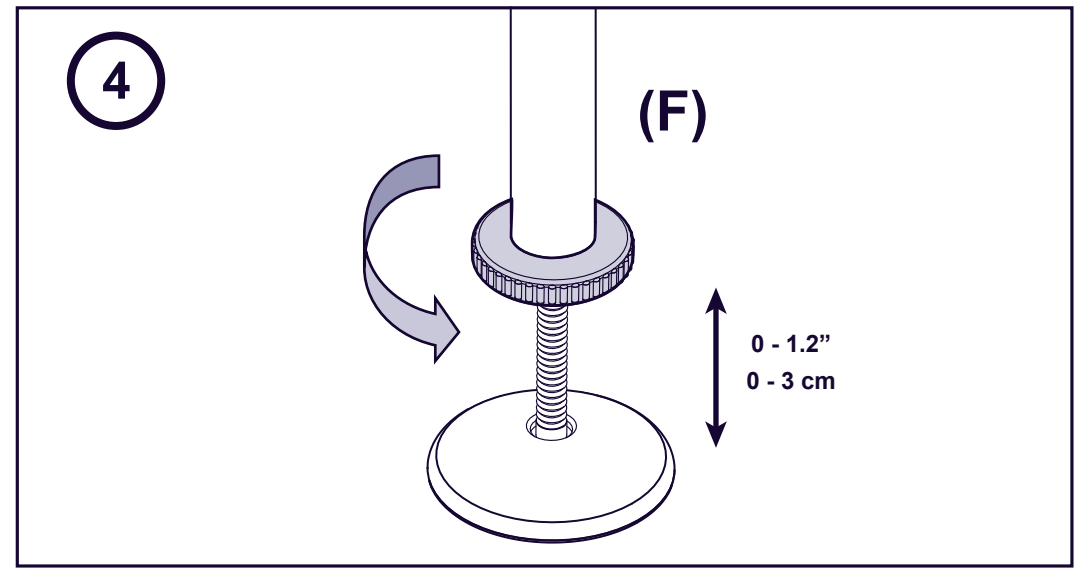
EN

Leg adjustment. The legs (F) can be extended by up to 1.2". To do this, turn the thread on the legs counterclockwise. The feet must be positioned vertically towards the floor.

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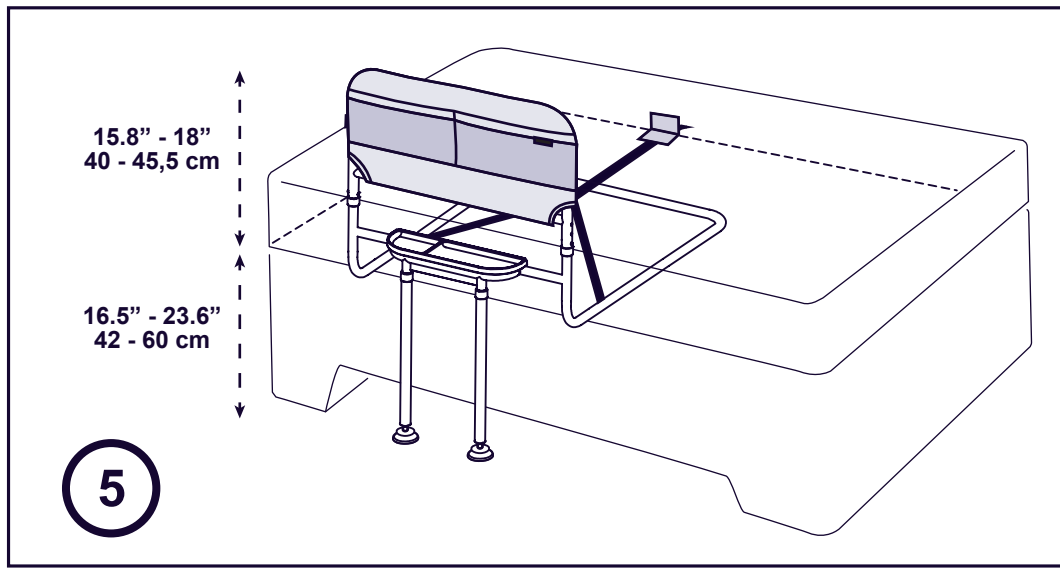
EN

Height adjustment. Select one of the 3 settings of the top rail (A) and adjust both parts to the same height using the safety click. Do the same with the legs (F), adjusting them to one of the 8 settings. Check that the safety click has fully extended through the desired hole and is securely fastened.

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EN

WARNINGS

- Read the instructions carefully before use to ensure proper installation.
- This product should only be installed by physically and mentally capable adults.
- Do not use two bed rails together on the same side of the bed to avoid being trapped between the two bed rails. The bed rail can be used on both sides of the bed if necessary to assist in getting in or out of bed from either side.
- The bed rail is for adult use only. It is NOT intended for use as a child safety system.
- This product is to be used as a support. Do not allow the user's full weight to fall on it. This bed rail is not a backrest.
- Do not use this product if any parts are missing or broken. Regularly check that all materials are in good condition and that none are missing or broken. Regularly check that the rail is correctly fitted to the bed and that the safety system is working properly.
- Do not use the bed rail without the cover. If the cover is being washed, keep an eye on the space between the bars. Do not put your head or any part of your body that could get caught between the bars.
- The bed rail is designed to prevent falls from the bed and to help users with sufficient mobility to get in and out of bed. If the user suffers from an injury, illness or is taking medication, he/she should consult a doctor before using it.
- If the bed rail is not used correctly, serious accidents can occur.
- Do not exceed the maximum supported user weight of 320 lb.
- Any serious incident related to the product should be reported to the manufacturer and the competent authority of the Member State in which they are user and patient set.

CLEANING AND MAINTENANCE

- Clean the bed rail with a damp cloth.
- The polyester fabric cover can be hand washed.
- Any replacement parts should be obtained from the manufacturer.

WARRANTY

KMINA warrants this device to be free from defects. The warranty shall remain in force for 3 years from the date of original purchase by the consumer. The warranty does not extend to parts exposed to natural wear and tear that require periodic replacement.

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