

# Modular Dynamic AFO

## Care & Use

To reshape, spot heat the carbon fiber with a heat gun up to 250° F (121° C) (higher temperatures will cause delamination). **Do not heat or reshape the PEEK rods or the carbon fiber PEEK attachments.** Attach padding kit. Select a laced shoe with a 5/8 in. (16 mm) heel and removable insole. Make sure to place the foot plate under the insole or foot orthotic in the shoe. For best results, use a custom foot orthotic.

## Warranty

1 year warranty\*

**Swedish Patent No. 531 069**

**U.S. Patent No. 7,766,851**

*Fillauer*®

[www.fillauer.com](http://www.fillauer.com)

## Fillauer LLC

2710 Amnicola Highway  
Chattanooga, TN 37406  
423.624.0946  
Fax 423.629.7936



© 2017 Fillauer LLC  
PM295 07-11-12/01-31-17