---

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Manufacturer takes responsibility for proper usage of the product only when the product was purchased from qualified REH4MAT employee or at specialized medical store!

REH4MAT Sp. z o.o. reserve the right to introduce technical and trading changes in the content of the instruction without warning.

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Product marked with the CE sign is Class I medical device, of rule 1, consistent with the requirements of Regulation (EU) 2017/745 of the European Parliament and of the Council of 5 April 2017 on medical devices.

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REH4MAT Sp. z o.o. Company introduced quality management system ISO 13485 regarding production, sales and service. The system is certified by Det Norske Veritas.

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## INTRODUCTION

FixCast Braces® are innovative pediatric orthoses, providing perfect support, immobilization and stabilization. Many sizes allow to fit brace even in a few months old baby. A particular braces are equipped with our innovative fabric MEMORYcast™, which is a thermoplastic panel for heat molding. Contrary to ordinary plaster cast, high plasticity of MEMORYcast™ allows to heat and mold it many times according to doctor's recommendations. Furthermore, the brace provides perfect lightweight immobilization. The compression and the constant temperature of the joints, reduce pain and improve healing. Due to this fact, FixCast Braces® are the best solution in case of orthopedic and neurological conditions in kids.

## INDICATIONS FOR USE

Torso support provides stabilization and relieving the trunk and spine. The support provides compression and stabilization of spine and proprioceptive input. Also, it provides warmth reducing inflammation. It is used in case of rheumatic, neurological and orthopedic disorders and injuries.

## CONTRAINDICATION

The **FixCast Braces®** shouldn't be used in case of:

- heat induced epilepsy,
- neoprene allergy or tends to skin inflammation

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## HOW THE ORTHOSIS MADE OF MEMORYCAST™ WORKS

This fabric is made of cover, loop and thin colorful neoprene layer and thermoplastic panel for heat molding. The panel has the same properties as rigid nylon and softens when temperature is approx. 60°C. It's very easy to do that, there is enough to place the brace in hot water, microwave or oven. Our thermoplastic panel is safe and biodegradable. Contrary to ordinary plaster cast, high plasticity of MEMORYcast allows to heat and mold it many times according to doctor's recommendations. Furthermore, the brace provides perfect lightweight immobilization. Due to this fact, FixCast Braces are the best solution in case of orthopedic and neurological conditions in kids.

**ATTENTION!** Do not bend the product before it is fully heated and the plate is fully plastic. Do not flat or bend the plate after the short initial heating. If there is even minimal resistance, stop shaping and put the product in the hot water to heat the plate more efficiently.

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The heating of MEMORYcast™ can take place in three different ways. However, it is recommended to heat the product using boiling water (METHOD I).

### **METHOD I - HOT WATER**

1. Prepare a dish (big enough to put the product in) and boil water in a kettle.
2. Pour **boiling water** into the prepared dish.
3. Loose straps and Velcro tapes.
4. Put a brace in **boiling water**. Remember to immerse the hole MEMORYcast™ panel. Wait approx. 1 min. The brace should be heated to 60 °C.
5. When a product is elastic, take it out of water and drain carefully. **Be careful not to burn yourself with hot water! Check if a brace is not too hot. If it is still hot, wait till its temperature will be safe for body but still plastic.**

**ATTENTION!** It is important to use boiling water. If water is not hot enough, the plate may be damaged during modeling.

**ATTENTION!** Do not bend the product before it is fully heated and the plate is fully plastic. Do not flat or bend the plate after the short initial heating. If there is even minimal resistance, stop shaping and put the product in the hot water to heat the plate more efficiently.

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## METHOD II - MICROWAVE OVEN

1. Loose straps and Velcro tapes.
2. Set a microwave oven on maximal power (usually 800 - 1000W).
3. Heat up the brace in 10 sec. cycles and each time check if the brace is elastic enough to form if easily. In each 10 sec. cycle place the brace in a different position and in a different place on a rotating tray. This is to evenly heat the brace. The brace will be ready to shape after few (or maximum after a dozen or so) cycles, depending on the kind of the brace and the power of the microwave oven.
4. When a thermoplastic panel MEMORYcast™ will be soft enough to make a form, check if the brace is not too hot. **Be careful not to burn yourself! If it is still hot, wait till its temperature will be safe for body but still plastic.**

## METHOD III - OVEN, HALOGEN OVEN OR OTHER

1. Set the temperature on approx. 150°C.
2. If a device will reach this temperature, loose straps and Velcro tapes and put it inside.
3. Heat up the brace for 5 min. After this time, take it out of the device and check if a thermoplastic panel MEMORYcast™ is soft. If not, heat it up for while.
4. When a thermoplastic panel MEMORYcast™ will be soft enough to mold it, check if the brace is not too hot. **Be careful not to burn yourself! If it is still hot, wait till its temperature will be safe for body but still plastic.**

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**ATTENTION!** There is possible to put the brace on, when the MEMORYcast™ thermoplastic fabric is able to shape but is not too hot. Despite the fact, that MEMORYcast™ panel does not have directly contact with skin and is covered with neoprene, there is ABSOLUTELY necessary to check its temperature before putting in on the skin! Do not put too hot orthosis on the skin, because the risk of burning occurs! In case of the brace is too hot after heating, there is necessary to wait for cooling it down.

There is possible to maintain the therapeutic limb's position when thermoplastic panel in not too hot and is possible to shape. There is necessary to wait only a few minutes after putting the brace on for cooling the MEMORYcast™. It maintains the patient's anatomic shape.

Wrap around design of the brace and Velcro straps provide wide range of circumferential adjustment and perfect fitting and stabilization.

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**ATTENTION!** There is possible to put the brace on, when the MEMORYcast™ thermoplastic fabric is able to shape but is not too hot. Despite the fact, that MEMORYcast™ panel does not have directly contact with skin and is covered with neoprene, there is **ABSOLUTELY** necessary to check its temperature before putting in on the skin! Do not put too hot orthosis on the skin, because the risk of burning occurs! In case of the brace is too hot after heating, there is necessary to wait for cooling it down.

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- The manual consists of essential information which is necessary for proper product's using, maintaining and cleaning. Please, keep the manual for future.
- Before using the products, read this manual carefully. Following the instructions included in the manual will provides product's safety and effectiveness.
- Proper use of the product requires diagnosis by physican or physical therapist.
- The Reh4Mat brace must be fitted by an authorized vender, physician or educated psyhiotherapist.
- Control using the product, especially in children, people with memory dysfunction, psychiatric and people with sensory disturbances.

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**Do not:**

- Use the product against its purpose of use
- Use the product in the wrong size
- Use the product by people and children with mental disorders without guardian control
- Use the product in people who are allergic to neoprene
- Use the product in case of damage or shape's deformation
- Attempt to service this pillow yourself
- Attempt to change the construction of the product
- Use the accessories which are not recommended by manufacturer

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ATTENTION! For single patient use only.

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ATTENTION! Pain is always the sign of warning. If the pain maintains or increases it is advisable to consult your doctor or physical therapist.

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ATTENTION! Before using the product, read this manual carefully. Before using the product, check if it has no signs of wear – no damage or tearing of straps, stitching. Only an undamaged product and proper use guarantee its right and safe working. The following information about the product should be kept in order to be able to use the instructions later.

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ATTENTION! Using of the product not according to the manual may cause serious complications

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ATTENTION! Never leave the product in a hot vehicle or other places where the product would be exposed to high temperatures.

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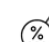
**ATTENTION!** Product works correctly only when the device is adjusted according to patient's height as well as weight. Correct adjustment is the most important due to the fact that only then the product is safe. It is forbidden to force child to keep specific position because he/she will feel uncomfortable and it may lead to wrong posture.

If the symptoms occur (for example skin allergies, intertrigo, bedsores), stop using the product immediately, take the user out of the product and lay them down. If the undesirable symptoms persist, contact a physician immediately. If the symptoms have disappeared, inform the attending physician about this fact in order to make a decision about the further rehabilitation process.

**ATTENTION!** Serious incidents involving this device should be reported both to manufacturer and to the competent authority in your country.

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**LABELLING:**

-  Declaration that the product meets the essential requirements
-  Catalog number
-  Batch number
-  Medical device
-  Please read the manual instructions before use
-  Attention!
-  Producer
-  Production date
-  Store in a dry place
-  Protect from sunlight
-  Storage temperature limitation
-  Relative humidity storage

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