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IMPORTANT

Read the instructions carefully before using this product and keep them if necessary.

User: If you are unable to read or understand any warnings, precautions or instructions, please contact your health care professional or distributor before using this product or risk and damage may result.

Consult your physician or health care professional before using any assistive device. He/she will advise and inform you on the most appropriate use in your case.

Distributor: this instruction manual must be given to the users of this product.

PRODUCT DESTINATION

The Rotating Bath Seat is a medical device. It is a technical bath aid for the elderly, people with reduced mobility, disabled, injured or who can easily lose their balance.

PRODUCT DESCRIPTION

SAFETY PRECAUTIONS

Failure to follow the safety precautions may result in the risk of serious injury.

- Don't exceed the maximum supported weight of 135 kg.
- Check all components are secured before use.
- Make sure that the product is in good working order.
- Visually check the product.
- Check the stability of the chair before use.
- Only use on a level ground.
- Check the chair for wear and tear.
- Do not lean out of the device to avoid rocking.
- In case of damaged, broken, worn or missing parts, contact the dealer.

MOUNTING

1. Remove contents from carton.
2. Assemble legs by inserting the height adjustment legs into the white rubber tip. Make sure to put the tips on the end without the brass push pin. Repeat for 3 other legs.
3. Insert the height adjustment leg into large outer leg by depressing the brass push pin. Repeat for 3 other legs.
4. Lay seat upside-down on the flat surface. Insert each leg into the seat depressing the brass push pin on the outer leg and inserting into

the bottom of the seat. A click can be heard when the leg is securely in place. Repeat for the 3 other legs.

5. Turn the bath bench right-side-up. Attach the soap tray by placing it between the legs and inserting the cross braces into the receptacles on the white leg collars.

USE

ADJUSTING LEG HEIGHT

- Before use, adjust height of each leg to your specific needs, making sure all legs are the same height. When desired height is reached, lock legs into place by aligning buttons into proper holes.
- To be sure legs are locked into place, push down on or pull up the legs.

ROTATING SEAT

To rotate seat, pull up on handle until seat is free of one of the 4 locked positions and rotate to desired position. Seat can rotate in both directions. Note : Built into the seat frame are 4 locking positions which will secure the seat in place to allow the user to safely stand up from or sit down on the seat. Make sure the seat is in one of the locked positions before standing or sitting.

REPLACING SEAT

The rotating seat and the two ball bearing rings that move the seat can be replaced if necessary. To remove seat and ball bearing rings, turn the lock knob (RTL12061-KNOB) under the seat (RTL 12061 SEAT) and ball bearing rings (RTL12061-RING). Replace seat, ball bearing rings and lock knob. Tighten until secure.

CARE AND MAINTENANCE

Failure to care for and maintain your product may result in the risk of serious injury.

- Clean the product regularly with lukewarm water and a soft cloth. Do not use abrasive products, detergents or solvent-based products.
- Dry with a clean, dry soft cloth.
- The product can be cleaned in an aqueous decontaminant machine at 80°C.
- It is recommended that the device is serviced at least annually.
- If the device is subject to heavy use and/or use in a multi user environment then we recommend the device is serviced more frequently.
- In case of malfunction, do not repair the product yourself. Consult an authorised dealer.

CHANGE OF OWNERSHIP

The product can be reused. It must first be cleaned and disinfected in accordance with hygiene instructions. When transferring your product, remember to hand over all the necessary technical documents to the new user. Beforehand, the product must be inspected by an authorised specialist.

RECYCLING

When the product has become unusable and you have to throw it away: please contact your specialist dealer. If you wish to carry out recycling for yourself, ask your local waste management company for disposal guidelines.

STORAGE

Failure to observe the storage conditions can lead to deterioration of the product and thus to the risk of serious injury.

- Do not store your product for an extended period of time near a heat source or in the sun (e.g. behind a window or near a radiator) or near a cold source.
- Keep away from all flames and sources of sparks.
- Respect the conditions for storing your product :
 - In a dry and temperate place.
 - Protect your product by packaging from dust, corrosion (e.g. abrasive elements, sand, sea water, salty air).
 - Store all removed parts together in the same place (or locate them if necessary) to avoid mixing them with other products when reassembling.
 - All components must be stored without any load (do not place any objects that are too heavy on the components of the product, do not jam anything...).

PERIOD OF USE

The life expectancy of this product is 5 years under normal use, safety and maintenance conditions for a user. Beyond this period, the product can be used for as long as it is still in good condition.

If the device is used in a multi user environment (such as a nursing home or hospital) then service life may be reduced.

FEATURES TECHNICAL

Overall dimensions	Width 38 cm x depth 38 cm x height 41-58 cm
Seat Dimensions	Width 38 cm x depth 36 cm x height 2 cm
Rotating seat : 360° with 90° lock	
Seat Material	Polyurethane
Height-adjustable legs	17.5 cm of adjustment in increments of 2.5 cm
Weight of bathing seat	3 kg
Maximum supported weight	135 kg

GUARANTEE

This product is guaranteed for two years from the date of purchase by the user. IDENTITIES will provide replacement parts for any part that has broken due to a manufacturing, design or material defect. Contact the distributor from whom you purchased the product in case you need to make a claim. Failures resulting from accidental damage, misuse, alteration, or wear and tear are not covered by this warranty. Contact the dealer from whom you purchased the product to apply the warranty.

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