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INSTRUCTIONS MANUAL EN

IMPORTANT

Read the instructions carefully before using this product and keep them if necessary.

User: If you are unable to read or understand any warnings, precautions or instructions, please contact your health care professional or distributor before using this product or risk and damage may result.

Consult your doctor or health care professional before using a walking aid. He/she will advise and inform you on the most appropriate walking technique for you.

Distributor: this leaflet must be given to the users of this product.

PRODUCT DESTINATION

The Neo Pop umbrella rollators are walking aids for people with reduced mobility. These products are intended for use indoors and outdoors and can be folded up for easy storage or transport.

PRODUCT DESCRIPTION

1. Handle
2. Locking pad
3. Knob
4. Wheel
5. Plastic skid

SAFETY PRECAUTIONS

Failure to follow the safety precautions may result in the risk of serious injury.

Do not exceed the maximum weight limit of 130 Kg.

Make sure all parts are securely fastened and in good working condition.

Regularly check the screws and fasteners of the rollator and tighten if necessary.

Before use, ensure that the rollator is fully open.

Make sure both handles are securely locked and at the same height. Test the stability of the walker before applying full weight to the handles.

Do not use the seat when the device is on an inclined or uneven surface.

Replace the wheels when they are very worn or damaged.

Caution: The rollator is intended solely as a walking aid and should not be used as a transport device. Do not sit on the rollator.

Do not apply more than 2kg of downward force on the locking pad.

Do not use the rollator on stairs or escalators.

Only use accessories and parts provided by the manufacturer.

Do not use the rollator to walk backward.

Do not make any adjustments to the rollator while it is in use.

All wheels must always remain in contact with the ground.

Be careful when going up curbs or using inclined, uneven, or slippery surfaces. Failure to follow this guideline may result in a significant risk of falls or injury.

Do not use the walker on wet, icy, or snowy slopes.

Beware of pinch points when folding and unfolding the device.

In case of damaged, broken, or worn parts, contact the retailer.

MOUNTING & USE

Opening the rollator:

As shown in Figure 1, pull the handles as far apart from each other as possible.

As shown in Figure 2, push the locking pad down to lock the walker in the open position.

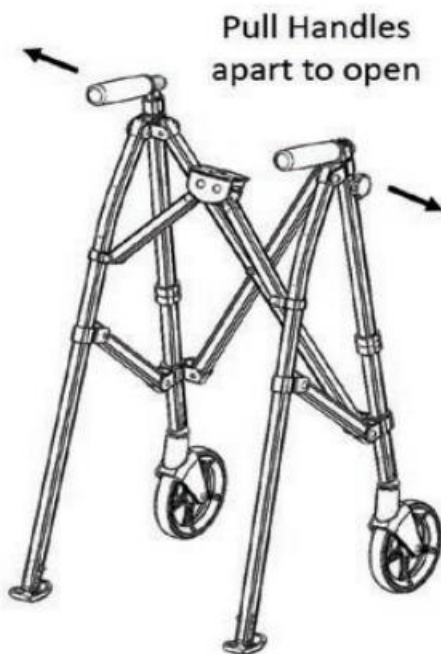
Figure 1

Figure 2

Closing the rollator:

As shown in Figure 3, pull up on the locking pad to begin closing the walker.

As shown in Figure 4, push the handles together to close the walker.

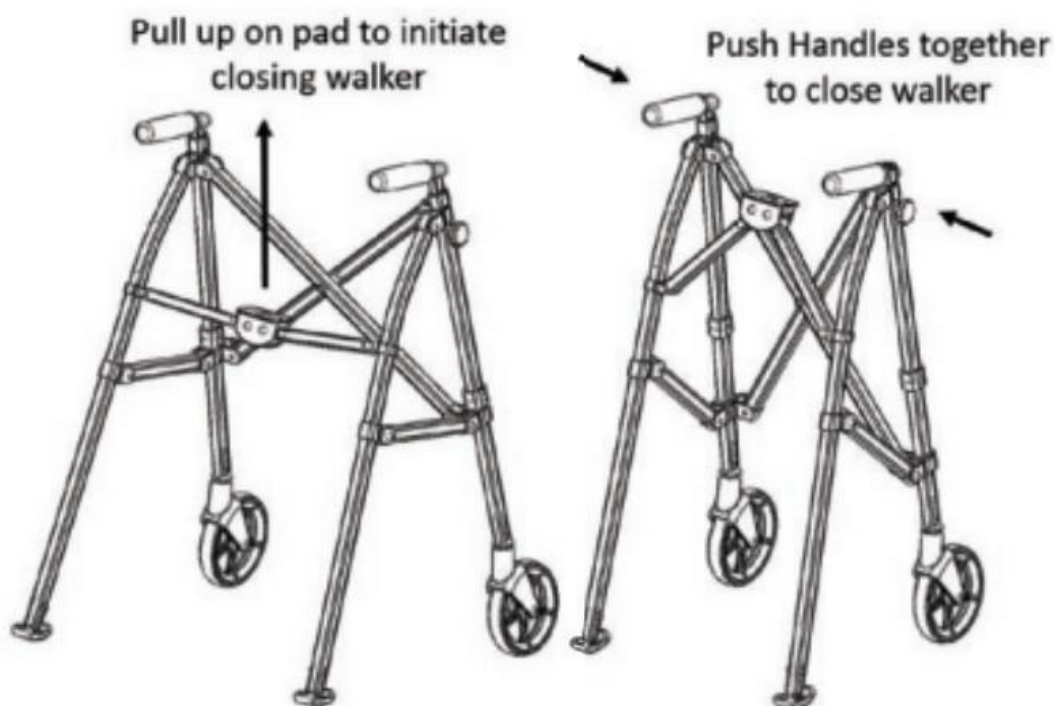
Figure 3

Figure 4

Height Adjustment:

As shown in Figure 5, remove the knob and bolt from the front leg tube.

Position the handle up or down to the desired height.

Align the hole in the handle tube with the hole in the front leg tube, reinsert the bolt, and tighten the knob as shown in Figure 5.

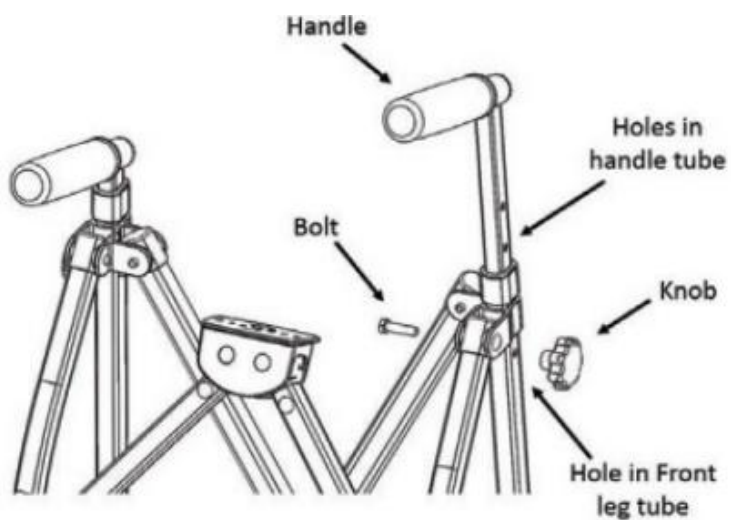
Figure 5

Skid replacement :

This information is for future reference. Do not replace the plastic feet (skids) on a newly purchased walker.

As shown in Figure 6, remove the bolts from the rear leg tube.

Pull the plastic feet (skids) downward to remove the worn plastic foot (skid).

Replace the worn plastic feet (skids) with new plastic feet (skids), and thread the bolts back into the new plastic feet (skids) as shown in the image below.

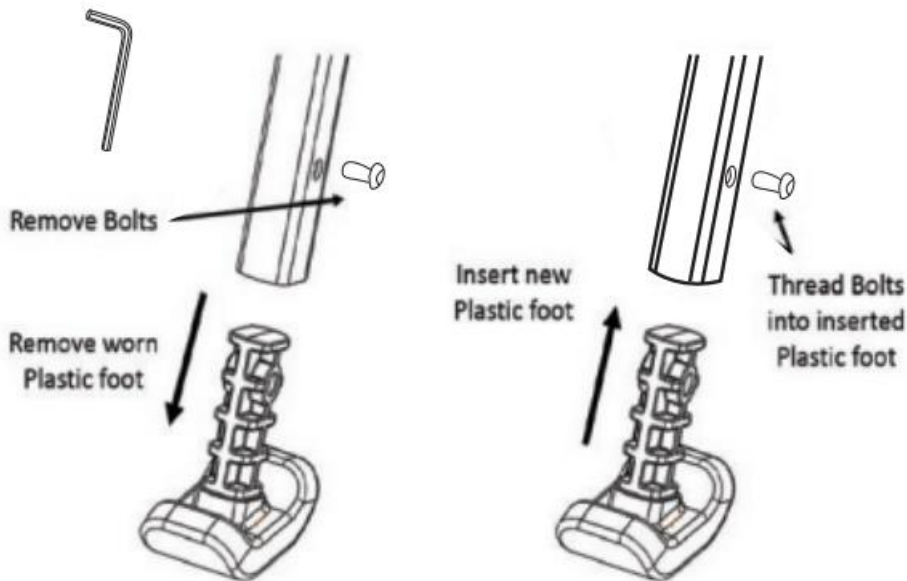
Figure 6

CARE AND MAINTENANCE

Failure to care for and monitor your product may result in the risk of serious injury.

- Clean the rollator regularly with warm water and a soft cloth. Do not use abrasive products, detergents or solvents.
- Dry with a clean, dry soft cloth.
- Make sure there is no accumulation of dirt or debris around the axle, brakes or wheels.
- Do not use cleaning products containing phenol or chlorine as this could damage the product.
- Lubricate the wheels with a little oil if they squeak.
- Periodically check the wear and strength of all screws and bolts. Replace all broken, damaged or worn parts immediately.
- Check that the brakes are working before each use.
- Check the parking brakes by pushing the handles into the locked position, then trying to push the rollator.
- If there is a malfunction, do not repair the rollator yourself. Consult an authorized dealer.
- **Checklist :**

To keep the rollator in good condition, check before each use, and check regularly as follows

Check the item	Anytime	Weekly	Monthly
Clip and knob for height adjustment	X		
Fasteners at wheels		X	
Cleanliness			X
Screws and fasteners		X	
Locking Pad condition and operation	X		
Stability of frame			X
Condition of the wheels and pads		X	

CHANGE OF OWNERSHIP

The product can be reused. It must first be cleaned and disinfected in accordance with hygiene instructions. When transferring the rollator, remember to hand over all the necessary technical documents to the new user. Beforehand, the product must be inspected by an authorized specialist.

RECYCLING

When the product has become unusable and you have to throw it away: please contact your specialist dealer. If you wish to carry out recycling for yourself, ask your local waste management company for disposal guidelines.

STORAGE

Failure to observe the storage conditions may lead to deterioration of the product and thus to the risk of serious injury.

Do not store your rollator for an extended period of time near a heat source or in the sun (e.g. behind a window or near a radiator) or near a cold source.

Keep away from all flames and sources of sparks.

Observe the conditions for storing and stowing your rollators.

- In a dry and temperate place
- Protect your rollator by packaging it from dust, corrosion (e.g. abrasive elements, sand, sea water, salty air).
- Store all removed parts together in the same place (or locate them if necessary) to avoid mixing them with other products when reassembling.
- All components must be stored without any load (do not place any objects that are too heavy on the rollator elements, do not jam anything...).

TECHNICAL SPECIFICATIONS

Overall dimensions	Width 63 cm x Depth 60 cm x Handles Height 83,5 to 101 cm in 5 positions
Folded dimensions	Width 17 cm x Depth 13 cm x Height 91,5 cm
Dimensions between legs	57 cm
Dimensions between handles	40 cm
Gross Weight	3.2 kg
Maximum weight	130 kg

PERIOD OF USE

The life expectancy of this product is 5 years under normal use, safety and maintenance conditions. Beyond this period, the product can be used for as long as it is still in good condition.

WARRANTY

This product is guaranteed for two years from the date of purchase by the user. IDENTITIES will provide replacement parts for any part that has broken due to a manufacturing, design or material defect. Contact the distributor from whom you purchased the product in case you need to make a claim. Failures resulting from accidental damage, misuse, alteration, or wear and tear are not covered by this warranty. Contact the dealer from whom you purchased the product to apply the warranty.

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