

FAULT FINDING

Should your Arctic bed fail to function, please check the following.

1. Check the Charger is plugged into a 13amp socket

2. Check to see if any of the following lights are showing on the charger.

Red - Charging Amber - Charged Green - Fully Charges

3. Check the connections of handset to the bed, the handset can be fitted to either the left or right hand side of the bed.

4. Ensure the bed frame is free of obstacles, ensuring all frames can lay flat to enable connections can be made.

Having checked the above, if the bed is still not functioning please contact


Centrobed on 01233 635353

Please could you ask for the Repairs Department



Arctic Handset Instructions

Each button has it's own function described below.

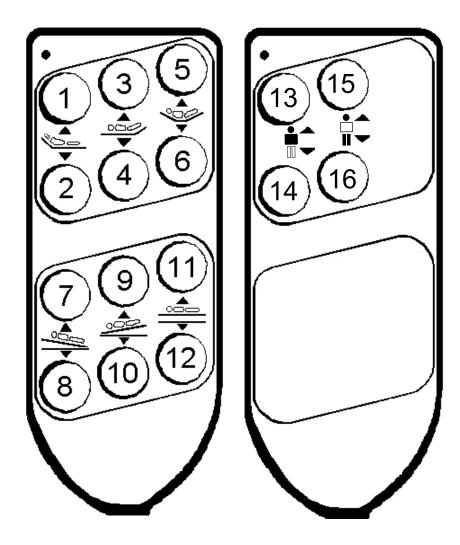

- 1. Raise to a sitting position
- 2. Lower from a sitting position
- 3. N/A
- 4. N/A
- 5. Positive Trendelenburg
- 6. Decrease Trendelenburg
- 7. Raise left side
- 8. Lower left side

- 9. Raise right side
- 10. Lower right side
- 11. Raise Bed
- 12. Lower bed
- 13. Left raise knee break
- 14. Left lower knee break
- 15. Right raise knee break
- 16. Right lower knee break