PreventID Dermatophyte (Ref. No KSTD001GP)

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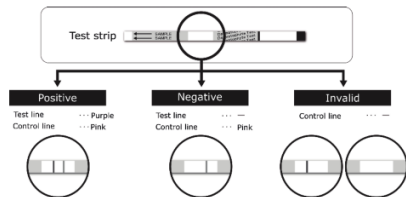
Test Interpretation (Fig. 5)

Positive: If a pink line appears in the control line zone and a purple line appears in the test line zone, it indicates a dermatophyte derived antigen positive result.

Negative: If a pink line appears in the control line zone and no visible line appears in the test line zone, it indicates a negative result. If a line appears in the test line zone after 30 minutes or longer, it indicates a negative result.

Invalid: If no pink line appears in the control line zone after 5 to 30 minutes, the test is invalid.

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References:

1. Ameen M, Lear JT, Madan V, Mohd Mustapa MF, Richardson M. British Association of Dermatologists' guidelines for the management of onychomycosis 2014. Br J Dermatol 2014; 171: 937-958.
2. Guidelines of care for superficial mycotic infections of the skin: onychomycosis. Guidelines/Outcomes Committee. American Academy of Dermatology. J Am Acad Dermatol 1996; 34: 116-121.
3. British Association of Dermatologists: Guidelines for treatment of onychomycosis, Roberts DT., Taylor WD., Boyle J., Br. J. Dermatol., 148, 402-410, 2003
4. Screening for tinea unguium by Dermatophyte Test Strip, Y. Tsunemi, et al., Br. J. Dermatol. , 170, 328-331, 2014
5. Clinical study of Dermatophyte Test Strip, an immunochromatographic method, to detect tinea unguium dermatophytes, Y. Tsunemi, M. Hiruma, , J Dermatol. 2016 Dec;43(12):1417-1423.

Temperature limitation	Manufacturer
In vitro diagnostic device	Lot number
Catalogue number	Expiry date
Keep away from sunlight	Do not reuse
Read user instructions	Contains sufficient for <n> tests

Status: 2019-02-28