




Instructions for Use

ICEFORM® SLEEVE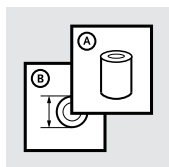

		3
EN	Instructions for Use	4
DE	Gebrauchsanweisung	5
FR	Notice d'utilisation	6
ES	Instrucciones para el uso	7
IT	Istruzioni per l'uso	8
NO	Bruksanvisning	9
DA	Brugsanvisning	10
SV	Bruksanvisning	11
EL	Οδηγίες Χρήσης	12
FI	Käyttöohjeet	13
NL	Gebruiksaanwijzing	14
PT	Instruções de Utilização	15
PL	Instrukcja użytkowania	16
CS	Návod k použití	17
TR	Kullanım Talimatları	18
RU	Инструкция по использованию	19
JA	取扱説明書	20
ZH	中文说明书	21
KO	사용 설명서	22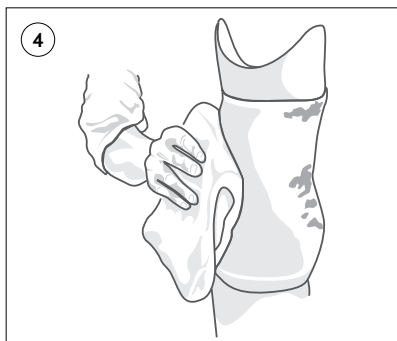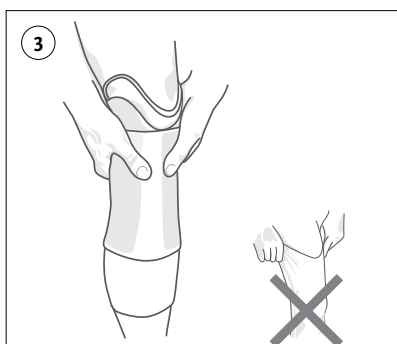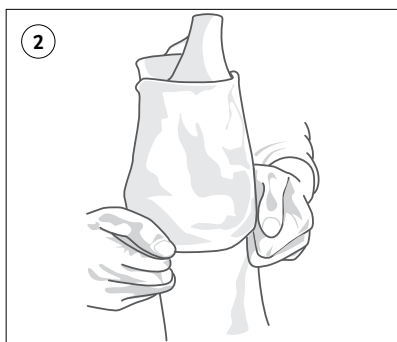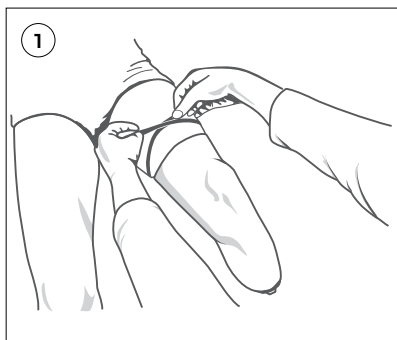

ENGLISH

1. Measure circumference 15cm (6") above mid-patella and choose the correct size. See sizing chart below (**Figure 1**).

Size	Measurement (cm)	Measurement (inches)
35	35 - 41	13¾ - 16"
41	41 - 49	16 - 19¼"
49	49 - 55	19¼" - 21¾"
55	55 - 62	21¾ - 24½"
62	62 - 76	24½ - 30"

2. Turn the ICEFORM sleeve inside out, half way, with TPE of distal end half-exposed. Stretch the sleeve over the brim of the socket and roll distal end completely down (**Figure 2**).
3. Roll down upper part of the sleeve until socket brim is completely exposed. Step into the socket and make sure the residual limb is positioned correctly in the socket. Roll the sleeve up with both hands. Do not pull on the edges as this may place strain on the skin (**Figure 3**).
4. Wash the ICEFORM sleeve daily, especially on the inside. To do so, remove the liner, turn the liner inside out and wash it with pH balanced, 100% fragrance- and dye-free soap. Be sure to rinse with plenty of water, and wipe dry with a towel. The sleeve can also be rolled down onto the socket and washed (**Figure 4**).
5. Sealant loops on the inside of the sleeve should be located on different places on the thigh every other day to prevent skin irritation at the location of the sealant loops.

All ICEFORM sleeves are minimum 35cm long (14").

Note: Excessive heat may damage the sleeve. Take care not to scrub the surface of the sleeve, since that can roughen the surface which can cause skin irritation.

Note: It is not recommended to use soap/creams/lotion that contain animal fats or oil or hydrocarbon oils.

WARRANTY: ICEFORM Sleeve comes with a 3 months warranty against defects in material and workmanship.

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EN – Caution: Össur products and components are designed and tested according to the applicable official standards or an in-house defined standard when no official standard applies. Compatibility and compliance with these standard is achieved only when Össur products are used with other recommended Össur components. If un-usual movement or product wear is detected in a structural part of a device at any time, the patient should be instructed to immediately discontinue use of the device and consult his/her clinical specialist. This product has been designed and tested based on single patient usage. This device should NOT be used by multiple patients. If any problems occur with the use of this product, immediately contact your medical professional.

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