Instructions for Use


Flex-Run™ Junior

Flex-Run™

Cheetah® Xceed

		3
EN	Instructions for Use	4
DE	Gebrauchsanweisung	9
FR	Notice d'utilisation	14
ES	Instrucciones para el uso	20
IT	Istruzioni per l'uso	26
NO	Bruksanvisning	31
DA	Brugsanvisning	37
SV	Bruksanvisning	42
EL	Οδηγίες Χρήσης	47
FI	Käyttöohjeet	53
NL	Gebruiksaanwijzing	58
PT	Instruções de Utilização	64
PL	Instrukcja użytkowania	70
CS	Návod k použití	76
TR	Kullanım Talimatları	81
RU	Инструкция по использованию	86
JA	取扱説明書	91
ZH	中文说明书	96
KO	사용 설명서	101
AR	تعليمات الاستخدام	107

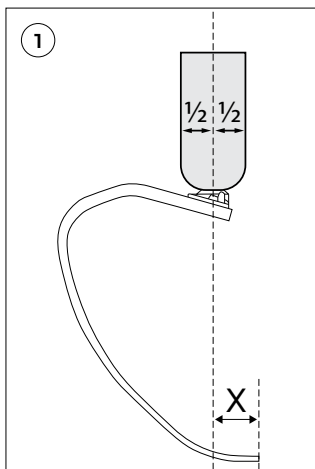ENGLISH

Medical Device

PRODUCT DESCRIPTION

Thank you for choosing an Össur Product.

The device is a prosthetic foot for sports activities. This document covers the following devices.

Flex-Run Junior

Flex-Run (Standard and Large)

Cheetah Xceed.

INDICATIONS FOR USE

- Lower limb amputation.
- Lower limb deficiency.

CONTRAINDICATIONS

None known.

INTENDED USE

This device is a modular component for a lower limb prosthesis that replaces the function or aesthetics of a missing limb. Compatibility of components and the overall prosthesis must be evaluated by a medical professional.

This device is for single patient use.

The device is intended for high to extreme impact levels, e.g. running, sprinting and recreational sports.

GENERAL SAFETY INSTRUCTIONS

Inform your patient about everything in this document that is required for safe use of this product.

WARNING: In cases of functional change or functional loss, the patient should stop using the device and contact a healthcare professional.

WARNING: If the device shows signs of damage or wear hindering its normal functions, the patient should stop using the device and contact a healthcare professional.

PRODUCT SELECTION

Please refer to the selection chart below to determine the appropriate category.

WARNING: Do not exceed weight limit. Risk of device failure.

Incorrect category selection may also result in poor device function.

Flex-Run Junior

Weight kg	15-24	25-34	35-45	46-55	56-67	68-84
Weight lbs	33-53	54-75	76-99	100-121	122-147	148-185
High to Extreme Impact Level	1	2	3	4	5	6

Flex-Run

Weight kg	37-44	45-52	53-59	60-68	69-77	78-88	89-100	101-116	117-130
Weight lbs	81-96	97-115	116-130	131-150	151-170	171-194	195-220	221-256	257-287
High to Extreme Impact Level	1	2	3	4	5	6	7	8	9

Cheetah Xceed

Weight kg	45-52	53-59	60-68	69-77	78-88	89-100	101-116	117-130	131-147
Weight lbs	99-115	116-130	131-150	151-170	171-194	195-220	221-256	257-287	288-325
High to Extreme Impact Level	1	2	3	4	5	6	7	8	9

ASSEMBLY INSTRUCTIONS

CAUTION: Epoxy can cause allergic reactions when in direct contact with body.

CAUTION: There may be a cutting hazard due to sharp edges or splinters on the device.

Attaching pyramid (Flex-Run Junior and Flex-Run)

If the foot is supplied without an adapter a suitable Össur adapter set must be used – see table below for compatibility.

WARNING: Do not exceed adapter weight limit - see table below.

Attachment plate must be added when using male or female adapter.

Roughen top surface of the foot and attachment plate.

Mix epoxy and apply to contact surfaces on foot and attachment plate.

Bond both parts together and clamp securely using a center bolt and the adapter.

After resin has set, remove bolt and adapter, then trim off excess resin

Clean parts, assemble, using threadlocker and torque as per following specifications.

Part number	Description	Weight limit	Center Bolt Torque	Bolt Size	Compatibility
FWX00030	Male Pyramid Adapter Set for cat 1 - 6	88 kg	41 Nm	M10	Both
CM150161	Female Pyramid Adapter Set for cat 1 - 6	88 kg	41 Nm	M10	Both
CM150162	Female Pyramid Adapter Set for cat 7 - 9	130 kg	43 Nm	M12	Flex-Run
FWX00031	Junior Male Pyramid Adapter Set for cat 1-3	45 kg	28 Nm	M8	Flex-Run Junior
FWX40001	Attachment Plate, for cat 1-6	-	-	M8/ M10	Both
FWX40002	Attachment Plate, for cat 7-9	-	-	M12	Flex-Run

Female Pyramid Adapter Set Screw torque is 15 Nm.
Flex-Run Large is compatible with Össur 4-hole adapters.

Attaching adapters (Cheetah Xceed)

Cheetah Xceed is compatible with Össur 4-hole adapters. The device should only be used with adapters with weight limit of 166 kg.

Nike Soles

The device can be fitted with a Nike spike pad or Nike sole. Refer to instructions for relevant sole/pad for assembly instructions.

ALIGNMENT

Bench Alignment

Sagittal plane

1. Put the ground contact point of the foot module in front of the alignment reference line according to dimension X (**Figure 1**) in the table below.
2. Find the applicable socket flexion.

Coronal plane

1. Set the applicable socket abduction/adduction
2. Set for up to 5° of external rotation.
3. Set the correct height - see Compression in table below to compensate for vertical compression.

Device	Dimension X	Compression
Flex-Run Junior	25-40 mm	10-20 mm
Flex-Run	25-40 mm	30-40 mm
Cheetah Xceed	50-60 mm	30-40 mm

Dynamic Alignment

Physical ability and the length of residual limb will have an effect on alignment. Optimal alignment reduces energy expenditure, and reduces muscle and ligament strain.

Adjust alignment to improve gait symmetry.

The A-P alignment and vertical deflection from socket to foot can have an effect on stride length.

USAGE

Troubleshooting

Make sure socket fit and suspension are optimal. Examine running symmetry at patient's preferred running speed.

If the duration of loading the prosthesis is longer compared to the contralateral side:

- The foot module is too soft
- The prosthesis is too short
- The foot is aligned too far anteriorly

If the duration of loading the prosthesis is shorter compared to the contralateral side:

- The foot module is too stiff
- The prosthesis is too long
- The foot is aligned too far posteriorly.

NOTE: If you move the A-P position of the foot, there will be an effect on hip flexion, stride length and trunk motion.

Maintenance

The device must be examined for splinters and/or delamination regularly by a medical professional. Interval should be determined based on patient activity.

Cleaning and care

Clean with a soft cloth and a mild soap.

ENVIRONMENTAL CONDITIONS

The device can be used in temperatures between -15°C to 50°C.

The device is waterproof.

A waterproof device can be used in a wet or humid environment and temporarily submerged in shallow fresh water.

Dry thoroughly after contact with fresh water.

No salt water or chlorinated water allowed.

Clean with fresh water in case of accidental exposure to salt water or chlorinated water.

REPORT A SERIOUS INCIDENT

Important notice to users and/or patients established in Europe:

The user and/or patient must report any serious incident that has occurred in relation to the device to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

DISPOSAL

The device and packaging must be disposed of in accordance with respective local or national environmental regulations.

LIABILITY

Össur does not assume liability for the following:

- Device not maintained as instructed by the instructions for use.
- Device assembled with components from other manufacturers.
- Device used outside of recommended use condition, application, or environment.

Compliance

This device has been tested according to ISO 10328 standard to two million load cycles.

The load level specification is shown in the tables below.

The device service life cannot be determined since load varies depending on the sport and patient activity level.

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