## Instructions for Use

---

Flex-Foot Cheetah<sup>®</sup>


Cheetah<sup>®</sup> Xtend

Cheetah<sup>®</sup> Xtreme






Cheetah<sup>®</sup> Xcel

Cheetah<sup>®</sup> Xpanse

Cheetah<sup>®</sup> Junior

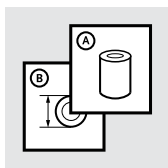		3
EN	Instructions for Use	4
DE	Gebrauchsanweisung	9
FR	Notice d'utilisation	15
ES	Instrucciones para el uso	21
IT	Istruzioni per l'uso	26
NO	Bruksanvisning	32
DA	Brugsanvisning	37
SV	Bruksanvisning	42
EL	Οδηγίες Χρήσης	47
FI	Käyttöohjeet	52
NL	Gebruiksaanwijzing	57
PT	Instruções de Utilização	62
PL	Instrukcja użytkownika	67
CS	Návod k použití	72
TR	Kullanım Talimatları	77
RU	Инструкция по использованию	82
JA	取扱説明書	88
ZH	中文说明书	93
KO	사용 설명서	98
AR	تعليمات الاستخدام	103
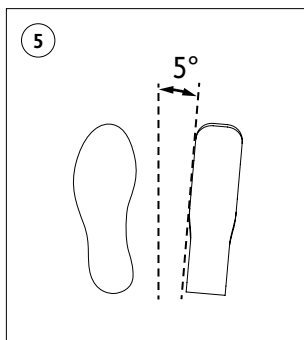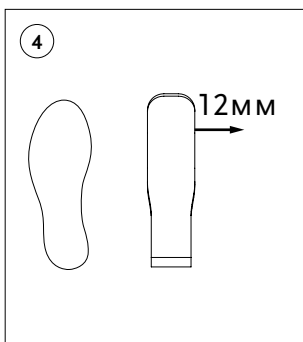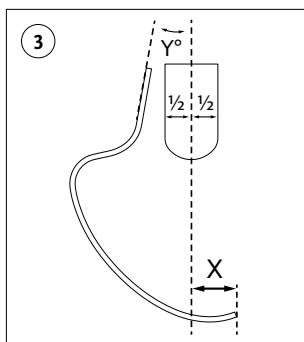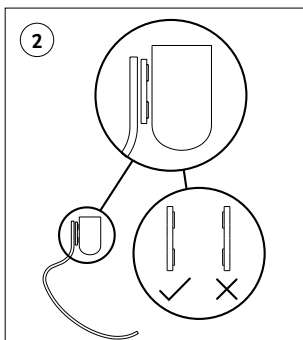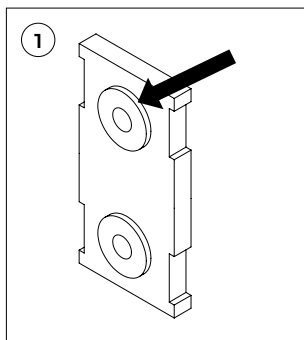
# ENGLISH

---

Medical Device

## PRODUCT DESCRIPTION

Thank you for choosing an Össur Product.

The device is a prosthetic foot for sports activities. This document covers the following devices.

Flex-Foot Cheetah

Cheetah Xtend

Cheetah Xtreme

Cheetah Xcel

Cheetah Xpanse

Cheetah Junior

## INDICATIONS FOR USE

- Lower limb amputation.
- Lower limb deficiency.

## CONTRAINDICATIONS

None known.

## INTENDED USE

This device is a modular component for a lower limb prosthesis that replaces the function or aesthetics of a missing limb. Compatibility of components and the overall prosthesis must be evaluated by a medical professional.

This device is for single patient use.

The device is intended for high to extreme impact levels, e.g. running, sprinting and recreational sports.

- Cheetah Xpanse: also for long jump.

## GENERAL SAFETY INSTRUCTIONS

Inform your patient about everything in this document that is required for safe use of this product.

**WARNING:** In cases of functional change or functional loss, the patient should stop using the device and contact a healthcare professional.

**WARNING:** If the device shows signs of damage or wear hindering normal function, the patient should stop using the device and contact a healthcare professional.

## PRODUCT SELECTION

Please refer to the selection chart below to determine the appropriate category.

**WARNING:** Do not exceed weight limit. Risk of device failure.

Incorrect category selection may also result in poor device function.

Cheetah Junior:

<b>Weight kg</b>	15-24	25-34	35-45	46-55
<b>Weight lbs</b>	33-53	54-75	76-99	100-121
<b>Category</b>	1	2	3	4

All other:

<b>Weight kg</b>	45-52	53-59	60-68	69-77	78-88	89-100	101-116	117-130	131-147
<b>Weight lbs</b>	99-115	116-130	131-150	151-170	171-194	195-220	221-256	257-287	288-325
<b>Category</b>	1	2	3	4	5	6	7	8	9

## REQUIRED COMPONENTS

The foot can be attached to the socket with direct lamination or by using one of the following options:

Lamination Connector Kit - see assembly instructions below.

Alignment Adapters - refer to Instructions for relevant alignment adapter.

## ASSEMBLY INSTRUCTIONS

**CAUTION:** Epoxy can cause allergic reactions when in direct contact with body.

**CAUTION:** There may be a cutting hazard due to sharp edges or splinters on the device.

### *Nike Soles*

The device can be fitted with a Nike spike pad or Nike sole. Refer to instructions for relevant sole/pad for assembly instructions.

### *Lamination connector*

The lamination connector must be laminated directly onto the posterior wall of the socket.

**WARNING:** Before starting, make sure to orient the lamination connector the correct way. The shoulders of the inserts (marked with arrow) should be oriented towards the socket (**Figure 1 & 2**).

1. Put the lamination connector in the middle third of the socket to make height adjustment easier.
2. Make sure the socket surface is flat. A flat surface below the lamination connector distributes the load from the foot evenly onto the socket.
3. Use a friction pad between the lamination connector and the foot.
4. Apply thread locker and torque bolts as shown in the table below.

	<b>Lamination connector kits</b>	
<b>Device</b>	<b>Lamination connector kit (FSX50009)</b>	<b>Lamination connector kit, pediatric (CHX50001)</b>
Flex-Foot Cheetah	x	
Cheetah Xtend	x	
Cheetah Xtreme	x	
Cheetah Xcel	x	
Cheetah Junior		x
Cheetah Xpanse	x	
Weight limit	147 kg	55 kg
Bolt size	3/8"	5/16"
Torque values [Nm]	45 Nm	28 Nm

## ALIGNMENT

### BENCH ALIGNMENT

#### Sagittal plane

1. Use an alignment jig to determine the vertical socket position.
2. Make sure the load line bisects the socket.
3. Make sure:
  - that the load line falls through the ground contact point
  - or that the load line is posterior to the toe end per dimension X below (**Figure 3**).
4. Clamp the foot into the correct flexion position Y (**Figure 3**).

#### Coronal Plane

1. Set the applicable socket abduction/adduction.
2. Offset the foot module 12 mm away from centerline (**Figure 4**).  
Cheetah Junior: No offset.
3. Set up for 5° of external rotation (**Figure 5**).
4. Set the correct height - see compression in table below to compensate for vertical compression.

Device	Dimension X	Flexion Angle Y	Compression
Flex-Foot Cheetah	50 mm	7°	25 mm
Cheetah Xtend	63 mm	7°	38 mm
Cheetah Xtreme	63 mm	7°	38 mm
Cheetah Xcel Large	110 mm	9°	38 mm
Cheetah Xcel Small	100 mm	7°	38 mm
Cheetah Xpanse Large	110 mm	48°	38 mm
Cheetah Xpanse Small	110 mm	54°	38 mm
Cheetah Junior	35 mm	7°	15 mm

### **Dynamic Alignment**

Physical ability and the length of residual limb will have an effect on alignment. Optimal alignment reduces energy expenditure and reduces muscle and ligament strain.

Adjust alignment to improve gait symmetry.

The A-P alignment and vertical deflection from socket to foot can have an effect on stride length.

### **Cutting the Foot**

Make sure the user has sufficient clearance during knee flexion.

Use a hack saw or band saw to trim the foot.

**NOTE:** The foot must be flush with or extend above the lamination connector.

## USAGE

### **Troubleshooting**

Make sure socket fit and suspension are optimal. Examine running symmetry at patients preferred running speed.

If the duration of loading the prosthesis is longer compared to the contralateral side:

- The foot module is too soft
- The prosthesis is too short
- The foot is aligned too far anteriorly

If the duration of loading the prosthesis is shorter compared to the contralateral side:

- The foot module is too stiff
- The prosthesis is too long
- The foot is aligned too far posteriorly

**NOTE:** If you move the A-P position of the foot, there will be an effect on hip flexion, stride length and trunk motion.

### ***Maintenance***

The device must be examined for splinters and/or delamination regularly by a medical professional. Interval should be determined based on patient activity.

Additionally, the Cheetah Xpanse must be examined for splinters and/or delamination every three months by a medical professional if used for long jump.

### ***Cleaning and care***

Clean with a soft cloth and a mild soap.

### **ENVIRONMENTAL CONDITIONS**

The device can be used in temperatures between -15 °C to 50 °C.

The device is waterproof.

A waterproof device can be used in a wet or humid environment and temporarily submerged in shallow fresh water.

Dry thoroughly after contact with fresh water.

No salt water or chlorinated water allowed.

Clean with fresh water in case of accidental exposure to salt water or chlorinated water.

### **REPORT A SERIOUS INCIDENT**

#### **Important notice to users and/or patients established in Europe:**

The user and/or patient must report any serious incident that has occurred in relation to the device to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

### **DISPOSAL**

The device and packaging must be disposed of in accordance with respective local or national environmental regulations.

### **LIABILITY**

Össur does not assume liability for the following:

- Device not maintained as instructed by the instructions for use.
- Device assembled with components from other manufacturers.
- Device used outside of recommended use condition, application, or environment.

### ***Compliance***

This device has been tested according to ISO 10328 standard to two million load cycles.

The load level specification is shown in the tables below.

The device service life cannot be determined since load varies depending on the sport and patient activity level.

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