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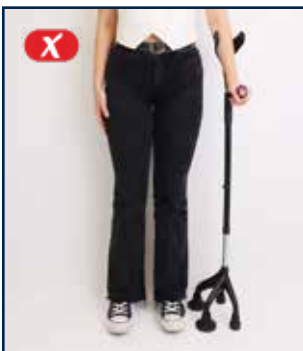
The **4 Legs Crutch** is a sturdy, steel and aluminium crutch that serves as a walking aid. It is designed with 4 tips on the base (2 wide and 2 narrow) for extra support or extra help when walking. It is designed to be used individually and by people with physical disabilities, who are in periods of convalescence or rehabilitation for any injury.

ASSEMBLY AND USE

1. Stand upright on a firm, flat surface, allowing the arm that is to hold the crutch to hang naturally at your side.
2. Join both tubes together and slide the aluminium tube until the crutch is at the correct height.
3. Position the wide end caps always facing outwards, and the tube can be rotated 180° so that the crutch can be used on the right or left arm.
4. The arms should be slightly bent with the hand in the handle (no more than 15°) and the handle at the level of the hip.
5. Attach the height adjustment clip to start using the crutch.

TURES

- It has 9 adjustments, the minimum height is 30" to the handle and the maximum height is 38.5" to the handle.
- The approximate weight of each crutch is 3 lbs.
- PA6 + 50GF (Base). Steel (Lower tube). Aluminium (Upper tube). Rubber end caps and handle.

WARNINGS

Do not use this product without reading and understanding the instructions. If necessary, contact a professional, dealer or service personnel before using this product to avoid possible injury.

Before use, you must ensure that:

- All parts are tight, nothing is loose, and there are no rattles or cracks in the metal.
- Only one KMINA 4 Leg Crutch is used without a second crutch on the other arm.
- The user does not suffer from impairments, disabilities or is under the effects of any medication, etc. that the doctor, therapist and/or retailer considers to interfere with the safe use of this product.

During use, ensure that:

- You do not hold the crutch in case of sudden impact or jolts. If this happens, release your hands and support them on the floor.
- Do not exceed the maximum weight capacity for this crutch, which is 360 lbs.
- The height of the crutch is correctly adjusted.
- Any safety hazards including slippery, uneven or soft surfaces and objects in the path are avoided.
- Using this crutch does not eliminate the risk of falling.
- The toe clips or cleats should be inspected frequently for damage or wear.
- The crutch should be kept in a clean condition. It is recommended that the handle and forearm be washed regularly with neutral soap.
- Replace any broken, damaged or worn parts with manufacturer's parts.
- Failure to follow the above instructions may result in serious injury.
- Any serious incident involving the product should be reported to the manufacturer and to the competent authority of the Member State in which the user and patient are established.

This product can be cleaned and disinfected with commercially available products as well as with a damp cloth. The padding can be hand washed or machine washed in cold water. Always keep the product clean and wash it regularly.

MULETA 4 PATAS - NEGRA	K10065
MULETA 4 PATAS - AZUL	K10069

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