KMINA

USER MANUAL
MANUAL DE USUARIO
BENUTZERHANDBUCH
MANUEL D'UTILISATION
MANUALE UTENTE
ユーザーマニュアル

BLACK | NEGRO | SCHWARZ | NOIR | NERO | 黒

K10091

WHITE | BLANCO | WEISS | BLANC | BIANCO | 白

K10092

KMINA - ADVANCE ROLLATOR
KMINA - ANDADOR ADVANCE
KMINA - ADVANCE ROLLATOR
KMINA - DÉAMBULATEUR ADVANCE
KMINA - DEAMBULATORE ADVANCE
KMINA - ADVANCE ロレーターローラー歩行器

1 COMPONENTS / COMPONENTES / KOMPONENTEN / COMPOSANTS / COMPONENTI/コンポーネント

	ENGLISH	ESPAÑOL	DEUTSCH	FRANÇAIS	ITALIANO	日本語
1.	Adjustable handle	Empuñadura regulable	Verstellbarer Griff	Poignée réglable	Impugnatura regolabile	調節可能ハンドル
2.	Backrest	Respaldo	Rückenlehne	Dossier	Schienale	背もたれ
3.	brake lever	Freno maneta	Bremshebel	Levier de frein	Leva del freno	ブレーキレバー
4.	Storage bag	Bolsa almacenaje	Aufbewahrungstasche	Sac de rangement	Borsa portaoggetti	収納バッグ
5.	Solid front wheels	Ruedas delanteras macizas	Vorderräder aus Vollgummi	Roues avant pleines	Ruote anteriori piene	ソリッド前輪
6.	Solid rear wheels	Ruedas traseras macizas	Bremshebel aus Vollgummi	Roues arrière pleines	Ruote posteriori piene	ソリッド後輪
7.	Rear wheel folding	Plegado de ruedas delanteras	Faltung der Hinterräder	Pliage des roues arrière	Piegatura delle ruote posteriori	後輪の折りたたみ
8.	Front wheel folding	Plegado de ruedas traseras	Faltung der Vorderräder	Pliage des roues avant	Piegatura delle ruote anteriori	後輪の折りたたみ
9.	Seat	Asiento	Sitz	Siège	Sedile	座面
10.	Removable cane holder	Soporte para bastón extraíble	Abnehmbarer Stockhalter	Support de canne amovible	Supporto per bastone rimovibile	取り外し可能な杖ホルダー
11.	Nuts	Tuercas	Muttern	Écrous	Dadi	ナット

2 MEASURES / MEDIDAS / MESSUNGEN / MESURES / MISURE / 測定方法

WIDTH	HEIGHT	LENGTH	FOLDING DIMENSIONS
58 cm (23") 65 cm (with cane holder) (25.5")	78 - 95 cm (30.7" - 37")	75 cm (29.5")	27x78x75 cm (10.6"x30.7"x29.5")

SEAT HEIGHT	SEAT DIMENSIONS	Ø WHEELS	COMPACT FOLDING
54 cm (21")	40 x 25 cm (16" x 10")	20 x 3 cm (8" x 1.2")	62x33x23,5 cm (24.4"x13"x9.3")

MAX. USER WEIGHT	NET WEIGHT
136 kg (300 lb)	≈ 8,3 kg (18.3 lb)

EN The **KMINA rollator** has been designed for people who have balance problems, weakness in the legs, or difficulty maintaining an upright posture while walking. It features a hand-brake system.

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3 ASSEMBLY / MONTAJE / VERSAMMLUNG / MONTAGE / MONTAGGIO/ 組み立て

EN | 1. Remove the walker from the box and take off the folding restraints from the front and rear wheels.

2. Lower both front wheels and secure them by inserting and tightening the nuts provided. Repeat the same process with the rear wheels.

3. Unfold the seat by applying pressure to both ends until it is fully secured.

4. Attach the adjustable handle by aligning the metal tubes on both parts of the walker. Press the locking pins and slide the metal tubes to the desired height. Make sure both sides are adjusted to the same height.

5. Attach the cane holder to the tab located on the side of the walker.

6. Finally, insert the basket into the tubes located at the front of the walker.

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EN | Pull the handle of Seat until the walker is folded.

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EN | 1. To fold the walker completely, first remove Storage bag from the front.

2. Take out Adjustable handle.

3. Pull the handle of Seat until the walker is folded.

4. Loosen the four nuts of Front wheel folding and Rear wheel folding, and fold Front wheels and Rear wheels upwards.

5. Tighten the nuts again and fasten the hook-and-loop straps so that the structure of the folded walker becomes more rigid.

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EN | Tighten the locking pins located on the sides of the Adjustable handle. Slide the metal tubes to the desired height. Make sure both sides are adjusted to the same height. You will know the height is completely fixed when you see the pin in one of the eight holes.

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EN |
Service brake: While the walker is being pushed by the user, pull the Brake lever upwards on both sides.

Parking brake (when parking and using the Seat function): When you want to park the walker or use the Seat function, push the Brake lever downwards. It will stay locked in place and the walker will not move.

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EN

REFERENCE

KMINA - ROLLATOR (BLACK)	K10091
KMINA - ROLLATOR (WHITE)	K10092

MATERIALS

Aluminium, plastic and Nylon fabric.

WARNINGS

- Make sure the walker is completely open and in the correct position before use.
- Be careful when folding and unfolding the walker to avoid hurting yourself.
- Make sure that the handle is properly adjusted to the correct height. Check the stability of the walker before use.
- Test the brake mechanism before use. If the wheels turn while the brakes are locked, refer back to how to adjust the brake pressure. When the brakes are released, the wheels should turn freely.
- Use only on even surfaces, never on a slope or uneven terrain. Using the walker in any other way than intended could cause damage or injury.
- Do not leave the walker unattended without locking the brakes.
- Do not stop or turn on a ramp or slope. Do not make sharp turns. On downhill slopes, apply the brakes to prevent the walker from going faster than you.
- Use the basket only to transport objects without exceeding the maximum load: 11 lb.
- Do not hang bags from the handles, the walker may tip over.
- Do not exceed the maximum user weight: 300 lb.

- Do not expose the product to temperatures above 40°C for a prolonged period of time; otherwise the aluminum tubes will heat up and could cause burns.
- According to the manufacturer's recommendations this product is suitable for recycling.
- Inspect the walker before each use to ensure that there is no damage or excessive wear.
- Before use, check that the walker is secure and stable; and all fastenings and fittings are tight and secure. If you find any signs of wear or damage to your walker, DO NOT USE IT.
- Retain these instructions for future reference.
- Any serious incident involving the product must be reported to the manufacturer and to the competent authority of the Member State in which the user and patient are established.
- Keep your walker clean using a damp cloth. Do not use abrasive cleaning materials. Do not clean or disinfect the walker using steam.
- Do not remove the sticker provided on the product. If necessary, make a note of the product lot in order to be able to indicate the manufacturer in case of an accident.

KMINA warrants this walker to be free from defects. The warranty will remain in effect for 3 years from the date of original purchase by the consumer. The warranty does not extend to parts exposed to natural wear and tear that require periodic replacement, such as brakes, wheels, handles and bags. This warranty does not cover:

- Replacement parts supplied by anyone other than KMINA or the authorized dealer.
- Any failure of the unit during the warranty period that is not caused by a defect in material or workmanship or if the failure is caused by misuse.
- KMINA is not responsible for accidental damage or damage to property.

In case of return, properly pack the walker preferably in its original packaging, otherwise KMINA will not be responsible for possible damages caused during shipment.

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