USER MANUAL
MANUAL DE USUARIO
BENUTZERHANDBUCH
MANUEL D'UTILISATION
MANUALE UTENTE
ユーザーマニュアル

BLACK | NEGRO | SCHWARZ | NOIR | NERO | ZWART | 黒

K10096

WHITE | BLANCO | WEISS | BLANC | BIANCO | WIT | 白

K10097

KMINA - ROLLATOR SHOPPING CART
KMINA - ANDADOR - CARRO COMPRA
KMINA - ROLLATOR - EINKAUFSTROLLEY
KMINA - DÉAMBULATEUR - CHARIOT DE COURSES
KMINA - DEAMBULATORE - CARRELLO SPESA
KMINA - ROLLATOR - BOODSCHAPPENTROLLEY
KMINA - 歩行器 - 買い物カート

K10096

K10097

	ENGLISH	ESPAÑOL	DEUTSCH	FRANÇAIS	ITALIANO	NEDERLANDS	日本語
1.	Adjustable handle	Empuñadura regulable	Verstellbarer Griff	Poignée réglable	Impugnatura regolabile	Verstelbare handgreep	調節可能ハンドル
2.	Backrest	Respaldo	Rückenlehne	Dossier	Schienale	Rugleuning	背もたれ
3.	brake lever	Palanca Freno	Bremshebel	Lévier de frein	Leva del freno	Remhendel	ブレーキレバー
4.	Storage bag	Bolsa almacenaje	Aufbewahrungstasche	Sac de rangement	Borsa portaoggetti	Opbergtas	収納バッグ
5.	Solid front wheels	Ruedas delanteras macizas	Vorderräder aus Vollgummi	Roues avant pleines	Ruote anteriori piene	Massieve voorwielen	ソリッド前輪
6.	Solid rear wheels	Ruedas traseras maciza	Bremshebel aus Vollgummi	Roues arrière pleines	Ruote posteriori piene	Massieve achterwielen	ソリッド後輪
7.	Rear wheel folding	Plegado de ruedas delanteras	Faltung der Hinterräder	Pliage des roues arrière	Piegatura delle ruote posteriori	Inklappen van de achterwielen	後輪の折りたたみ
8.	Front wheel folding	Plegado de ruedas traseras	Faltung der Vorderräder	Pliage des roues avant	Piegatura delle ruote anteriori	Inklappen van de voorwielen	後輪の折りたたみ
9.	Seat	Asiento	Sitz	Siège	Sedile	Zitting	座面
10.	Lifting lever	Palanca de elevación	Hubhebel	Lévier de levage	Leva di sollevamento	Hefhendel	昇降レバー

WIDTH	HEIGHT	LENGTH	COMPACT FOLDING
55 cm (21.5")	83 - 93 cm (32.5" - 36.5")	70 cm (27.5")	55 x 70 x 47 cm (21.5" x 27.5" x 18.5")

SEAT HEIGHT	SEAT DIMENSIONS	Ø WHEELS	BAG CAPACITY
52 cm (20.5")	39,5 x 32 cm (15.5" x 12.5")	20 x 3,5 cm (8" x 1.4")	Folded bag: 28 L Unfolded bag: 80 L

MAX. USER WEIGHT	NET WEIGHT
136 kg (300 lb)	≈ 10,5 kg (23 lb)

EN The **KMINA Walker-shopping cart** is designed for people with balance issues, leg weakness, or difficulty maintaining an upright posture while walking. It also includes a practical storage system, ideal for carrying groceries comfortably and safely.

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3

ASSEMBLY / MONTAJE / VERSAMMLUNG / MONTAGE / MONTAGGIO / MONTAGE / 組み立て

EN | 1. Fully loosen the nuts located at the Front wheel folding and Rear wheel folding joints. Lower both front wheels and secure them by inserting and tightening the nuts again. Repeat the same operation with the rear wheels. If you want to fold the walker, repeat this step in reverse order.

2. Install the Adjustable handle by aligning the metal tubes on both parts of the walker. Tighten the locking pins and slide the metal tubes to the desired height. Make sure both sides are set to the same height.

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EN | Tighten the locking pins located on the sides of the Adjustable handle. Slide the metal tubes to the desired height. Make sure both sides are adjusted to the same height. You will know the height is completely fixed when you see the pin in one of the five holes.

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EN |

-Service brake: While the walker is being pushed by the user, press the brake lever upward.
Parking brake (when parking and using the seat function): When you want to park the walker or use the seat function, press the brake lever downward. It will lock in place and the walker will not move.

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EN | The storage bag for groceries, when folded, is fully integrated into the walker's seat. To use it, lift the seat and unfold the bag. To store it, repeat the process in reverse order. If you want to remove the bag completely for washing, detach the side hook-and-loop fasteners.

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Folded bag capacity: 28 L

Unfolded bag capacity: 80 L

KMINA - ROLLATOR - SHOPPING CART (BLACK)	K10096
KMINA - ROLLATOR - SHOPPING CART (WHITE)	K10097

MATERIALS

Aluminium, plastic and Nylon fabric.

WARNINGS

- Make sure the walker is completely open and in the correct position before use.
- Be careful when folding and unfolding the walker to avoid hurting yourself.
- Make sure that the handle is properly adjusted to the correct height. Check the stability of the walker before use.
- Test the brake mechanism before use. If the wheels turn while the brakes are locked, refer back to how to adjust the brake pressure. When the brakes are released, the wheels should turn freely.
- Use only on even surfaces, never on a slope or uneven terrain. Using the walker in any other way than intended could cause damage or injury.
- Do not leave the walker unattended without locking the brakes.
- Do not stop or turn on a ramp or slope. Do not make sharp turns. On downhill slopes, apply the brakes to prevent the walker from going faster than you.
- Use the basket only to carry items and do not exceed the maximum capacity.
- Do not hang bags from the handles, the walker may tip over.
- Do not exceed the maximum user weight: 300 lb.
- Do not expose the product to temperatures above 40°C for a prolonged period of time; otherwise the aluminum tubes will heat up and could cause burns.
- According to the manufacturer's recommendations this product is suitable for recycling.
- Inspect the walker before each use to ensure that there is no damage or excessive wear.
- Before use, check that the walker is secure and stable; and all fastenings and fittings are tight and secure. If you find any signs of wear or damage to your walker, DO NOT USE IT.
- Retain these instructions for future reference.
- Any serious incident involving the product must be reported to the manufacturer and to the competent authority of the Member State in which the user and patient are established.
- Keep your walker clean using a damp cloth. Do not use abrasive cleaning materials. Do not clean or disinfect the walker using steam.
- Do not remove the sticker provided on the product. If necessary, make a note of the product lot in order to be able to indicate the manufacturer in case of an accident.

KMINA warrants this walker to be free from defects. The warranty will remain in effect for 3 years from the date of original purchase by the consumer. The warranty does not extend to parts exposed to natural wear and tear that require periodic replacement, such as brakes, wheels, handles and bags. This warranty does not cover:

- Replacement parts supplied by anyone other than KMINA or the authorized dealer.
- Any failure of the unit during the warranty period that is not caused by a defect in material or workmanship or if the failure is caused by misuse.
- KMINA is not responsible for accidental damage or damage to property.
- Any modification carried out by the user or by third parties without the approval of IMPROVING MOBILITY DEVELOPMENT S.L. will void the warranty coverage.

In case of return, properly pack the walker preferably in its original packaging, otherwise KMINA will not be responsible for possible damages caused during shipment.

KMINA - ANDADOR - CARRO COMPRA (NEGRO)	K10096
KMINA - ANDADOR - CARRO COMPRA (BLANCO)	K10097

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