USER MANUAL | MANUAL DE USUARIO | BENUTZERHANDBUCH |  
MANUEL D'UTILISATION | MANUALE UTENTE | ユーザーマニュアル

red**dot** winner 2025

|                                     |        |
|-------------------------------------|--------|
| BLUE/AZUL/BLAU/BLEU/BLU/青           | K10084 |
| WHITE/BLANCA/WEISS/BLANCHE/BIANCA/白 | K10085 |

**KMINA - PRO WHEELCHAIR**  
**KMINA - PRO SILLA DE RUEDAS**  
**KMINA - PRO ROLLSTUHL**  
**KMINA - PRO FAUTEUIL ROULANT**  
**KMINA - PRO SEDIA A ROTELLE**  
**KMINA - PRO 車椅子**

|     | ENGLISH                          | ESPAÑOL                    | DEUTSCH                        | FRANÇAIS                                  | ITALIANO                                        | 日本語          |
|-----|----------------------------------|----------------------------|--------------------------------|-------------------------------------------|-------------------------------------------------|--------------|
| 1.  | Adjustable handlebar             | Manillar regulable         | Verstellbarer Lenker           | Guidon réglable                           | Manubrio regolabile                             | 調節可能なハンドル    |
| 2.  | Padded backrest                  | Respaldo acolchado         | Gepolsterter rückenlehne       | Dossier rembourré                         | Schienale imbottito                             | クッション付き背もたれ  |
| 3.  | Armrests                         | Reposabrazos               | Armlehnen                      | Accoudoirs                                | Braccioli                                       | アームレスト       |
| 4.  | Padded seat                      | Asiento acolchado          | Gepolsterter Sitz              | Siège rembourré                           | Seduta imbottita                                | クッション付き座面    |
| 5.  | Storage basket                   | Cesta de almacenaje        | Aufbewahrungskorb              | Panier de rangement                       | Cestino portaoggetti                            | 収納バスケット      |
| 6.  | Front wheels                     | Ruedas delanteras          | Vorderräder                    | Roues avant                               | Ruote anteriori                                 | 前輪           |
| 7.  | Foldable footrest                | Reposapiés abatible        | Klappbare Fußstütze            | Repose-pieds rabattable                   | Repose-pieds rabattable                         | 折りたたみ式フットレスト |
| 8.  | Brake lever                      | Palanca de freno           | Bremshebel                     | Levier de frein                           | Leva del freno                                  | ブレーキレバー      |
| 9.  | Storage Basket Hooks             | Enganches cesta almacenaje | Haken für Aufbewahrungskorb    | Crochets pour panier de rangement         | Ganci per cestino portaoggetti                  | 収納バスケット用フック  |
| 10. | Folding lock                     | Bloqueo de plegado         | Faltverriegelung               | Verrou de pliage                          | Blocco di chiusura                              | 折りたたみロック     |
| 11. | Rear wheels                      | Ruedas traseras            | Hinterräder                    | Roues arrière                             | Ruote posteriori                                | 後輪           |
| 12. | Folding button                   | Botón de plegado           | Faltknopf                      | Bouton de pliage                          | Pulsante di chiusura                            | 折りたたみボタン     |
| 13. | Structure                        | Estructura                 | Struktur                       | Structure                                 | Struttura                                       | 構造           |
| 14. | Handlebar Height Adjustment Tabs | Pestañas altura manillar   | Höheneinstelllaschen am Lenker | Ergots de réglage de la hauteur du guidon | Linguette regolazione dell'altezza del manubrio | ハンドル高さ調整用タブ  |
| 15. | Accessory hook                   | Gancho accesorios          | Zubehörhaken                   | Crochet pour accessoires                  | Gancio accessori                                | アクセサリー用フック   |

| WIDTH                                                                             |  | HEIGHT                                                                            |  | LENGTH                                                                            |  | FOLDING DIMENSIONS                                                                |  |
|-----------------------------------------------------------------------------------|--|-----------------------------------------------------------------------------------|--|-----------------------------------------------------------------------------------|--|-----------------------------------------------------------------------------------|--|
| 67 cm (26")                                                                       |  | 95 -106 cm (37"- 42")                                                             |  | 110 cm (43")                                                                      |  | 67x95x61 cm (26"x37"x24")                                                         |  |
|  |  |  |  |  |  |  |  |
| SEAT HEIGHT                                                                       |  | SEAT DIMENSION                                                                    |  | BACKREST HEIGHT                                                                   |  | Ø REAR WHEEL                                                                      |  |
| 46 cm (18")                                                                       |  | 46 x 40 cm (19"x16")                                                              |  | 33 cm (13")                                                                       |  | 60 cm (24")                                                                       |  |
| MAX. USER WEIGHT                                                                  |  | WEIGHT WITHOUT WHEELS                                                             |  |                                                                                   |  | TOTAL WEIGHT                                                                      |  |
| 114 kg (250 lb)                                                                   |  | 12,8 kg (28 lb)                                                                   |  |                                                                                   |  | 17 kg (37 lb)                                                                     |  |

**EN | The KMINA - PRO Wheelchair**, is based on a minimalist design, inspired by baby stroller and high-performance bicycle technology. Its strong aluminum frame, with oval profiles, adds value through detailed craftsmanship. It has been designed to be self-propelled by the user using the large-diameter rear wheels, or pushed by another person using the handlebar. It supports a maximum weight of 250 lb. This wheelchair is intended for independent patients, elderly people, individuals with mobility problems, or those with physical disabilities who still have enough autonomy to propel themselves safely. ***This wheelchair is NOT designed for patients with complete lower limb immobility, quadriplegia, or involuntary spasmodic movement disorders that make it difficult or impossible to maintain a seated position.***

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R.1

R.2

**EN** | Unfold the foldable **footrest**, first unfolding the main tube (R.1), then the footplate (R.2). It remains fixed once unfolded.

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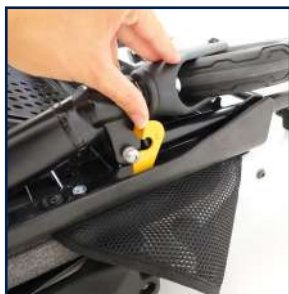
2

**EN | Unfold the structure of the wheelchair (A).** First, loosen the folding lock of the chair. Then, press the folding button and push the padded seat downward until it is fully unfolded and stable.

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3

**EN | Lower the armrests** until they are in a horizontal position relative to the floor.

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4

**EN | The wheelchair features a quick-release system for the rear wheels (B).**

To install them, align the metal tube of the wheels with those on the structure of the wheelchair and push the rear wheels onto the sides of the chair. Then, press the button in the center of the wheel until it is fully secured.

To remove them, press the button in the center and pull the wheel out.

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5

**EN | Attach the handlebar (C).** To do so, open the handlebar height adjustment tabs located on the structure of the wheelchair. Insert the handlebar with the padded part facing the back of the chair.

To adjust the height, align the holes in the handlebar with the tabs' pins. There are three height positions available.

This handlebar has been designed for easy operation with one hand.

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6

**EN | Attach the storage basket (D).** To do so, insert it from the back of the wheelchair and hook the elastic straps onto the storage basket hooks located on the sides of the rear part of the structure.  
Secure the basket to the structure using the side fasteners.

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## EN |

1. **Raise the armrests** until they are perpendicular to the ground.
2. Adjust the **handlebar to the lowest height**. Open the handlebar height adjustment tabs and push the handlebar downward. Close the tabs again to fix it in place.
3. **Fold the footrest** in two steps: first the tray, then the main tube. Once folded, it will remain fixed.
4. Press the **folding lock button** and, with one hand, pull the seat upward. Use the other hand to hold the handlebar. To keep the structure folded, use the folding lock.
5. If desired, remove the rear wheels by pressing the center button on each and pulling them out.

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**EN** | The handlebar and the lower frame of the wheelchair structure function as grips for transportation. If you fold the wheelchair with the rear wheels attached, you can transport it by pushing the handlebar. If you remove the wheels, you can transport the wheelchair by holding the handlebar with one hand and the structure with the other.

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**EN** | The wheelchair allows for **quick release** of the rear wheels. To remove them, press the button located in the center of the wheel and pull it out. To reinsert them, align the metal tubes, bring the wheels to the sides of the chair, and press the central button to secure them.

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**EN** | Raise the armrests to assist with the user's transfer in and out of the wheelchair.

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**EN** | To tilt the wheelchair backwards, press down on the lower frame of the wheelchair with one foot. Keep both hands on the handlebar at all times.

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**EN** | The maximum slope before tipping, for the use of parking brakes, is 10° on a rearward slope and 8° on a forward slope.

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**EN** | To deactivate the brake, push the brake lever upwards.

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**EN** | To activate the brake, push the brake lever all the way down.

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## MATERIALS

Structure: Aluminum, steel, and stainless steel components.  
 Backrest, seat, and plastic components: Polyamide and polypropylene.  
 Armrests: High-density polyurethane foam.  
 Textiles (cushions and basket): Waterproof synthetic fabric.

## WARNINGS

- Do not use the wheelchair without reading the instructions carefully.
- IMPROVING MOBILITY DEVELOPMENT S.L. is not responsible for misuse of the wheelchair or knowingly ignoring the instructions.
- To avoid entrapment risks, refer to Annex I.
- This wheelchair cannot be used as a seat in any type of vehicle.
- Do not use this wheelchair on streets, roads, or any surface with vehicular traffic.
- Do not attempt to use the wheelchair on ramps without proper assistance.
- Before placing or removing the user, the wheelchair must be on a smooth, stable surface.
- Do not apply weight to the footrest when lifting or seating the user.
- Do not lean on or drop weight onto the top of the backrest, this may cause the wheelchair to tip over.
- Do not stand on the wheelchair.
- Sudden direction changes on a ramp or slope may cause instability or tipping.
- With the brakes engaged, keep the wheelchair away from potential brake-lever activation.
- Remember: this wheelchair is designed to transport people, not objects.
- When reaching for items requiring the seated user to lean, ensure the front wheels are pointing forward to stabilize the chair.
- Once seated, do not lean or stretch excessively. There exists a risk of falling or injury.
- The wheelchair's temperature may rise if exposed to heat sources such as direct sunlight for extended periods.
- If storing the wheelchair for a long period, do not leave it outdoors.
- Do not exceed the maximum weight capacity of 250 lb.
- Do not hit the wheelchair during movements, when loading or unloading it from the car, when going up or down steps, or in any other situation.
- Do not drop the wheelchair from high or low heights, either alone or with a user seated in it. Seek assistance such as ramps or lifts and use force carefully to ensure it never falls.
- Following manufacturer recommendations, this product is suitable for recycling at dedicated collection points.
- Any serious incident with the product must be reported to the manufacturer and relevant authority in the user's country.
- Keep these instructions for future reference.

## MAINTENANCE AND CLEANING

- Inspect the wheelchair before each use to ensure no excessive wear or damage. If any is detected, DO NOT USE.
- Regularly check that all parts are secure and that wheels rotate properly.
- If the footrest becomes loose, use a 13 mm open-ended wrench and a 5 mm Allen key.
- Keep the wheelchair clean. Aluminum and steel tubes can be wiped with a damp cloth.
- Common disinfectants may be used repeatedly to clean bodily fluid stains.
- Fabric parts (backrest, seat, and basket) can be detached and hand-washed (maximum 30 °C) with neutral soap. They can withstand about 50 washes.

## WARRANTY

IMPROVING MOBILITY DEVELOPMENT S.L. warrants this wheelchair to be free from defects. The warranty will remain in effect for 3 years from the date of original purchase by the consumer. The warranty does not extend to parts exposed to natural wear and tear that require periodic replacement, such as brakes, wheels and handgrips. This warranty does not cover:

- Replacement parts supplied by anyone other than IMPROVING MOBILITY DEVELOPMENT S.L. or the authorized dealer.
- Any failure of the unit during the warranty period that is not caused by a defect in material or workmanship or if the failure is caused by misuse.
- IMPROVING MOBILITY DEVELOPMENT S.L. is not liable for accidental damage or damage to property

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**EN** | The following indicates the potential entrapment areas in the wheelchair. Figures 1 and 2 show the specific areas and points, which are described in the table below.

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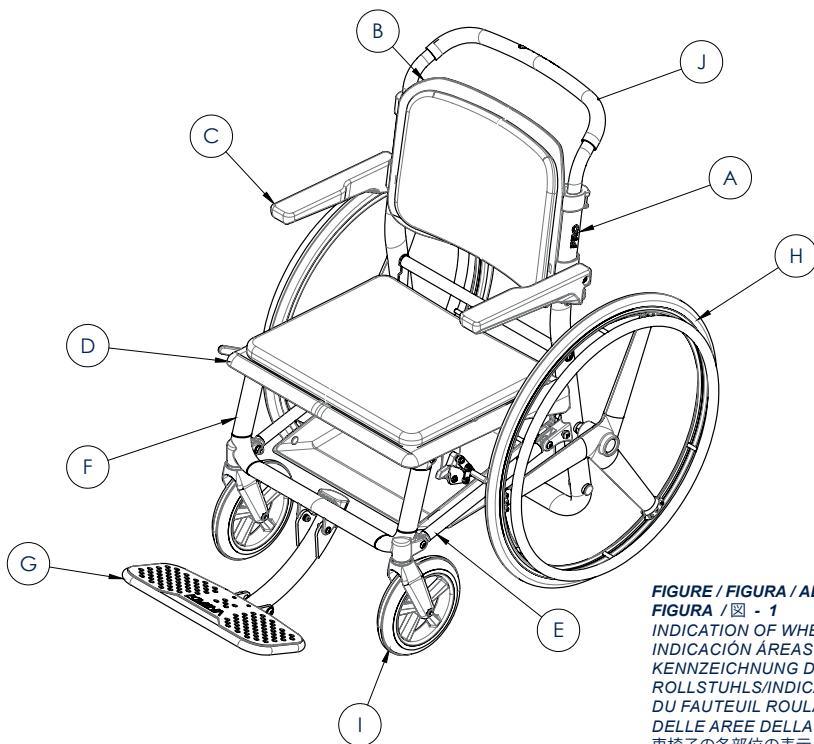
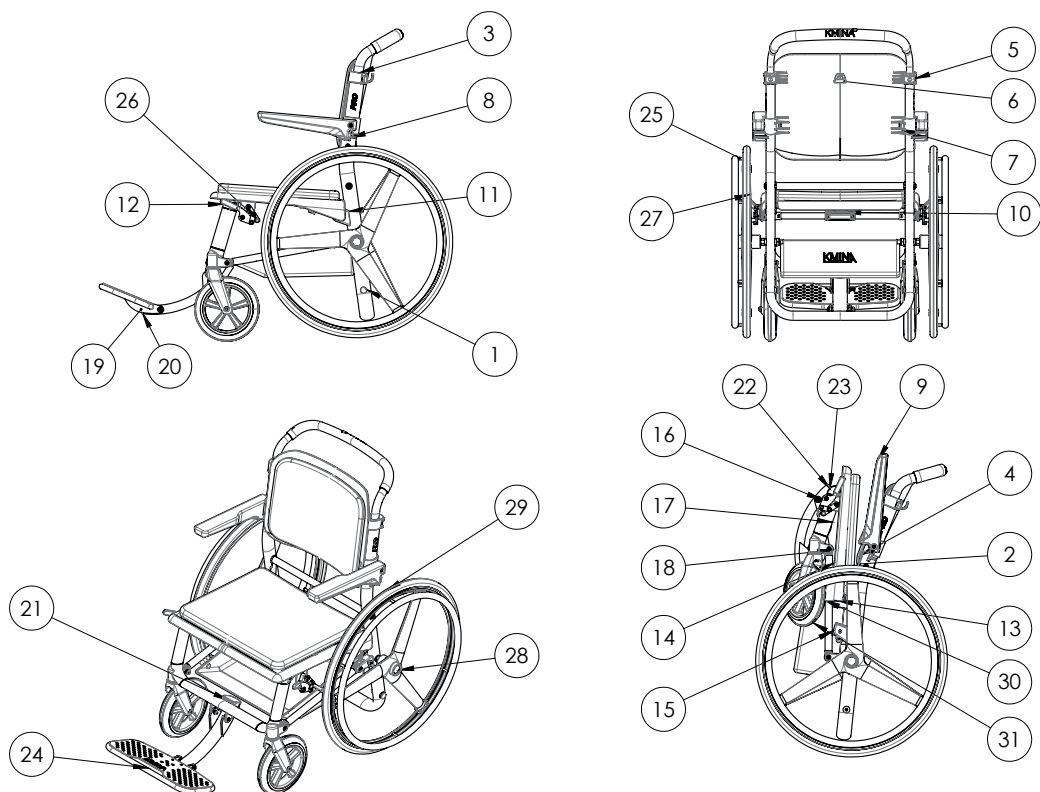
| AREA                                 | N° | DESCRIPTION                                                                                |
|--------------------------------------|----|--------------------------------------------------------------------------------------------|
| (A)<br>REAR FRAME                    | 1  | Axle support interior. The risk is eliminated with a cap.                                  |
|                                      | 2  | Between seat and rear frame. Folding movement. Cushioned by pads.                          |
| (B)<br>BACKREST                      | 3  | Pinching between handlebar tube and backrest. Height adjustment movement.                  |
|                                      | 4  | Between seat and backrest. Folding movement. Cushioned by pads.                            |
|                                      | 5  | Between handlebar adjustment gate and backrest. Handlebar height adjustment movement.      |
|                                      | 6  | Backrest hook.                                                                             |
|                                      | 7  | Backrest bracket ribs.                                                                     |
| (C)<br>ARMRESTS                      | 8  | Between the structure's rotation lock plate and the armrest pad. Armrest folding movement. |
|                                      | 9  | Between seat and armrests. Folding movement. Cushioned by pads and padding.                |
| (D)<br>SEAT                          | 10 | Between folding lock bar and seat. Unfolding movement.                                     |
|                                      | 11 | Between rear structure hinge and seat hinge. Unfolding movement.                           |
|                                      | 12 | Between front frame hinge and seat, cushioned by rubber pad. Unfolding movement.           |
|                                      | 13 | Inner tube of seat frame at the back.                                                      |
| (E)<br>SIDE BARS                     | 14 | Between side bar and seat. Folding movement.                                               |
|                                      | 15 | Between side bar and rear frame. Folding movement.                                         |
| (F)<br>FRONT FRAME                   | 16 | Between brakes and front frame. Folding movement.                                          |
|                                      | 17 | Between seat and front frame. Folding movement.                                            |
|                                      | 18 | Between lower front frame hinge and seat. Folding movement.                                |
| (G)<br>FOOTREST                      | 19 | Between tube and footrest platform. Footrest folding movement.                             |
|                                      | 20 | Inside of footrest tube.                                                                   |
|                                      | 21 | Between footrest and front frame. Unfolding movement.                                      |
|                                      | 22 | Between footrest platform and seat. Folding movement.                                      |
|                                      | 23 | Between footrest tube and seat. Folding movement.                                          |
|                                      | 24 | Front "handle" of the footrest tray.                                                       |
| (H)<br>REAR WHEELS<br>SELF-PROPELLED | 25 | Between push ring and wheel.                                                               |
|                                      | 26 | Between brake and wheel. Brake operation.                                                  |
|                                      | 27 | Between seat and spokes. Wheel rotation movement.                                          |
|                                      | 28 | Wheel axle with rear frame. Wheel mounting movement.                                       |
|                                      | 29 | Between wheel and armrests. Cushioned by armrest padding.                                  |
| (I)<br>FRONT WHEELS                  | 30 | Wheel (and fork) against side bar. Folding action.                                         |
|                                      | 31 | Wheel (and fork) against basket. Folding action. Cushioned by soft fabric.                 |

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## READ BEFORE RETURNING, SAVE THE PLANET.

- EN** Returns generate more than 15 million tons of CO<sub>2</sub> annually.
- ES** Las devoluciones generan más de 15 millones de toneladas de CO<sub>2</sub> al año.
- DE** Retouren erzeugen mehr als 15 Millionen Tonnen CO<sub>2</sub> jährlich.
- IT** I resi generano oltre 15 milioni di tonnellate di CO<sub>2</sub> all'anno.
- FR** Les retours génèrent plus de 15 millions de tonnes de CO<sub>2</sub> par an.
- FR** Les retours génèrent plus de 15 millions de tonnes de CO<sub>2</sub> par an.
- JA** リターンは年間1,500万トン以上のCO2を排出する。

- EN** If pieces are missing or have arrived damaged, before returning, **contact us**.
- ES** Si faltan piezas o están en mal estado, antes de devolver **contacta** con nosotros.
- DE** Wenn Teile fehlen oder beschädigt sind, **kontaktieren** Sie uns vor der Rücksendung.
- IT** In caso di parti mancanti o danneggiate, prima della restituzione, **contattaci**.
- FR** Si des pièces sont manquantes ou endommagées, avant de retourner nous **contacter**.
- JA** 部品が不足していたり、状態が悪い場合は、返品する前に弊社までご連絡ください。

### IMPROVING MOBILITY DEVELOPMENT S.L.

Avda. Mare Nostrum 1, Bloque 1, Pta 1 6  
46120, Alboraya, Valencia, Spain  
admin@kmina.com | www.kmina.com

27/01/2026 V.2.