# Yack

## Il montascale a ruote

*Wheeled stair climber*

### Nuova serie N970

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### New N970 series

*In addition to the well-established features of the previous N960, the new series implements two further functions, namely the calculation of the user weight and the height of the step of the stair.*

*Entering these two parameters directly from the control panel allows the new Yack to optimize movement on the stair to the maximum by controlling the braking levels and positions in an "intelligent" way*

*If the parameters are not entered, the device will behave in a standard way.*

*To complete the improvements, there are the new wheels with softer compound which, in addition to the "intelligent braking", give the N970 series high levels of ride comfort and safety for both the patient and the operator.*

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### *Electronic at the service of comfort*

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*The display placed on the control panel allows the monitoring of the stair climber ensuring safety during its operations. Messages on the interactive display guide the operator and assist him to guarantee a proper use of the machine ensuring safety. When powered up the display shows the level of battery charge, the inclination level and the climb and descent speed.*

*The N970 series implements two new functions that involve entering the user weight and step height directly on the control panel. These two parameters optimize movement on the stair, significantly improving ride comfort.*

*Yack is equipped with an emergency button, through which the machine is electrically isolated; the machine re-starting procedure is carried out step by step by messages on the display.*

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*Wheeled stair climber designed to transport persons who are directly seated on-board the machine. Extremely compact, it is easily manoeuvrable also in reduced places. The option of disassembling it into several parts makes the machine extremely easy to transport.*

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- Max load capacity 130 kg.
- New electronic making the machine even more reliable with better performances.
- Multifunctional display on control panel with self-diagnosis system integrated in the machine and stable information for a machine assisted-guide.
- Emergency stop button.
- Safety belt.
- Equipped with padded seat, folding armrests and highly adjustable headrest.
- Folding footrest.
- Max height step 21 cm.

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*The new Yack is equipped with acoustic alarms and information messages shown on the display that simplify the use and make it even more safe:*

- *Low battery signal.*
- *Excessive load signal.*
- *Incorrect engine absorption.*
- *Incorrect functioning of the electronic board.*
- *Incorrect functioning of brakes.*
- *Excessive inclination of the machine during use.*
- *Stand by with acoustic alarm every 30 seconds.*

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- *Max load capacity 120 Kg.*
- *Climber suitable for stairs with max. 26 cm. height stairs.*

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- *Max load capacity 130 Kg.*
- *Climber suitable for stairs with max. 24 cm. height stairs.*

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*The adaptable platforms make N972 a wheeled stair Climber almost universal, able to hold a wide range of wheelchairs and be transported in security. Easy and handy to use, it can be dismantled in more parts to improve the transport facility.*

- Max load capacity 130 kg.
- New electronic making the machine even more reliable with better performances.
- Multifunctional display on control panel with self-diagnosis system integrated in the machine and stable information for a machine assisted-guide.
- Emergency stop button.
- Safety belt.
- Width adjustable lower plates anchored to the body of the machine.
- Width and height adjustable wheelchair upper hooks with adjustable clamps.
- Adjustable and removable headrest
- Max height step 21 cm.

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*This version allows to easily accommodate many wheelchair types and the carrying in very narrow stairs, with reduced manoeuvre spaces. N973 allows to normally use the wheelchair in flat surfaces maintaining the climber hooked. The option of disassembling it into several parts makes the machine extremely easy to transport. It mainly serves to hook wheelchairs that cannot be hooked to N972 universal version, especially the paediatric wheelchairs whose dimensions considerably interfere with the stair climber size.*

Technical specifications

- Max load capacity 130 kg.
- New electronic making the machine even more reliable with better performances.
- Multifunctional display on control panel with self-diagnosis system integrated in the machine and stable information for a machine assisted-guide.
- Emergency stop button.
- Safety belt.
- Height and width adjustable wheelchair upper hooking bar with adjustable clamps.
- Connecting axis positioned on the climber base to insert in the specific hooks.
- Adjustable headrest.
- Max height step 21 cm.

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- External electronic battery charger. The battery can be charged either inside or outside the device, allowing it to be powered with a supplementary battery pack.

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The new Yack is equipped with acoustic alarms and information messages shown on the display that simplify the use and make it even more safe:

- Low battery signal.
- Excessive load signal.
- Incorrect engine absorption.
- Incorrect functioning of the electronic board.
- Incorrect functioning of brakes.
- Excessive inclination of the machine during use.
- Stand by with acoustic alarm every 30 seconds.

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