

## QUICK START GUIDE

---



1. Unpack the scooter. You will have the following parts:
  - 1 x scooter unit
  - 1 x battery pack (pre-installed)
  - 2 x key
  - 1 x charger and mains cable
  - 2 x armrest

2. Unlock the folding mechanism by pressing the red lever in the centre of the scooter forwards, towards the tiller, then simultaneously push the tiller slightly forwards so it opens up slightly.



3. Holding the tiller with one hand and the seat with the other, pull them apart so the scooter unfolds.

**CAUTION: ENSURE HANDS ARE FREE FROM ENTRAPMENT WHILST USING THE FOLDING MECHANISM**



4. Press down on the base of the scooter to ensure the folding mechanism is locked in position.
5. Pull up the strap on the battery so the plastic insulation strip can be removed, then refit the battery pack as shown.



## 6. Install the armrests

Press the button on the back of the seat and insert the peg of the armrest into the hole on the side of the frame. Release the button and check the armrest is secure. Repeat for the other armrest.



7. Pull on the catch (arrowed left) on the tiller to unlock. Then pull up the tiller until it is set at the correct height for the user. Push the catch to lock the tiller.

8. Insert the key into the ignition and turn to start the scooter.



## 9. CHARGE BEFORE USE

Plug the mains lead into the charger, and then plug the round connector from the charger into the charging socket at the back of the scooter. Finally switch the charger on at the mains.

The scooter can be charged either unfolded or folded (refer to section 9.3 for folding instructions).