

1) COMPONENTS / COMPONENTES / KOMPONENTEN /
COMPOSANTS / COMPONENTI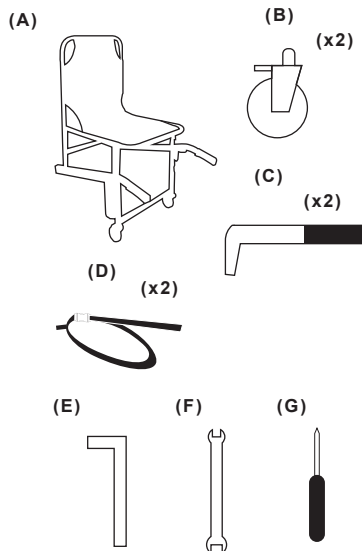

2) ASSEMBLY / MONTAJE / VERSAMMLUNG / MONTAGE / MONTAGGIO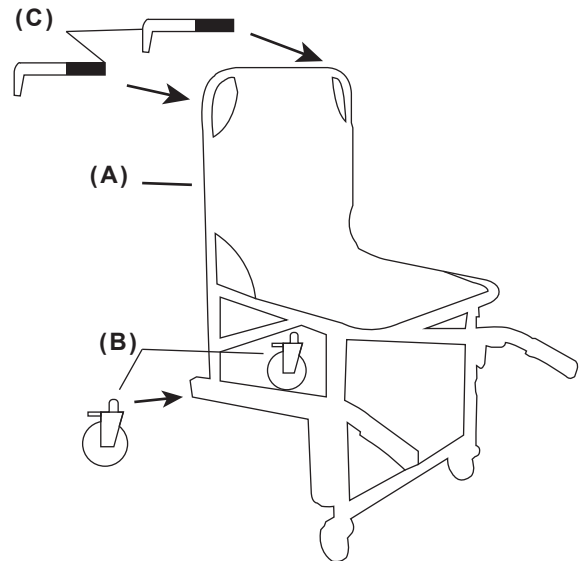

EN

REF.
PEPE - MANUAL STAIRLIFT CHAIR P40031

The PEPE Manual Stairlift Chair is a chair designed to help transport patients up and down stairs with as little effort as possible. It has two safety belts for a completely secure hold and two folding levers at the front for easy transfer.

ASSEMBLY

1. Install the two rear wheels (B) by inserting them into the rear holes. To screw them on, use the screws provided on the wheels and tighten them using the screwdriver (G) and the open-end wrench (F) to secure the nut. Make sure that the brakes are facing outwards.
2. Insert the handlebars (C) into the holes on the side of the backrest and tighten them with the screws and nuts provided. Tighten them with the Allen wrench (E) and the open-end wrench (F) so that they are fixed.
3. Pass the strap of the belt through the slots to adjust it and place one at waist level and the other at foot level, making sure that the patient is well secured.

INSTRUCTIONS FOR USE

- To fold the chair. Pull the lever on the back of the chair with one hand and with the other hand, push the seat upwards until it folds completely.
- To take a patient up or down the stairs. Press the button on the lower handles of the chair to extend them. Each of the two caregivers takes one end of the chair (upper and lower handles) and lifts the chair with the patient up or down the stairs.

MATERIAL

- Structure: aluminum.
- Accessories: steel.
- Fabric: PVC.

WARNINGS!

- When using the chair to transport patients, make sure that the seat belt is securely locked to ensure safety.
- Be careful not to tear the surface of the chair.
- Do not jump or play on the chair.
- Do not exceed the maximum user weight which is 220 lb.
- Do not remove the sticker that comes on the product. If necessary, make a note of the product lot so that you can indicate the manufacturer in the event of an accident.
- Any serious incident involving the product must be reported to the manufacturer and to the competent authority of the member State in which the user and patient are established.

CLEANING AND MAINTENANCE

- Clean the product with neutral soaps. Do not use corrosive or abrasive products.
- Regularly check that all parts are present and that none are missing. If any parts are missing DO NOT use the chair.
- When not in use, store the chair in a dry place where there is no possibility of corrosion.
- This product can be transported in any vehicle.

WARRANTY INFORMATION

The company warrants this device to be free from defects. The warranty will remain in effect for 3 years from the date of original consumer purchase. The warranty does not extend to parts exposed to natural wear and tear that require periodic replacement.

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