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The **PEPE Folding Crutch** is a sturdy, lightweight crutch that provides stability for those who need extra support and balance. Its foldable condition makes it perfect for storage and transport with an easy folding system. It has been designed to be used by people who are convalescing or undergoing rehabilitation.

HEIGHT ADJUSTMENT

- Unfold the folding crutch by simultaneously separating the 2 parts. To unfold and fold it, hold the entire tube so that it does not jump causing damage or scares and screw the fitting thread between the two tubes before adjusting the height.
- Remove the height adjustment clip and slide the aluminum tube until the crutch is at the correct height.
- To set the correct height, stand upright on a firm, flat surface, letting your arm hang naturally at your side. The arms should be slightly flexed with the hand on the handle (no more than 15°) and the handle at the level of the trochanter.
- Make sure the clip or push button is properly positioned and fits into one of the index holes on the tube.
- Replace the clip to start using the crutch. To fold back, proceed in the reverse order of unfolding.

WARNINGS

Do not use this equipment without first reading and understanding these instructions. If needed, contact health care professional, dealer or technical personnel before attempting to use, otherwise injury may occur.

Before using, make sure that:

- All the pieces are well adjusted. Nothing is unadjusted or uneven and there are no cracks in the metal.
- You do not have any disability or are under the influence of any medicine, etc., that a medical professional or therapist would deem to be in conflict with the safe use of the product.

When using, make sure that:

- Adjust the height and that all parts are adjusted correctly.
- Don't hold the crutch in case of sudden impact or jolts. If that is the case, we release our hands and put them on the ground.
- Don't exceed the maximum weight capacity for these crutches, which is 220 lb.
- Regulate the height of the crutch correctly (3'5" to 3'10") with the 7-button adjustment range.
- Avoid any safety risk including slippery, uneven or smooth surfaces and objects on the road.
- Using this crutch does not avoid the risk of falls.
- Inspect the ferrule or plug often to verify that it is not damaged or worn.
- Don't modify the mechanical characteristics of the crutch.
- The crutch must be kept in a clean condition.
- Replace any broken, damaged, or worn parts. Use only manufacturer's parts.
- Failure to follow the above instructions may result in serious injury.
- Stairs without handrails are high risk and should be used with the utmost caution.
- Any serious incident related to the product must be reported to the manufacturer and to the competent authority of the Member State in which the user and patient are established.

The **product can be cleaned** and disinfected with commercially available products, as well as with a damp cloth. Always keep the product clean and wash it regularly.

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