**USER MANUAL
MANUAL DE USUARIO
BENUTZERHANDBUCH
MANUEL D'UTILISATION
MANUALE UTENTE
ユーザーマニュアル**

	K10074
--	---------------

	K10075
--	---------------

**KMINA - ELECTRIC WHEELCHAIR
KMINA - SILLA DE RUEDAS ELÉCTRICA
KMINA - ELEKTRISCHER ROLLSTUHL
KMINA - FAUTEUIL ROULANT ÉLECTRIQUE
KMINA - SEDIA A ROTELLE ELETTRICA
KMINA - 電動車椅子**

	ENGLISH	ESPAÑOL	DEUTSCH	FRANÇAIS	ITALIANO	日本語
1.	Armrest	Reposabrazos	Armlehne	Accoudoir	Bracciolo	アームレスト
2.	Control unit with Joystick	Mando de control con joystick	Steuergerät mit Joystick	Commande avec joystick	Comando con joystick	ジョイスティック付き操作装置
3.	Pocket	Bolsillo	Tasche	Poche	Tasca	ポケット
4.	Padded seat	Asiento acolchado	Gepolsterter Sitz	Siège rembourré	Seduta imbottita	クッション付き座面
5.	Belt	Cinturón	Gurt	Ceinture	Cintura	ベルト
6.	Footrest	Reposapiés	Fußstütze	Repose-pieds	Poggiapiedi	フットレスト
7.	Front wheels	Ruedas delanteras	Vorderräder	Roues avant	Ruote anteriori	前輪
8.	Pneumatic rear wheels	Ruedas traseras neumáticas	Pneumatische Hinterräder	Roues arrière pneumatiques	Ruote posteriori pneumatiche	空気入り後輪
9.	Armrest folding button	Botón plegado reposabrazos	Klappknopf der Armlehne	Bouton de pliage accoudoir	Pulsante pieghevole bracciolo	アームレスト折りたたみボタン
10.	Backrest with rear pocket	Respaldo con bolsillo trasero	Rückenlehne mit Rückentasche	Dossier avec poche arrière	Schienale	背もたれ
11.	Push handle	Manillar	Schiebegriff	Poignée de poussée	Schienale con tasca posteriore	背もたれ
12.	Motors	Motores	Motoren	Moteurs	Motori	モーター
13.	Clutch lever	Palanca de embrague	Kupplungshebel	Levier d'embrayage	Leva della frizione	クラッチレバー
14.	Battery	Batería	Batterie	Batterie	Batteria	バッテリー
15.	Folding mechanism	Mecanismo de plegado	Faltmechanismus	Mécanisme de pliage	Meccanismo di chiusura	折りたたみ機構
16.	Anti-tip wheels	Ruedas antivuelco	Kippschutzräder	Roues anti-basculement	Ruote antiribaltamento	転倒防止キャスト
17.	Charger	Cargador	Ladegerät	Chargeur	Caricabatterie	充電器

EN | This electric wheelchair is equipped with a basic toolkit to assist with routine maintenance, adjustments, and minor repairs. Below is a list of the included tools and their typical applications:

	TOOL	QUANTITY	USE
1.	Bag	1	Storage of all tools.
2.	Allen Key	3	Adjust bolts or screws on armrests, footrests, frame joints, and controller mount.
3.	L-Shaped Hex Key	1	Fine adjustments on small bolts (e.g., controller mount, battery bracket).
4.	Open-End Wrench	3 (different sizes)	Tighten/loosen hex nuts on wheel axles, frame joints, and structural components.
5.	T-Wrench (Socket Wrench)	1	Apply greater torque to large bolts (motor mounts, wheel axles, frame parts).
6.	Screwdriver	1	Fasten/remove screws on control panel, battery compartment, and seat attachments.
7.	Clamp	3	Attach the control cable to the frame if it is switched to the left armrest.
8.	Strap	1	Secure the folded structure.

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MAX. USER WEIGHT	WEIGHT WITHOUT BATTERY	WEIGHT WITH BATTERY
136 kg (300 lb)	24,5 kg (54 lb)	29,5 kg (65 lb)

EN | The KMINA - Electric Wheelchair has been designed to provide comfort and safety to the user during their movements, in a faster and effortless way. It supports a maximum weight of 300 lb and is intended for independent patients, elderly people, or individuals with reduced mobility.

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1

EN | Unfold the electric wheelchair. Open the folding mechanism and, using one hand, push the push handle while pulling the seat outward with the other hand.

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2

EN | Press the lever on the handlebar structure to lift it and start assembly.

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3

EN | Install the joystick control unit. To do so, first loosen the nut located under the right armrest. Insert the control unit into the right or left armrest, and adjust it to the user's preferred position. Finally, tighten the nut to secure it.

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3

EN | 1. Press the armrest folding buttons.

2. Loosen the folding mechanism located at the back of the wheelchair.

3. Place one hand on the push handle and the other on the seat, and apply force until the wheelchair is fully folded.

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EN | Press the armrest folding button located on the sides of the armrests. Raise the armrests to assist with the user's transfer in and out of the wheelchair.

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EN | The clutch lever allows switching between electric driving mode and manual driving mode.

■ **Electric driving (D):** When the brake lever is down, in the electric driving position, the wheelchair cannot be pushed and must be driven electrically as normal.

■ **Manual driving (N):** When the brake lever is up, in the manual driving position. The wheelchair is in brake release mode and can be pushed.

If you want to change the driving mode, you must switch the clutch lever on both motors to the same position.

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EN | Before driving, familiarize yourself with local pedestrian traffic rules and practice. Start by practicing forward movement at the lowest speed, then practice making “S” turns, and finally try reversing. Make sure all functions are working properly and the battery is fully charged before using it on the road.

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EN |

1. Sit on the wheelchair and, if needed, lift the armrests fully upward to make the seat transfer easier. Fasten the safety belt.
2. Press the power button; the remaining buttons on the control unit will light up. You will be able to see the wheelchair's battery level. Each time you press a button, it will beep. Press the mute button if you do not want it to beep.
3. The speed adjustment buttons allow you to set the speed level from 1 (slowest) to 5 (fastest).
4. Move the joystick in any direction. The further it moves from the center, the faster the electric wheelchair will go. When reversing, pay attention to your surroundings.
5. To stop the electric wheelchair, release the joystick so that it comes to a smooth stop.
6. If a system fault occurs, the battery and speed level indicators will blink intermittently. If you are in an emergency, press the emergency button; it will emit a sound to call for help.

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- EN** | 1. Turn off the electric wheelchair by pressing the power on/off button.
 2. Connect the charger directly to the wheelchair controller.
 3. If you wish to charge the battery outside the wheelchair, release it from the frame by pressing the button. It is located under the seat. Connect it to the charger.
 4. Plug the charger into an electrical outlet to begin charging the battery.
 5. While charging, the charger indicator light will be orange. It will turn green when charging is complete.

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EN | The wheels of this wheelchair are equipped with inflatable tyres to ensure optimal performance and greater comfort during use. To inflate them, unscrew the cap from the valve located on the wheel. Connect the pump and inflate until the recommended pressure is reached (2.2 - 2.5 Bars). Remove the pump and screw the valve cap back on.

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MATERIALS

Magnesium and aluminium alloy, fabric.

WHEEL PRESSURE

The recommended tire pressure is 2.2 - 2.5 bar.

WARNINGS

- Familiarize yourself with local pedestrian traffic regulations before using the product.
- DO NOT operate this vehicle without first reading this manual.
- Practice in a safe environment before actual use, especially when learning to turn, brake, or reverse.
- DO NOT use the wheelchair under the influence of alcohol, drugs, extreme fatigue, or mental confusion.
- DO NOT allow children or unconscious persons to use the wheelchair.
- Avoid driving at night or in low visibility conditions.
- ALWAYS use the seatbelt and keep your feet on the footrests while driving.
- When getting on or off, sit as far back as possible to prevent tipping.
- DO NOT place weight on the footrests or use the armrests for support during transfers.
- While seated, avoid any position or movement that could alter your center of gravity, as this may cause the chair to tilt.
- DO NOT carry passengers, tow objects, or tie pet leashes to the chair.
- DO NOT exceed the maximum permitted weight of 300 lb.
- DO NOT attempt to climb curbs higher than 2" or cross gaps wider than 4".
- Avoid sudden movements, emergency braking, and sharp turns.
- When reversing, ensure there are no obstacles behind you. Use the lowest speed.
- DO NOT use the wheelchair if parts are damaged or missing.
- DO NOT drive with the motor unlocked (manual clutch mode); if so, always be accompanied. Any assistant helping the electric wheelchair user must understand and control its operation.
- DO NOT drag or push the chair with unnecessary force.
- Never use the wheelchair near train tracks, bodies of water, open flames, or escalators.
- In case of a fall, structural damage, or detected faults, contact the provider before using again.
- NEVER transport the electric vehicle with a person seated in it.
- DO NOT use the electric vehicle for any purpose other than intended.
- DO NOT use or expose the electrical system to adverse weather conditions such as rain, snow, fog, strong winds, or extreme temperatures.
- Avoid driving on soft, wet, muddy, sandy, or snowy terrain.
- Reduce speed on uneven terrain or in crowded areas.
- DO NOT drive near lakes, rivers, or stagnant water and avoid going through puddles.
- DO NOT use the wheelchair in the shower, bathtub, swimming pool, or sauna.
- DO NOT smoke while using this electric vehicle. Dropping lit cigarettes on the padded seating system may cause a fire leading to damage, serious injury, or death.
- Use extreme caution when using oxygen near electrical circuits or combustible materials. Contact your oxygen supplier for safety information.
- If the electric vehicle begins to smoke, spark, or catch fire, stop using it immediately and request repairs.
- When climbing a slope, drive straight up; both front wheels should touch down at the same time. To reduce the risk of tipping, DO NOT drive at an angle or allow only one wheel or one side to climb first. Use particular caution when climbing any slope.
- DO NOT get off the chair, reverse, or leave the vehicle parked on a slope.
- When descending slopes, DO NOT switch the clutch lever to manual mode; braking will not function.
- Braking distance on slopes may be significantly longer than on flat ground.
- DO NOT drive on slopes steeper than 12° or on surfaces with a lateral tilt greater than 3°.
- ALWAYS select a low speed when descending slopes (move the joystick to the center position).
- DO NOT use portable electronic devices while driving the chair.
- Avoid accessories or loose clothing that may interfere with wheels or gears—there is a risk of entrapment, especially when folding the chair.
- ALWAYS turn off the chair before getting in or out, entering elevators, and after parking.
- If the chair moves unexpectedly or control is lost, turn off the system immediately.
- DO NOT use the power button to stop the chair while in motion.
- Disconnect power if the chair will not be used or is parked for a long time.
- DO NOT tamper with the motor—it may damage the chair.
- DO NOT place fingers between moving parts. Wearing gloves is recommended when handling parts of the chair.
- IMPROVING MOBILITY DEVELOPMENT S.L. is not responsible for self-inflicted damage, damage to third parties, or property damage, whether accidental or due to bad faith.
- IMPROVING MOBILITY DEVELOPMENT S.L. is not responsible for theft or loss of parts or of the electric wheelchair.
- The CE marking will be void if components or accessories not approved by IMPROVING MOBILITY DEVELOPMENT S.L. are added or replaced.
- DO NOT connect any electrical devices to the vehicle unless expressly certified for this use by IMPROVING MOBILITY DEVELOPMENT S.L.
- The electric wheelchair cannot be transported on airplanes.

CHARGING AND MAINTENANCE OF THE BATTERY

- Fully charge the battery before first use (8 to 14 hours). It must be fully charged at first use to reach 90% of its capacity. After 4-5 charging cycles, the battery will reach 100% and last longer.
- DO NOT use car batteries as replacements.
- If the battery level shows only one bar, charge the chair.
- Batteries must be stored outside the chair if it will not be used for an extended period.
- DO NOT charge or store the chair outdoors, in confined spaces, or under high temperature, direct sunlight, flammable gas, mist, or steam.
- Ensure the charger's plug is dry and intact before connecting to the chair.
- Only use the original charger provided by our company. Using a different brand may shorten the warranty period and an unknown charger could cause damage or injury.
- DO NOT touch the charging terminals with wet hands.
- Charge the battery at least once a week if the chair is not used regularly.
- DO NOT place the charger on the seat or cover it during charging.
- Ensure the chair is turned off before charging.
- Avoid frequently discharging the battery completely.
- In case of charging failure or overheating, disconnect the charger immediately and contact technical support.
- Dispose of used batteries in accordance with local regulations.
- DO NOT attempt to open or repair the battery yourself.
- Inform the transport company when shipping the chair that it includes a lithium battery.
- DO NOT use the chair if the battery is fully discharged.

USE AND MAINTENANCE OF THE CHARGER

- Before using the battery charger, carefully read all instructions.
- The charger is designed exclusively for lithium-ion batteries. DO NOT use it with other types of batteries.
- DO NOT use the charger in environments with corrosive liquids or gases, flammable surfaces, poorly ventilated or humid areas, or near open flames.
- Keep out of reach of children.
- Do not start charging if you detect faulty connections, damaged cables, or signs of malfunction.
- If any anomaly is detected, immediately disconnect the charger from the power supply.
- In case of emergency or for any repairs, contact a qualified technician.
- Do not modify the circuit or update the software without authorization.
- The charger cable is flexible SVT, minimum 18 AWG.
- The charger has an LED indicator: Green means connected to power, no battery connected or charging completed; Red means charging; Flashing indicates battery error or charger failure.
- Possible failure causes: Poor or no connection between charger and battery, incompatible or defective battery, incorrect polarity, internal charger or battery faults, fully charged battery, charger malfunction.

MAINTENANCE AND CLEANING

- Perform regular inspections. To keep the electric wheelchair in good condition, it is recommended to perform visual and functional checks before each use. All parts should be inspected regularly and at least once a month.
- Check the seat for looseness or abnormal vibrations before driving.
- Ensure that the armrests and footrests are firmly attached and DO NOT wobble.
- Periodically check the condition of tires, brakes, cables, and electrical connections.
- Check the inflation of the rear tires; if noticeably deformed while riding, inflate them.
- Verify that all screws are tight and wheels are not worn.
- Ensure that the joystick functions normally and is not loose or misaligned.
- If you notice unusual noises, vibrations, or power loss, contact an authorized technician.
- Improper supervision or maintenance could result in injury, damage, or death due to ingestion or choking hazards from small parts or materials.
- Electrical components damaged by corrosion must be replaced immediately.
- To avoid electric shock, inspect the plug and cord for cuts or frayed wires. Replace damaged cables immediately.
- Check daily if the battery needs charging.
- When performing maintenance, repairs, or lifting any part of the electric vehicle, be aware of the weight of individual components, especially batteries. Always use proper lifting posture and ask for help if needed.
- Store the product in a clean, dry, well-ventilated place free from corrosive gases and with relative humidity ≤80%.
- Keep the chair clean and free of accumulated dust and dirt.
- Clean the frame with a damp cloth. DO NOT use direct water or submerge any parts.
- DO NOT use solvents, alcohol, or abrasive products on the controls or chassis.
- Avoid liquids coming into contact with the joystick or electrical system.
- If the chair has not been used for a long time, take dust prevention measures.
- The seat has a cover that can be washed. Remove it from the chair and separate the fabric for washing.

PERFORMANCE PARAMETERS

- Maximum speed: 5 mi/h*.
- Maximum slope: 12°.
- Braking method: Electromagnetic.
- Lithium battery: 24V/30Ah.
- Motor: 350W (x2).
- Charger: 24V/3A.
- Charger cable: 104".
- Control panel: 24V/35A.

*The maximum speed may vary by up to 10% depending on the user's weight, slope, terrain type, and specific motor tolerances.

WARRANTY

IMPROVING MOBILITY DEVELOPMENT S.L. guarantees that this product is free from defects.

- The warranty for the motor and control unit remains valid for 2 years from the original date of purchase.
- The warranty for the battery and charger remains valid for 1 year from the original date of purchase.
- The product warranty remains valid for 3 years from the original date of purchase.

The warranty does not cover parts subject to natural wear that require periodic replacement. This warranty does not cover:

-Replacement parts supplied by anyone other than IMPROVING MOBILITY DEVELOPMENT S.L. or the authorized distributor.

-Any failure of the unit during the warranty period not caused by a defect in materials or workmanship, or failures caused by misuse.

-IMPROVING MOBILITY DEVELOPMENT S.L. is not liable for self-caused damage, third-party damage, or property damage, whether accidental or intentional.

-IMPROVING MOBILITY DEVELOPMENT S.L. is not liable for theft or loss of parts or the product itself.

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