**USER MANUAL
MANUAL DE USUARIO
BENUTZERHANDBUCH
MANUEL D'UTILISATION
MANUALE UTENTE**

**KMINA - ELECTRIC TOILET FRAME
KMINA - ESTRUCTURA INODORO ELÉCTRICA
KMINA - ELEKTRISCHE TOILETTENSTRUKTUR
KMINA - STRUCTURE DES TOILETTES ÉLECTRIQUES
KMINA - STRUTTURA DELLA TOILETTE ELETTRICA**

EN The **KMINA Electric Toilet Frame** has been designed to provide additional support for toilet functions, preventing accidental slips or falls. It has a loader and a lifting mechanism that facilitates access to the toilet and the incorporation of the user after use. It has been designed to be used by the elderly, people with mobility difficulties, and/or patients convalescing or recovering from hip or knee surgery, etc.

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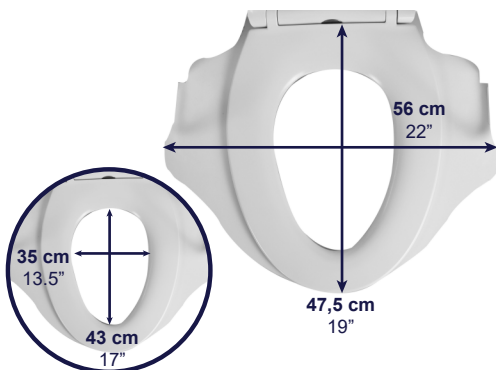
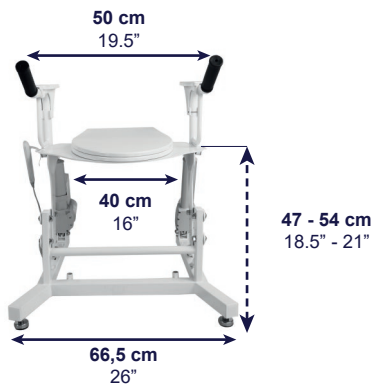
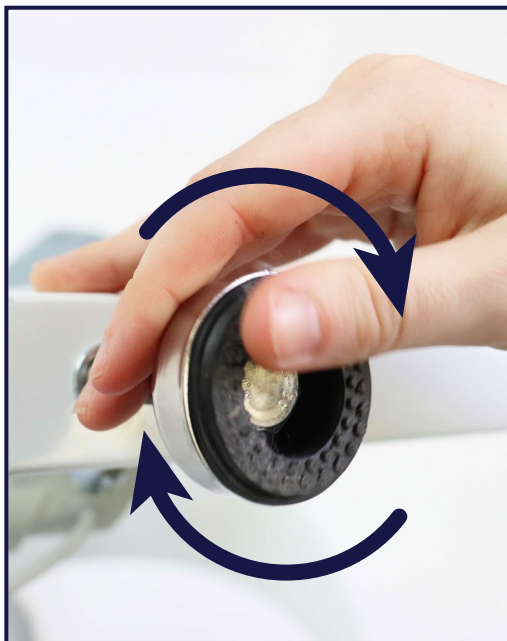
EN Turn the frame (A) on its side and insert the legs (B) into the holes in the frame. Screw the 4 legs at the same height and tighten the nut with the open-end wrench (N) to make sure they are securely fastened.

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EN Remove the white tube (E) from the side projection of the frame (A) and insert the plastic washer (G) and then insert the tube. Subsequently, place each of the handles (C) and screw the screws (F) using the Allen wrench (I).

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EN Place the armrest (D) on the handle (C), pressing on it until you hear it engage with the handle.

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EN Screw the armrests (D) for safety. To do this, use the large screws (P) that come with a small nut and, with the help of the narrow Allen wrench (O) tighten them securely.

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EN Remove the trim (J) from the cover fastening (I) to position it. Screw the cover fastening into the holes of the central structure, inserting the small screws (K). Use the small Allen key (L). Finally, place the trim on top.

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EN Insert the lid (H) into the projections of the central frame(A) until you hear the click that indicates that it is correctly installed. To remove the lid, press the button in the middle and pull upwards.

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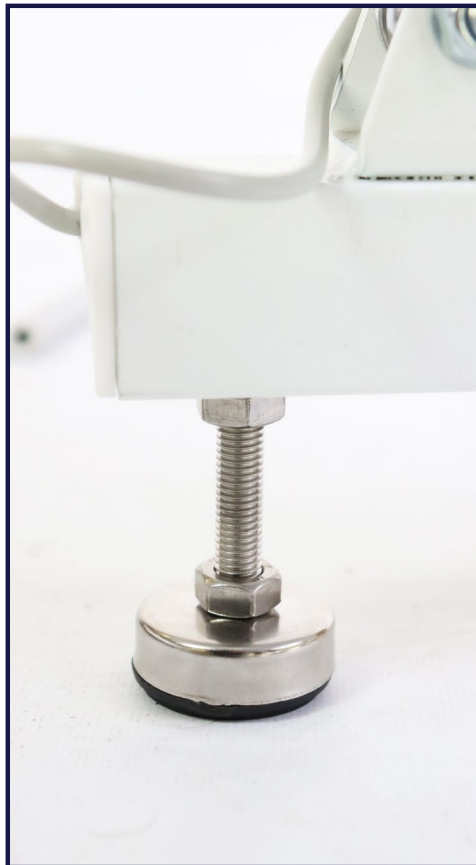
- EN** To adjust them:
1. Turn them counterclockwise.
 2. Once you have the desired height, turn the nut until it is positioned at the top of the leg.
 3. Check that it is securely fastened.

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K30037

MATERIALS

Frame and handles: steel.
 Seat: polypropylene (PP).
 Armrests: PVC.
 Handles: rubber.

WARNINGS

- Read the instructions carefully before installation.
- Users with limited physical capabilities must be supervised or assisted when using the device.
- Do not exceed the maximum weight supported by the device, which is 330 lb.
- The handles are intended to serve as a support for standing and/or sitting, using both at the same time. Do not drop your entire body weight on only one of the handles.
- Do not stand on this product. It is not a climbing aid.
- Do not apply 100% of the weight only on one armrest, use only half of the body weight on each of the armrests. In case of overloading, the armrest may break.
- Check that all components are properly assembled and screwed together before use.
- Keep away from heat sources.
- Any serious incident involving the product must be reported to the manufacturer and to the competent authority of the Member State in which the user and the patient are established.
- Do not remove the sticker on the product. If necessary, write down the batch of the product in order to be able to inform the manufacturer in case of accident.

CLEANING AND MAINTENANCE

- Clean regularly by hand with a damp cloth. Do not use abrasive cleaning agents or rinse with water.
- Disinfect the product by wiping all accessible surfaces with disinfectant (after disassembly, if possible). All cleaning products used must be effective, compatible with each other and protect the materials to be cleaned.
- If the structure is to be used by several people, it must be disinfected at each use.
- For disinfection in hospital environments, follow the instructions of a hygiene specialist.

WARRANTY

KMINA guarantees that this device is free from defects. The warranty shall remain in force for 3 years from the date of original purchase by the consumer. The warranty does not extend to parts exposed to natural wear and tear requiring periodic replacement. This warranty does not cover:

- Replacement parts supplied by anyone other than KMINA or the authorized dealer.
- Any failure of the unit during the warranty period that is not caused by a defect in material or workmanship or if the failure is caused by misuse.
- KMINA is not liable for accidental damage or damage to property.

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