USER MANUAL
MANUAL DE USUARIO
BENUTZERHANDBUCH
MANUEL D'UTILISATION
MANUALE UTENTE
ユーザーマニュアル

PEPE - FOLDING SCOOTER
PEPE - SCOOTER PLEGABLE
PEPE - KLAPPBARER ROLLER
PEPE - SCOOTER PLIANT
PEPE - SCOOTER PIEGHEVOLE
PEPE - スクーター

PEPE - AUTOMATIC FOLDABLE SCOOTER
PEPE - SCOOTER PLEGABLE AUTOMÁTICO
PEPE - AUTOMATISCH FALTBARER SCOOTER
PEPE - SCOOTER PLIABLE AUTOMATIQUE
PEPE - SCOOTER PIEGHEVOLE AUTOMATIC
PEPE - 自動折りたたみスクーター

BLACK / NEGRO / SCHWARZ / NOIR / NERO / 黒

P10042

BLACK / NEGRO / SCHWARZ / NOIR / NERO / 黒

P10048

WHITE / BLANCO / WEISS / BLANC / BIANCO / 白

P10043

WHITE / BLANCO / WEISS / BLANC / BIANCO / 白

P10049

1

COMPONENTS / COMPONENTES / KOMPONENTEN / COMPOSANTS / COMPONENTI / コンポーネント

REF: P10048 - P10049

	ENGLISH	ESPAÑOL	DEUTSCH	FRANÇAIS	ITALIANO	日本語
1.	Handlebar	Manillar	Lenkstangen	Guidon	Manubrio	ハンドルバー
2.	Control panel	Panel de control	Bedienfeld	Panneau de commande	Pannello di controllo	コントロールパネル
3.	Front light	Luz delantera	Vorderes Licht	Feu avant	Luce anteriore	フロントライト
4.	Footrest	Reposapiés	Fußstütze	Repose-pieds	Poggiapiedi	フットレスト
5.	Front wheels	Ruedas delanteras	Vordere Räder	Roues avant	Leva di assistenza	前輪
6.	Removable battery	Batería extraíble	Abnehmbare Batterie	Batterie amovible	Batteria rimovibile	着脱式バッテリー
7.	Rear wheels	Ruedas traseras	Hinterräder	Roues arrière	Ruote anteriori	後輪
8.	Motor	Motor	Motor	Moteur	Motore	モーター
9.	Charging port	Puerto de carga	Anschluss zum	Port de charge	Porta di ricarica	充電ポート
10.	Folding system	Sistema de plegado	Klappbares System	Système de pliage	Sistema pieghevole	折りたたみ式
11.	Seat	Asiento	Sitz	Siège	Sedile	シート
12.	Backrest	Respaldo	Rückenlehne	Dossier	Schienale	バックレスト
13.	Charging cable	Cable de carga	Ladekabel	Câble de chargement	Cavo di ricarica	充電ケーブル
14.	Removable armrests	Reposabrazos extraíbles	Abnehmbare Armlehnen	Accoudoirs amovibles	Braccioli rimovibili	取り外し可能なアームレスト
15.	Folding lever	Mando de plegado	Klapphebel	Lever de pliage	Leva di piegatura	折りたたみレバー

2

MEASURES / MEDIDAS / MESSUNGEN / MESURES / MISURE / 測定方法

WIDTH	HEIGHT	HANDLEBAR HEIGHT	LENGTH	FOLDING DIMENSIONS
51 cm (20")	86 cm (34")	65 - 88,5 cm (25.5" - 35")	96 cm (37.8")	51x73x42cm (20"x29"x16.5")
SEAT HEIGHT	SEAT DIMENSIONS	BACKREST HEIGHT	Ø REAR WHEEL	Ø FRONT CASTOR
57,5 cm (23")	42x38 cm (16.5"x15")	33 cm (13")	18 cm (7")	18 cm (7")
MAX. USER WEIGHT		WEIGHT WITHOUT BATTERY		WEIGHT WITH BATTERY
120 kg (265 lb)		REF: P10042 - P10043: 20,5 kg (45 lb) REF: P10048 - P10049: 24kg (52 lb)		REF: P10042 - P10043: 22,5 kg (49.5 lb) REF: P10048 - P10049: 26kg (57 lb)

EN | The PEPE - Folding Scooter has been designed to offer comfort and safety to the user when moving around. It supports a maximum weight of 265 lb and is designed for independent patients, elderly people or people with reduced mobility.

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EN | ■ To unfold the scooter, hold the front handle and the rear handle at the same time to bring them together. Push down to unfold the scooter.

■ To fold the scooter hold the front lift handle and the rear lift handle at the same time to push them together. Then pull upward.

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EN | Switch off the scooter by turning the key. It is recommended to lower the armrests to the lowest possible height. To do this, pull the knobs outward, lower the armrests to the minimum, and release the knobs so they lock into place. You can also remove the armrests completely. Place the scooter on a flat surface.

- To unfold the scooter, press the **Folding button** to position II. When the scooter is completely unfolded, return the Folding button to its central position. To unfold the scooter using the **Folding lever**, leave the Folding button in its central position and press and hold button A.
- To fold the scooter, press the **Folding button** to position I. When the scooter is completely folded, return the Folding button to its central position.
- To fold the scooter using the **Folding lever**, leave the Folding button in its central position and press and hold button B.

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EN | Manual folding option with specific tools, not included.

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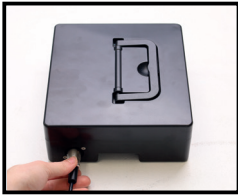
- EN** | ■ First turn off the scooter's power switch before charging.
 ■ Insert the plug at the output end of the charger into the charging port on the scooter or the charging port on the battery. Then connect the plug at the input end to the power socket and turn on the charger switch.
 ■ While charging, the indicator light on the charger will be red. It will change to green when charging is complete.
 ■ Only use the specific charger for this scooter, otherwise the battery may be damaged.
 ■ If there is no indication on the power meter when the scooter is switched on, check the fuse for a faulty fuse.

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- EN** | To remove the battery, pull the carrying handle and lift it vertically upwards to remove the battery compartment. Batterie can be charged separately.

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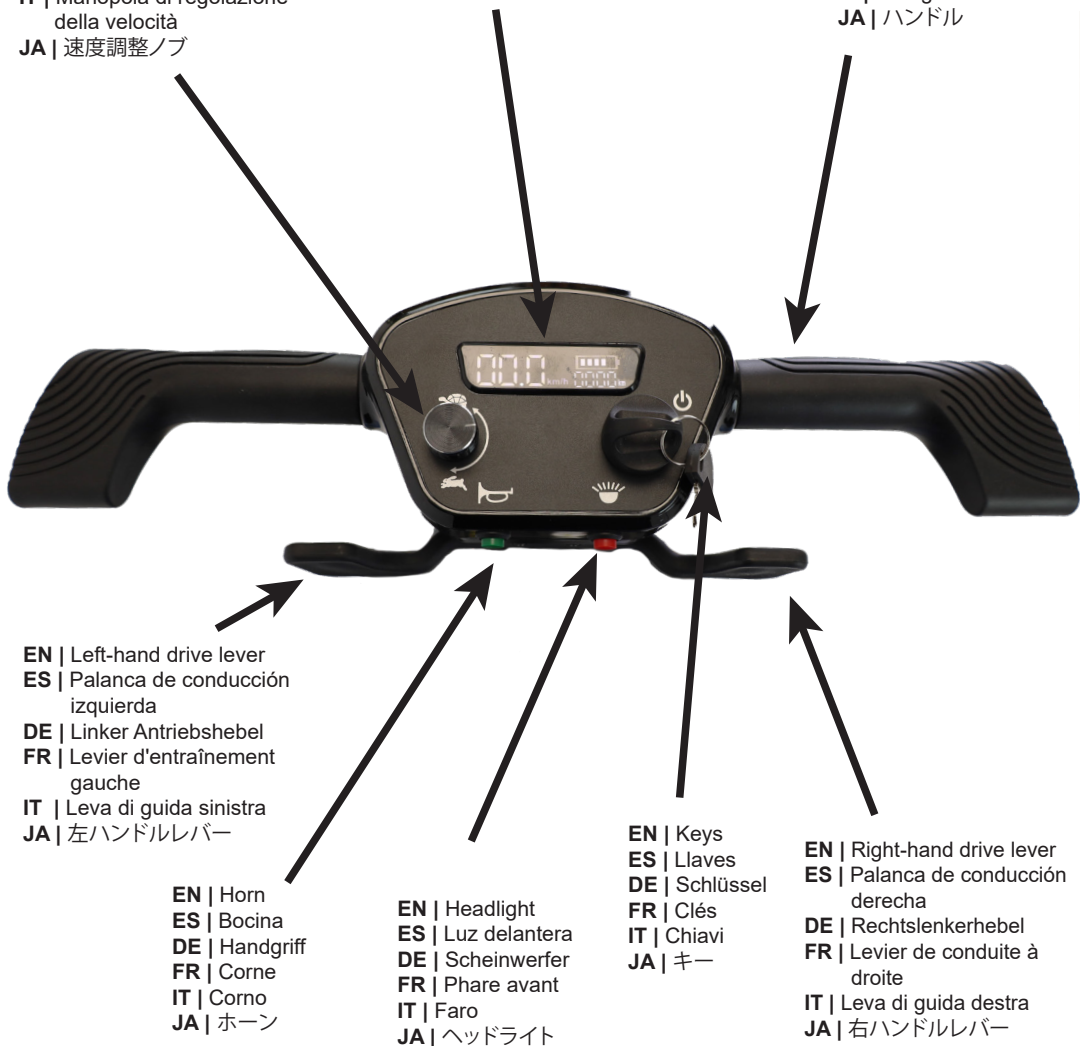
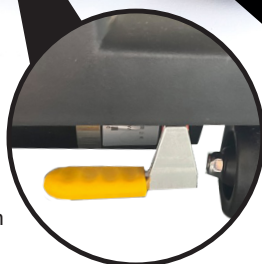
EN | The brake lever allows switching between electric and manual driving modes.

■ **Electric driving:** When the brake lever is on, in electric position, the scooter remains in braking state and cannot be pushed. At this time, the scooter can be driven electrically in a normal way.

■ **Hand pushing:** When the brake lever is off, in the manual position, the scooter is in brake release state and can be pushed. This function can only be activated when the scooter is on and in standby mode. During this time, you cannot drive the scooter.

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EN | Loosen and remove the nuts located on the sides of Seat. Insert the armrests and then reinsert and tighten the nuts.

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EN | Before drive, familiarise yourself with local pedestrian traffic regulations and practise operation. Make sure that the brakes are working properly, all functions are normal and the battery is fully charged before using it on the road.

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P10048 P10049

EN | The first time you unfold the scooter, connect the battery to the rear of the scooter using the cable provided. Make sure the scooter is switched off and unfold it using the remote control by pressing button A, or with the folding button in position II. Then place the battery on the scooter tray and use it normally.

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- EN** | 1. When sitting on the scooter, activate the key in the electric lock to start the scooter.
2. Adjust the speed button according to road conditions, environment and personal preferences.
3. Hold the handlebar with both hands and slowly move the driving lever forward with your right fingers to advance the scooter.
4. When backing back, pay attention to the conditions in the environment. Use your left fingers to slowly move the drive lever forward. When parking, release the lever to stop the scooter gently.
5. To stop the scooter, release the driving lever so that the scooter returns to the center position and stops gently.

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Aluminium and plastic (ABS).

WARNINGS

- Make sure you are aware of the current legislation for the use of the scooter in your area or zone.

- DO NOT expose the scooter's electrical system to damp environments for long periods (rain, snow).
- DO NOT use in adverse weather conditions such as rain, snow, fog, high winds or extreme temperatures.
- DO NOT operate under the influence of alcohol, excessive fatigue or mental confusion.
- DO NOT operate under the influence of alcohol or drugs.
- Avoid driving on terrain such as beaches, muddy areas or slippery surfaces.
- The scooter is designed for one person only. Do not carry passengers or use it to transport or tow objects.
- The rider should remain seated with both hands on the handlebars. Avoid sharp turns or sudden weight shifts to prevent tipping.
 - Reduce speed in rough terrain or congested areas.
 - Always turn off the ignition switch after parking and remove the key.
 - If you do not use the scooter for more than a week, disconnect the battery.
 - When descending slopes, do not switch the brake lever to the 'manual' position to enter the 'manual driving' mode, as there will be no braking effect. This is to ensure the safety of the user and the mobility scooter.
 - If you detect faults, take the scooter to a professional technician.
 - Read the safety instructions carefully and fully understand their contents before using the scooter.
 - Reduce your speed in uneven terrain or congested areas.
 - Always turn off the ignition switch after parking and remove the key.
 - If you do not use the scooter for more than a week, disconnect the battery.
 - When descending slopes, do not change the brake lever while using the scooter.
 - DO NOT attempt to open, repair or modify the internal structure of the motor.
 - DO NOT use gasoline or other chemicals to clean the housing or other parts of the electric scooter.
 - Check daily to see if the battery needs charging.
 - Inspect the brakes before riding to ensure that they are working properly.
 - Regularly check the lever fixing bolts for looseness and ensure that the lever is in the neutral position.
 - Avoid getting the electrical system wet; use the scooter only after it is completely dry.
 - Check the condition of the battery regularly and clean the terminals if necessary.
 - If the scooter is not used for an extended period of time, charge the battery approximately once a month.
 - Regularly inspect the insulation of the cables and ensure that all connectors make good contact.
 - Check the front and rear tyres for oscillation.
 - Regularly inspect the condition of the scooter, such as brakes, battery, tyres and electrical connections.
 - Do not cross puddles deeper than 2,5".
 - Your surroundings may interfere with the operation of devices that emit electromagnetic fields (shop alarm systems, automatic doors, etc.).
- The braking distance on slopes may be significantly greater than at ground level.
- If lighted cigarettes are thrown onto a padded seating system, a fire may result causing damage, serious injury or death.
- Occupants of the electric vehicle are at particular risk of death or serious injury from such fires and the gases they produce, as they may not be able to move away from the electric vehicle.
- DO NOT smoke while operating this electric vehicle.
- Improper supervision or maintenance could result in injury, damage, or death due to ingestion or suffocation caused by parts or materials.
- Make sure you have all your belongings checked before turning on and using the scooter.
- Improper use of this product may result in injury or damage.
- DO NOT attach or tie a leash to the scooter to walk your pet.
- If you have any questions regarding warnings, cautions, or instructions, contact a healthcare professional or your dealer before attempting to use this equipment.
- Turn off the electric vehicle before getting on, off, or handling unwieldy objects.
- Never leave the electric vehicle on a slope.
- If you must brake in an emergency, simply release the drive lever and allow the mobility device to come to a complete stop.
- Do not switch off the mobility device while it is in motion unless you have no other option.
- Never transport the mobility device with the occupant seated in the vehicle.
- Avoid storing or using the electric vehicle near fire or combustible products.
- DO NOT exceed the maximum permissible load.
- When servicing, repairing or lifting any part of the mobility device, be aware of the weight of the individual components, especially the batteries. Ensure that you always adopt the correct lifting position and ask for assistance if necessary.
- Ensure that moving parts of the mobility device, such as wheels or the seat lift, do not cause injury, especially when children or animals are nearby.
- The scooter is NOT recommended for use by minors.
- DO NOT expose the mobility scooter to direct sunlight for long periods of time. Metal surfaces and parts such as the seat may overheat.
- DO NOT connect any electrical device to the electric vehicle that has not been expressly certified by IMPROVING MOBILITY DEVELOPMENT S.L. for this purpose.
- Improper use could cause the electric vehicle to smoke, spark or catch fire. Damage, serious injury or death could result from fire.
- DO NOT use the electric vehicle for any purpose other than that for which it was intended.
- If the electric vehicle starts to smoke, spark or burn, stop using it and have it repaired immediately.

- To avoid electric shock, check the plug and cord for cuts or frayed wires. Replace cut or frayed cords immediately.
- Corroded electrical components due to exposure to water or other liquids can cause damage, serious injury or death.
- Reduce exposure of electrical components to water or other liquids.
- Electrical components damaged by corrosion should be replaced immediately.
- Electrical vehicles that are frequently exposed to water or other liquids may require more frequent replacement of electrical components.
- Never cover the lighting system with textiles.
- DO NOT insert fingers between scooter parts. The use of gloves is recommended when handling scooter parts. Use extreme caution with metal and heavy parts.
- A fault in the electrical system may cause unusual behaviour such as the light coming on steady or not coming on at all.
- Only climb gradients that do not exceed the maximum safety slope and only with the backrest in an upright position.
- Avoid braking sharply or accelerating on slopes.
- Never attempt to negotiate an obstacle when going up or down a slope.
- Always approach obstacles in a straight line. Make sure that the front and rear wheels go over the obstacle in one go, do not stop right in the middle.
- Avoid varying your centre of gravity as well as sudden changes of direction when the mobility device is in motion.
- Always distribute the weight evenly when carrying the mobility device. Always try to keep the centre of gravity of the electric vehicle in the middle and as close to the ground as possible.
- Be aware that the electric vehicle will brake or accelerate if the driving speed changes while in motion.
- Drive in narrow spaces at minimum speed and with due caution.
- There is an increased risk of rollover when making turns.
- Reduce speed before making turns. Only accelerate when you have completed the turn.
- Note that the seat height has a major influence on the centre of gravity. The higher the seat height, the greater the risk of tipping over.
- Drive very carefully on soft surfaces, especially on uphill and downhill slopes.
- During the process, pay particular attention to the tipping stability of the electric vehicle.
- DO NOT attempt to carry out maintenance work that is not described in this user manual. Such repair and/or maintenance must be carried out by a qualified technician. Contact a dealer.
- For safety reasons and to prevent accidents resulting from inadvertent wear and tear, it is important that this electric vehicle is serviced annually under normal operating conditions.
- Under difficult operating conditions, e.g. daily driving on steep slopes or in the case of use in medical care with frequent changes of wheelchair users, it is advisable to carry out intermediate checks on the brakes, accessories and running gear.
- Spare parts must match the original parts.
- Always state the serial number of the electric vehicle to make it easier to order the correct spare parts.
- Only use batteries that have been approved by IMPROVING MOBILITY DEVELOPMENT S.L. for this electric vehicle.
- Continued use of an electric vehicle that has not been adjusted to the correct specifications could cause erratic vehicle behaviour resulting in damage, serious injury or death.
- After any adjustment, repair or maintenance and before use, ensure that all fasteners are assembled and tightened securely.
- Drive very carefully on soft surfaces, especially on uphill and downhill slopes.
- During the process, pay particular attention to the tipping stability of the electric vehicle.
- DO NOT attempt to carry out maintenance work that is not described in this user manual. Such repair and/or maintenance must be carried out by a qualified technician. Contact a dealer.
- For safety reasons and to prevent accidents resulting from inadvertent wear and tear, it is important that this electric vehicle is serviced annually under normal operating conditions.
- Under difficult operating conditions, e.g. daily driving on steep slopes or in the case of use in medical care with frequent changes of wheelchair users, it is advisable to carry out intermediate checks on the brakes, accessories and running gear.
- Spare parts must match the original parts.
- Always state the serial number of the electric vehicle to make it easier to order the correct spare parts.
- Only use batteries that have been approved by IMPROVING MOBILITY DEVELOPMENT S.L. for this electric vehicle.
- Continued use of an electric vehicle that has not been adjusted to the correct specifications could cause erratic vehicle behaviour resulting in damage, serious injury or death.
- After making handle angle adjustments and before use, the handle must be firmly locked in position.
- Otherwise, the scooter could fall, resulting in personal injury and/or damage to the vehicle. Gently push or pull on the handle to ensure that the handle is securely attached to the adjustment plate.
- Never drive around obstacles at a steep angle.
- Before overcoming obstacles, place the backrest in an upright position.
- DO NOT attempt to get off the scooter on a slope or incline.
- Protect the battery charger from moisture. Always charge the batteries in a dry environment.
- Do not use the battery charger if it has been dropped or damaged.
- NEVER attempt to recharge the batteries by connecting the cables directly to the battery terminals.
- Only use an extension cable if absolutely necessary. If you must use an extension cord, make sure it is in perfect condition.
- Under no circumstances may this electric vehicle be used as a vehicle seat or to transport the user in a vehicle.
- Risk of twisting when lifting heavy parts. Use the correct lifting techniques.
- When cleaning electronic components, observe their protection class with regard to water ingress.
- Make sure that no water splashes onto the plug or wall socket.
- DO NOT touch the socket with wet hands.

- All cleaning agents and disinfectants used must be effective, compatible with each other and must protect the materials to be cleaned.
- Check that all wheels, bearings, handlebars, seat and other parts are in good condition.
- Never use corrosive fluids (alkalis, acids, etc.) or abrasive cleaning agents.
- We recommend using a normal household cleaner, such as dishwashing liquid, unless otherwise specified in the cleaning instructions.
- Do not use solvents (cellulose strippers, acetone, etc.) that change the structure of the plastic or dissolve the attached labels.
- Always make sure that the product has dried completely before using it again.
- When the scooter is not in use, it is recommended that it be kept indoors or protected outdoors with a cover and properly supervised.
- The product must not be cleaned in automatic washing systems, using steam or pressure cleaning equipment.
- Dirt, sand and salt water may damage the bearings and steel parts may rust if the surface is damaged.
- This product may contain substances that could be harmful to the environment if it is disposed of in places (landfill sites) that are not legally permitted.
- DO NOT dispose of the battery with household waste.
- DO NOT dispose of the battery in a fire.
- The battery must be taken to a suitable collection point. There is no charge for returning the battery and it is required by law.
- Only dispose of the discharged battery.
- Cover the lithium battery before disposal.
- IMPROVING MOBILITY DEVELOPMENT S.L. is not liable for its own damage, damage to third parties or damage to property, whether accidental or due to bad faith.
- IMPROVING MOBILITY DEVELOPMENT S.L. is not liable for theft or loss of parts or the scooter.
- The CE marking will be invalidated if components or accessories are replaced or added that have not been approved by IMPROVING MOBILITY DEVELOPMENT S.L. for this product.
- In this case, the company adding or replacing the components or accessories shall be responsible for the conformity assessment/ CE marking or registration of the electric vehicle as a special design and also for the related documentation related.
- This scooter is suitable for air travel.

BATTERY CHARGING AND MAINTENANCE

1. Frequent full charging: Be sure to charge the battery daily, even after short rides.
2. Regular deep discharge: Every two months, allow the battery to fully discharge and then recharge it.
3. Avoid high current discharges: Do not overcharge the scooter and avoid excessive strain in hot or cold weather.
4. Proper storage: Keep the scooter in a cool, dry place away from direct sunlight.

MAINTENANCE AND CLEANING

- Use a soft, slightly damp, wrung out cloth. In addition, inspect and maintain the electric scooter regularly.
- Avoid abrasive chemicals.

PERFORMANCE PARAMETERS

- Maximum speed: 4.3 mi/h.
- Maximum slope: 9°.
- Braking distance: ≈ 39" on dry pavement.
- Braking method: Electromagnetic.
- Lithium battery: 24V/10AH.
- Motor: 24V/180W.
- Charger: 24V/2A.
- Charger cable: 60".
- Control panel: 24V/45A

WARRANTY

IMPROVING MOBILITY DEVELOPMENT S.L. guarantees that this device is free from defects.

- The warranty on the motor and control panel will remain valid for 2 years from the date of original purchase by the consumer.
- The warranty on the battery and charger shall remain in effect for 1 year from the date of original purchase by the consumer.
- The product warranty shall remain in effect for 3 years from the date of original consumer purchase.
- The warranty does not extend to parts subject to natural wear and tear that require periodic replacement. This warranty does not cover:
 - Replacement parts supplied by anyone other than IMPROVING MOBILITY DEVELOPMENT S.L. or the authorised distributor.
 - Any failure of the unit during the warranty period that is not caused by a defect in material or workmanship or if the failure is caused by misuse.
- IMPROVING MOBILITY DEVELOPMENT S.L. is not responsible for its own damage, damage to third parties or damage to property, whether accidental or in bad faith.
- IMPROVING MOBILITY DEVELOPMENT S.L. is not liable for theft or loss of parts or the scooter.

PEPE - SCOOTER PLEGABLE (NEGRO)	P10042
PEPE - SCOOTER PLEGABLE (BLANCO)	P10043
PEPE - SCOOTER PLEGADO AUTOMÁTICO (NEGRO)	P10048
PEPE - SCOOTER PLEGADO AUTOMÁTICO (BLANCO)	P10049

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EN |

Serial Number	Fault Phenomenon	Cause of Fault	Elimination Methods
1	The driving lever is defective or the maximum speed is low	<ul style="list-style-type: none"> ① Battery voltage is too low ② The cable in the driving lever is loose ③ The driving lever is stuck or fails 	<ul style="list-style-type: none"> ① Fully charge the battery ② Clean or replace the driving lever ③ Contact the supplier
2	The motor does not work when turned on	<ul style="list-style-type: none"> ① Battery voltage is too low ② The cable in the driving lever is loose ③ Motor failure 	<ul style="list-style-type: none"> ① Fully charge the battery ② Contact the supplier ③ Consult a professional or repair station
3	Insufficient mileage on a single charge	<ul style="list-style-type: none"> ① Insufficient charging or charger failure ② Battery is aging or damaged ③ Many slopes, headwind, frequent braking, heavy load 	<ul style="list-style-type: none"> ① Fully charge or replace the charger ② Contact the supplier ③ Try to avoid these bad conditions
4	The charger does not work	<ul style="list-style-type: none"> ① Charger plug is disconnected or connector is loose ② Charger or scooter plug fuse is burned ③ Battery wires are disconnected 	<ul style="list-style-type: none"> ① Secure plugs and connectors ② Replace the fuse ③ Connect the wires
5	Abnormal noise during driving	<ul style="list-style-type: none"> ① Screws or nuts are loose ② Wheel bearing are worn 	<ul style="list-style-type: none"> ① Tighten loose screws or nuts ② Replace the bearings
6	Scooter power display shows nothing	<ul style="list-style-type: none"> ① Battery voltage is too low ② The cable in the driving lever is loose 	<ul style="list-style-type: none"> ① Charge the battery correctly ② Connect the wires
7	Power is interrupted during driving	<ul style="list-style-type: none"> ① Battery thermal protector activated ② Motor overheating protection 	<ul style="list-style-type: none"> ① Reset the thermal protector ② Let the motor cool down before driving
8	Other faults	Other faults not covers in the guide above	Contact the supplier or repair station. Do not disassemble the scooter yourself

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READ BEFORE RETURNING, SAVE THE PLANET.

- EN** Returns generate more than 15 million tons of CO₂ annually.
- ES** Las devoluciones generan más de 15 millones de toneladas de CO₂ al año.
- DE** Retouren erzeugen mehr als 15 Millionen Tonnen CO₂ jährlich.
- IT** I resi generano oltre 15 milioni di tonnellate di CO₂ all'anno.
- FR** Les retours génèrent plus de 15 millions de tonnes de CO₂ par an.
- FR** Les retours génèrent plus de 15 millions de tonnes de CO₂ par an.
- JA** リターンは年間1,500万トン以上のCO₂を排出する。

- EN** If pieces are missing or have arrived damaged, before returning, **contact us**.
- ES** Si faltan piezas o están en mal estado, antes de devolver **contacta** con nosotros.
- DE** Wenn Teile fehlen oder beschädigt sind, **kontaktieren** Sie uns vor der Rücksendung.
- IT** In caso di parti mancanti o danneggiate, prima della restituzione, **contattaci**.
- FR** Si des pièces sont manquantes ou endommagées, avant de retourner nous **contacter**.
- JA** リターンは年間1,500万トン以上のCO₂を排出する。

IMPROVING MOBILITY DEVELOPMENT S.L.
Avda. Mare Nostrum 1, Bloque 1, Pta 16
46120, Alboraya, Valencia, Spain
admin@kmina.com | www.kmina.com