KMINA

USER MANUAL
MANUAL DE USUARIO
BENUTZERHANDBUCH
MANUEL D'UTILISATION
MANUALE UTENTE
ユーザーマニュアル

46 cm (18")	K10078
43 cm (17")	K10079

KMINA - ALUMINIUM WHEELCHAIR
KMINA - SILLA DE RUEDAS ALUMINIO
KMINA - ROLLSTUHL AUS ALUMINIUM
KMINA - FAUTEUIL ROULANT EN ALUMINIUM
KMINA - SEDIA A ROTELLE IN ALLUMINIO
KMINA - アルミ製車椅子

1 COMPONENTS / COMPONENTES / KOMPONENTEN / COMPOSANTS / COMPONENTI/コンポーネント

	ENGLISH	ESPAÑOL	DEUTSCH	FRANÇAIS	ITALIANO	日本語
1.	Handle	Empuñadura	Handgriff	Poignée	Maniglia	Maniglia
2.	Backrest	Respaldo	Rückenlehne	Dossier	Schienale	Schienale
3.	Armrest	Reposabrazos	Armlehne	Accoudoir	Bracciolo	Bracciolo
4.	Rear wheel	Rueda trasera	Hinterrad	Roue arrière	Ruota posteriore	Ruota posteriore
5.	Assist lever	Palanca de ayuda	Assistenzhebel	Lever d'assistance	Leva di assistenza	Leva di assistenza
6.	Brake lever	Palanca de freno	Bremshebel	Lever de frein	Leva del freno	Leva del freno
7.	Front castor	Rueda delantera	Vorderrad	Roue avant	Ruota anteriore	Ruota anteriore
8.	Footrest	Reposapiés	Fußstütze	Repose-pieds	Poggiapiedi	Poggiapiedi
9.	Seat	Asiento	Sitz	Siège	Sedile	Sedile
10.	Belt	Cinturón	Gurt	Ceinture	Cintura	Cintura
11.	Backrest lever	Palanca respaldo	Hebel für die Rückenlehne	Lever du dossier	Leva dello schienale	背もたれレバー
12.	Pocket	Bolsillo	Tasche	Poche	Tasca	Tasca
13.	Tools	Herramientas	Werkzeuge	Outils	Strumenti	用具

2 MEASURES / MEDIDAS / MESSUNGEN / MESURES / MISURE / 測定方法

WIDTH	HEIGHT	LENGTH	FOLDING DIMENSIONS
K10078: 71 cm (28") K10079: 68 cm (26.8")	92 (36")	103 cm (40.5")	35x87,5x76 cm (13.7"x34"x30")

SEAT HEIGHT	SEAT DIMENSIONS	BACKREST HEIGHT	Ø REAR WHEEL	Ø FRONT CASTOR
50 cm (19.6")	K10078: 46x42cm(18"x16.5") K10079: 43x42cm(17"x16.5")	40 cm (15.7")	60 cm (23.6")	18 cm (7")

MAX. USER WEIGHT	NET WEIGHT	TOTAL WEIGHT (with footrest)
136 kg (300 lb)	≈ 12 kg (26 lb)	≈ 13,5 kg (30 lb)

EN The **KMINA - Aluminium Wheelchair** has been designed to be self-propelled by the user using the large rear wheels, or pushed by another person using the handles. It supports a maximum weight of 136 kg, is made of aluminum, and is intended for independent users, elderly people, individuals with mobility issues or physical disabilities who still have enough autonomy to self-propel safely.

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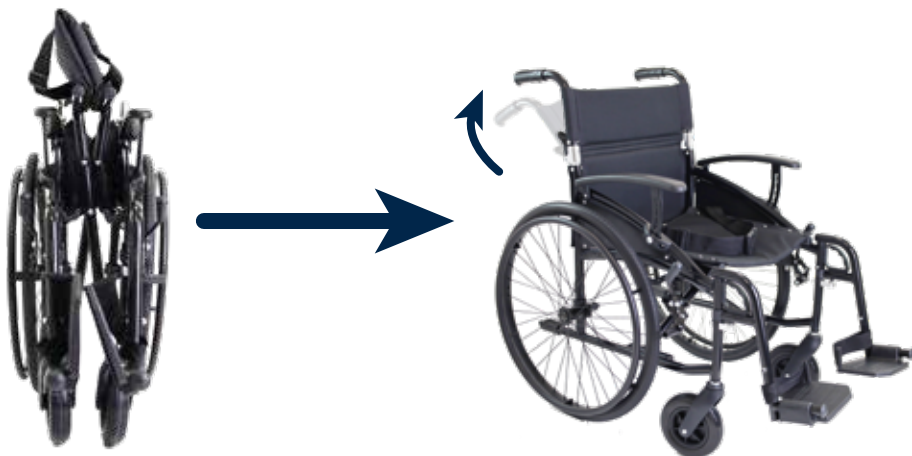
- EN**
1. To remove the footrests, press the lever, rotate the footrest, and detach the metal tubes.
 2. Fold the backrest by pushing the backrest lever downward.
 3. Pull the seat upward to fold the chair frame.

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2

3

EN **Press the button** located on the side of the wheelchair and lower the armrest, to assist the user's transfer into and out of the chair. It has 5 adjustment settings.

Make sure the area is clear before pressing the button to avoid the risk of entrapment.

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EN The footrest height can be adjusted for greater comfort in foot support. An Allen key and a hex wrench are required for this, both included with the wheelchair.

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EN The maximum slope for the use of parking brakes is 11,85° for a rear slope and 11,5° for a front slope.

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EN To deactivate the brake, push the brake lever upwards.

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EN To activate the brake, push the brake lever all the way down.

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EN

MATERIALS

- Structure: Aluminium.
- Armrests: Polyurethane.
- Handles: PVC.
- Plastic finishes: ABS.
- Fabric: Nylon fabric.
- Wheels: PP.
- Front castors: Aluminium and steel.

REFERENCE

KMINA - ALUMINIUM WHEELCHAIR (18")	K10078
KMINA - ALUMINIUM WHEELCHAIR (17")	K10079

WARNINGS

- Do not use the wheelchair without carefully reading the instructions.
- This chair may not be used as a seat in vehicles of any kind.
- Always transfer the wheelchair user to a seat approved by the vehicle manufacturer and use the available safety devices.
- Do not use this wheelchair on streets, roads or any surface with vehicle traffic.
- Do not attempt to use the wheelchair on ramps without proper assistance.
- Before inserting or removing the user, the wheelchair must be on a smooth and stable surface.
- Do not apply weight on the footrests when inserting or lifting the user into or out of the wheelchair.
- Do not lean or drop weight on the top of the wheelchair backrest, it may cause the wheelchair to tip over.
- Do not stand on the wheelchair.
- Sudden changes in direction when using a ramp or slope can cause instability and/or the possibility of tipping.
- Keep in mind that this chair is intended for transporting another person, and not for personal transport.
- Never stretch so far that you have to move around in the seat or out of the chair.
- When you have to reach for objects that require you to lean, make sure the front wheels point forward, as this will stabilize the chair.
- The temperature of the wheelchair may increase when exposed for long periods of time to external heat sources, such as sunlight.
- If you will be storing the wheelchair for an extended period of time, do not leave it exposed to prolonged weathering.
- Do not exceed the maximum supported weight, which is 300 lb.
- In accordance with the manufacturer's recommendations, this product is suitable for recycling at designated recycling points.
- Any serious incident involving the product should be reported to the manufacturer and to the competent authority of the Member State in which the user and patient are established.
- Retain these instructions for future reference.

MAINTENANCE

- Inspect the chair before each use to ensure that there is no damage or excessive wear. If you find any signs of wear or damage to your chair, DO NOT USE IT.
- Regularly check that all parts are sound before operation and that the casters turn smoothly. If you notice that the casters are loose, tighten them using the wrench provided.
- Keep the wheelchair clean, the steel tubes can be easily cleaned with a damp cloth. The fabric parts should be washed by hand (max. 30°) and with neutral soap.

WARRANTY

KMINA warrants this wheelchair to be free from defects. The warranty will remain in effect for 3 years from the date of original purchase by the consumer. The warranty does not extend to parts exposed to natural wear and tear that require periodic replacement, such as brakes, wheels and handgrips. This warranty does not cover:

- Replacement parts supplied by anyone other than KMINA or the authorized dealer.
- Any failure of the unit during the warranty period that is not caused by a defect in material or workmanship or if the failure is caused by misuse.
- KMINA is not liable for accidental damage or damage to property.

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