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MADE IN ITALY

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Class I Medical Device in accordance with Annex VIII,
Regulation (UE) 2017/745 (rule 1 and rule 13).

Rev. 17 - February 2024

The information contained in this manual belongs exclusively to **KSP Italia S.r.l.**; therefore, the written authorisation of KSP Italia S.r.l. is required prior to whole or partial reproduction thereof. The information must only be used for the purpose for which it was intended.

1. Table of Contents

Summary

1.	Table of Contents	56
2.	Labelling.....	57
3.	Declaration of Conformity	58
4.	The Manual	60
5.	Whom it is for	61
6.	General warnings.....	62
7.	Graphic Symbols	64
8.	Description and Intended Use	67
9.	EASY UP N3100 Technical Specifications.....	69
10.	EASY UP N3200 ELEKTRO Technical Specifications	72
11.	Adjustments and Commands	77
12.	Transport, Handling and Unpacking	87
13.	Assembly.....	89
14.	First Start-Up	90
15.	Use	92
16.	Transport	99
17.	Foreseeable improper use.....	100
18.	Instructions for emergency situations.....	101
19.	Problems-Causes-Remedies	102
20.	Maintenance.....	103
21.	Periodic Maintenance.....	105
22.	Disposal	106
23.	Warranty.....	107
24.	Assistance, Home service	108
25.	Reports – Alert.....	108
26.	Notes	108

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**COMPANY WITH
QUALITY SYSTEM
CERTIFIED BY DNV GL
= ISO 9001 =**

ksp@kspitalia.com
www.kspitalia.com

Declaration of Conformity (in accordance with ISO/IEC 17050-1)

1) No. 049/24 - **Date 21/02/2024**

2) Issuer's name: **KSP ITALIA SRL**
Issuer's address: **VIA DELL'ARTIGIANATO 1, 06031 BEVAGNA (PG), ITALY Tel. 0742. 36.19.47**
Fax 0742.36.19.46 www.kspitalia.com, e-mail: ksp@kspitalia.com

N° EUDAMED SRN: **IT-MF-000009316**

3) Object of the declaration: **STANDING EASY UP MANUAL LIFTER N3100**

4) The Manufacturer KSP Italia declares under his sole responsibility that the medical device above described complies with all applicable requirements of the following legislation and fulfils all applicable provisions thereof (and any other relevant UE legislation providing for the issuance of EU declaration of conformity):

Documents No.	Title	Edition/Date of issue
5) Regulation (EU) 2017/745	Medical Devices Regulation	Emission: 5 April 2017
Regulation 2011/65/EU	Restriction of the use of certain hazardous substances in electrical and electronic equipment	Emission: 8 June 2011

Additional information:

- 6) Medical devices designed and manufactured with quality management system compliant to ISO 13485.
CE Marked Medical device in accordance with Annex II and Annex III, Regulation (UE) 2017/745.
Class I medical device as for rule 1 and rule 13, Regulation (UE) 2017/745, Annex VIII.
Registered at the Italian Ministry of Health with number: 2269362.

BASIC UDI-DI (GMN): **805577318STANDING-MANUQF**

Signed for and on behalf of:

KSP Italia Srl

Bevagna, li 21/02/2024

7) Claudio Emanuelli,
Legal Representative

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**COMPANY WITH
QUALITY SYSTEM
CERTIFIED BY DNV GL
= ISO 13485 =**

**COMPANY WITH
QUALITY SYSTEM
CERTIFIED BY DNV GL
= ISO 9001 =**

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ksp@kspitalia.com
www.kspitalia.com

Declaration of Conformity (in accordance with ISO/IEC 17050-1)

- 1) No. 050/24 - Date 21/02/2024
- 2) Issuer's name: KSP ITALIA SRL
Issuer's address: VIA DELL'ARTIGIANATO 1, 06031 BEVAGNA (PG), ITALY Tel. 0742.36.19.47
Fax 0742.36.19.46 www.kspitalia.com, e-mail: ksp@kspitalia.com

N° EUDAMED SRN: IT-MF-000009316
- 3) Object of the declaration: **STANDING ELEKTRO EASY UP, ELECTRICAL LIFTER N3200**
- 4) The Manufacturer KSP Italia declares under his sole responsibility that the medical device above described complies with all applicable requirements of the following legislation and fulfils all applicable provisions thereof (and any other relevant UE legislation providing for the issuance of EU declaration of conformity):

Documents No.	Title	Edition/Date of issue
5) Regulation (EU) 2017/745	Medical Devices Regulation	Emission: 5 April 2017
Regulation 2011/65/EU	Restriction of the use of certain hazardous substances in electrical and electronic equipment	Emission: 8 June 2011

Additional information:

- 6) Medical devices designed and manufactured with quality management system compliant to ISO 13485.
CE Marked Medical device in accordance with Annex II and Annex III, Regulation (UE) 2017/745.
Class I medical device as for rule 1 and rule 13, Regulation (UE) 2017/745, Annex VIII.
Registered at the Italian Ministry of Health with number: 2269390.

BASIC UDI-DI (GMN): **805577318STANDING-ELETPH**

Signed for and on behalf of:

KSP Italia Srl

Bevagna, li 21/02/2024

- 7) Claudio Emanuelli,
Legal Representative

4. The Manual

This use and maintenance manual, together with the instructions provided on the labels, is the *information provided by the manufacturer* in conformity with Directive Regulation MDR (UE) 2017/745.

Each device must be accompanied by the information necessary for safe use, keeping in mind the training and knowledge of potential users.

This manual is an integral part of the medical device and must be carefully preserved. It is intended for users, the owner, those responsible for safety, and maintenance personnel.

It provides information about technical specifications, correct use, transport, storage, maintenance and disposal of the device as well as the associated safety recommendations; however, this manual is no substitute for an experienced user.

The manufacturer will promptly notify owners of the product of any changes to the instructions that affect the safety of the patient or user. Other changes or additions are excluded from the obligation to give notice.

If this manual or the labels and/or markings on the product are completely or partially damaged, faded or illegible, you must immediately request a copy from your reseller or the manufacturer.

This manual refers to the products STANDING EASY UP N 3100 e N 3200 ELEKTRO. Where not indicated otherwise, the instructions are to be understood as applicable to both standings. The peculiarities of one or the other model will be explicitly highlighted.

5. Whom it is for

The manual has been written taking into account the characteristics, knowledge, cultural level and training of the user.

The user must have the following characteristics:

- Older than age 18
- Physically strong and emotionally healthy
- Capable of evaluating hazardous situations and acting calmly and thoughtfully
- Excellent knowledge of spoken and written English
- Understands the meanings of the symbols and markings

We would also like to point out that an additional condition necessary for the use of the device is that it's positioning and accessories are adjusted by a health professional or by someone who has taken part in specific training on the proper use of the device by the manufacturer or authorized dealer thereof

6. General warnings

To verify the suitability of the environment where the device will be employed, use the information in this manual or contact technical support.

WARNING!!! The device is not suitable to accommodate agitated patients.

In any case, the device:

- Is intended only for indoor use, and may not be used outdoors
- It must be used on level ground, which is not wet or slippery
- The patient on board can be moved only in the version with the 4 pivoting wheels with brakes.

Upon delivery of the device

- *If the device has been damaged in transport when you receive it, do not use it, but contact the technical support department to have it checked and repaired.*
- *If requested, authorized personnel will perform a series of demonstrative tests and issue a specific document attesting to its use.*
- *The device must be operated by a person who has the physical capacity to control the mass of the device and the person transported, especially in cases of emergency or malfunctioning*
- *The operator must be familiar with the entire contents of this manual.*
- *It is essential that the operator/assistant perform preliminary tests to verify the suitability of the device to carry out the tasks for which it is intended*
- *Before using the device each day, check that it is in good condition and that all of its operational and safety devices are working properly.*

WARNING!!! THE DEVICE IS POWERED BY A HIGH VOLTAGE 220 VAC. DO NOT KEEP LIQUIDS OR LIQUID CONTAINERS ON THE TABLE, ESPECIALLY WHILE HANDLING.

ANY SPILLAGE ONTO THE CONTROL PANEL OR POWER SOURCE MAY POSE AN ELECTRICAL HAZARD.

General Warnings:

- Never use the stair climber on surfaces that are wet, smooth or slippery, waxed, icy or similar, that do not ensure the operator's stability.
- Carpets and mats may be hazardous.
- When the device is powered on, do not open or touch the internal parts which are electrically operated.

WARNING!! Use of the device by untrained individuals and/or failure to follow the manufacturer's instructions may result in danger.

WARNING!! Undergo the appropriate training prior to use.

WARNING!! If the presence of an assistant is necessary, the operator should never leave the device unless in emergency situations.

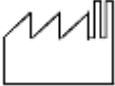
Disposal

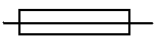
The standing is an electrical and electronic device that must be disposed of by KSP Italia or the APIRAEE consortium, whose mandate is the disposal of electrical and electronic waste.

In this regard, check the section on disposal. In countries other than Italy, the disposal must take place according to the local regulation and transposition of the waste electrical and electronic equipment (WEEE) Directive or related regulation.

7. Graphic Symbols

Symbol	Meaning	Notes
	<i>Mark of conformity to European Directives</i>	
	<i>Safe work load</i>	
	<i>WEEE</i>	
	<i>Date of manufacture</i>	
	<i>Alternating current</i>	
	<i>Operating instructions</i>	
	<i>Dangerous voltage</i>	
	<i>Warning sign</i>	
	<i>Warning or generic hazard</i>	
	<i>Warning of dangerous voltage</i>	

Symbol	Meaning	Notes
	<i>Moving parts</i>	
	<i>Danger of crushing feet</i>	
	<i>Prohibition sign</i>	
	<i>Prohibition against the operator wearing slippers</i>	
	<i>Prohibition against wearing shoes with heels</i>	
	<i>Mandatory behaviour sign</i>	
	<i>Generic mandatory behaviour sign</i>	
	<i>Refer to the instruction booklet</i>	
	<i>Non-skid shoes required</i>	
	<i>Fuse</i>	

Use and Maintenance Manual

Description and Intended Use

KSP Italia Srl

Symbol	Meaning	Notes
IP _{N₁} N ₂	Protection grade	N1 = - 0 Unprotected - 1 Protected against the penetration of foreign solid bodies of Ø 50 mm and larger - 2 Protected against the penetration of foreign solid bodies of Ø 12.5 mm and larger - 3 Protected against the penetration of foreign solid bodies of Ø 12.5 mm and larger - 4 Protected against the penetration of foreign solid bodies of Ø 12.5 mm and larger - 5 Protected against dust - 6 Totally protected against dust N2 = - 0 Unprotected - 1 Protection against the vertical fall of drops of water - 2 Protection the vertical fall of drops of water when the HOUSING is inclined up to 15° - 3 Protected against rain - 4 Protected against sprays of water - 5 Protected against jets of water - 6 Protected against powerful jets of water - 7 Protected against the effects of temporary immersion in water - 8 Protected against the effects of continuous immersion in water NOTE: When a characteristic digit is not required, it is replaced with the letter "X" ("XX" if both digits are omitted).
	UDI DI assegnato al Dispositivo	
	Do not use outside, keep dry!	
	Fabbricante	
	Modello	
	Numero di serie	
	Dispositivo Medico	

8. Description and Intended Use

The products EASY UP N 3100 and N 3200 ELEKTRO are support devices for the disabled which allow people with reduced motor skills to bring them into an upright position and to remain there for a period of time with the assistance of an operator. These are Class I medical devices in accordance with Annex VII of Regulation (EU) 2017/745, and are built in compliance with national and international regulations regarding medical devices and aids for the disabled. The products have been duly registered in the Database and Repertory of Medical Devices of the Ministry of Health.

The product is particularly suitable for patients with the following conditions, though not limited thereto:

- Quadriparesis (quadriplegia)
- Multiple sclerosis
- Cerebral palsy
- Advanced demyelinating disorders
- Lower limb paralysis (paraplegia)
- Meningomyelocele
- Vertebral fractures

The device is indicated for subjects with reduced motor skills who require assistance in order to assume and maintain an upright position with the aid of trained personnel or otherwise, provided that in the latter case the patient possesses sufficient autonomy and physical prowess to perform such operations. The assessment of this is required for the buyer or the person responsible for prescribing the device.

The device consists of a metal structure with two telescopic columns, a wooden platform for foot support, and a wooden table.

The device is available in the manual version N 3100 or the electric version N 3200 ELEKTRO.

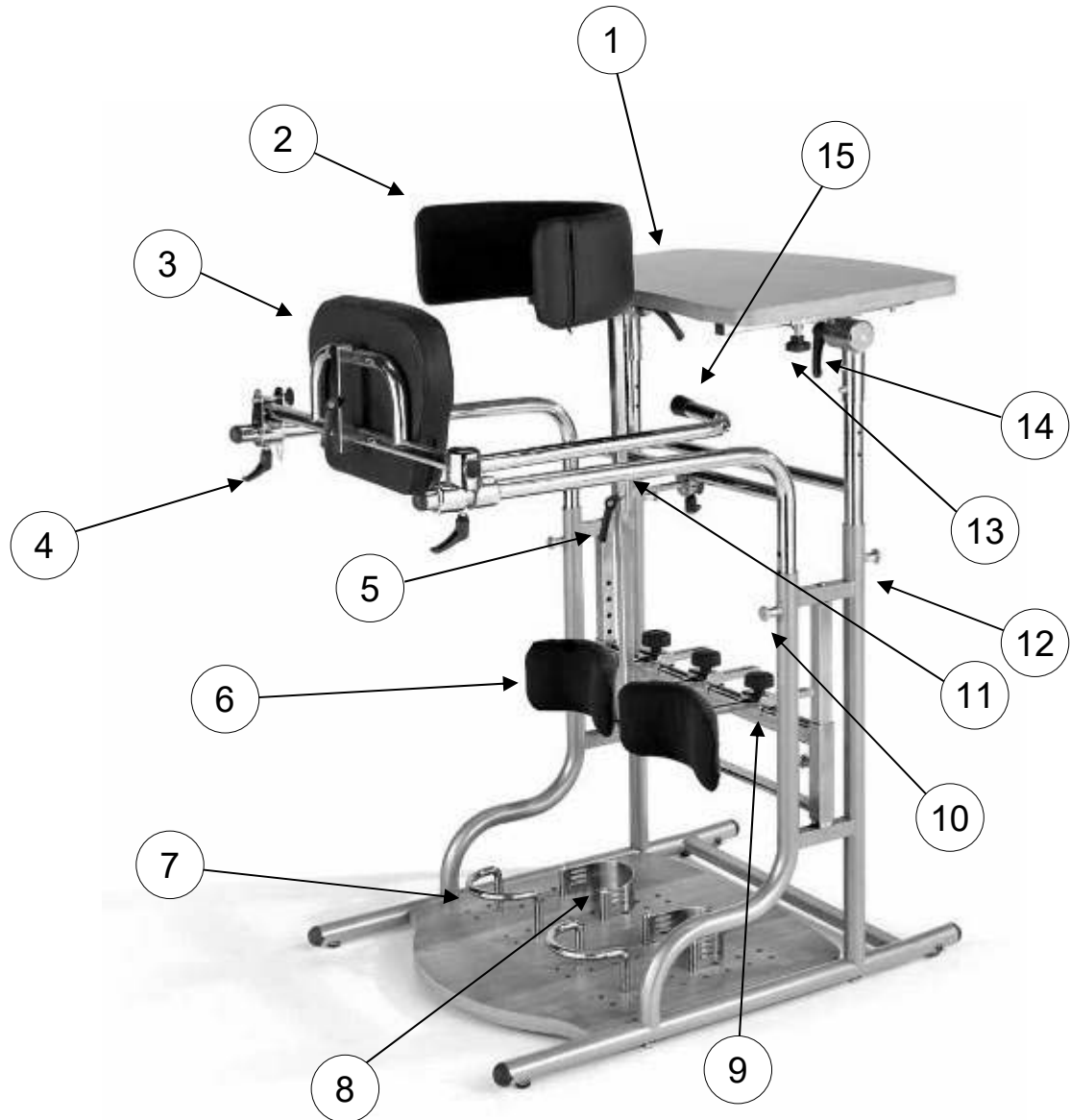
In the version N3100 ascending motions are done manually with the help of an assistant, or autonomously by the patient by using the lateral handlebars.

The version N3200 ELEKTRO is equipped with a lifting mechanism with a hoisting sling, which through the use of an electric motor brings the patient to an upright position, and it has double-brake wheels to allow for movement while the patient is onboard.

Both versions may be equipped with accessories to help maintain an upright posture (headrest, pelvic support, lateral pelvic thrust, etc) as shown in the following chapters.

The use of these devices must be within the limits and according to the procedures explicitly described in this manual from the manufacturer. The producer of the device is free from any liability for damages or injury resulting from the improper use of the equipment by untrained personnel, modifications or unauthorized actions, the use of unauthorized spare parts, exceptional events, or the total or partial non-compliance with the instructions presented in this manual, which must be enclosed in the event of a transfer of the product to third-parties.

9. EASY UP N3100 Technical Specifications

1-Table

2-Pectoral support

3-Rear thrust

4-Adjustment knobs to regulate the forward movement of the back thrust

5-Adjustment knobs to regulate the height of the pectoral support

6-Knee pads

7-Heel stopper

8- Foot stopper

9- Kneepad adjustment knobs

10- Latch device to adjust the height of the handlebars

11- Hand support bars

12- Latch device to adjust the height of the table

13- Knob to adjust the forward movement of the table

14- Knob to adjust the tilt of the table

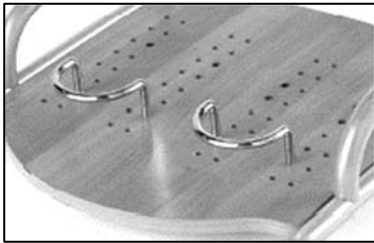
15- Handle for moving the rear thrust

N3100 ACCESSORIES

N 3001

Kneepad padded with variable ergonomics, independently adjustable in height, width and depth.

N 3002

Extractable heel stopper adjustable laterally or forwards/backwards.

N 3003

Pectoral support adjustable in height and depth.

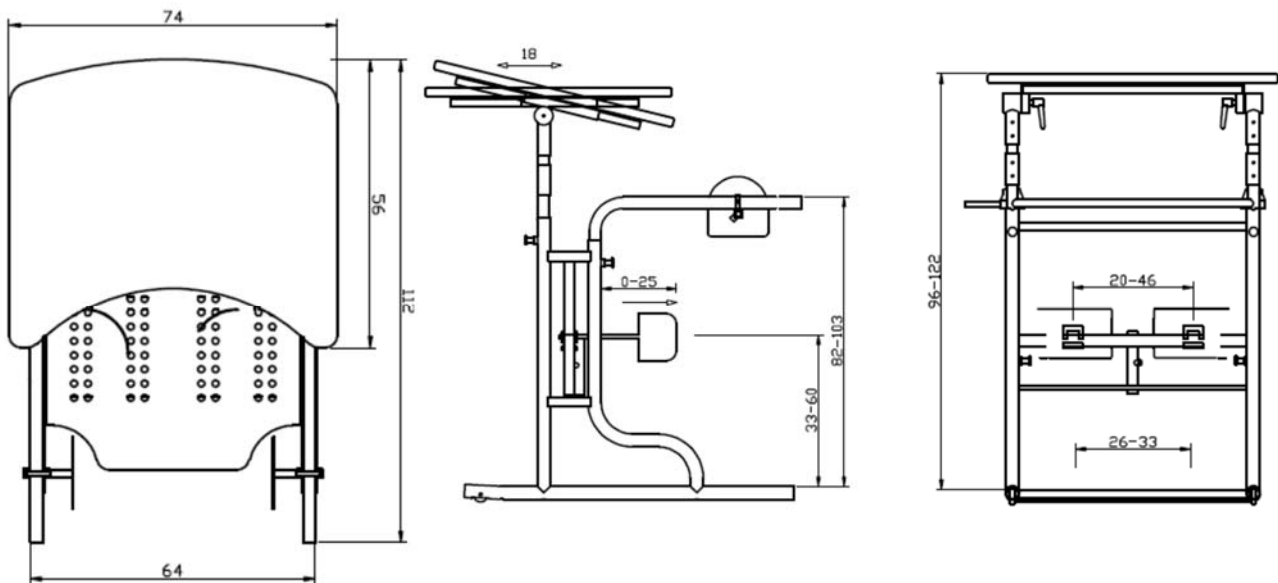
N 3004

Rear thrust adjustable forwards/backwards position with safety lock and arranged for the accessories mounting. Handle for opening and closing of the rear thrust.

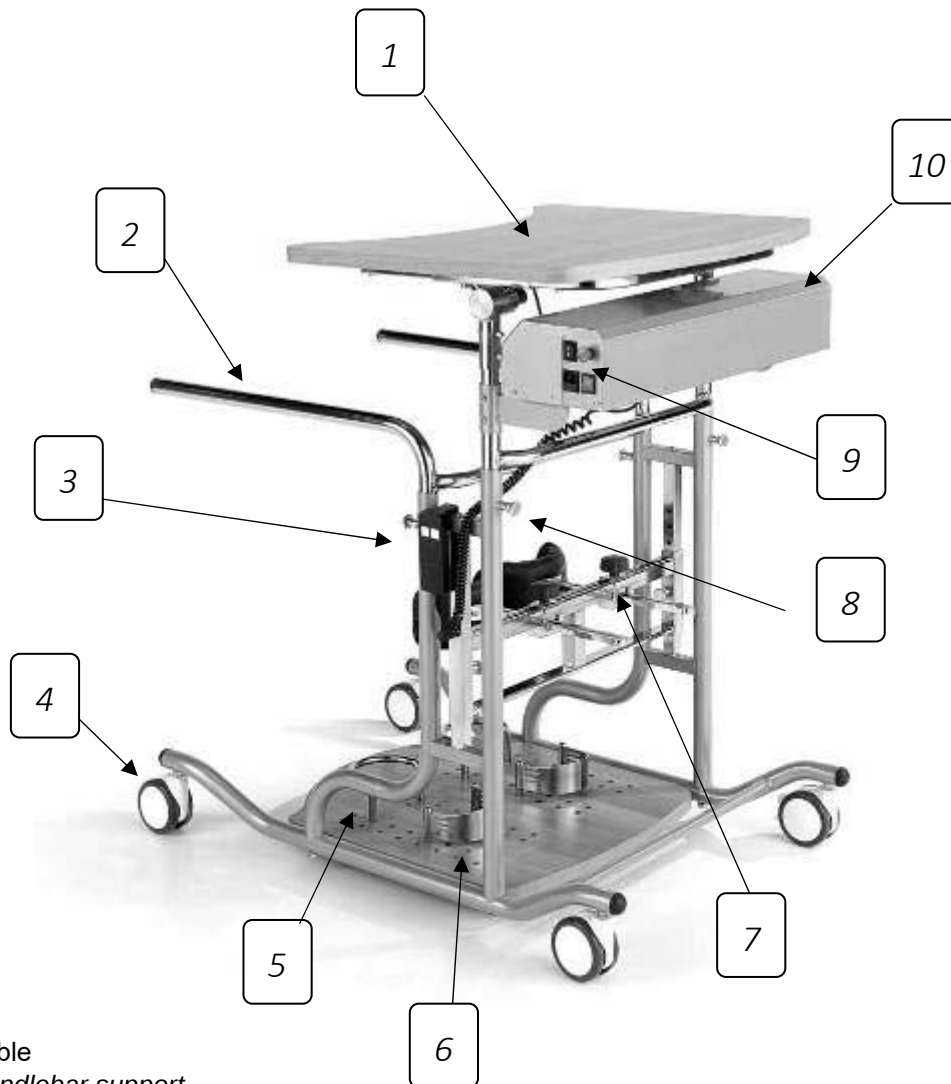
EASY UP N3100 Technical Specifications

- Adjustable sizes, from medium to large for adults.
- Twin column support frame with telescopic slide to independently adjust the height of the handrails and table. Finished polyester and epoxy.
- Base complete with 2 wheels (40 mm D.) to allow for movement.
- Walnut finish wood platform with holes for the placement of the heel and foot stoppers.
- Walnut finish wooden table, adjustable in height, depth, and forwards/backwards tilt.
- Maximum patient weight: 95 Kg
- Maximum patient height: 190 cm
- Temperature and storage in a dry environment: $-10^{\circ} \div 50^{\circ}\text{C}$
- Usage temperature: $10^{\circ} \div 50^{\circ}\text{C}$
- Device weight: 47 Kg

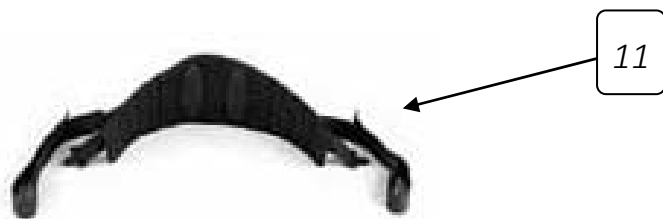
Measurements and Dimensions

10. EASY UP N3200 ELEKTRO Technical Specifications

- 12- Table
- 13- Handlebar support
- 14- Remote control connected by a wire
- 15- Pivoting brake wheels
- 16- Heel stoppers
- 17- Foot stoppers
- 18- Knobs to adjust the depth of the kneepads
- 19- Latch device to adjust the height of the table
- 20- Ignition and control panel
- 21- Electric actuator
- 22- hoisting sling

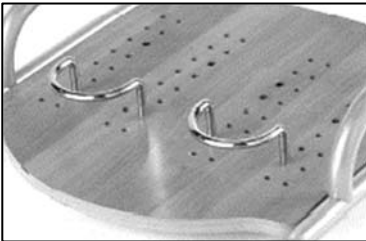
EASY UP N3200 ELEKTRO ACCESSORIES

N 3001

Kneepad padded with variable ergonomics, independently adjustable in height, width and depth.

N 3002

Extractable heel stopper adjustable laterally or forwards/backwards.

Lifting harness, complete with pelvic belt made from a stain-resistant fabric which is washable, hypoallergenic and non-toxic

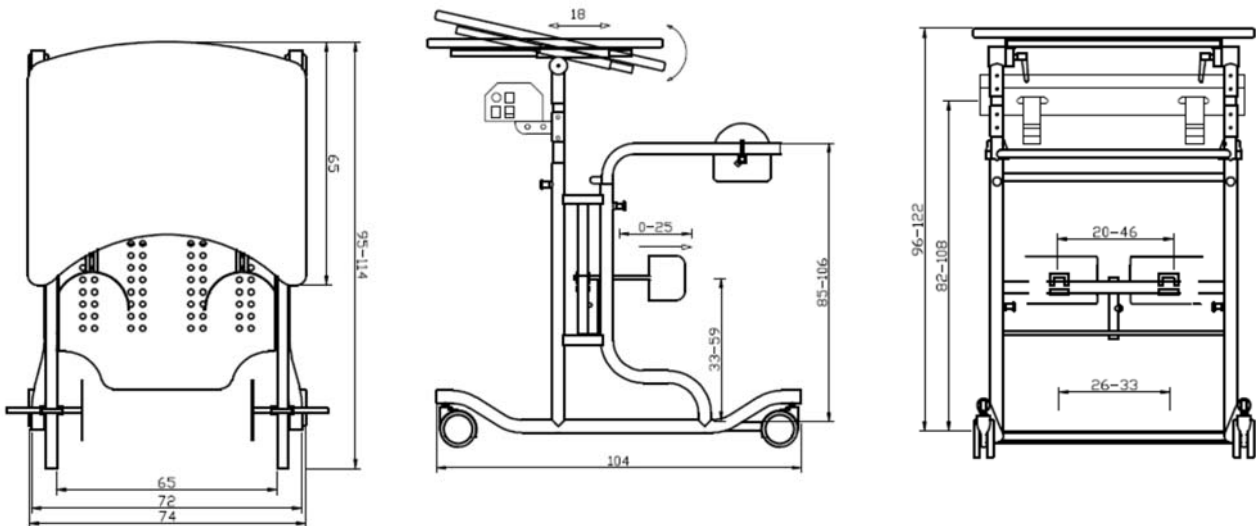
N 3201 – Standard size

N 3102 – Large size

EASY UP N3200 ELEKTRO TECHNICAL SPECIFICATIONS

- Adjustable sizes, from medium to large for adults.
- Twin column support frame with telescopic slide to independently adjust the height of the handrails and table. Finished polyester and epoxy.
- Internal gas pushers in the telescopic columns for easy height adjustment of the table
- Base complete with 4 pivoting 100 mm D. wheels to for movement, equipped with double-brakes.
- Walnut finish wood platform with holes for the placement of the heel and foot stoppers.
- Walnut finish wooden table, adjustable in height, depth, and forwards/backwards tilt.
- Electric assembly for the upward and downward movement of the lifting sling. 230V-300W motor and electronic control panel. Complete with push-button panel (non-wireless) and power cord.

- Front end-of-stroke switch
- An additional button to control ascent and descent.
- Emergency button.
- Device weight: 57 Kg
- Maximum patient weight: 125 Kg
- Maximum patient height: 190 cm
- Temperature and storage in a dry environment: $-10^{\circ} \div 50^{\circ}\text{C}$
- Usage temperature: $10^{\circ} \div 50^{\circ}\text{C}$

EASY UP N3100 - N3200 ELEKTRO ADDITIONAL ACCESSORIES

N 3005 Pelvis support

N 3006
Support for pelvis, back and head.

N 3007
Headrest and rear thrust

N 3008
Anterior removable foot stopper, adjustable in forwards/backwards
and lateral position, complete with strap foot stopper.

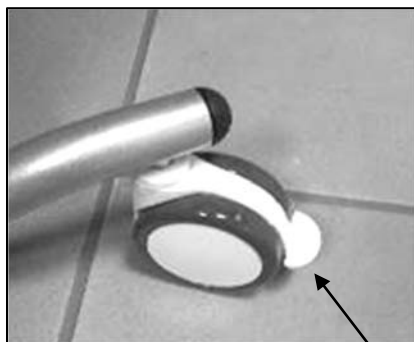
N 3009
Lateral pelvis thrust

N 3004
Rear thrust adjustable in forwards/backwards position with
safety lock and arranged for the accessories mounting.
Handle for opening and closing of the rear thrust.

11. Adjustments and Commands

WHEELS

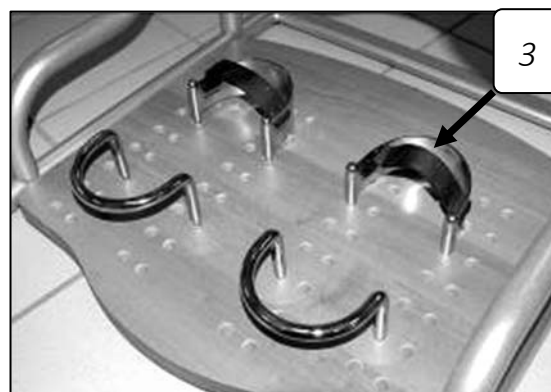
The electric version N3200 Elektro has four pivoting wheels with double brakes allowing for rotation and orientation

To activate the brake, push the lever (1) downwards, and conversely raise the lever (1) in order to release the brake.

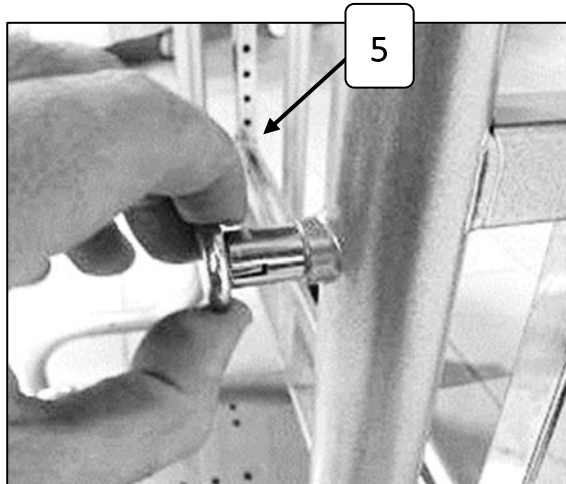
Adjusting the foot stoppers

The base of the device is equipped with a series of holes (4) designed to accommodate the heel stoppers (2) and foot stoppers (3) to allow for safe ascending motions. Insert the said devices in order to accommodate the distance between the feet and the size of the patient.

Adjusting the height of the handlebars

The height of the handlebars can be adjusted using the latch device

b. Grasp the head of the locking mechanism (5)

b. Pull it outwards

c. Turn the device

d. Release it so that it remains in the extended position and then repeat this operation for the both locking mechanisms

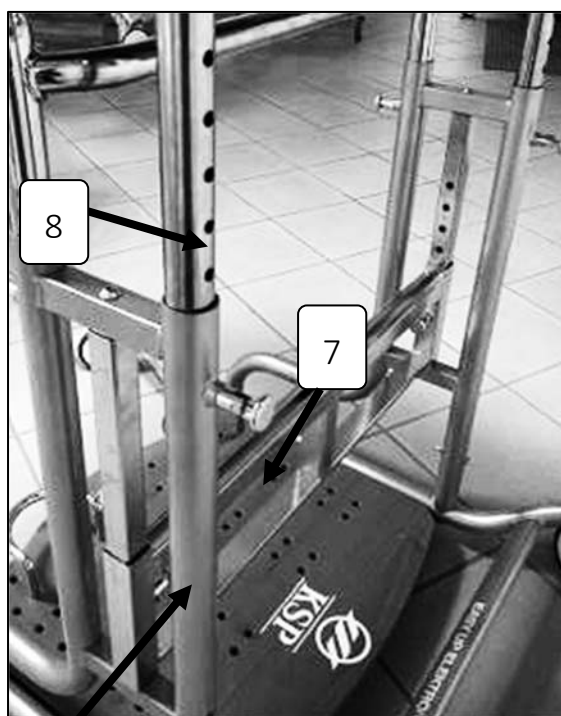
- g. Raise the handlebars to the desired height*

- h. Turn both latch devices so that the return into the holes (6)*

Adjusting the height of the table

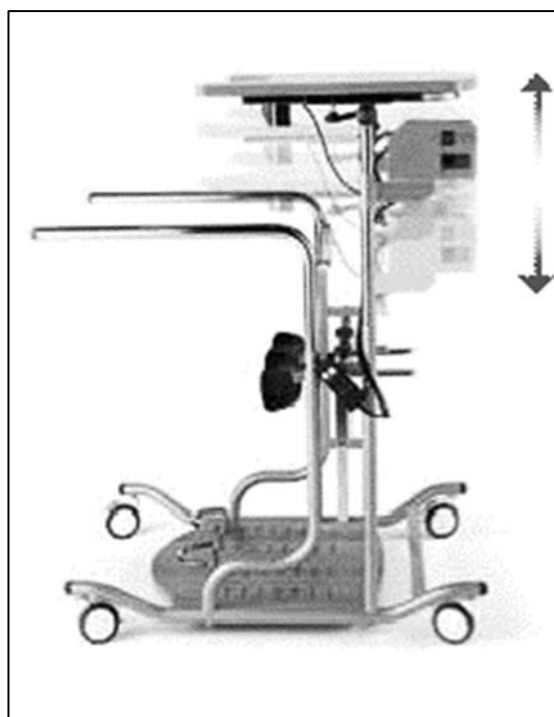
To raise the table, pull out the latch mechanism (7) as shown in the preceding paragraph and lift the top part of the frame by sliding the telescopic columns until they have reached the desired position.

Then re-insert the latch mechanism (7) to lock the table into the desired position.

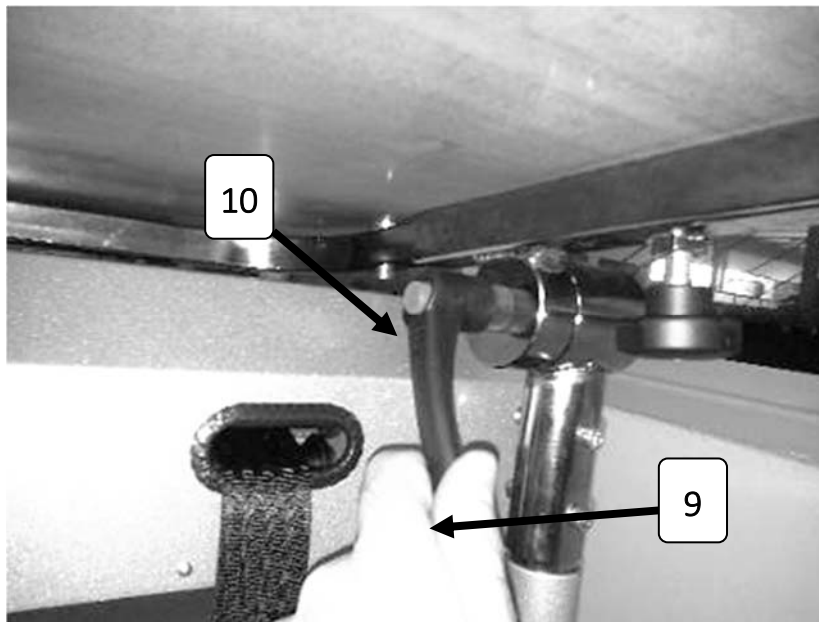
The version N 3200 Elektro is equipped with to gas pushers inserted into the support column (8B) of the table. These facilitate the raising of said section of the device in order to adjust the height thereof, while compensating for the added weight of the electric actuator.

8B

Adjusting the tilt of the table

- g. Grasp the handle (9) located below the table*
- h. Turn the lever counter-clockwise to loosen the table lock*
- i. If necessary, press the button (10) and return the lever to starting position in order to further loosen the grip*

- j. Repeat this operation on both handles until the table can be moved by hand*
- k. Adjust the table to the desired height*

- l. Secure the table in the desired position by turning the handle (9) clockwise until it has been completely fastened.*

Adjusting the position of the table

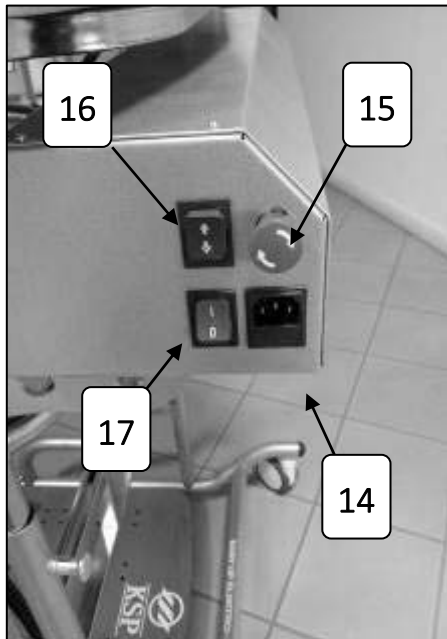
To adjust the depth of the table, turn the knobs (11) located under the table to loosen the locking mechanism and allow for movement.

Once the knobs have been loosened, slide the plane along the tracks until the desired position has been reached.

Once the adjustment has been made, turn the knobs (9) to firmly secure the table.

Ignition Interface

1. Connect the power supply cord into the socket (14)
2. To start the device, turn the ignition switch to position "1" (17)

Command Interface

The model N3200 Elektro may be operated both from the side panel of the engine compartment, as well as with the wired control panel.

With the side control panel, you can actuate the movement of the belts with the button (16) on the engine compartment.

2. If using the wired control panel, press the buttons (18).

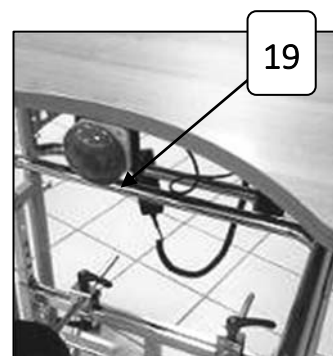
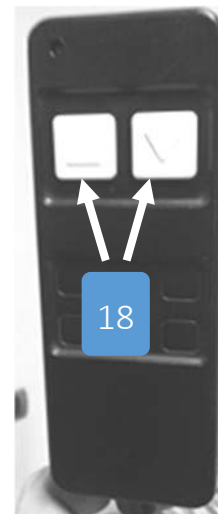
Emergency buttons

The device is equipped with two engine lock systems in the event of an emergency.

One is placed in the front with the control commands, and is intended primarily for caregivers.

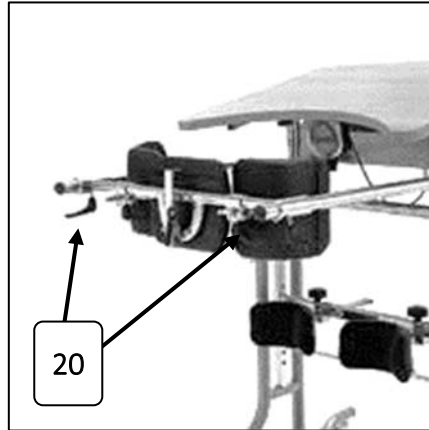
The other is located under the table, and is intended for the patient, being more readily available for said individual, and it also serves as a safety mechanism when standing up.

3. The emergency help button (15) is activated by push and can be reset by clockwise rotation.
4. The patient emergency/end-of-stroke button (19) is activated by push, without resetting, and only ascending motions.

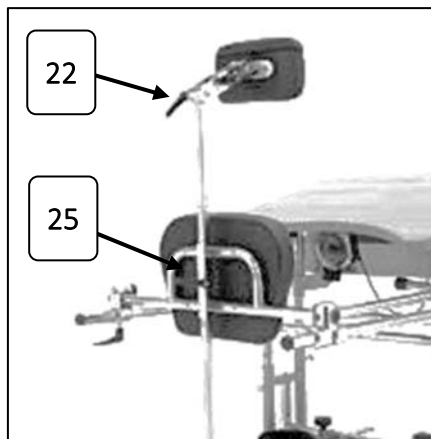
Adjusting Accessories

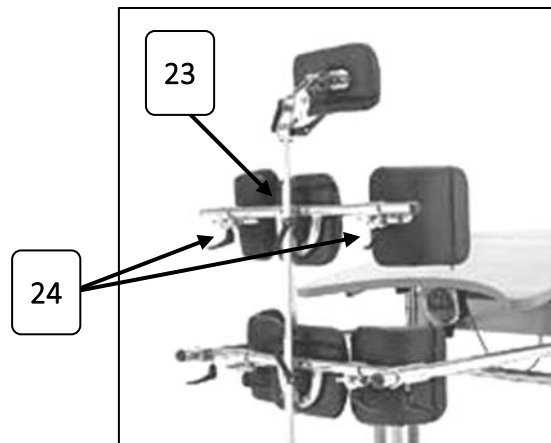
- g. Handles to adjust the depth of the rear pelvis support (20).

- h. Handles to adjust the tilt (22) and height (25) of the headrest accessory.

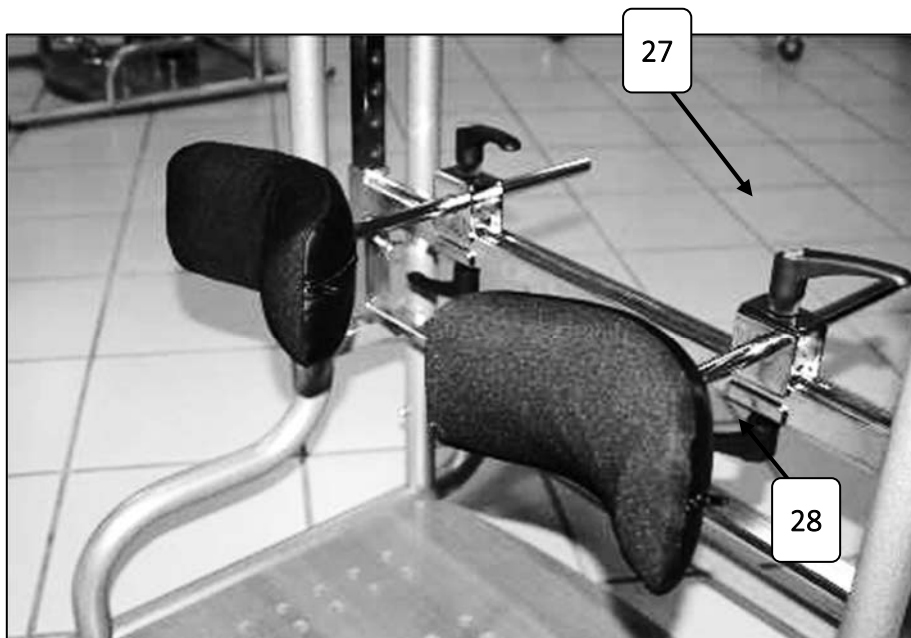
- i. Handle to adjust the height (23) and depth (24) of the back support.

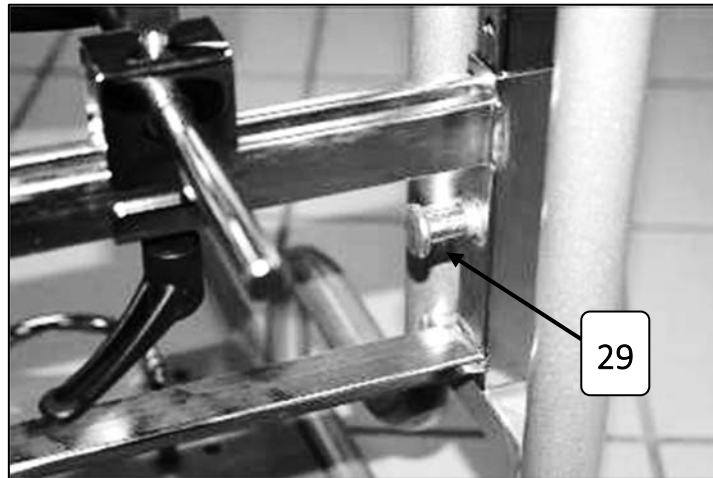
- j. Handle to adjust the lateral supports (26).

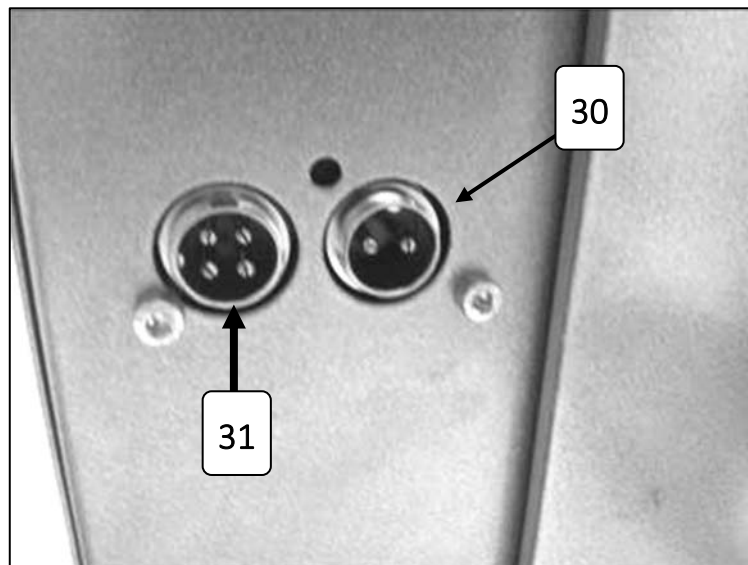
- k. Handles (27) and (28) knobs to regulate the distance-depth of the kneepads, latch mechanism (29) to regulate height.

- I. Connectors of the front end-of-stroke button (30) and control panel (31).

12. Transport, Handling and Unpacking

Packaging and Transport

The device is contained in a cardboard box on a pallet that adequately protects against the vibration occurring during normal transport. Transportation of the package should be done while maintaining a vertical position thereof and avoiding bumps or jerks. Upon receipt of the device, check the status of the integrity of the packaging.

If you note any damages, immediately notify KSP Italia and check the operability of the device after it has been inspected by Technical Support Staff.

The device is shipped with already assembled and does not require any additional assembly operations by the user other than geometric adjustments to suit the particular physical traits of the patient, as well as possibly coupling the wheels with the base.

To remove the device, cut the tape sealing the package and remove the upper portion thereof. With at least one other person, lift the device from the box and put it on the ground. The product is now ready for use.

Handling and Unpacking

To protect the operator from hazards deriving from the manual handling of loads, the person assigned to unpacking the device must be informed of the risks of said operations. This paragraph lists the basic operations for correct handling.

To lift the load correctly without risking harm to your spinal column, you must:

- 9- Keep your back straight
- 10- Keep your torso erect
- 11- Your body should be crouched
- 12- The weight lifted must be kept as close to the body as possible

Figure 4

Figure 5

- 13- For lifting loads that weigh more than 25 kg (if the operator is female, the load limit is 15 kg) ask others for help or use suitable equipment

Use and Maintenance Manual

Transport, Handling, Unpacking, and Storage

KSP Italia Srl

- 14- Do not store the products above shoulder height
- 15- Bring the package to a suitable area for the unpacking and assembly of the parts of the machine
- 16- Open the packing without using sharp or pointed objects that could damage the contents

Figure 6

It is a good idea to save the packing to reship the device (such as to the Technical Support) because the original packing ensures safe transport.

13. Assembly

Prerequisites: The device must not have damage or dents from transport. Improper adjustments of the device or its accessories may result in harm to the patient. Adjustments should be made by a qualified professional who the person prescribing the treatment.

Packaging content

The device comes packed and pre-assembled, therefore the purchaser is not required to perform any special assembly operations, except for as may be required for any accessories.

The accessories are listed in the section "Technical Specifications", and are assembled by inserting their respective tubes into the appropriate slots and by fastening the using the adjustment knobs and handles listed in the "Adjusting accessories" section.

In the version on wheels, the device is sent with its wheels not attached, and it's necessary for the purchaser to insert them by simply holding the components and inserting them into the appropriate slots.

Check that the insertion has been done correctly by turning the wheel around its axis and making sure that it is not able to be removed.

14. First Start-Up

Prerequisites: The device must be correctly assembled and adjusted to the anthropometric characteristics of the patient.

WARNING!! Place the knee pads below the patella near the upper end of the tibia.

EASY UP N 3100

2) Check-up, Adjustments

Prior to use, assistance personnel should:

- Adjust the height and depth of the knee pads
- Adjust the height of the handlebars
- Adjust the height and depth of the pectoral support
- Adjust the height of the table
- Adjust the height and depth of the lateral support and back support accessories
- Insert and adjust the heel stoppers and foot stoppers
- Make sure everything has been securely and effectively fastened

EASY UP N3200 ELEKTRO

In addition to the adjustments to the EASY UP N3100, also do the following:

- Fasten the harness and apply tensile force to verify the hold of the couplings
- Connect the power cord
- Turn on the device and run the motor to check its functionality
- Activate the emergency stop and end-of-stroke buttons to be sure that they are operational
- Perform three upward and downward operations with a non-disabled operator with the same general features as the patient.

We'd like to point out that the standing is a device for the disabled and that its use is only allowed for trained personnel familiar with its specific use. Said education and training must be provided by KSP Italia or by entities/persons authorized by the manufacturer, followed by a test and qualification certificate. The lack of the aforementioned documentation implies unfitness to use the equipment and relieves the manufacturer from any liability for damages to property or people attributed thereto.

15. Use

Pre-requisites: The device must be correctly assembled and pass the necessary tests prior to its first use. Support personnel must have written authorization to use the equipment and must have carefully read and understood the preceding paragraphs.

During the use of the device, the safety and well-being of the person being transported are of top priority. Therefore, follow the safety instructions provided at the beginning of this manual and all other recommendations received throughout the description of this device.

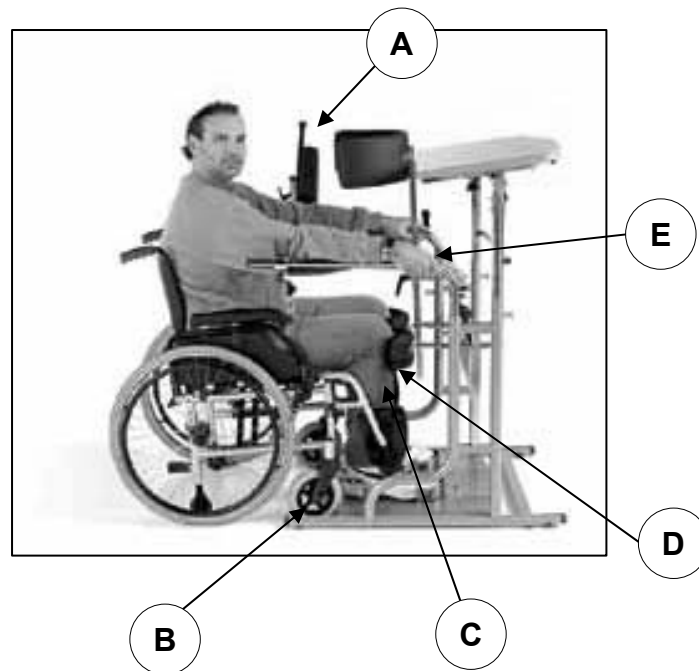
Sudden movements by the transported person should absolutely be avoided.

Ignition, Check-up, and Adjustments

Check that the adjustments have been made appropriate to the needs of the transported person and perform the checks listed in the "maintenance" section upon each use.

EASY UP N3100

ASCENDING MOTIONS

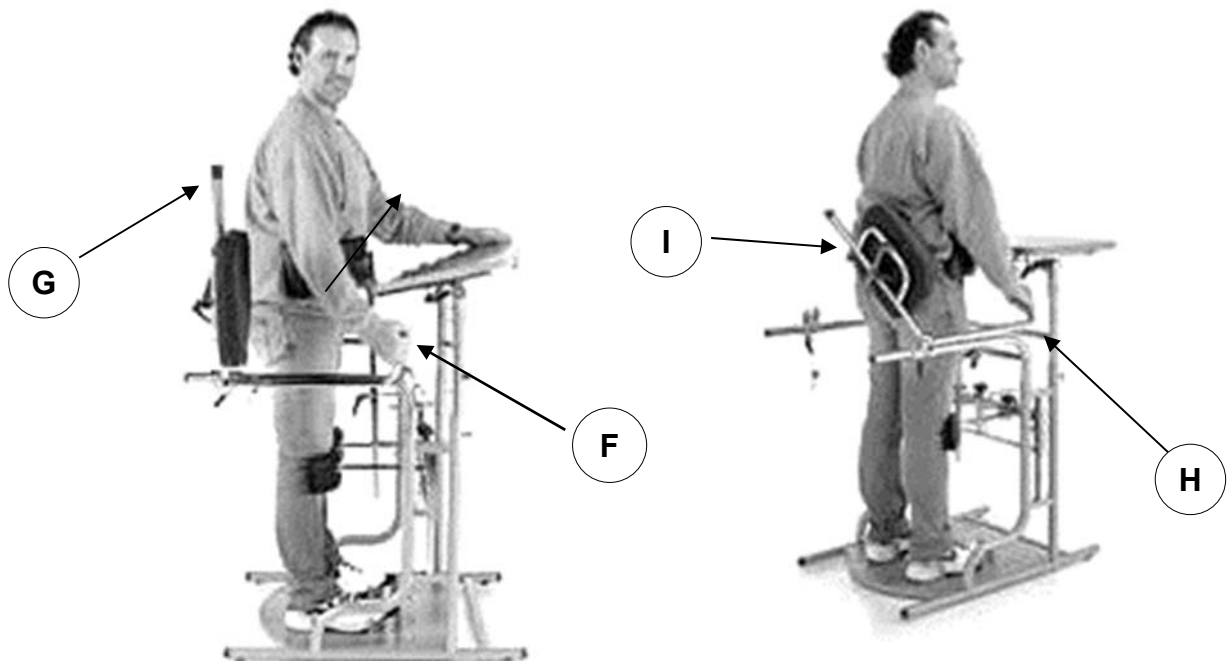
First be sure to brake the wheels of the device (if present in the version being used).

- f. raise the rear thrust to allow access to the patient
- g. bring the wheelchair close to the device until the front wheels are in contact with the lower platform of the standing, then apply the brakes to the wheelchair
- h. place your feet onto the platform of the standing with your heels pressed against the heel stoppers
- i. bring your knees in contact with the kneepads
- j. firmly grip the handlebars of the standing

Apply force to the handlebars and on your legs (to the extent possible depending on the condition of the patient) and bring yourself to an upright position or follow the instructions provided by medical personnel to raise the patient.

WARNING!!! Ascending and descending operations depend on the state of the patient's health. Depending on his or her mobility, the operation may take place with or without the assistance of a caregiver, and may also entail risks for the patient's safety. Consult medical personnel for the correct procedure.

F-G Grab the rotating handle to open/lock the rear thrust

H-I Turn the handle inwards to position the rear thrust

This is how the device is correctly used

DESCENDING MOTIONS

To get down from the device, proceed as follows:

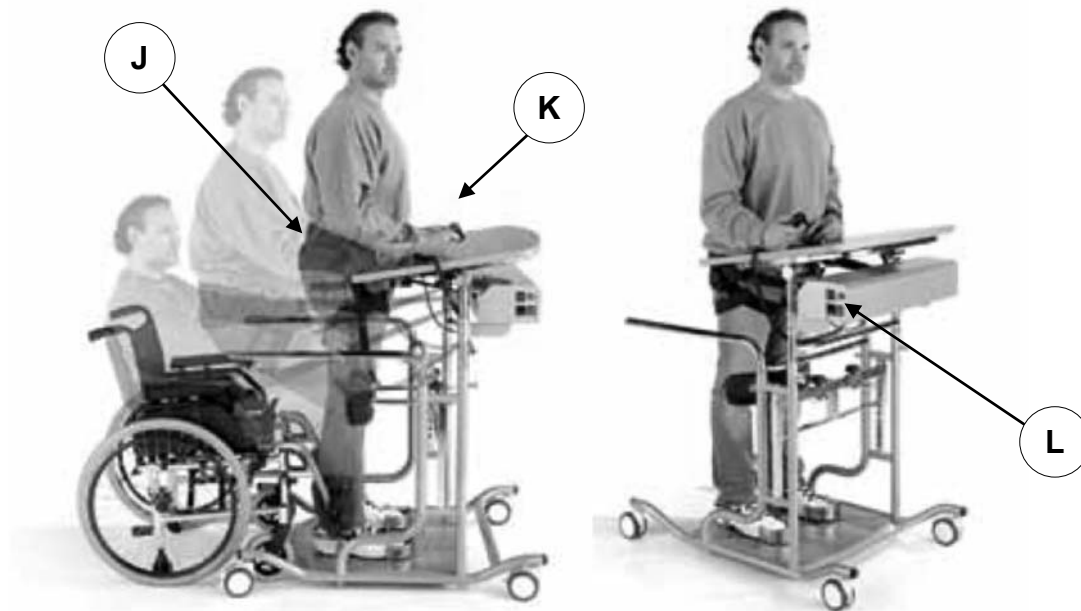
- apply the brakes to the wheelchair
- move the wheelchair until the front wheels are in contact with the platform of the device and then apply the brakes to the wheelchair
- turn the handle outwards, and open the rotating rear thrust (3)
- firmly grasp the handrails and crouch down until sitting in the wheelchair
- remove your feet from the heel stoppers and foot stoppers
- release the brakes on the wheelchair and proceed as you wish

WARNING!!! The methods for descending motions depend on the type of disability affecting the patient. For the correct methods, consult medical staff for assistance.

EASY UP N3200 ELEKTRO

ASCENDING MOTIONS

As with the instructions for the EASY UP N3100 model:

- i. be sure that the device's pivoting wheels have been locked
- j. fasten the hoisting sling around the patient as explained in the following paragraphs
- k. bring the wheelchair close to the device until the front wheels are in contact with the platform of the device, then apply the brakes to the wheelchair
- l. fasten the hoisting sling (j) around the patient
- m. place the patient's feet onto the platform of the standing with his/her heels pressed against the heel stoppers
- n. bring the knees in contact with the kneepads
- o. firmly grip the handlebars of the device
- p. operate the lifting mechanism with the remote control (k) or the control panel (l)

DESCENDING MOTIONS

WARNING!! Lowering from the standing is a very critical operation. Make sure a trained professional is available to help you in the event of difficulties.

To get down from the device, proceed as follows:

- be sure that the wheels are locked
- place the wheelchair behind the patient and apply the brakes
- lift and remove any back supports and/or accessories
- activate the engine with the wired push-button panel or the control panel on the engine compartment to release the belts that hold the hoisting sling and to move the patient backwards
- operate the engine until the patient is sitting in the wheelchair again
- remove the feet from the heel stoppers and foot stoppers
- unfasten and remove the hoisting sling
- release the brakes on the wheelchair and proceed as you wish

Application of the hoisting sling

8. Move the hoisting sling behind the patient's back and extend the thigh strap

9. Pass the thigh strap under the thigh

10. Pass it over the thigh and connect it with the first part

11. Fasten the ends of the thigh strap with the Velcro mechanism

12. Repeat this operation on both thighs

13. Now grab the lifting strap and hook it to the harness

14. Activate the engine and lift the patient

16. Transport

To transport the stair climber safely, you must use the packing initially provided, if possible.

The steps to do so are the same as those for unpacking, but in reverse.

For small movements of the device within a building, you can use the pivoting wheel (if present in the version), and move it by pushing on the handrails.

The versions which are not equipped with wheels can be moved by applying force to the handrails and moving the device with the use of the front wheels (1).

17. *Foreseeable improper use*

Users must refrain by behaviours which, even if they seem to be risk-free, are not explicitly recommended in this manual.

Foreseeable instances of improper use may include:

- d. Normal carelessness and not a deliberate intention to use the machine improperly

Correct education/training and the selection of suitable personnel is essential. Education and training must be provided by KSP Italia or by agencies/persons authorized by the manufacturer, with verification of what they learned and a certificate of qualification. The lack of the above documentation indicates unsuitability for use and relieves the manufacturer of liability for property damage and personal injury caused by that person.

The standing is not suitable for outdoor use, but rather only for use inside the home (home, nursing homes, hospital/healthcare facilities or the like).

- e. Following the “path of least resistance” to performing a task

The device is only to be used with people. Therefore, transporting objects is prohibited. The same goes for transporting persons who are holding objects.

- f. Foreseeable behaviour of certain categories (minors, the disabled, etc.)

In this case as well, the selection of personnel and their education/training plays an essential role. Never allow unauthorized personnel to manoeuvre the device. In the absence of authorized personnel, remove the power-on jack and keep it in a secure place. Only authorize its use by personnel who have excellent knowledge of written and spoken English and who are able to read the manual and understand its contents and their importance.

18. Instructions for emergency situations

Possible emergency situation	Action to be taken
Damage to the cable while in use	• Change the cable • If a cable is not available in the vicinity, request technical assistance
Generic breakdown during movement, followed by an inability to proceed	• Return the patient to his/her wheelchair • Contact technical support
The person being transported faints or is nauseated	• Stop immediately and implement the necessary measures • If necessary, return the patient to his/her wheelchair

**The person being transported must be calm and relaxed while in the device.
It is not suitable for agitated patients.**

19. Problems-Causes-Remedies

<i>Problem</i>	<i>Probable Cause</i>	<i>Possible Remedy</i>
- The device is not accepting ASCEND/DESCEND commands	- The power cord is not correctly inserted - The fuse has tripped	- Inspect and plug in the power cord correctly - Replace the fuse - Contact technical support
- Movements are intermittent	- Signal transfer contacts to the remote control are dirty or rusted	- Turn off the power, turn off the remote control and clean the contacts with a dry cloth lightly moistened with alcohol - Contact technical support
- The hoisting sling will not couple correctly	- Belt is out of order	- Contact tech. support
- During operation, unusual noises are heard	- Transmission problems - Foreign objects caught in the device	- Stop the device and remove the foreign objects - Contact technical support

IMPORTANT:

If, despite the checks and remedies, the device continues to have problems that limit its correct operation (even partially), the operator is required to contact an authorized service centre, or the manufacturer, immediately.

20. Maintenance

This chapter describes the only maintenance that the user is allowed to perform. All maintenance must be performed with the cable disconnected from the power grid and the switch set in the OFF position.

Cleaning

Painted parts must be cleaned with soft, damp cloth.

Chromed parts can be cleaned with a commercial spray. Avoid spraying directly on the machine. Do not spray water or other substances that could damage the surfaces.

Wheel cleaning (before each use)

Remove any pieces of metal and sticky residues that may be deposited on the surface.

Check the wheels for wear (before each use)

Periodically check the integrity of the wheels. In the event of damage or excessive wear, contact technical support.

Checking the safety belt/ hoisting sling (before each use)

Before each use, always check the condition of the seat belts and buckles.

Support systems check (prior to first use)

Before each use, always check the integrity of the table, their adjustments, and their couplings. Check that the levers and knobs are fastened correctly.

Safety systems check (before each use)

Before each use, check the functionality of the emergency button on the control panel, as well as the emergency button/end-of-stroke button under the table.

Visual structural check (weekly)

At least weekly, check that the structure of the standing is intact, that the steering column or metal parts are not bent, cracked or have signs of failure and that there is not excessive play in the moving parts.

Fuse changes (when needed)

The device is equipped with two fuses on both power lines. To change these devices, you must disconnect the power beforehand, remove the fuse box container located below the 220 AC outlet, then proceed with the replacement.

21. Periodic Maintenance

Keeping the device in good working order means having regular maintenance performed annually by an authorized service centre. For a list of service centres, contact the company's representative or the company directly, at:

KSP Italia

Zona Industriale, Via dell'Artigianato, 1
06031 - Bevagna (PG) - Italy
Tel. +39 0742 361947 - Fax +39 0742 361946
ksp@kspitalia.com - www.kspitalia.com

Periodic maintenance includes the following actions:

- check of the electrical system
- check of the emergency and end-of-stroke buttons
- check of the motor
- check of the wheel drive system
- check for wheel wear safety
- check of the various parts
- check of the integrity of the frame and accessories
- check of the lifting system
- check of the mechanical transmission
- check of the seat belts/ hoisting sling

22. Disposal

Proper separate collection for the subsequent environmentally compatible recycling, treatment or disposal of abandoned equipment contributes to mitigating possible negative effects on the environment and health and promotes the recycling of the materials that comprise the equipment.

The barred dumpster symbol shown on the equipment indicates that the product must be collected separately from other waste at the end of its useful life.

The EASY UP Elektro N3200 is an electrically powered medical device and must be disposed of as non-professional domestic WEEE. Non-electrical devices, such as the Standing Easy Up N3100, are not subject to the regulation of electrical products and must be disposed according to the national and regional laws in force at the time of disposal (or according to the laws in force in the country in which the Device is located).

Illegal disposal of the product entails the application of administrative sanctions pursuant to current legislation.

ATTENTION! This part relating to disposal is valid and therefore applies only to users of the products who are located in Italy (not abroad or the European Union).

Batteries must be disposed of separately in accordance with the legislation on batteries and accumulators.

The operations to be carried out for disposal are shown below:

- Remove the battery pack from the device
- Remove the batteries from the battery pack
- Dispose of batteries according to current legislation.

In countries other than Italy, the disposal must take place according to the local regulation and transposition of the waste electrical and electronic equipment (WEEE) Directive or related regulation.

23. Warranty

The warranty is intended for the product covered in this manual and applies to any lack of conformity as defined by Legislative Decree of 02 February 2002, No. 24.

It shall be valid for **24 months (12 months** for Professional or Extensive Use) starting from the date of delivery to the end user reported in the tax documentation. Said documentation must be presented when requesting interventions and proof of purchase must be provided.

The purchaser or owner of the sales contract has **two months** from the time of discovery to report the conformity defect; the report must be made in writing in a registered letter with return receipt or a telegram to KSP Italia or the authorized reseller.

Any repair work must be carried out by the authorized dealer's technical assistance service; Intervention on the product by unauthorized personnel will result in the immediate termination of the warranty.

Warranty means the free replacement or repair, within established deadlines, of the component parts of the appliance which, at the sole discretion of the Manufacturer, are found to be defective due to manufacturing defects.

The warranty does not cover damage caused by incorrect maintenance and/or failure to comply with the rules of use and the warnings/instructions contained in this Manual, as well as transport failures (scratches, dents, etc.), damage from falls, damage caused by neglect, tampering, inability to use the product and repairs carried out by unauthorized personnel; damage connected to incorrect installation of the product if this is carried out directly by the User or unauthorized personnel, from dirt, stains, liquids and abrasions of the covering, from power surges and blackouts, or alterations resulting from environmental and climatic conditions and all damage caused by phenomena not dependent on normal operation. In any case, the warranty does not cover defects due to normal wear and tear, negligence and careless use, improper and/or incorrect use, as specified in this Manual.

A component replaced under warranty does not in any case extend the overall warranty of the product including the replaced part.

24. Assistance, Home service

KSP ITALIA provides a home Technical Assistance Service which can be activated by contacting the authorized dealer or KSP Italia directly.

ATTENTION! This part relating to assistance is valid and applicable only for users of the product located in Italy (not abroad or the European Union).

25. Reports – Alert

Any accident, malfunction and/or dysfunction, alteration of the characteristics and performance of the device that has caused or is potentially capable of causing harmful events for the Operator/User, must be promptly reported by contacting the retailer at the references indicated in the purchase title, or directly the Manufacturer, in order to adopt all suitable intervention measures, including device "recall" campaigns.

The Operator, Purchaser or Patient must report any serious incident occurring in relation to the Device to KSP Italia and to the competent authority of the member state in which the user is established.

26. Notes