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Modelli

**K05/06-2, K05/06-2R, K10/20-2, K10/20-2R, K05/K30-2IR,
K30-2R, K55/56-2R, K59-3R, K65-3, K67-4R, K69-5R,
K91/S-2R, K91PA/PAB-2R, K901/902/905/915-2R**

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Instructions and maintenance manual

Rev.14 on 29th April 2025

Class I Medical Device pursuant to Annex VIII Regulation EU 2017/745

The information in this manual is the exclusive property of KSP Italia Srl and may not be reproduced, in whole or in part, or used for any purpose other than that for which it was provided, without written authorisation from KSP Italia Srl.

1. Declaration of conformity	64
2. The Manual	66
3. Intended use	69
4. Safety requirements	70
5. Transport, handling and unpacking	75
6. Assembly	81
7. Technical specifications	84
8. Using the device	108
9. Charging the batteries	109
10. Troubleshooting	116
11. Maintenance	117
12. Disposal	120
13. Warranty	120
14. Support – Home service	121
15. Reports – Alert	121

1. Declaration of conformity

Declaration of Conformity

(in accordance with ISO/IEC 17050-1)

2) No. 113/25 - Date 11/02/2025

2) Issuer's name: KSP ITALIA SRL
Issuer's address: VIA DELL'ARTIGIANATO 1, 06031 BEVAGNA (PG), ITALY Tel. 0742. 36.19.47
Fax 0742.36.19.46 www.kspitalia.com, e-mail: ksp@kspitalia.com

N° EUDAMED SRN: IT-MF-000009316

3) Object of the declaration: **LIFT/RELAX MOTORIZED ARMCHAIR, models K10/2 - K20/2 - K05/2 - K06/2 - K65/3.**

4) The Manufacturer KSP Italia declares under his sole responsibility that the medical device above described complies with all applicable requirements of the following legislation and fulfils all applicable provisions thereof (and any other relevant UE legislation providing for the issuance of EU declaration of conformity):

Documents No.	Title	Edition/Date of issue
5) Regulation (EU) 2017/745	Medical Devices Regulation	Emission: 5 April 2017
Regulation 2011/65/EU	Restriction of the use of certain hazardous substances in electrical and electronic equipment	Emission: 8 June 2011

Additional information:

6) Medical devices designed and manufactured with quality management system compliant to ISO 13485.
CE Marked Medical device in accordance with Annex II and III, Regulation (UE) 2017/745.
Class I medical device as for rule 1 and rule 13, Regulation (UE) 2017/745, Annex VIII.
Registered at the Italian Ministry of Health with respective number:
K10-2: 2269227; K20-2: 2269228; K05-2: 2269229; K06-2: 2269231; K65-3: 2269225.

BASIC UDI-DI (GMN): 805577318POLTRONE-0000W5

Signed for and on behalf of:

KSP Italia Srl

Bevagna, 11/02/2025

7) Claudio Emanuelli,
Legal Representative

Declaration of Conformity

(in accordance with ISO/IEC 17050-1)

1) No. 125/25 - Date 11/04/2025

2) Issuer's name: KSP ITALIA SRL
Issuer's address: VIA DELL'ARTIGIANATO 1, 06031 BEVAGNA (PG), ITALY Tel. 0742. 36.19.47
Fax 0742.36.19.46 www.kspitalia.com, e-mail: ksp@kspitalia.com

N° EUDAMED SRN: IT-MF-000009316

3) Object of the declaration: **LIFT/RELAX MOTORIZED ARMCHAIR WITH ROLLER, models K10/2R - - K20/2R - K30/2R - K30/2IR - K05/2R - K05/2IR - K06/2R - K55/2R - K56/2R - K59/3R - K91/2R - K900/2R - K901/2R - K902/2R - K91S/2R - K91PA/2R - K91PA/2R - K905/2R - K91PAB/2R - K915/2R - K67/4R - K69/5R**

4) The Manufacturer KSP Italia declares under his sole responsibility that the medical device above described complies with all applicable requirements of the following legislation and fulfils all applicable provisions thereof (and any other relevant UE legislation providing for the issuance of EU declaration of conformity):

Documents No.	Title	Edition/Date of issue
5) Regulation (EU) 2017/745	Medical Devices Regulation	Emission: 5 April 2017

Regulation 2011/65/EU	Restriction of the use of certain hazardous substances in electrical and electronic equipment	Emission: 8 June 2011
-----------------------	---	-----------------------

Additional information:

6) Medical devices designed and manufactured with quality management system compliant to ISO 13485.
CE Marked Medical device in accordance with Annex II and III, Regulation (UE) 2017/745.
Class I medical device as for rule 1 and rule 13, Regulation (UE) 2017/745, Annex VIII.
Registered at the Italian Ministry of Health with respective number:

CODICE KSP	N° REPERTORIO	CODICE KSP	N° REPERTORIO	CODICE KSP	N° REPERTORIO
K10-2R	2269306	K20-2R	2269308	K30-2R	2269312
K30-2IR	2269336	K05-2R	2269344	K06-2R	2269345
K55-2R	2269349	K56-2R	2269350	K59-3R	2269303
K91-2R	2269346	K900-2R	2273404	K901-2R	2269301
K91S-2R	2391845	K91PA-2R	2391840	K905-2R	2316832
K902-2R 07AR: 2741426 K902-2R 07BE: 2741429 K902-2R 07BL: 2741430 K902-2R 07GI: 2741431 K902-2R 07RO: 2741433 K902-2R 07tO: 2741434 K902-2R 07VE: 2741437		K91PAB-2R 04BL: 2682160 K91PAB-2R 04RO: 2682163 K91PAB-2R 04TO: 2682164 K91PAB-2R 04MA: 2682161 K91PAB-2R 04PA: 2682162 K91PAB-2R 04AR: 2682159		K915-2R 04BL: 2682154 K915-2R 04RO: 2682157 K915-2R 04TO: 2682158 K915-2R 04MA: 2682155 K915-2R 04PA: 2682156 K915-2R 04AR: 2682153	

BASIC UDI-DI (GMN): 805577318POLTRONE-ROLLEC

Signed for and on behalf of:

KSP Italia Srl
Bevagna, 11/04/2025

7) Claudio Emanuelli,
Legal Representative

2. The Manual

This use and maintenance manual, together with the instructions provided on the label, incorporates the information provided by the Manufacturer in conformity with Regulation EU 2017/745.

Each device must be accompanied by the information necessary to guarantee its safe use, bearing in mind the training and knowledge of the potential Users.

The manual is an integral part of the device. It must therefore be stored with extreme care and always included, also in the event of transfer of the product to third parties.

It is written for operators/users, the owner, Users and maintenance personnel.

The manual provides indications on technical specifications, correct use of the device, methods of transport, preservation, maintenance and disposal, and on the related safety precautions.

Any changes to the instructions provided by the Manufacturer that may be relevant for the safety of the patient and/or operator/user will be promptly communicated to the owners/Users of the product, through all channels useful for these purposes.

The Manufacturer is not required to provide notification of any other changes and/or additions.

If this manual or the labels and/or markings on the product are found to be damaged, even partially, faded or illegible in parts or entirely, you must immediately request another copy from your retailer or the Manufacturer.

This manual refers to the devices listed on the title page and in the “Declaration of Conformity” section. Unless otherwise indicated, the instructions and precautions refer to all models. Particular information is given in reference to the model concerned.

The images in the manual may differ slightly from the product in your possession (e.g., colour, shape, etc.), without affecting the validity of the operating and safety information in this document.

2.1 Definitions

The meanings of terms used in this manual are given below:

Medical Device: A device intended by the Manufacturer for use on humans for the purpose of treating, diagnosing or mitigating a disease.

End User: The person who uses the medical device; in this manual it is the person/patient who uses the Armchairs.

Operator: A person in charge of operating the Armchair if different from the End User (it can coincide with the End User if he/she is also capable of operating the Armchair).

2.2 Graphic Symbols

Graphic symbols are used in this manual and on the device and their meanings are given in the following table.

Symbol	Meaning	Notes
	Mark of conformity to European standards	On the device
	Electrical material disposal symbol	On the device
	Date of manufacture	On the device
	Manufacturer	On the device
	Model	On the device
	Serial number	On the device
	Instructions for use	On the device
	Keep dry	On the device
	Indoor use	On the device
	Type B applied parts	On the device
	Maximum weight of end user	On the device
	General warning sign	Pay attention where this sign is used
	Caution: hazardous voltage	Pay attention where this sign is used

	Caution: limb crushing hazard	Pay attention where this sign is used
	Caution: Moving mechanical parts	Pay attention where this sign is used
	General prohibition sign	Prohibition sign
	Do not use shoes with heels	Prohibition sign
	Keep away from sources of heat.	Prohibition sign
	Generic mandatory behaviour sign	Mandatory behaviour
	See the instruction booklet	Mandatory behaviour
	On/off	Other signs
	Alternating current	Other signs
	Fuse	Other signs
	Dangerous voltage	Other signs
	Use in indoor environments	On the device
	Wash by hand	Fabric maintenance
	Dry flat	Fabric maintenance
	Do not iron	Fabric maintenance

3. Intended use

This manual has been written taking into account the characteristics, knowledge, educational level and training of the operator/end user.

The operator controlling the chair should:

- Have full knowledge of the product;
- Be over 18 years of age;
- Be in good mental and physical health (by way of indication, not under the influence of alcohol or drugs, not suffering from mental illness, such as conditions involving hallucinations, loss of balance and the like; in case of doubt, contact the Manufacturer);
- Be capable of assessing hazardous situations and acting calmly and thoughtfully;
- Have excellent knowledge of spoken and written Italian in order to understand the manual and learn its content;
- Know the meanings of symbols and markings;
- Not have difficulty standing upright or staying in balance.

CHECK THE SUITABILITY OF THE ENVIRONMENT BEFORE EACH USE!

The Operator/end user is always fully responsible for the conformity/compatibility of the premises where the product is used.

THE ARMCHAIR IS NOT SUITABLE FOR OUTDOOR ENVIRONMENTS BUT ONLY DOMESTIC AND INDOOR USE.

The device is a disability aid that allows people with reduced mobility to sit down or get up easily thanks to a lift mechanism.

It is a Class I medical device as defined by Annex VIII of Regulation EU 2017/745, constructed in conformity with Italian and international law on medical devices and disability aids and duly listed in the Medical Device Registry of the Italian Ministry of Health.

The Armchair ensures the complete safety and comfort of the user, minimising the efforts of care workers helping with sitting down and getting up.

The Armchair is suitable for people with motor difficulties, in particular with difficulty sitting down or, once seated, getting up, due, for example, to hip pain. The Armchair, with its base-back-footrest movements, facilitates these actions.

It consists of a metal and wooden structure operated by one or more DC motors powered by a power supply. The movement is provided by means of a system of levers in the structure, which, depending on the model, operate the sections through drive motors controlled by the push-button panel.

The Armchair is exclusively designed for supporting persons, as indicated in this document.

It must be used within the limits and according to the methods explicitly described by the Manufacturer in this Manual. The Manufacturer is therefore considered relieved of all liability for damages arising from improper use of the product by uninstructed persons, as well as for any unauthorised modifications or repairs, including the use of spare parts other than those acquired directly from KSP (or approved by it), exceptional events and total or partial failure to observe the instructions given in this manual.

4. Safety requirements

4.1 General warnings

Use this device in a manner consistent with its intended use and in accordance with the instructions and modes of use indicated. KSP Italia S.r.l. assumes no liability for damage to property or persons caused by incorrect and/or improper use of the device or by use for a purpose other than that specified by the Manufacturer.

In order to use the Armchair, the operator/user must be familiar with this Manual and the instructions for use, and aware of the risks related to “improper” and incorrect use.

Therefore, use of the device must be considered prohibited to persons with insufficient knowledge of its modes/features of use. In the event of any doubts and/or uncertainties regarding its operation and use, the operator/user must contact the authorised retailer and/or the Manufacturer directly for any clarifications and/or explanations, or, where necessary and expressly requested, specific support in the terms and modes specified in the purchase documents.

BEFORE USING THE DEVICE, CAREFULLY READ THIS MANUAL AND THE INSTRUCTIONS

The following actions should not be performed as they can compromise the conformity and/or features of the device:

- Incorrect installation
- Improper use
- Use of third-party parts and/or accessories not approved by the Manufacturer
- Servicing or tampering by unauthorised personnel
- Insufficient or improper maintenance

The following instructions and precautions must be observed during installation/assembly, use and maintenance of the device in order to ensure the fulfilment of the requirements for the safety of the operator and the end user, and the proper functioning of the product.

- o To prevent improper use by unauthorised persons, when the device is not in use, unplug the cable from the power outlet and the armchair and then remove the power supply, where present, or place the cable in the side pocket;
- o Any alteration, replacement or work on the device not carried out by personnel authorised by KSP Italia S.r.l. invalidates the warranty and, in any case, exempts the Manufacturer from all liability for any direct and/or indirect damage that may result to people or property;
- o Use a suitable power cable supplied by KSP Italia. Use only the specified power cable and/or power supply certified for the country of use. Periodically check the integrity of the cable and/or battery charger. Insert the power supply plug fully into the socket on the back of the device;
- o Use a supply voltage of 220-230Vac 50/60Hz;
- o To prevent hazards to people or property, comply with all the rated data and labels on the product. Consult the manual before connecting the device;
- o Avoid exposed circuits. Do not touch exposed connections or components connected to the power supply;
- o Do not operate if a fault is suspected or the casing is damaged;
- o If you suspect that the device may be faulty and/or damaged, do not use it but have it checked by specialised personnel authorised by the Manufacturer;
- o Cleaning and maintenance must only be carried out after disconnecting the plug from the mains power supply;
- o Avoid contact with or seepage into the device of liquids or powders;
- o Do not operate in potentially explosive atmospheres and/or near flammable mixtures;
- o Avoid exposure to excessive heat. The temperature of use should be between +5°C and +40°C;
- o Do not use the Armchair outdoors. It has been designed and built for use in enclosed environments sheltered from atmospheric agents;
- o Use the device only with original parts supplied by the Manufacturer;
- o Make sure that the specifications of the mains electrical system conform to the power requirements of the device indicated on the label and in this manual;
- o Do not use other devices on the User while seated in the Armchair;
- o Do not use the device in environments with high electromagnetic fields as this could cause malfunctions;
- o Carry out the maintenance on the device as specified by the Manufacturer;
- o Portable and mobile radio communication equipment may affect the operation of the device;
- o Caution – using controls and adjustments or performing procedures other than those specified herein can cause risks for the User and the operator.
- o The product is designed to seat one person at a time.
- o Do not allow children to use or play on the appliance.

- o Do not use the remote control and/or unplug/plug in the power cable with wet hands. Do not sit or slide with your weight on the backrest or the footrest. Overturning hazard.
- o Do not place the Armchair on or near curtains or similar furnishings. These can end up between the moving parts, be dragged by the movement and lead to dangerous situations.

Electrical precautions

- o Insert the plug fully into the electrical socket. Risk of short circuit or fire.
- o When the device is not being used, make sure that the plug is removed and that the push-button panel is properly stored in the provided pocket, if present. Risk of fire, electric shock, short circuit and cable accidents with children.
- o When removing the plug from the socket, never pull on the cable but hold the plug directly near the socket. Risk of short circuit, electric shock or fires caused by cable damage.
- o In the event of a power outage (e.g., blackout or activation of circuit breakers in the building's electrical system), turn off the device and disconnect the plug from the power outlet. Risk of accidents.
- o Do not use if the electrical cable, plug or socket are damaged. Risk of electrocution, short circuit or fire.
- o Do not damage, break, alter, bend, pull, twist or wind the electrical cable during use. Risk of electrocution, short circuit or fire.
- o Do not place or stack heavy objects on it. Risk of electrocution, short circuit or fire.
- o Do not damage the plug with pins, water or dirt of any kind. Risk of fire, electrocution or short circuit.
- o Do not place the power cable near heat sources or on hot surfaces. Risk of fire, faults or damage to people.
- o Do not use while wearing wet clothing. Risk of injury or electrocution.
- o Do not use the Armchair when it is wet. Risk of injury or electrocution.
- o Do not use with a voltage other than that indicated on the label and do not connect the plug to a power strip. Risk of fire or short circuit.
- o Do not use at voltages and frequencies other than those indicated in this manual.
- o During house cleaning involving the use of water (e.g., steam cleaning, washing floors, etc.), disconnect the appliance from the mains power supply and ensure that it does not come into contact with liquids.

Precautions when using the product

- o When using the reclining backrest or the footrest, make sure that there are no objects and/or people in the vicinity that might be harmed. Risk of injury.
- o Avoid sitting down forcefully on the mechanism. Risk of injury.
- o Use of the Armchair by children or disabled people without supervision is prohibited. Risk of injury.

Other general precautions

- o Do not use components other than those included in the packaging upon delivery.
- o Use only in a domestic environment.
- o Do not expose to temperatures, humidity or uses other than those indicated in this manual.
- o Do not assemble in a different way than that shown in this manual.
- o Do not use materials that are dangerous to the health of the end user.
- o Assemble the product with care not to create risks due to cutting of electrical cables and/or mechanical shearing.
- o Do not sit, climb or jump on the backrest, armrests or footrest. Risk of injury or damage to the product.
- o Do not put hands or feet under the Armchair or in the spaces between the backrest and the seat or the armrest. Risk of injury.
- o Do not place the Armchair close to heaters or other sources of heat. Risk of fire or damage.
- o In the event of failure or malfunction of the device, immediately discontinue its use. Turn off the device, unplug it and contact the authorised retailer or Manufacturer for repair. Risk of electric shock, short circuit or fire.
- o Do not use the device if the leather or fabric covers are torn and the internal mechanisms are protruding. Risk of injury. Contact your authorised retailer or the Manufacturer for repair.

Interference with other devices

The device does not interfere with other devices, except in terms of space.

Electromagnetic immunity and emissions are within legal limits and such as not to cause interference or create dangerous situations if used as intended.

The Manufacturer assumes no liability for damage, accidents or injuries caused by failure to comply with the requirements and indications/safety guidelines provided in this Manual.

Therefore, KSP Italia will not accept any liability for damages resulting from improper and/or abusive use of the product, nor will it be liable for any damages that result from wear and tear, negligence, neglect, handling, incorrect/irregular installation and/or connection of the products, or from improper and/or different use, whether by the operator/end user or by any third party not authorised to use it.

4.2 Contraindications and possible side effects

The Armchair may only be used by persons familiar with the device and its use. It is essential that the operator knows how to carefully assess the following warnings/contraindications.

The Armchair is not suitable for seating people who are "agitated" or suffering from diseases or on drugs that can cause sudden movements of those affected, such as Alzheimer's, or Users with mental disorders that cause "agitation" and similar effects. In case of doubt, contact the Manufacturer to verify the suitability of the device for the resident User.

- o The device must be operated only by personnel trained and instructed in treatment techniques.

- o Do not use the device on Users suffering from serious cardiological or neurological disorders.
- o The use of this electromedical device for minors under 18 years of age is prohibited.
- o Never use the Armchair on surfaces that are wet, smooth or slippery, waxed, icy or similar and cannot guarantee its stability or the grip of the base on the floor.
- o Carpets and mats can pose a hazard.
- o Make sure that no one leans on the footrest or backrest sections when the User is on the Armchair. Overturning hazard.
- o Ensure that the User is seated properly and is not imbalanced with his/her weight on the footrest or backrest sections.
- o Never insert hands into the moving mechanism when the Armchair is switched on.
- o The expected lifetime of the device is 8 years.

4.3 Improper use


Operators and Users must refrain from behaving in ways that, even if apparently harmless, are not explicitly recommended in this manual.

Foreseeable improper uses derive from:

- e) Normal carelessness and thoughtless intention to use the machine incorrectly.
Use of the Armchair outdoors; only domestic and internal use of the device is permitted. Use of the Armchair in wet environments or while rooms are being sanitised.
- f) Instinctive reactions during use in the event of malfunctions, accidents and failures.
In the event of malfunctions/accidents/failures, never abandon the Armchair with the user seated in it. Request assistance from other staff and do not leave the User alone unless unavoidable.
- g) Behaviour deriving from the “path of least resistance” when performing a task. The Armchair is only designed for seating people. Therefore, supporting objects or animals is prohibited. The same applies to supporting people who are holding objects, goods or animals.
- h) Foreseeable behaviour of certain categories (children, the disabled, etc.). Here too, the selection of personnel and their education/training plays an essential role. Never allow unauthorised personnel to operate the device and when authorised personnel are not present, remove the electrical plug. Only personnel with good knowledge of the product and of the use and maintenance manual are authorised to use it.

4.4 Residual risks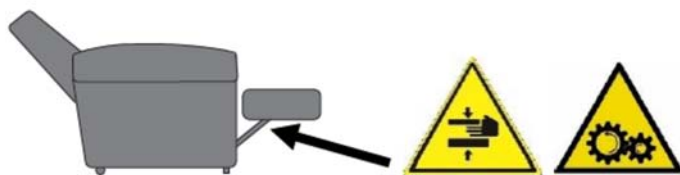

Hazard: hand crushing

Source: moving parts

Hazardous situation: moving parts by hand in the base of the Armchair where the mechanisms are located.

Likelihood: extremely low.

Harm: possible harm with recovery in less than 30 days.

Remedy: a label indicating the need to read the manual; instructions and warnings on residual risks in the manual.

Total risk: extremely low, acceptable.

5. Transport, handling and unpacking

5.1 Packing and Transport

The device is contained in cardboard packaging that protects it from normal transportation vibrations. The package must be transported in the vertical position and avoiding shocks and jerks. Check the integrity of the packaging upon arrival. If the packaging is damaged, immediately notify the carrier who delivered it.

Before operating an Armchair from packaging that showed signs of damage, the product must be checked by the technical support department or by personnel authorised by KSP Italia.

- If, on arrival, the device is found to have damage caused by transport, do not use it but contact the technical support service to have it checked and repaired.
- It is advisable for the User to use the device during training with no one seated on it, in order to acquire maximum familiarity of use.
- The Armchair must be used by a person in good physical condition, capable of controlling the movements of the device and performing the necessary operations and movements correctly and safely.
- The User must be fully acquainted with the contents of the manual and the symbols.
- It is essential for the User to check his/her ability to perform the operations with preliminary tests.

- Daily use of the Armchair must only commence after ensuring that operating and safety parts are in proper working order, as indicated in the paragraph on maintenance and preliminary checks.

5.2 Package contents

The Armchair is delivered pre-assembled in the packaging and ready for use upon connection to the power supply.

The package contains:

- Armchair
- Button control panel (already connected)
- Power cable (already connected or included power supply)
- Use and maintenance manual

To protect the operator from hazards deriving from the manual handling of loads, see the paragraph on technical specifications, which shows the weights of the individual parts and the entire product.

The person assigned to unpacking must be trained in the risks of such operations.

Remember that to lift the load correctly without risking harm to your spinal column, you must:

- Keep your back straight
- Keep your trunk upright
- Keep the weight lifted as close to your body as possible

As at the time of drafting of this version, the law provides that when handling loads weighing over 25 kg (15 kg for female operators), help should be sought from other people or special lifting and transport equipment, such as trolleys or stairlifts, should be used.

Do not store the products above shoulder height.

Take the package to a suitable area for unpacking and assembly of the parts of Armchair.

Open the package without using sharp or pointed objects that could damage the contents.

It is a good idea to keep the original packaging materials for future reshipment of the device (e.g., to Technical support) as the original packaging will ensure safe transport.

5.3 Movement

Only move the Armchair using trolleys and suitable equipment. Risk of injury, falls and damage to the musculoskeletal system.

Movements in a domestic environment

When moving the Armchair, wrap the cable and take care not to step on it. In addition, be careful of impacts that could damage the Armchair.

Be careful when moving the Armchair on delicate floors. The wheels (if present) or the support feet may cause damage. Do not move the Armchair with objects or people on it, so as not to cause risk of falls and injuries. This could also damage the floor due to the weight.

To position the Armchair, make sure that the floor is sufficiently level and that it can support the weight.

In unlevel places, there may be risk of falls and injuries in addition to damage to the floor.

ATTENTION!!!

Before moving the Armchair, disconnect the power supply from the wall outlet and from the socket on the Armchair (where an external power supply is present), wind it and place it on the seat. Wind the push-button panel cable and place it in the side pocket.

Positioning

Place it in a position not directly exposed to sunlight to prevent the covering from fading. Otherwise, protect it from exposure using curtains on the windows.

Place it away from heat sources in order to avoid changes to the covering and to prevent it from catching fire.

ATTENTION!!!

Do not use in environments with high humidity, such as bathrooms, or where there is a risk of the Armchair getting wet.

MOVING THE ARMCHAIR INSIDE DOMESTIC ENVIRONMENTS

CAUTION! The path must be flat, with no slopes or steps, free from obstacles, dry, not slippery and without carpets or mats. Risk of bruising or accidents, and possible damage to the product. In addition, before taking the Armchair through corridors or doors, make sure that there is at least 20 cm clearance on either side. KSP is not liable for damage to persons or property due to incorrect handling, nor acknowledges repairs or replacement parts under the warranty as a result of such operations.

CAUTION! Movement must be carried out with no person or objects on the Armchair and using the utmost caution. It must always be done with the Armchair in the "sitting" position, as in the figure below. Risk of bruising or accidents, and possible damage to the product. KSP is not liable for damage to persons or property due to incorrect handling, nor acknowledges repairs or replacement parts under the warranty as a result of such operations.

"Sitting" position for moving the Armchair on its roller system or swivel wheels.

CAUTION! Before sitting on the Armchair, always check that the wheels are locked, or that the roller system is disengaged, so that the armchair rests on its feet and not on the wheels! Falling and bruising hazard!

WARNING! DO NOT USE THE ARMCHAIR WITH THE ROLLER INSERTED; ALWAYS CHECK THAT IT IS DISENGAGED BEFORE EACH USE.

Moving the Armchair without a roller system or swivel wheels

The Armchair can be moved without a roller system by:

- pushing, for adjustment movements of up to a maximum of 5 cm. In this case, always push at the base, in order not to apply the pushing force to the arms, footrest or back;
- for movements of more than 5 cm, use special equipment, such as transport trolleys for level paths or stairlifts. These operations should always be performed by trained personnel.

CAUTION! Do not lift the Armchair manually. Risk of bruising or accidents, and possible damage to the product. KSP is not liable for damage to persons or property due to incorrect handling, nor acknowledges repairs or replacement parts under the warranty as a result of such operations.

Moving the Armchair on fixed wheels

The Armchairs equipped with fixed swivel wheels can be moved on a flat surface after disengaging the brake on the front wheels. For this purpose, raise the brake as in figure (a).

At the end of the movement, re-engage the brake on both wheels by pushing the tab downwards, as shown in figure (b),

Moving the Armchair with a roller system

If the Armchair is equipped with a roller mechanism, proceed as follows.

16- Go to the rear of the Armchair

17- b. Lift the fabric cover or open the zip, depending on the model

a.

b.


c.

18- Press the lifting pedal down with your foot to its lowest position, as shown in figure (d). The pedal automatically remains in place (figure e).

19- Then push the Armchair to the desired location.

20- Once you reach the desired place, disengage the roller with the foot pedal by pushing it upwards, as shown in figures (f) and (g).

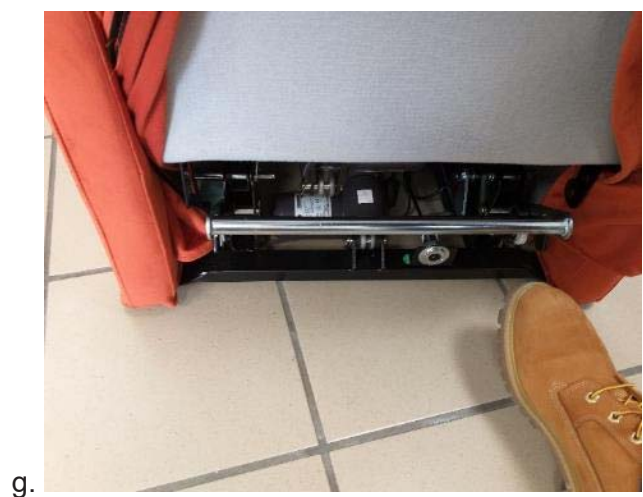Cover the motorised area with the fabric.

i.

j.

6- Assembly

6.1 Assembly

Arrange the contents of the package on a flat surface and check that all the parts are present and in good condition.


Make sure that the Armchair has no damage or dents caused by transport. If in doubt, contact your retailer or the Manufacturer

Connecting to the power supply

The Armchair is powered by a 220V AC domestic mains outlet; the max. power consumption is 2A.

Some Armchairs come with an external power supply, which, before being plugged into the mains supply must be connected to the armchair as follows.

a, b) Connect the cable (1) to the transformer (2) as in figure (b).

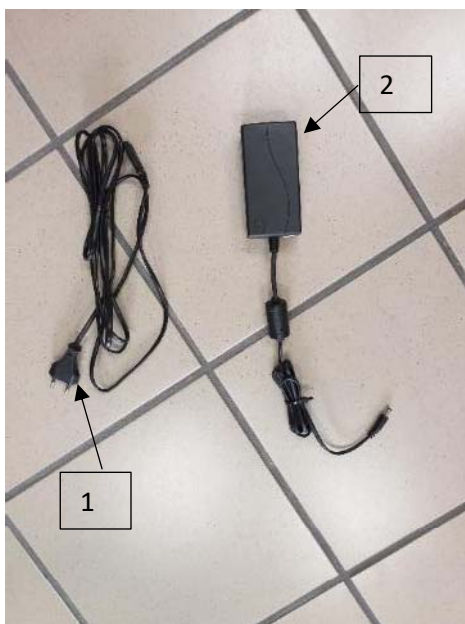
a.


b.

c, d) Raise the rear cover of the Armchair to access the power socket (3) on the edge of the Armchair.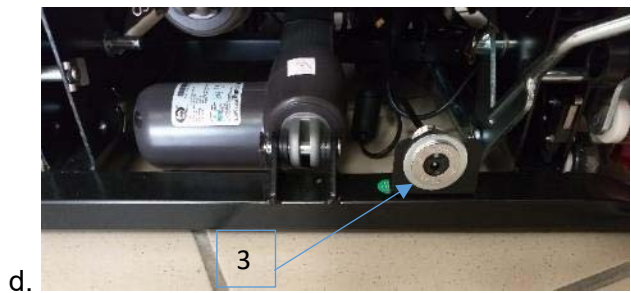

e, f) Grasp the Armchair connection plug (4) and insert it firmly into the socket in the Armchair (3).

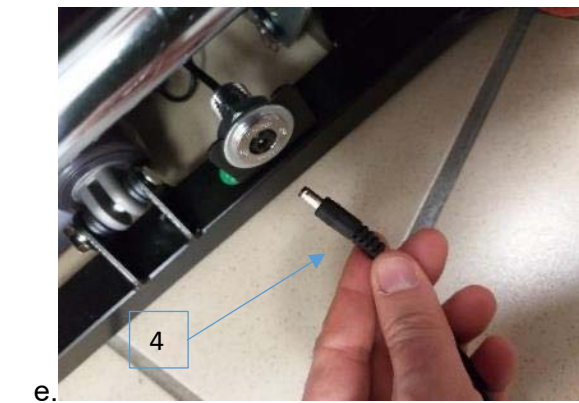g) Then connect the mains plug to the domestic wall outlet.

CAUTION! Make sure the building's electrical system conforms to the electrical safety regulations and is properly maintained. Do not use power strips. Make sure that the circuit breakers in the domestic electrical system (magnetothermal and differential) are working properly. The manufacturer is exempt from any liability for damages or accidents resulting from non-conformity of the electrical system in the place of installation.

The power supply has a green power LED (5). When the power is connected, this LED lights up. When there is no power, the LED is off.

Installation checks

After installation, check the following:

- With no one seated on the armchair, operate all the parts and make sure that the movements are performed properly and that there are no mechanical noises.

If you notice a smell of burning or electrical equipment, unplug it, and contact the Manufacturer. Do not use the product until you have received instructions from KSP Italia.

- Check that the green LED lights up when the power supply is connected.
- Repeat the movements and check their functioning with an able-bodied person seated in the Armchair.
- Repeat the operations with the intended user (end user) and check the suitability of the device for their health condition.

If in doubt, contact the Manufacturer.

7- Technical specifications

7.1 Models and components

This manual applies to the following models:


Model K05-2, K05-2R, K05-2IR

Two-motor Armchair with independent footrest and backrest movement.

- Control for moving to the backrest and footrest reclined position
- Control for moving to lifting position with base tilted forward
- Adjustment of intermediate settings between the two previous positions
- Independent backrest movement
- Weight capacity: 130 kg

Dimensions in cm: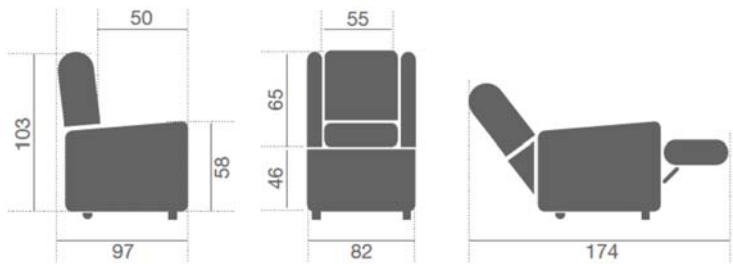

Movements;

Cod. K06-2							
Cod. K06-2R							


Model K06-2, K06-2R

Two-motor Armchair with independent footrest and backrest movement.

- Control for moving to the backrest and footrest reclined position
- Control for moving to lifting position with base tilted forward
- Adjustment of intermediate settings between the two previous positions
- Independent backrest movement
- Weight capacity: 130 kg

Dimensions in cm: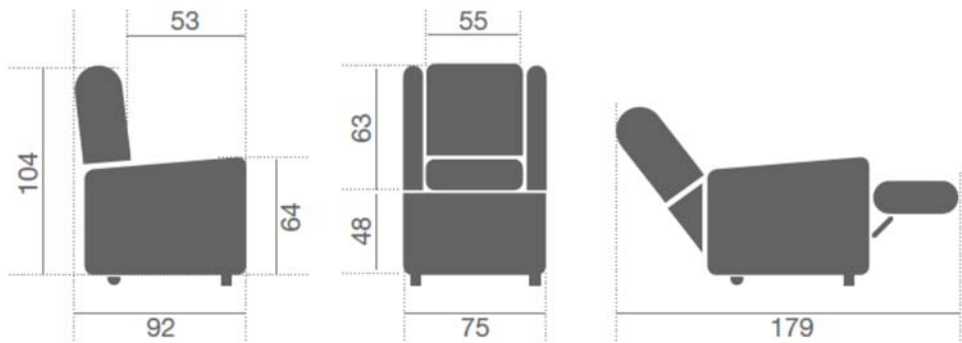

Movements;

Cod. K06-2							
Cod. K06-2R							


Model K10-2, K10-2R

Two-motor Armchair with independent footrest and backrest movement.

- Control for moving to the backrest and footrest reclined position
- Control for moving to lifting position with base tilted forward
- Adjustment of intermediate settings between the two previous positions
- Independent backrest movement
- Weight capacity: 130 kg

Dimensions in cm: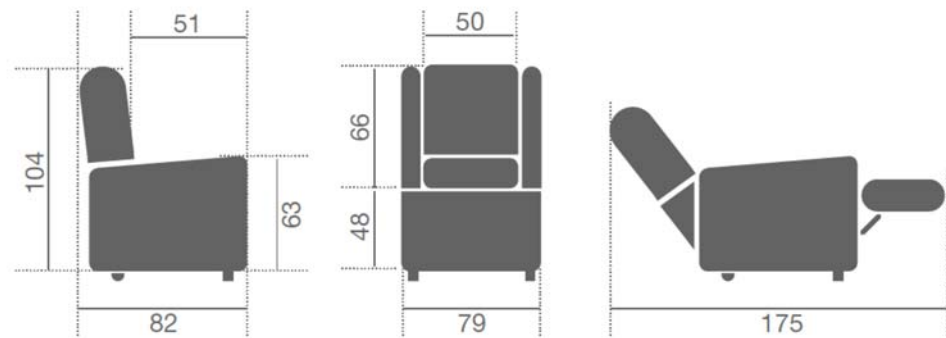

Movements

Cod. K10-2							
Cod. K10-2R							


Model K20-2, K20-2R

Two-motor Armchair with independent footrest and backrest movement.

- Control for moving to the backrest and footrest reclined position
- Control for moving to lifting position with base tilted forward
- Adjustment of intermediate settings between the two previous positions
- Independent backrest movement
- Weight capacity: 130 kg

Dimensions in cm: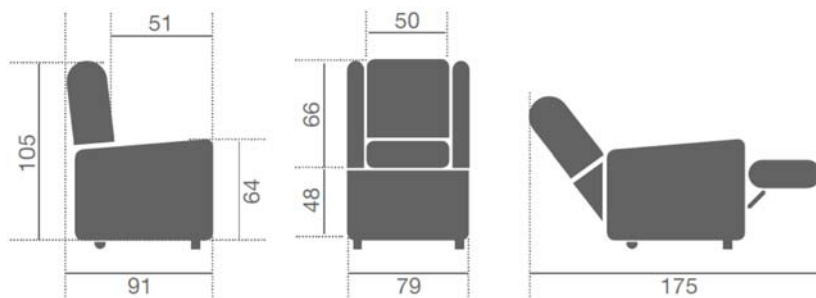


Movements;

Cod. K20-2							
Cod. K20-2R							


Model K30-2R, K30-2IR

Two-motor Armchair with independent footrest and backrest movement.

- Control for moving to the backrest and footrest reclined position
- Control for moving to lifting position with base tilted forward
- Adjustment of intermediate settings between the two previous positions
- Independent backrest movement
- Weight capacity: 130 kg
- Folding armrest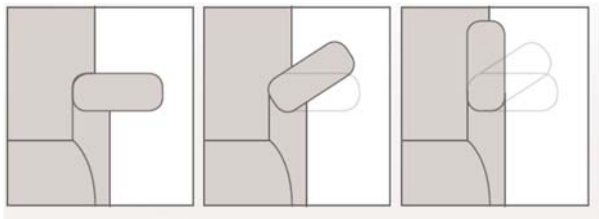

Dimensions in cm:

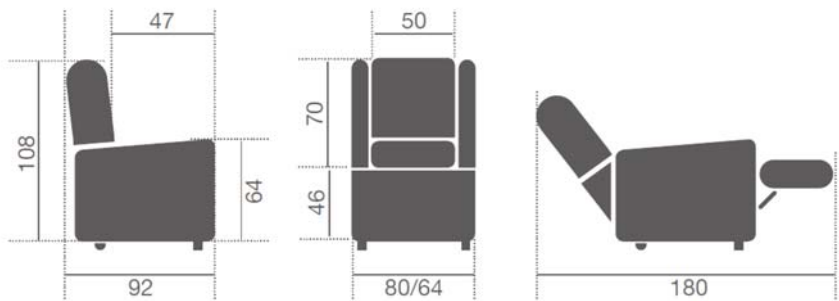
Movements;

Cod. K30-2R							
	●	●	●	●		●	●
Cod. K30-2IR							
	●	●	●	●		●	●


Model K55-2R

Two-motor Armchair with independent footrest and backrest movement.

- Control for moving to the backrest and footrest reclined position
- Control for moving to lifting position with base tilted forward
- Adjustment of intermediate settings between the two previous positions
- Independent backrest movement
- Weight capacity: 130 kg
- Folding armrest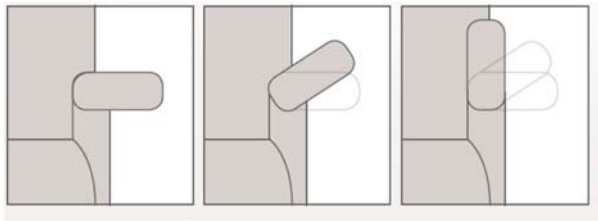

Dimensions in cm:

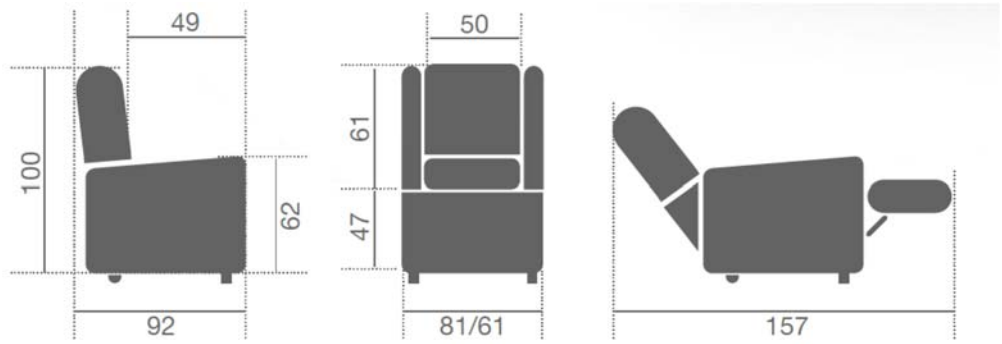Movements;

Cod. K55-2R						
	●	●	●	●		●


Model K56-2R

Two-motor Armchair with independent footrest and backrest movement.

- Control for moving to the backrest and footrest reclined position
- Control for moving to lifting position with base tilted forward
- Adjustment of intermediate settings between the two previous positions
- Independent backrest movement
- Weight capacity: 130 kg

Dimensions in cm: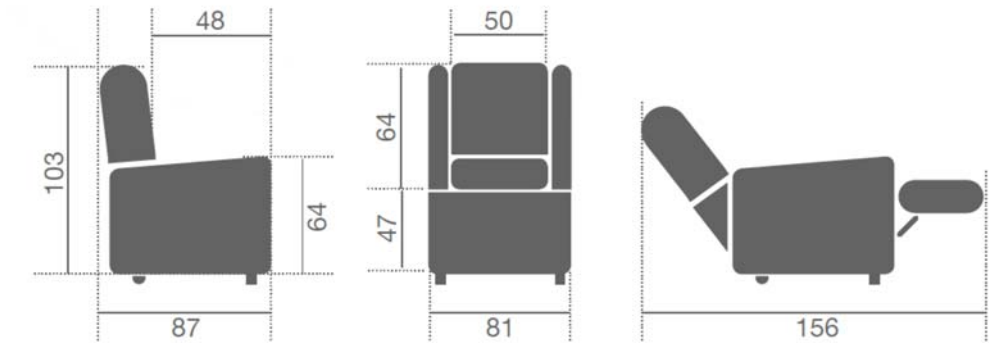

Movements;

Cod. K56-2R							
	●	●	●	●			●


Model K59-3R

Three-motor Armchair with independent footrest and backrest movement.

- Control for moving to the backrest and footrest reclined position
- Head section adjustment control
- Control for moving to lifting position with base tilted forward
- Adjustment of intermediate settings between the two previous positions
- Independent backrest movement
- Weight capacity: 130 kg

Dimensions in cm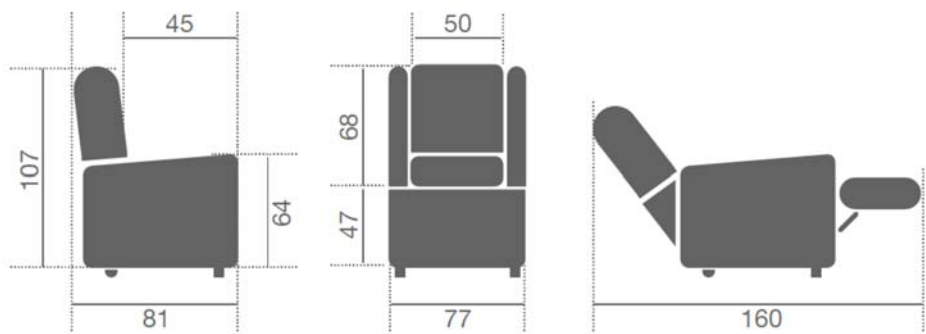

Movements

Cod. K59-3R							
	●	●	●	●	●		●


Model K65-3

Three-motor Armchair with independent footrest and backrest movement.

- Control for moving to the backrest and footrest reclined position
- Control for moving to lifting position with base tilted forward
- Adjustment of intermediate settings between the two previous positions
- Independent backrest movement
- Weight capacity: 250 kg

Dimensions in cm: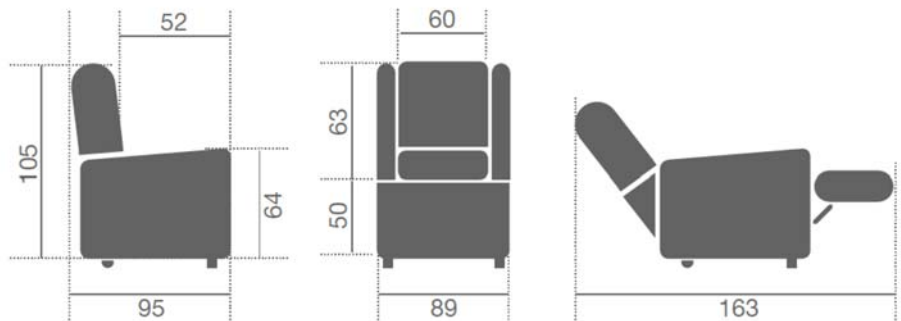

Positions

Cod. K65-3							
	●	●	●	●			●


Model K91-2R

Two-motor Armchair with independent footrest and backrest movement. For:

- Control for moving to the backrest and footrest reclined position
- Control for moving to lifting position with base tilted forward
- Adjustment of intermediate settings between the two previous positions
- Independent backrest movement
- Weight capacity: 130 kg

Dimensions in cm: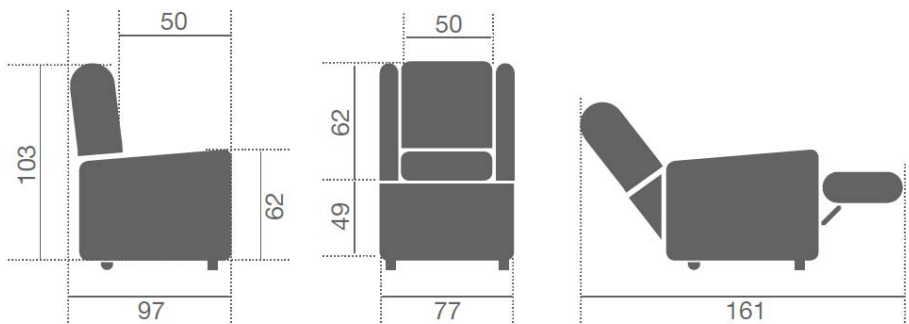

Movements

Cod. K91-2R							
	●	●	●	●			●


Model K901-2R

Two-motor Armchair with independent footrest and backrest movement.

- Moving to the backrest and footrest reclined position
- Moving to lifting position with base tilted forward
- Adjustment of intermediate settings between the two previous positions
- Independent backrest movement
- Weight capacity: 130 kg

Dimensions in cm:

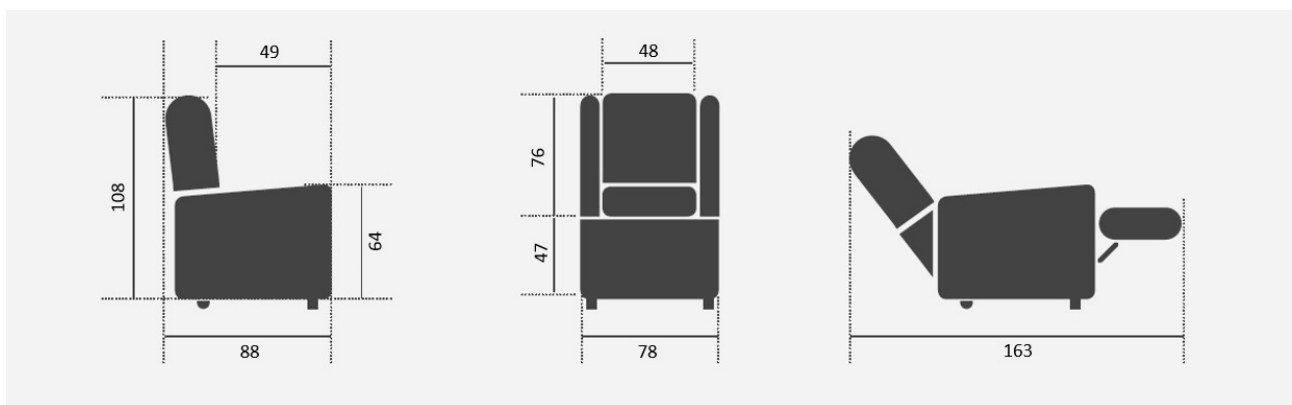Movements


K91S-2R

Two-motor Armchair with independent footrest and backrest movement.

- Moving to the backrest and footrest reclined position
- Moving to lifting position with base tilted forward
- Adjustment of intermediate settings between the two previous positions
- Independent backrest movement
- Weight capacity: 130 kg
- Small dimensions seat (cm 45x45)

Dimensions in cm:

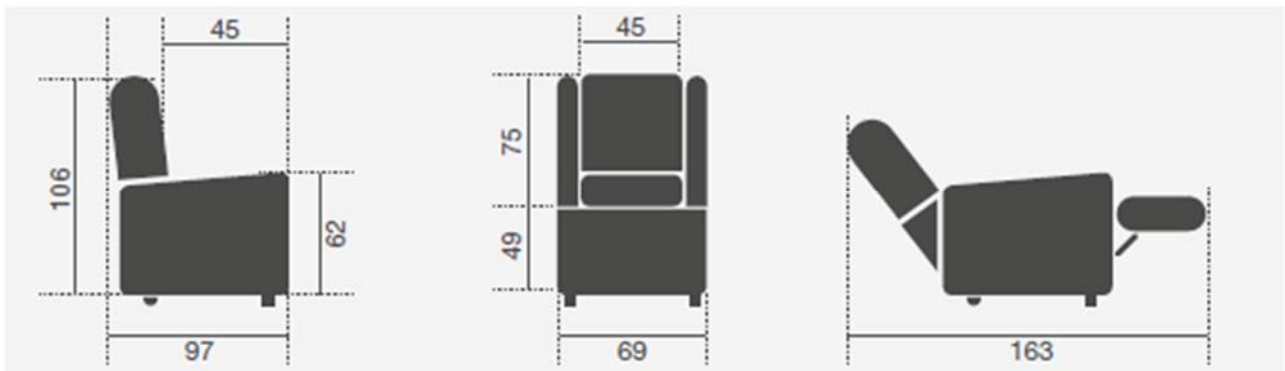Movements:


K91PA-2R

Two-motor Armchair with independent footrest and backrest movement.

- Moving to the backrest and footrest reclined position
- Moving to lifting position with base tilted forward
- Adjustment of intermediate settings between the two previous positions
- Independent backrest movement
- Weight capacity: 130 kg
- Footrest with extension at the end simultaneous to the opening command

Dimensions in cm:

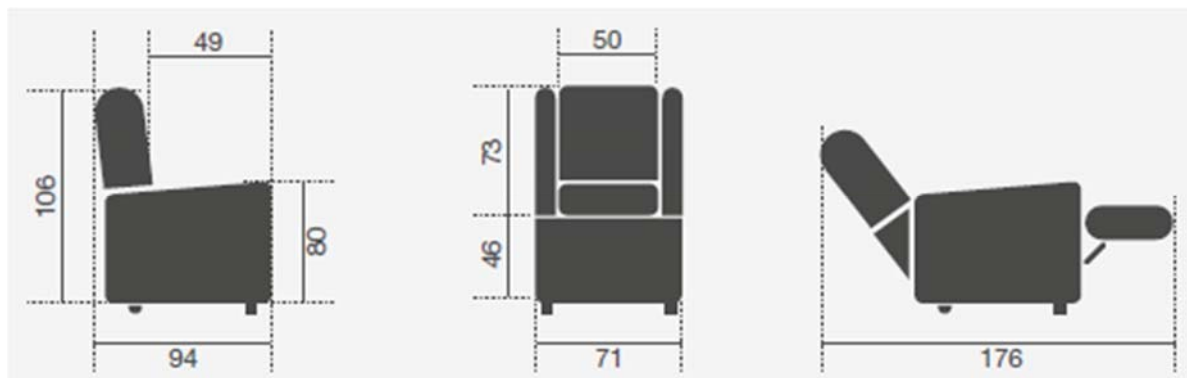Movements:


K91PAB-2R

Two-motor Armchair with independent footrest and backrest movement.

- Moving to the backrest and footrest reclined position
- Moving to lifting position with base tilted forward
- Adjustment of intermediate settings between the two previous positions
- Independent backrest movement
- Weight capacity: 130 kg
- Footrest with extension at the end simultaneous to the opening command
- Electric seat tilt

Dimensioni in cm:

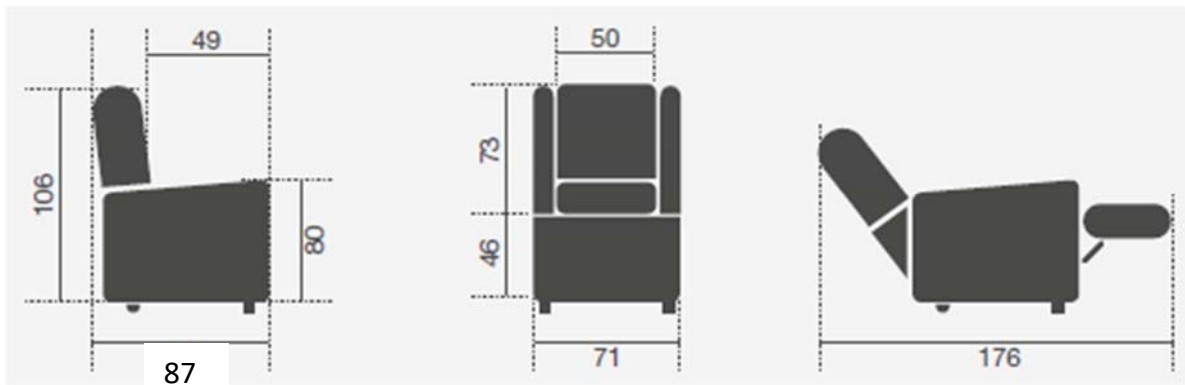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
Model K905-2R

Two-motor Armchair with independent footrest and backrest movement.

- Moving to the backrest and footrest reclined position
- Moving to lifting position with base tilted forward
- Adjustment of intermediate settings between the two previous positions
- Independent backrest movement
- Weight capacity: 130 kg

Dimensions in cm:

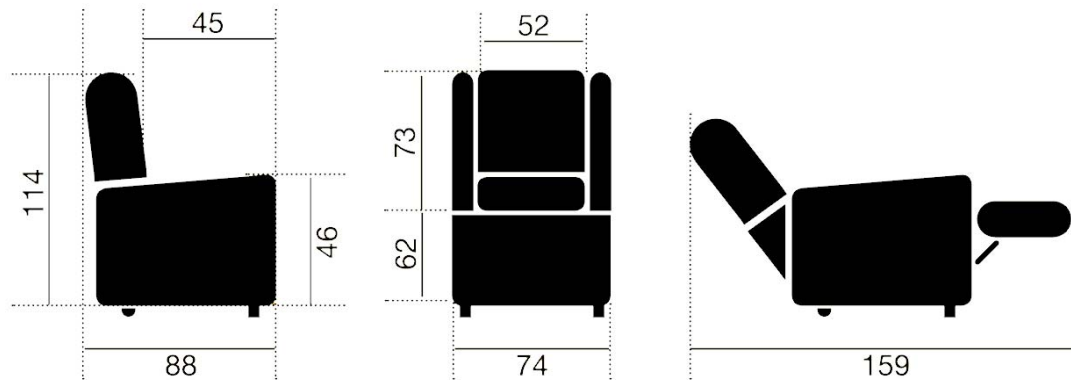Movements:


Model K915-2R

Two-motor Armchair with independent footrest and backrest movement.

- Moving to the backrest and footrest reclined position
- Moving to lifting position with base tilted forward
- Adjustment of intermediate settings between two previous positions
- Independent backrest movement
- Weight capacity: 130 kg
- Rear push bar, Removable armrest optional
- Bed System

Dimensions in cm:

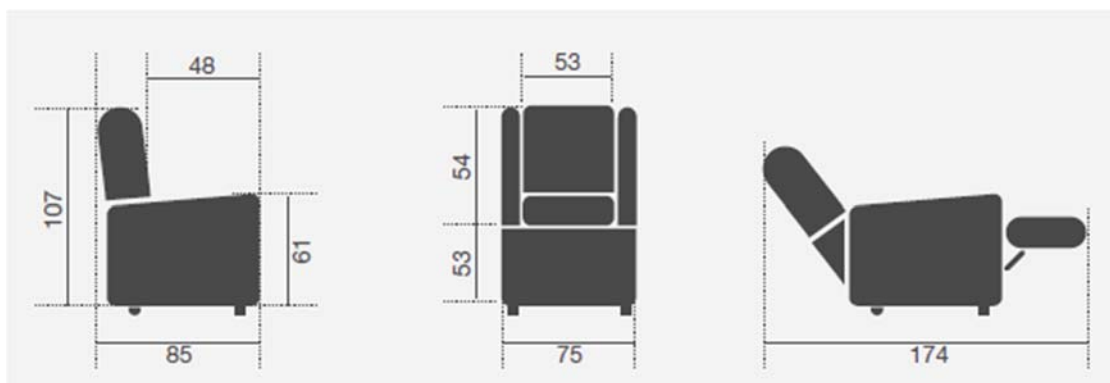Movements:


Model K902-2R

Two-motor Armchair with independent footrest and backrest movement.

- Moving to the backrest and footrest reclined position
- Moving to lifting position with base tilted forward
- Adjustment of intermediate settings between the two previous positions
- Independent backrest movement
- Weight capacity: 130 kg

Dimensions in cm:

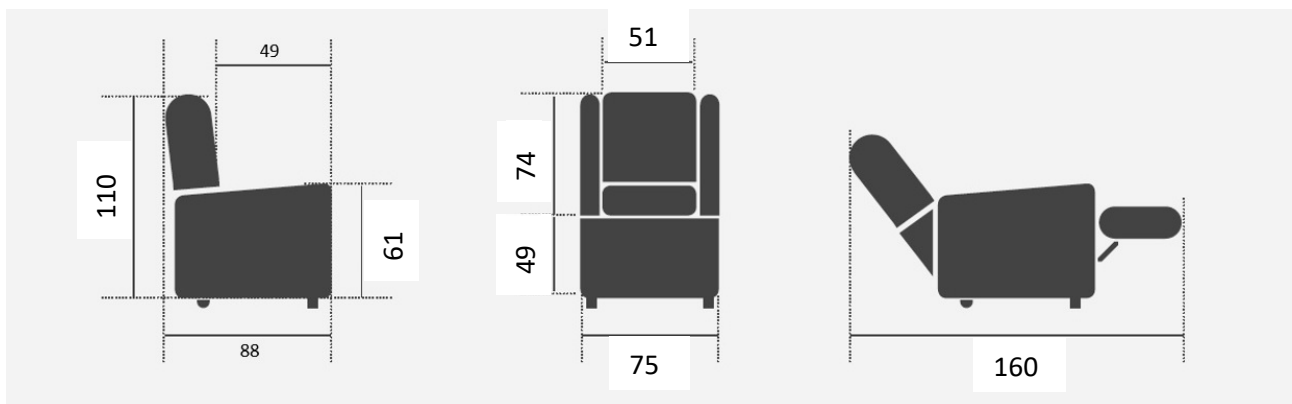Movements


Models K67-4R / K69-5R

Armchair with 4 or 5 motors, with independent footrest and backrest movement.

- Moving to the backrest and footrest reclined position
- Moving to lifting position with base tilted forward
- Adjustment of intermediate settings between the two previous positions
- Independent backrest movement
- Weight capacity: 180 Kg (K67) – 250 Kg (K69)
- Functions: LIFT / RELAX / BED / ELEVA / COMFORT / ZERO GRAVITY.
- Quality fabric with high wear resistance.
- Manual Roller system (K67) / Electric Roller System (K69).
- Removable sides and lateral headrests for space saving, patient transfer and emergency operations.
- Equipped with a rechargeable battery that allows it to be used anywhere in the room.

Dimensions in cm:

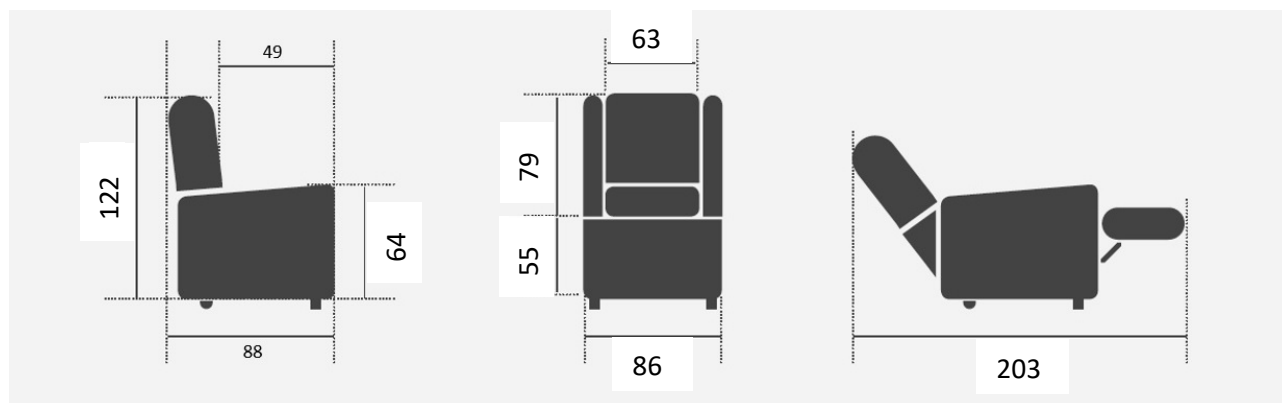Movements:

Other specifications:

Temperature of use	10-40 °C
Noise	<80 dBA
Whole body vibrations	<0.5 m/s ²

Power supply

Input voltage	220 VAC, 50/60 Hz
Maximum rated current	2 A (1.3 A for K901-2R, 3.3 A for K915-2R/ K67-4R/K69-5R)

Environmental conditions for transport and storage

Temperature: minimum -10°C – maximum 50°C, 15% - 70% RH

Environmental conditions of use

Temperature: minimum +5°C – maximum 40°C

Humidity: 15% - 70%

Atmospheric pressure: 700 – 1060 hPa

7.2 User interface

The user interface on the Armchair consists of the push-button control panel.

The push-button panels with only two buttons are for the models with a single motor (suffix "-1"), while those with 4 or more buttons are for the models with 2 or 3 motors (suffix "-2" or "-3").

All the buttons operate by sustained action and perform the following functions.



Generic two-engine Armchair

Button (16): simultaneously moves the backrest and footrest to the fully lifted position, with the backrest completely straight, the footrest fully retracted and the seat raised and tilted forward. It can be stopped at any intermediate position, and if pressed and held allows you to reach the position with the seat tilted forward, the position for lifting and getting out of the Armchair.


Button (17): simultaneously moves the backrest and the footrest to the fully reclined position. It can be stopped in any intermediate position, and pressed and held to reach the fully horizontal position with the backrest fully lowered and the footrest fully raised.

Button (18): moves the backrest independently, without moving the footrest or the seat. The button lowers the backrest.

Button (19): moves the backrest independently, without moving the footrest or the seat. The button raises the backrest.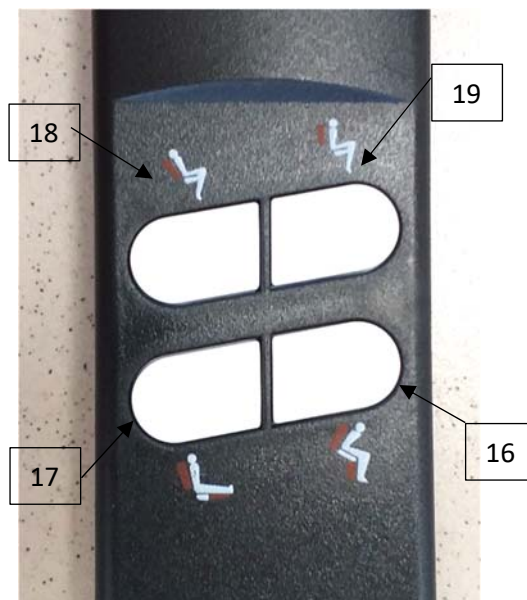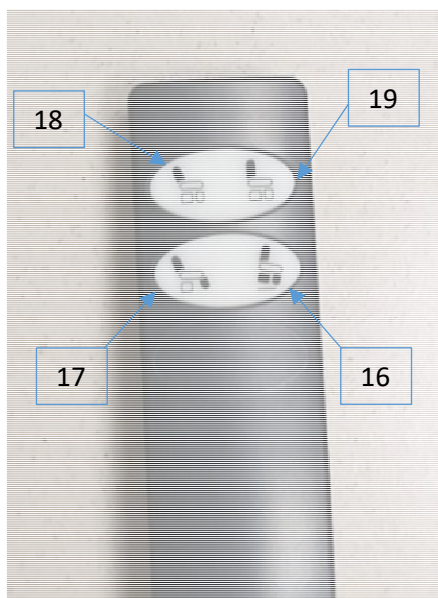

Armchair model K65-3 has the following push-button panel model.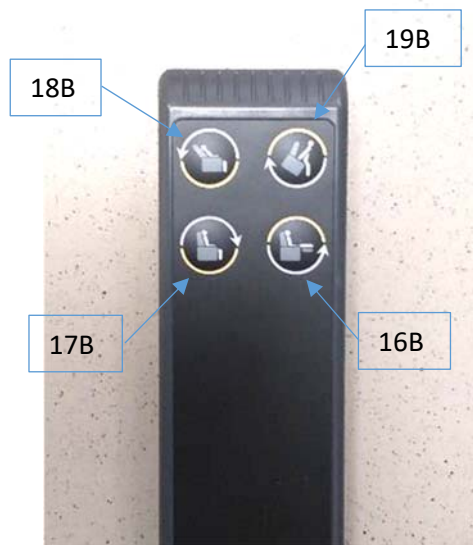

Button (16B) moves the footrest upwards.

Button (17B): moves the footrest downwards.

Button (18B): moves the backrest downwards.

Button (19B): moves the backrest upwards. Once the vertical position is reached, if you continue to press the button, the Armchair rises up to allow the User to get off or get on.

Armchair with adjustable head section

The K59-3R Armchair has an 8-button control panel as it has additional backrest motorisation in the head section.

Button (20): reset button. Brings the chair to a sitting position, with the footrest retracted, the seat lowered and the backrest upright. When pressed, the LED (30) turns red.

Button (21): stops the movement of the Armchair. To lock the Armchair, press and hold the button until the red LED flashes twice (28). To unlock the Armchair, press and hold the button again until the LED flashes twice (28).

The LED (29) lights up when any of the other buttons are pressed.

Button (22): raises the foot section of the Armchair.


Button (23): lowers the foot section of the Armchair. If the foot section is fully lowered and you continue to press the button, the Armchair rises up to allow the User to get off or on.

Button (24): lowers the back section.

Button (25): raises the back section.

Button (26): lowers the head section of the backrest.

Button (27): raises the head section of the backrest.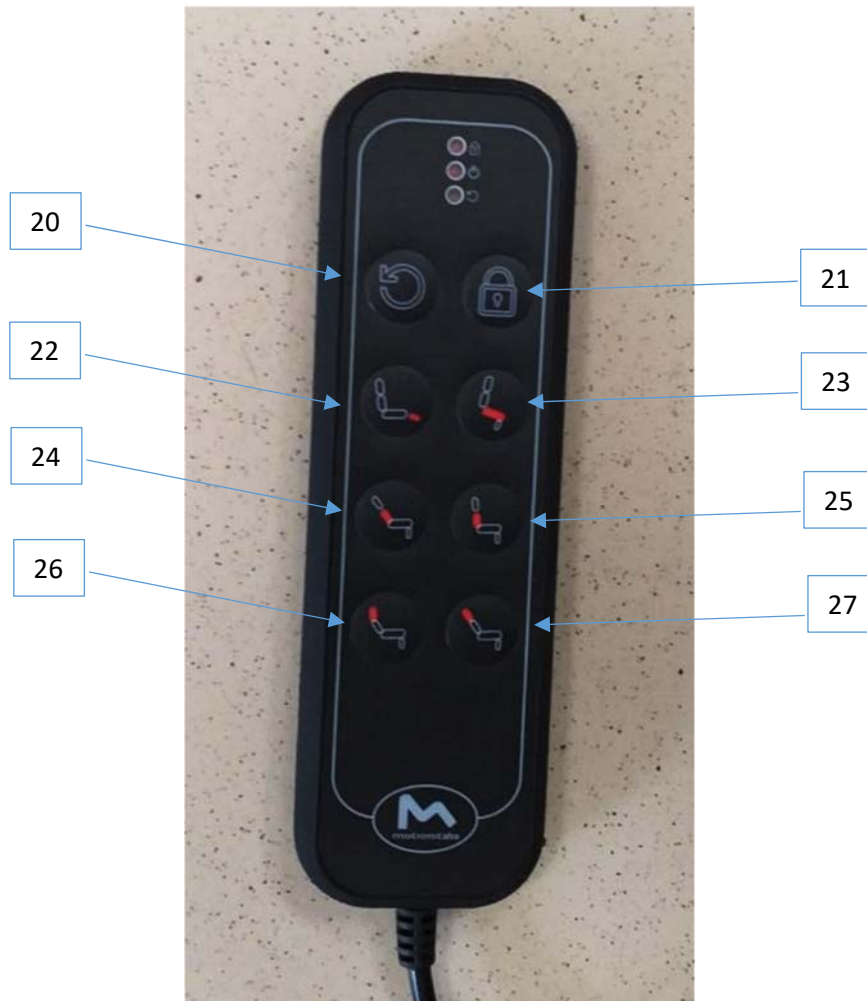

Armchair with 5 motor

The K69-5R Armchair has a 12-button control panel as it has 3 additional motorisation respect to the standard models.

Button (20): reset button. Brings the chair to a sitting position, with the footrest retracted, the seat lowered and the backrest upright. When pressed, the LED (32) turns red.

Button (21): stops the movement of the Armchair. To lock the Armchair, press and hold the button until the red LED flashes intermittently (32). To unlock the Armchair, press and hold the button again until the LED remains fixed (32).

Button (22): raises the foot section of the Armchair.

Button (23): lowers the foot section of the Armchair. If the foot section is fully lowered and you continue to press the button, the Armchair rises up to allow the User to get off or on.

Button (24): lowers the back section.

Button (25): raises the back section.

Button (26): raise the chair to the lift position (leaning forward).


Button (27): the armchair descends from the raised position, returning orthogonal to the ground.

Button (28): activates the Bed function, lowering the backrest until it assumes the lying position with simultaneous tilting of the armchair backwards.

Button (29): ascends the back section returning to a normal position, no longer lying down.

Button (30): raise the armchair in the Eleva position by lifting it from the wheels (not leaning forwards), which must be pressed until the end if you want to move the armchair by pushing it manually, therefore with the aid of the transport wheels.

Button (31): the armchair descends vertically, bringing it back to the ground and no longer on the rear transport wheels.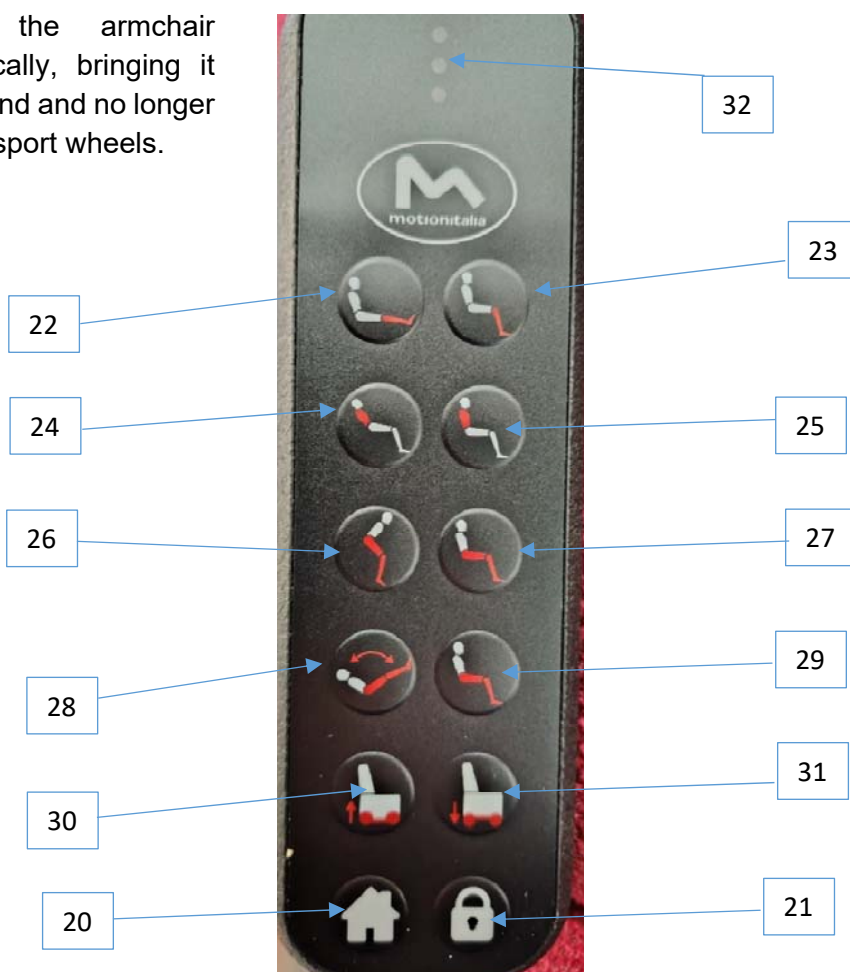

Armchair with 4 motor

The K67-4R Armchair has a 8-button control panel as it has 2 additional motorisation respect to the standard models.

Button (22): raise the chair to the lift position (leaning forward).

Button (23): the armchair descends from the raised position, returning orthogonal to the ground.

Button (24): activates the Bed function, lowering the backrest until it assumes the lying position with simultaneous tilting of the armchair backwards.


Button (25): ascends the back section returning to a normal position, no longer lying down.

Button (26): lowers the back section.

Button (27): raises the back section.

Button (28): raises the foot section of the Armchair.

Button (29): lowers the foot section of the Armchair. If the foot section is fully lowered and you continue to press the button, the Armchair rises up to allow the User to get off or on.

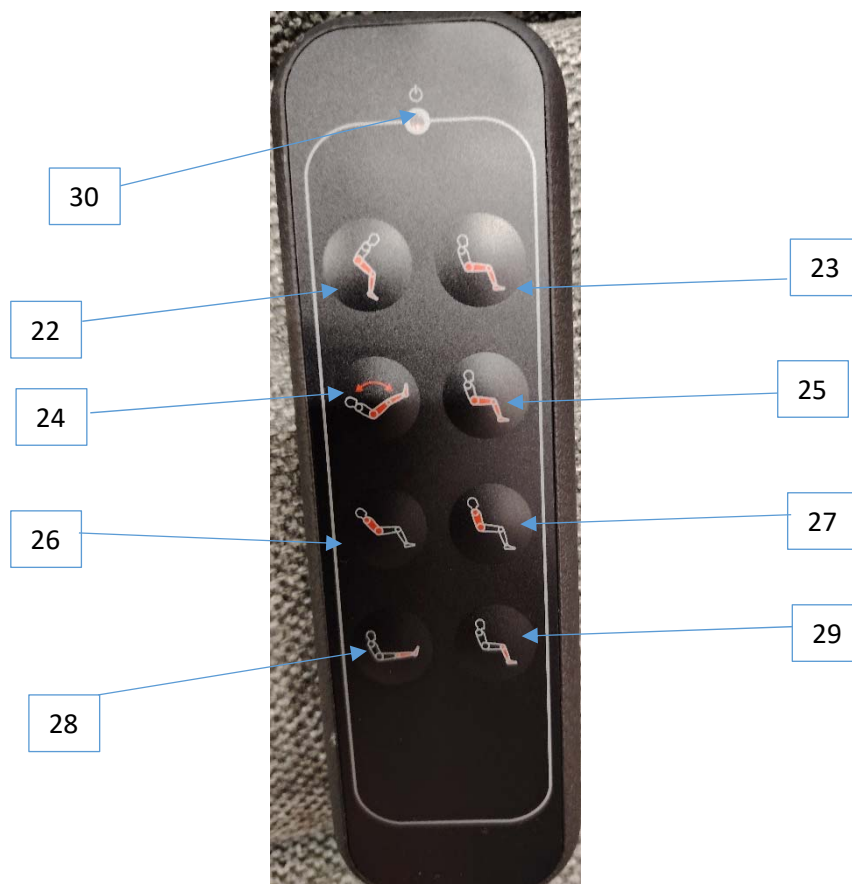WARNING! DO NOT USE THE ARMCHAIR WITH THE ROLLER INSERTED FOR SAFETY REASONS; ALWAYS CHECK THAT IT IS DISENGAGED BEFORE EACH USE.

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XX: Armchair model

Y: number of motors, can be 1, 2 or 3.

W: additional properties, if any. For example, i = fire retardant

Z: presence or absence of a roller system or wheels for moving the Armchair

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8- Using the device

General safety guidelines

Caution! When not using the product, keep the Armchair in a domestic environment. Do not leave it outdoors (on balconies, etc.), keep it away from direct exposure to the elements, such as rain, moisture or direct sunlight, which can affect the operation of the product. The storage environment must have hygienic conditions (by way of limited example: free from mould, insects, rats, pet droppings and the like).

Caution! Keep away from heat sources (radiators, stoves, fireplaces, portable heating elements such as irradiation or fan heaters, etc.) and from water (sinks, tanks, water containers such as cleaning buckets, etc.)!

Warning! The weight of the Armchair with the user seated on it can cause damage over time to certain floorings, such as parquet or floating floors. If in doubt, contact the Manufacturer.

8.1 First Start-Up

Check that the Armchair has been properly assembled. See the chapter on assembly for more details on the correct procedures.

On the first start-up, check the general state of the Armchair, in particular:

- That all movements are performed correctly
- That no smells of burning or electrical equipment are noticed
- That with a person seated, the movements occur without jerking or effort.
- Carry out all the checks indicated in the chapter on operator/user maintenance, both those before each use and those at daily and weekly intervals.

Check the following after each use:

- d. The integrity of the mechanical parts;
- e. The condition of the power supply: that it is not excessively hot;
- f. That there are no smells of overheated electrical material.

When using the Armchair, the safety and well-being of the person seated are the most important priorities. Then, follow the safety instructions at the beginning of this manual and all the advice received during the description of the device.

If the User has back problems, we recommend that you take particular care. In addition, the User must be calm and relaxed while seated on the Armchair.

The person seated should avoid abrupt movements.


The user must not be holding any kind of weight or object. Also ensure that user's clothing or limbs do not interfere with the movements of the Armchair.

The lifting and lowering operations must be performed as described in the relevant paragraph and with due attention to the greater sensitivity of the user.

With the user seated, the total weight of the Armchair is increased. The User's movements can change the centre of balance of the Armchair, particularly in the reclined position. The hip zone should always be kept above the seat.

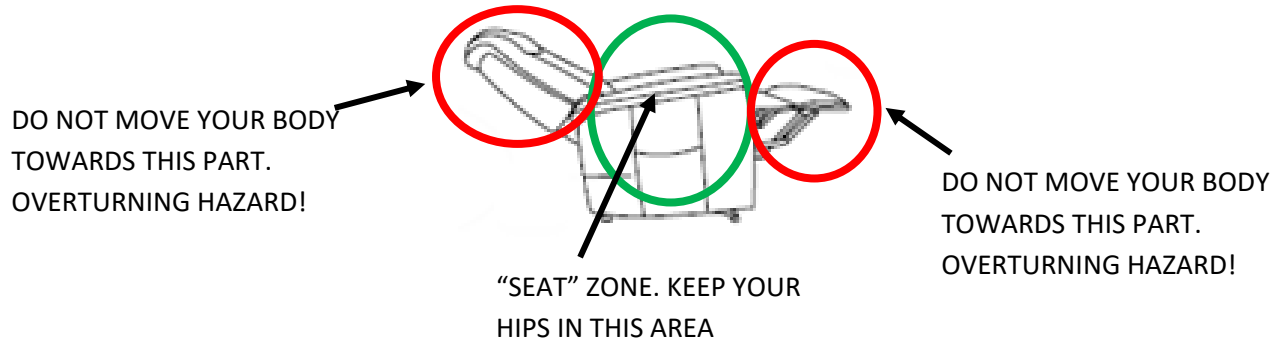NOTE: All the controls must be held down to operate.

Periodically check the condition of the backrest and footrest and make sure that there is no excessive play or loss of integrity. Otherwise, do not use the device. Disconnect the power (remove the power supply and store it in a safe place to make sure that no one uses the Armchair, if there is an external power supply) and contact the Manufacturer to check the need for repairs and/or replacement.

8.2 Mode of operation

8.2.1 Utilization


Once the armchair has been assembled, in accordance with the instructions in the relevant paragraph, positioned in a suitable place away from sources of heat and humidity, and connected to the power supply, the product is ready to use.


Check that the Armchair has been properly assembled. Refer to the chapter on assembly for more details on the correct procedures.

Lifting and lowering operations must always be carried out with the seat of the Armchair behind the user and with the Armchair raised or, in any case, with the footrest fully retracted.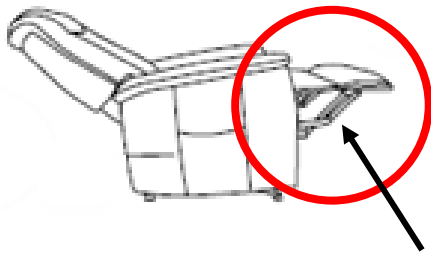


YOU MUST NOT SIT DOWN
ON THE ARMCHAIR WITH
THE FOOTREST EVEN
PARTIALLY EXTENDED

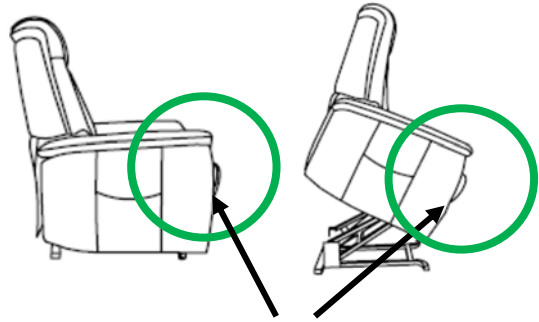YOU MAY SIT DOWN.
FOOTREST FULLY
RETRACTED.

CAUTION! BEFORE EACH USE, AND BEFORE MOVING THE PARTS, ENSURE THAT THERE ARE NO OBSTACLES WITHIN AT LEAST 1.5 M OF THE BACK OF THE ARMCHAIR. THE MOVEMENT OF THE PARTS CAN COLLIDE WITH OBSTACLES AND CAUSE SUDDEN MOVEMENTS, WITH DANGER OF FALLING.

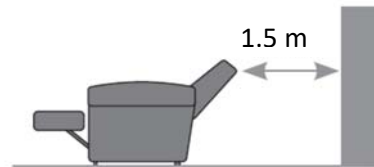CAUTION! Before sitting on an Armchair with a roller system, always check that the system is disengaged, and that the Armchair is resting on its feet and not on the wheels! Falling and bruising hazard!


“Sitting” position for moving the Armchair on its roller system.

CAUTION! Before sitting on an Armchair with fixed swivel wheels, always check that the brake is engaged and that the Armchair is immobile. Try to move it with your hands and check that the brakes are working properly. Falling and bruising hazard!

SITTING ON THE ARMCHAIR


- Place the Armchair in a raised or horizontal position with the footrest retracted, as in the figures below.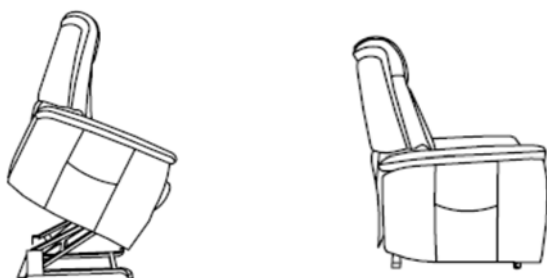

- Move close to the Armchair, with your back towards it, until the armchair is touching the back of your legs, as in the figures below. If the User is not self-sufficient, these operations can be performed with the help of an assistant.

CAUTION! Avoid the use of lifters for positioning/removing the user from the Armchair. The design of the Armchair is not compatible for use with these devices.

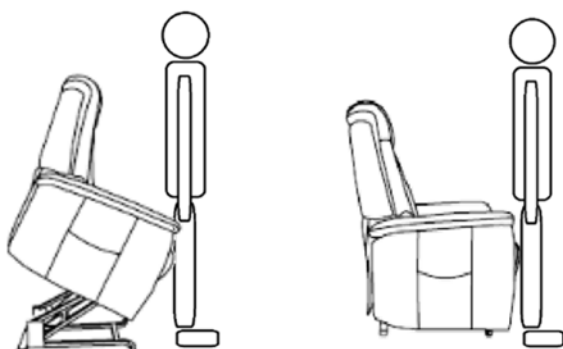- Sit down and adjust the Armchair to the position you prefer.

NO!

CAUTION! do not sit down or get up from the Armchair with the footrest even partially extended!

To stop the movements of the parts of the Armchair, release the control buttons.

ATTENTION!!! OBSERVE THE INTERVALS OF USE INDICATED ON THE LABEL.

XX MIN ON/ YY MIN OFF

THIS INDICATES THAT CONTINUOUS OPERATION OF UP TO XX MINUTES MUST BE FOLLOWED BY YY MINUTES WITHOUT OPERATION. THIS PREVENTS THE MOTORS FROM OVERHEATING. FAILURE TO OBSERVE THIS INDICATION LEADS TO RISK OF OVERHEATING AND FIRE.

For continuous usage of less than 2 minutes, observe a proportional rest period. For example, if 2 min ON/ 5 min OFF is indicated, for 12 minutes of operation, let the motors rest for at least 2.5 minutes.

The interval depends on the type of motor and the weight capacity, and may vary over time, without prejudice to the accuracy of the label.

IN ANY CASE, SEE THE PRODUCT LABEL FOR THE CORRECT TIME INTERVALS AND CONTACT THE MANUFACTURER IF IN DOUBT.

The following details are given for each type of Armchair:

MODEL	Capacity kg	Period of use
K05-2	130,00	1 minute ON / 8 minutes OFF
K05-2R	130,00	1 minute ON / 8 minutes OFF
K05-2IR	130,00	1 minute ON / 8 minutes OFF
K06-2	130,00	1 minute ON / 8 minutes OFF
K06-2R	130,00	1 minute ON / 8 minutes OFF
K10-2	130,00	1 minute ON / 8 minutes OFF
K10-2R	130,00	1 minute ON / 8 minutes OFF
K20-2	130,00	1 minute ON / 8 minutes OFF
K20-2R	130,00	1 minute ON / 8 minutes OFF
K30-2IR	130,00	1 minute ON / 8 minutes OFF
K30-2R	130,00	1 minute ON / 8 minutes OFF
K55-2R	130,00	1 minute ON / 8 minutes OFF
K56-2R	130,00	2 minutes ON / 18 minutes OFF
K59-3R	130,00	2 minutes ON / 18 minutes OFF
K65-3	250,00	2 minutes ON / 5 minutes OFF
K67-4R	180,00	2 minutes ON / 8 minutes OFF
K69-5R	250,00	2 minutes ON / 8 minutes OFF
K91-2R	130,00	2 minutes ON / 18 minutes OFF
K91S-2R	130,00	2 minutes ON / 18 minutes OFF
K91PA/PAB-2R	130,00	2 minutes ON / 18 minutes OFF
K901-2R	130,00	2 minutes ON / 18 minutes OFF
K902-2R	130,00	2 minutes ON / 18 minutes OFF
K905-2R	130,00	2 minutes ON / 18 minutes OFF
K915-2R	130,00	2 minutes ON / 8 minutes OFF

Follow these intervals unless otherwise indicated on the label:

2 MIN ON/ 18 MIN OFF

GETTING UP FROM THE ARMCHAIR

- To get up from the Armchair, the backrest must be completely raised and the footrest fully retracted.
- Bring the Armchair to one of the positions shown below.

- Get up from the Armchair.


CAUTION! BEFORE SITTING DOWN, CHECK THAT THE FLOOR IS NOT SLIPPERY AND THAT THERE ARE NO CARPETS OR THE LIKE THAT CAN CAUSE THE ARMCHAIR TO MOVE WHILE YOU ARE SITTING DOWN OR GETTING UP FROM IT. DANGER OF SUDDEN MOVEMENTS AND FALLING!

CAUTION! DO NOT PUSH OR MOVE THE ARMCHAIR WITH SOMEONE SEATED ON IT.

CAUTION! THE ARMCHAIR CONTAINS ELECTRICALLY OPERATED MECHANICAL MOVING PARTS. THEIR MOVEMENTS CAN BE A CAUSE OF IMPACT, CRUSHING OR SHEARING HAZARDS. KEEP OBJECTS, CHILDREN AND ANIMALS AWAY FROM THE DANGER ZONES INDICATED IN THE AREAS UNDERNEATH. IN ADDITION, OBSTACLES TO THE MOVEMENTS CAN CAUSE OVERHEATING OF THE MOTORS AND FIRE HAZARD.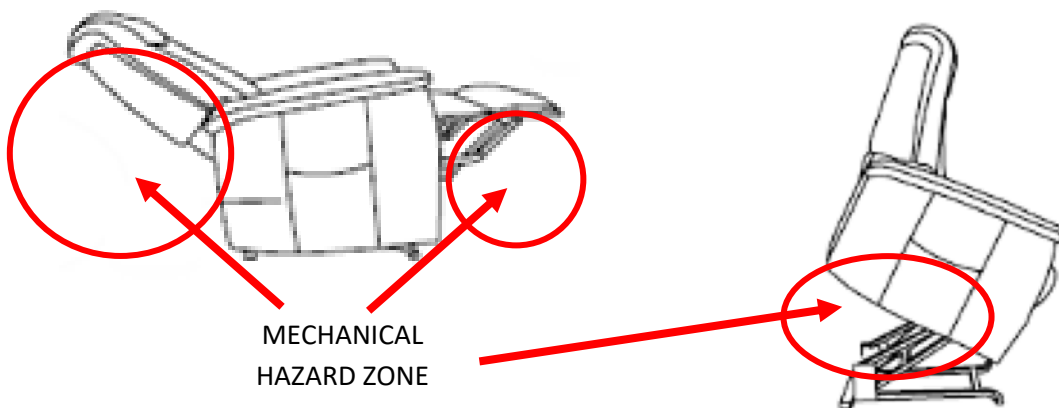

CAUTION! THE MOTOR AND MECHANISM ZONE IS COVERED WITH A FABRIC CURTAIN. CHECK THAT PETS DO NOT ENTER IT, AND IN ANY CASE, PREVENT CHILDREN OR OTHER PEOPLE FROM ACCESSING IT. CRUSHING AND/OR SHEARING HAZARD.

- The Armchair has been tested for the emission of magnetic fields (electromagnetic compatibility). However, it is recommended, for additional safety, to check that it causes no interference with any other medical devices used by the user, operator or other people nearby.
- The use of the Armchair is only permitted in indoor environments sheltered from the elements.
- The user must maintain a seated position, with his/her back in the backrest zone and legs in the footrest zone. It is prohibited to adopt positions other than that clearly indicated in this manual, such as (but not limited to) standing or kneeling on the Armchair.
- Do not use the Armchair to support things other than people.
- Do not use the Armchair on slippery, wet or damaged surfaces with carpets or wax polish. Danger of slipping while sitting down/getting up from the Armchair.
- Do not use the Armchair on uneven surfaces.

ATTENTION!!! Persons who are not self-sufficient should never be left alone in the Armchair.

CAUTION! Both before and during use, always ensure that the electrical cables do not end up between the moving parts of the Armchair! ELECTROCUTION hazard!

8.2.2 Instructions for emergency situations

Numerous hazardous situations can arise in the environment where the Armchair is installed, and it is impossible to foresee all the emergencies that may occur. General indications are provided here, always granting the User or assistant the flexibility to adopt alternative measures in response to the situation at hand.

- In the presence of an imminent risk or hazard due to the mechanical structure, such as loss of stability or resistance, jerking movements or strange noises, call the assistant or another person for help. Do not operate the motors or stop movements in progress. Disconnect the Armchair from the power supply and get up or help the user up from the Armchair.
- In the presence of an imminent risk or hazard due to an electrical part, such as the start of fire or risk of electrocution, call the assistant or another person for help. Do not operate the motors or stop movements in progress. Do not touch the electrical part but get up or help the user up from the Armchair.

- In the event that the user suddenly feels unwell, the position to adopt (whether with the backrest upright or fully reclined) depends on the state of health and the type of condition. Contact your physician.

CAUTION! The power disconnection or emergency stop function is performed by removing the electrical plug from the wall outlet. Make sure that the plug and wall outlet are nearby, in a clearly visible position and accessible to the operator/user.

The user must remain calm and relaxed while sitting on the Armchair. You must absolutely avoid abrupt movements by the person being transported.

The Armchair contains electronic parts. Therefore, in the event of fire, only use a CO₂ extinguisher.

POSSIBLE EMERGENCY SITUATIONS	ACTIONS TO TAKE
• Short circuit in the motors • Trace the electrical cables with the conductor cables exposed • Other electrical hazard	• call the assistant to obtain help, • do not operate the motors or stop movements in progress, • disconnect the Armchair from the electrical supply, • get up or help the user up from the Armchair.
• Failure of mechanical parts that may affect stability • Failure of mechanical parts that may affect the resistance of the structure	• call the assistant to obtain help, • do not operate the motors or stop movements in progress, • do not touch the electrical part, • get up or help the user up from the Armchair.
• The person being transported faints or is nauseated	• adopt the appropriate position or help the user to get up, depending on his/her condition, • if necessary, call for help.

9- Charging the batteries

No version of this device has batteries. The power supply must be of the type indicated on the label and in the technical section.

10-Troubleshooting

PROBLEM	PROBABLE CAUSE	POSSIBLE SOLUTION
The moving parts do not operate or visibly "force" the movement	Lack of power supply or obstruction to the movement	• Check that the plug is properly connected. • Check that there are no obstacles preventing movement. • Check that there are no objects between the moving parts. • If the problem persists, contact the Manufacturer's technical support.
The surface of the power supply or electrical parts is hot	Problem in the electrical parts	• If the product has been used continuously for less than 2 minutes, this has no effect on the machine. • Otherwise, if overheating occurs without have performed movements, unplug immediately and contact technical support.
Shutdown of the Armchair following prolonged use	Motors overheated	• This can be due to the activation of a self-resetting temperature limiter. • Unplug and wait at least two hours before reuse.
The roller system or wheels do not allow movements	Roller lowering mechanism or wheels jammed	• Check for any dirt or objects preventing wheel movement or lowering of the mechanism. • If the problem persists, contact the Manufacturer's technical support.
The push-button panel controls do not respond	Roller system not fully raised	• Make sure the roller system is fully raised. • In there is no roller system, or the roller system is fully raised, contact the Manufacturer's technical support.
Other	Other	If cases other than the above occur, remove the plug from the outlet, wait a while, plug in once more and turn the device on. Immediately after purchase, the machine may give off a strange smell due to its covering. If, despite this, the machine continues to have problems, contact an authorised retailer or the Manufacturer.

If, despite having carried out the checks and remedies described above, the device continues to present a fault that limits (even partially) its proper operating conditions, the operator or person in charge of its use must immediately contact the authorised Technical support service, the retailer or the Manufacturer.

In each of the cases mentioned above, from the moment a problem/defect appears, do not use the Armchair until it has been resolved/repared and you have received authorisation for use from the Manufacturer. Do not take any initiatives of your own other than those indicated above.

11-Maintenance

This chapter describes the maintenance operations that the user is allowed to perform.

All maintenance must be carried out with the cable disconnected from the power supply.

Before reselling or purchasing the used product, make sure that all maintenance has been carried out and that the product has been refurbished by the Manufacturer.

Before carrying out any inspection, maintenance or cleaning operation, disconnect the power supply and make sure the user has got up from the Device.

11.1 Cleaning

Fabrics (User/Operator - as necessary or with a change of use to another User)

For maintenance of the fabrics, use the special fabric cleaning kit available from authorised KSP Italia retailers.

Do not use neutral detergents, bleaches, trielin, petrol or other thinners, to avoid altering the covering.

Do not use spray polishes, paste creams or unsuitable products.

If the outer covering comes into contact with detergents (particularly those containing sulphides), shampoos or other substances, clean it immediately with a cloth slightly dampened with a solution of neutral soap and water and then dry it with a dry cloth.

Removable fabrics (User/Operator - as necessary or with a change of use to another User)

For removable fabric covers or accessories, follow the cleaning and maintenance labels.

Push-button panel (User/Operator - as necessary or with a change of use to another User)

Clean with a damp cloth. In the case of particularly stubborn dirt, use only neutral soap.

Never use solvents, petrol, diesel, trielin or alcohol.

Disinfecting (User/Operator - as necessary or with a change of use to another User)

Whenever necessary, the Armchair can be disinfected with suitable non-corrosive products that do not harm the fabrics.

Disinfectants must not contain phosphates, phosphorus or formaldehyde and must have a pH value of between 6 and 8.

If you have any doubts about the operation, safety or maintenance of the product, please contact KSP Italia.

Do not spray water or other substances that could damage the surfaces.

11.2 Checks to perform daily or before each use (user/operator)

Daily or before each use:

- Visual check of the general condition of the product.
- Check that the structure is not bent and has not suffered impacts.
- Check that the backrest is firmly inserted in the seat, making sure that there is no play, damage, signs of detachment or cracks between the seat and the backrest.
- Check that all movements occur smoothly, performing a movement cycle with no one on the Armchair, and, if present, lower the roller system to check that the Armchair moves.
- Visually check the condition of the electrical cables and push-button panel. In the case of wear, unplug the power connection, do not use the product and contact technical support.
- Confirm that the brakes on the wheels are working properly by engaging the brakes and trying to move the armchair, ensuring that it does not move.

11.3 Visual check of the structure (user/operator - weekly)

At least once a week, check that the structure of the Armchair is intact, that the backrest and footrest are not bent, cracked or show signs of failure and that there is no excessive play in the moving parts.

Any other operations or checks than those described in the above paragraph must be carried out by specialised personnel or in authorised KSP Italia service centres.

Take the utmost care:

- to carry out all maintenance and/or repair operations using appropriate tools and to avoid damage to the cables.
- not to make the connections in a different way than that shown in this manual.
- not to connect devices to the control unit with electrical and mechanical characteristics other than those indicated in this manual.
- not to connect the power cable to outlets with voltages other than those indicated in this manual.
- not to wet the electrical components.

The device must be disconnected from the mains supply before carrying out repairs, disassembly or any operation that brings the operator into contact with live parts (the plug for the wall outlet is the disconnection device).

CAUTION!

During maintenance, disconnect the plug from the power outlet. Do not perform these operations with wet hands. RISK OF ELECTROCUTION OR INJURY.

CAUTION!

Do not splash water on the device. RISK OF ELECTROCUTION AND DAMAGE TO THE DEVICE.

11.4 Storage conditions

For periods without use of more than 60 days, place the device in its initial packaging. Store the packaged product in a dry environment at room temperature.

Take care not to impact the product during handling. In particular, place the seat in a horizontal position, retract the footrest and remove the backrest.

Do not store in places directly exposed to sunlight or near heat sources to avoid changes to the covering and do not store the product outdoors.

Storage environment: Temperature: -10C° to 50°C Humidity: 10 to 70% (without condensation).

11.5 Periodic maintenance

To ensure that the Armchair continues working efficiently and retains its features, a routine maintenance programme must be carried out every three years by the Manufacturer or by qualified personnel authorised by KSP Italia.

For a list of service centres, contact the representative agent/retailer or the company directly, at:

KSP Italia Srl
Via Dell'Artigianato, 1
06031 Bevagna (PG) Italy
Tel. +39 0742 361947 - Fax +39 0742 361946
ksp@kspitalia.com - www.kspitalia.com

Periodic maintenance includes a check of the following:

- electrical system
- control buttons
- motors
- mechanical wear
- roller system and swivel wheels, depending on the model
- power supply efficiency
- movement system
- mechanical transmission
- load tests

ATTENTION!!! FOLLOWING ACCIDENTS, FALLS, IMPACTS OR SHOCK OF ANY KIND, THE OPERATION AND SAFETY OF THE DEVICE MAY BE AFFECTED. THE ARMCHAIR MUST UNDERGO A SERIES OF CHECKS BY THE MANUFACTURER/AUTHORISED CENTRES TO ENSURE ITS SUITABILITY FOR USE. USING IT WITHOUT AN INSPECTION BY THE MANUFACTURER/AUTHORISED CENTRE OR HAVING IT INSPECTED BY UNAUTHORISED ENTITIES EXEMPTS THE MANUFACTURER FROM ALL LIABILITY FOR SUBSEQUENT MALFUNCTIONS OR DEFECTS AND INVALIDATES THE WARRANTY.

12-Disposal

Proper separate collection for the subsequent environmentally compatible recycling, treatment or disposal of abandoned equipment contributes to mitigating possible negative effects on the environment and health and promotes the recycling of the materials that comprise the equipment.

The barred dumpster symbol shown on the equipment indicates that the product must be collected separately from other waste at the end of its useful life.

The Armchair is an electrically-powered medical device and its disposal is the responsibility of KSP Italia, at no charge to the buyer.

KSP Italia is a member of the mandatory consortium APIRAEE (www.apirae.it) and is registered in the National Register of Manufacturers of Electrical and Electronic Equipment with manufacturer code **IT08090000005523**

Illegal disposal of the product by the user is subject to the administrative sanctions referred to in Legislative Decree no. 152 of 3 April 2006.

13-Warranty

The warranty is valid for 24 months for a new product or 12 months for a used product or one (already assigned) for professional use; in all cases, the warranty starts from the delivery date given on the invoice and delivery documents.

The batteries are guaranteed for 6 months.

For the warranty to be effective, the buyer/end user (or other holder of the sales agreement) must report obvious defects within two months of delivery and hidden defects within two months of their discovery, and in any case not beyond the warranty period.

Complaints and service requests must be submitted in writing by registered post, e-mail or telegram to KSP Italia, or to the authorised retailer involved in the sale. All repair work must be carried out by the authorised retailer's technical support service or by KSP Italia personnel; work on the product by unauthorised personnel causes immediate termination of the warranty.

The warranty covers the free replacement or repair, under the agreed terms and conditions, of component parts of the device which, **at the manufacturer's sole discretion**, have material or factory defects.

The warranty does not cover damages caused by incorrect maintenance and/or failure to comply with the guidelines for use and the precautions/instructions given in this manual, as well as transport damage (scratches, dents, etc.), damage from falls, damage caused by carelessness, tampering, inability to use the product and repairs made by unauthorised personnel; damage due to incorrect installation of the product when performed directly by the user or by unauthorised personnel, damage caused by dirt, stains or liquids, scratches on the covering, damage caused by power surges and blackouts, alterations caused by environmental or climate conditions and all damage due to causes not resulting from normal operation. In any case, the warranty does not cover defects due to normal wear and tear, negligence and careless use, misuse and/or erroneous use, as specified in this Manual.

Under no circumstances will the replacement of a component under the warranty extend the overall warranty period of the product, including that of the part replaced.

14-Support – Home service

KSP ITALIA provides a Home Technical Support Service that can be requested by contacting the authorised retailer or KSP Italia directly.

15-Reports – Alert

Any accident, malfunction, breakdown and/or alteration of the characteristics and performance of the device that has caused or may potentially cause harm to the operator/user must be promptly reported, contacting the retailer through the contact details given in the purchase document, or the Manufacturer directly, so that all appropriate measures may be taken, including recall of the device.

The Operator, Buyer or Patient must report any serious accident that occurs in connection with the Device to KSP Italia and the competent authority of the Member State where the user resides.

KSP Italia Srl

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06031 Bevagna (PG) Italy

Tel. +39 0742 361947 - Fax +39 0742 361946

ksp@kspitalia.com - www.kspitalia.com