

MULTIFIT NS 2.4 mm (3/32")

Thickness Perforation	mm (inches) % (type)	2.4 (3/32) 0 (non perfo)	2.4 (3/32) 28 (micro+)
hermoforming conditions			
Optimum activation temperature (in water bath)	°C (°F)	65 (149)	65 (149)
Activation time (in water bath)	minutes	3 – 4	3 - 4
Transparent when activated		no	no
Working time	minutes	1 – 1 ½	0 ¾ - 1 ¼
Hardening time	minutes	3 ¾ - 4 ¼	1 ¾ - 2 ¼
Time to completion	minutes	20 - 21	11 – 12
Resistance to stretch		moderate	moderate
Drape		high	high
Memory (after 200 % elongation)		low	low
Maximum elongation when activated	%	1250	20
Memory (after maximum elongation)		low	low
Sticks to itself when activated and wet		temporarily	temporarily
Sticks to itself when activated, after drying		temporarily	temporarily
Adhesion (velcro strip) using heat gun		n.a.	n.a.
Aechanical properties at 21°C			
Flexural modulus	MPa	800	215
Elastic modulus	MPa	700	200
Tensile strength	MPa	7.0	3.5
Strain at break	%	7	12
Seneral properties			
Density	g cm⁻³	1.46	1.46
Hardness (shore D)	-	48	48
Surface feeling		smooth	smooth
Color		pearl white/ black	pearl white / black
Odor		none	none
Fatigue (60°)	cycles	750	2300
Fatigue (90°)	cycles	250	500
Biocompatible	•	yes	yes



MULTIFIT NS 3.2 mm (1/8")

Thickness	mm (inches)	3.2 (1/8)	3.2 (1/8)
Perforation	% (type)	0 (non perfo)	1.0 (macro)
hermoforming conditions			
Optimum activation temperature (in water bath)	°C (°F)	65 (149)	65 (149)
Activation time (in water bath)	minutes	3 - 4	3 - 4
Transparent when activated		no	no
Working time	minutes	1 ¼ - 1 ¾	1 ¼ - 1 ¾
Hardening time	minutes	4 ¾ - 5 ¼	4 ¾ - 5 ¼
Time to completion	minutes	19 - 20	19 – 20
Resistance to stretch		moderate	moderate
Drape		high	high
Memory (after 200 % elongation)		low	low
Maximum elongation when activated	%	1400	900
Memory (after maximum elongation)		low	low
Sticks to itself when activated and wet		temporarily	temporarily
Sticks to itself when activated, after drying		temporarily	temporarily
Adhesion (velcro strip) using heat gun		n.a.	n.a.
Aechanical properties at 21°C			
Flexural modulus	MPa	740	625
Elastic modulus	MPa	535	500
Tensile strength	MPa	7.0	5.0
Strain at break	%	18	5
General properties			
Density	g cm⁻³	1.46	1.46
Hardness (shore D)		48	48
Surface feeling		smooth	smooth
Color		pearl white / black	pearl white / black
Odor		none	none
Fatigue (60°)	cycles	750	500
Fatigue (90°)	cycles	750	500



TECHNICAL DATA SHEET

INFORMATION

The hardening time indicates the time period during which the material remains flexible, but no longer mouldable.

The time to completion indicates the length of time until the orthosis is finished and can be worn by the patient.

The memory indicates the ability of the material to regain its original shape after reheating.

The flexural modulus indicates the resistance of the material to a force causing it to bend.

The elastic modulus defines the ratio of the applied tensile stress to the change in shape of the material.

The tensile strength is the pulling force required to break the material.

The strain at break is the length increase of the material when stretched until failure.

The hardness indicates the resistance of the material to compression.

Fatigue indicates the minimum number of stress cycles the material sustains when bending over 90 degrees without failure.

The biocompatibility is studied according the guidelines of the International Organization for Standardization 10993 – Biological Evaluation of Medical Devices:

- o Primary skin irritation study.
- o Delayed dermal contact sensitization study.
- o Cytotoxicity study.

Note:

Although the information in this publication is believed to be accurate and reliable, the data shown are for guidance only. Orfit Industries gives no guarantees about the results and assumes no liability in connection with them. The properties reported here are intended primarily to facilitate comparison among Orfit products. Standard testing methods often allow alternative measuring methods. Therefore, data from other sheet manufacturers may not be directly comparable. For additional information, please contact Orfit Industries.



www.orfit.com



© ORFIT INDUSTRIES N.V.

Belgium T +32 (0)3 326 20 26 America T 516-935-8500 Ref.: 80020 – MULTIFIT NS.docx VERSION 1

LAST UPDATE: 2022-06-01