# biöwalker®

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Manufacturer takes responsibility for proper usage of the product only when the product was purchased from qualified Reh4Mat® sp. z o.o. employee or at specialized medical store.

REH4MAT® sp. z o.o. reserve the right to introduce technical and trading changes in the content of the instruction without warning.

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Product marked with the CE sign is Class I medical device, of rule 1, consistent with the requirements of Regulation (EU) 2017/745 of the European Parliament and of the Council of 5 April 2017 on medical devices.

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REH4MAT® sp. z o.o. Company introduced quality management system ISO 13485 regarding production, sales and service. The system is certified by Det Norske Veritas.

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Biowalker® is an active upright device for children with mobility impairment. The upright position is achieved by Patient and Parent or Carer. The product is made up of three parts: an adult vest, a child hip harness and calf shells with sandals. Hip harness maintains the Patient in the correct position and reduces the load of the lower limbs. The Patient's feet are placed on the comfortable sandals made of ABS. These sandals are assembled to calf shells of the Carer or Parent. They provide active, anatomic gait pattern with proper coordination between upper and lower limbs. You can adjust the height of the sandals.

Biowalker® reflects the physiological Patient's gait pattern and improves cerebral neuroplasticity and helps to restore psychomotor functions. Our upright mobility device prevents against the muscle weakness and contractures, increases the bones density and improves the vascular and respiratory system. What is more, Biowalker® provides Patient's independence and peer socialization. The device may be used only on harden, even surface.

REH4MAT® sp. z o.o. products are innovative and carefully manufactured of the high quality materials. Our key objective is customer's satisfaction. In case of any questions feel free to contact us by e-mail:

[biuro@reh4mat.com](mailto:biuro@reh4mat.com)

**CONTENTS:**

1. Purpose of use	19
2. Product description	19-20
3. Safety rules	21-22
4. Care and Maintenance	22
5. Using	23-30
6. Technical inspection	31

**1. PURPOSE OF USE**

Biowalker® is an active upright device which allows Patient to maintain standing position and walking together with Parent or Carer. The device is good for children who cannot stand and transfer by their own. Biowalker® is recommended for children from age 1 (physiological walking milestone). The product is made up of three parts: an adult vest, a child hip harness and calf shells with sandals.

**INDICATIONS:**

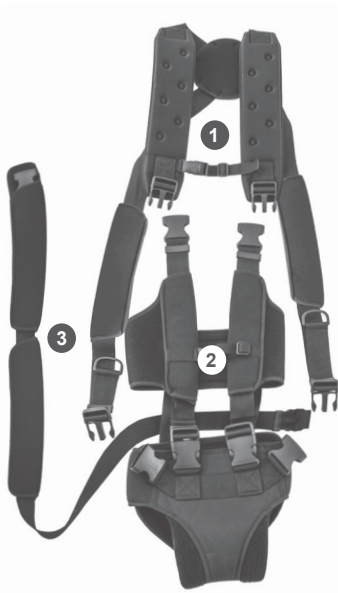
- Cerebral palsy (CP)
- quadriplegia/tetraplegia
- dyskinesia (athetosis, chorea)
- dystonia
- spinal bifidia
- paraplegia
- small muscles contractures
- multiple sclerosis (SM)
- abnormal gait pattern

**CONTRINDICATIONS:**

- severe muscles and tendons contractures
- sprain/strain in the lower limb area
- severe body asymmetry (high degree of scoliosis, pelvic and/or spinal deformations)
- Lower back pain of the Carer
- Pregnancy of the Carer
- Patient with unstable medical
- condition requiring fast repositioning for treatment

**WARNING!** If there are any doubt about Child's or Carer's ability to use the Biowalker®, please consult doctor or therapist.

**2. PRODUCT DESCRIPTION**

1. Adjusted adult vest
2. Trunk and hip harness for Patient
3. Hip belt, adjustable in circumference and height.
4. Calf shells with sandals

**CARER**

**PATIENT**

**CARER**

Mature person who knows this user manual. It can be **PARENT OR CARER**. This person is staying physically and mentally fit. Due to **CARER'S** walking, the **PATIENT** achieves natural gait pattern. Functional design of the **Biowalker®** provides parent-child proximity and improves rehabilitation process.

**PATIENT**

Child in age 1 or more with maximum weight 40 kilos (88 lb). **PATIENT'S** feet are assembled on the sandals which moves together with **CARER**.

The **PATIENT'S** height cannot be below the clavicles of the **CARER**.

### 3. SAFETY RULES

---

**WARNING!** Read this instruction carefully before use and keep for future reference. In other case you may hurt yourself and the Child.

**WARNING!** You can use the device only on the even, hardened surface.

**WARNING!** Only mature person, staying physically and mentally fit, may be a Carer.

**WARNING!** Carer cannot be under the influence of psychoactive substances, drugs or alcohol.

**WARNING!** If some problem of Patient or Carer occur during using the device (fainting, fatigue), stop using the products and consult your doctor immediately.

**WARNING!** Using the Biowalker® on stairs, may result in injury and is not recommended.

**WARNING!** Always check the product before using. If any signs of damage are observed, do not use it! Do not use the Biowalker® if any part is broken or missing. It may hurt you and the Child.

**WARNING!** Always ensure all straps are securely fastened. Check if straps do not unfasten due to Patient's and Carer's weight. Improper assemble may hurt the users after slipping.

**WARNING!** Do not change the design of this device. Do not attach, remove, sew and cut any elements. Doing any changes in this product or using not in accordance with instructions increases the risk of hurt the users.

**WARNING!** Adjust length of straps and tapes only in the sitting position. Do not adjust the products while standing! It may risk in severe injury of the users.

**WARNING!** Please follow instruction in "USING" (point 5 in this manual). Putting and removing the Biowalker® is possible only in the sitting position. While sitting or standing we recommend to use handrail or handle for better balance.

**WARNING!** Keep the device clean. Please follow the instructions in "CARE AND MAINTAINCE" (point 4 of this manual).

**WARNING!** Ensure Patient's hip harness and Carer's hip girdle are on the same level. Maintaining the hip harness above the Carer's pelvis may result in severe lower back pain of the Carer.

**WARNING!** Recommended shoes while using the Biowalker® are trainers (sneakers) for safety and stability reasons. Shoes with heels, large boots, sandals or slippers should not be worn.

**WARNING!** The Patient's body weight should be distributed on both feet. Patient cannot "hung" in the device. If the Patient's feet do not have a full contact with sandals, we do not recommend to use the Biowalker®.

**WARNING!** Straps cannot be fasten too tight or too loose. They cannot be curled or twisted.

**WARNING!** Always ensure the device is fitted and assembled to Patient and User properly before using.

Both, Patient and Carer, should breath freely and feel comfortable in the device.

**WARNING!** The Biowalker® should not have directly contact with skin. Redness on Users skin may be visible and is expected on weight bearing points. If however this redness persists, please review the set-up of the product to ensure it is not too tight or too loose. If the redness still visible, stop using Biowalker® immediately.

**WARNING!** Maximum Patient weight is 40 kilos (110 lb). Never exceed the maximum load!

**WARNING!** Before using the stroller, remove all paper and plastic packaging. The packaging is not a toy! Keep away from children and animals.

## WARNING!

If the device is assembled properly and Patient is above the Carer's clavicles it means the Patient is too high in relation to Carer and cannot use the Biowalker® (fig.1)

### WARNING!

Manufacturer takes responsibility only in case of purchasing the product in authorised distributor or medical shop.

### WARNING!

Manufacturer takes responsibility only for hidden faults and technical defects of the product which occurred during a proper use of the device.

### WARNING!

Use the Biowalker® only in accordance with the intended use. In other case using the product may result in severe injury and complications

### WARNING!

For single Patient use only. Using the device not according to the manual may cause serious complications.

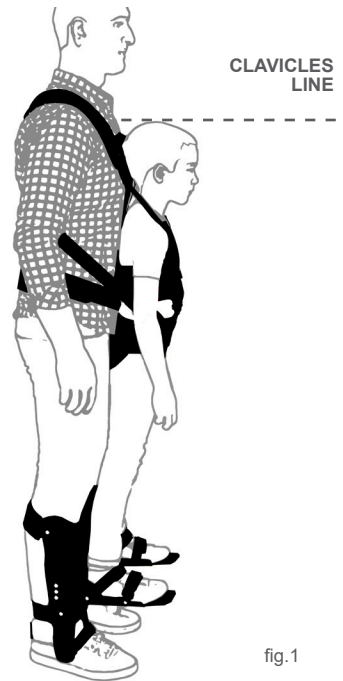
fig.1

## 4. CARE AND MAINTAINANCE

### Aadult vest, a child hip harness and soft fabric of calf shells (lining, straps):

- Wash by hands in warm water with soap in temperature of 30°C,
- Sluice carefully ,
- Do not iron and do not chlorinate,
- Do not dry clean and do not tumble dry,
- Protect from moisture,
- Protect from high temperature and dry away from sources of warmth.

### Sandals:

- Do not wash and do not sluice,
- Wipe with a clean, soft cloth,
- Use a mild cleaning agent like soap.

## 5. USING

### STEP 1 - PUTTING THE HARNESS ON PATIENT

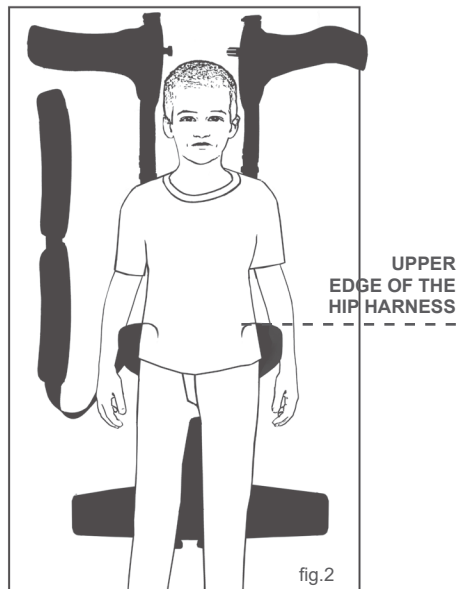
**1.1.** Lay the torso vest with the hip belt on a flat surface, such as a bed or a bench.


**1.2** Place the patient on the laid-out torso vest and hip belt so that the individual body parts of the patient (pelvis, torso, chest) are in the correct position, as indicated in the diagram (Figure 2).

**WARNING!** The correct positioning of the patient is determined by the hip belt. Its upper edge should be at the level of the anterior superior iliac spine of the patient

**Drawing how to place Patient on the unfold harness correctly.**

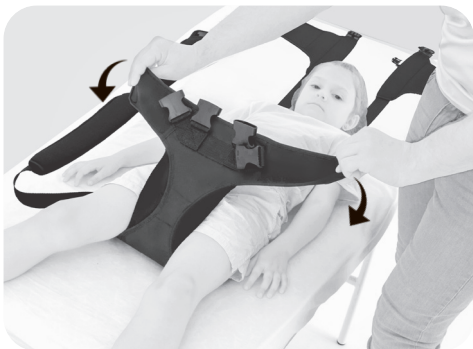**1.3.** First, fasten the hip belt.

**1.4.** Fasten the Velcro closures.

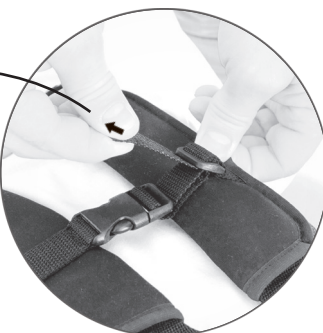
**1.5.** Next, fasten the buckles and connect the trunk belts with hip harness.

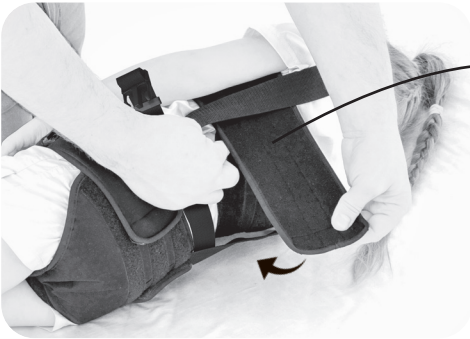
Adjust the required length of the chest's belts.

**1.6.** Connect two chest belts with the middle buckle. Adjust the required length of the middle strap if it is necessary.

**1.7.** To fasten the torso belt, which is part of the vest, gently roll the patient onto one side and fasten the Velcro. Remember to support the patient's pelvis and shoulders while turning them.



**WARNING!** Remember to ensure that the vest straps are positioned on the outside and not immobilized by the torso belt in any way. This could restrict the proper functioning of the product.

## STEP 2 - ADJUSTMENT OF THE BELTS STABILIZING THE PATIENT'S FEET

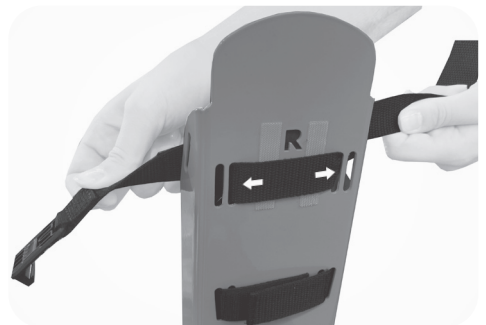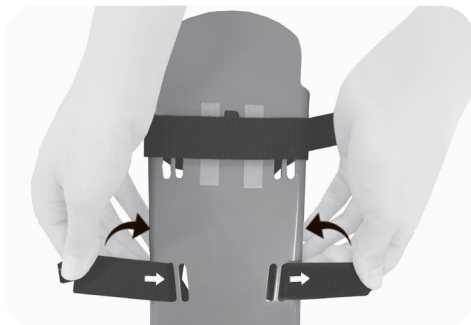**2.1.** Place the patient's foot on the foot/platform.

**2.2.** Adjust the spacing of the straps according to the width of the shoes, using various configurations (examples above).


Adjustments are made by weaving the straps through selected holes, thereby setting the foot in the abduction or adduction position.


**WARNING!** The feet are available as right (R) and left (L). The appropriate markings are located under the foot/patient platform.

**KROK 3 - PUTTING THE SANDALS**

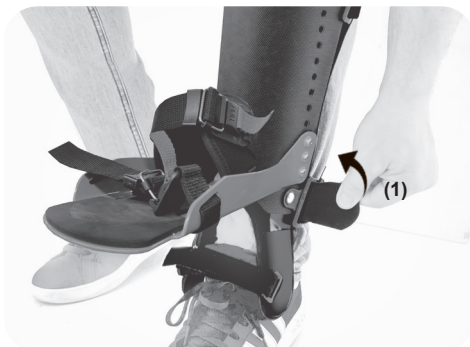**3.1.** Calf shells (el.3) should be place in the shin area.

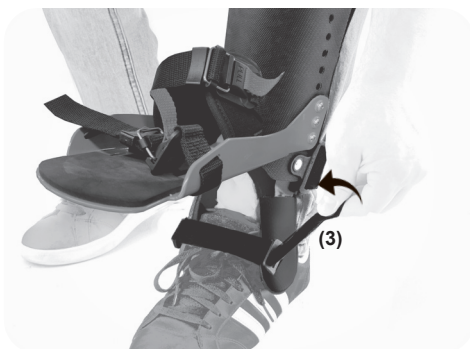**3.2.** Fasten the back Velcro straps by threading them through the buckles. First, fasten the lower Velcro strap (1) and then the upper one (2).

**3.3.** Fasten the strap which goes on your shoe.


**3.4.** Sandals are ready to use.


**WARNING!** Make sure that the strap relief elements are located in the middle of the straps.

### SANDALS HEIGHT ADJUSTMENT:

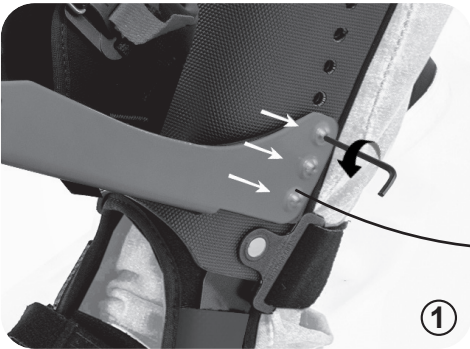①

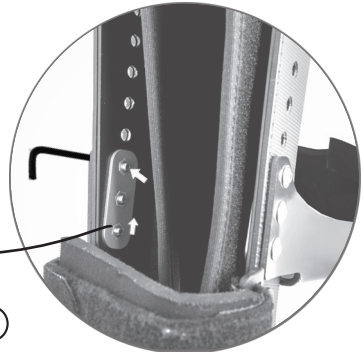②

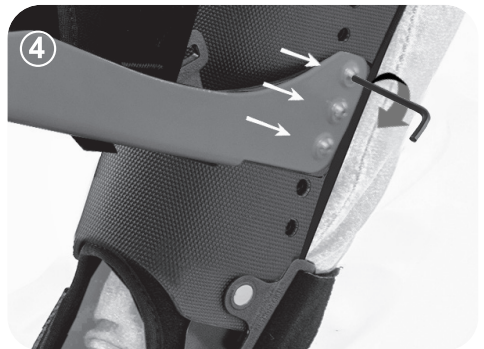
**WARNING!** If you need to change the height of the sandal, unscrew 3 screws on the left and right side using an Allen key.

Change the place of the thread washer and adjust the required height.

③

Adjust the required position of the sandals.

④

Screw the socket screws firmly.

**STEP 4 - CONNECTING THE CHILD TO THE CARER**

**4.1.** Place the straps but remember to place round element (1) between your scapulas.

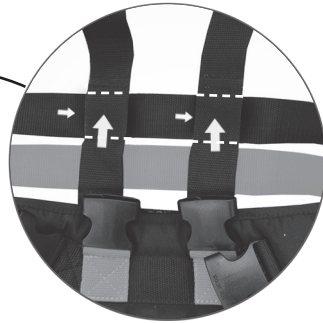
Depending on your body type, you can extend or shorten the length of the shoulder straps using adjusters.


**4.2.** Fasten the front middle strap.

**4.3.** Place your baby in a sitting position on your lap. Using the buckle, attach the upper element of the shoulder straps to the Patient's torso vest.

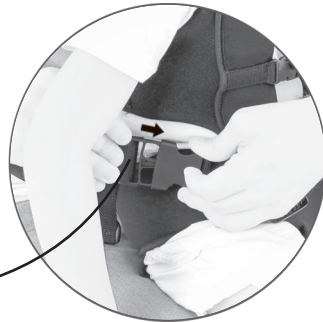**4.4.** Fasten the caregiver's lap belt. **WARNING!** It is possible to adjust the height of the hip belt.

**4.5.** Fasten the lower buckle to the hip harness.

**4.6.** Place the child's feet on the sandals.

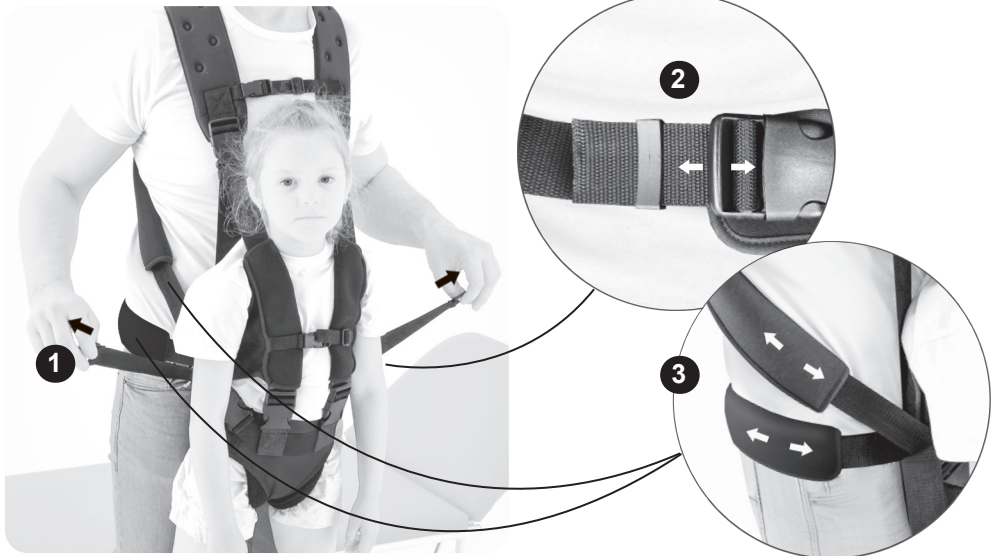
**WARNING!** Before you fasten the straps, remember to check if the child's heels touch the calf shells.

**4.7.** Fasten the straps that stabilize your child's foot. First, fasten the first strap running across the child's midfoot to the magnetic buckle (1). Perform the same operation with the upper strap (2) and similarly with the second foot. Then tighten the straps.

**WARNING!** You may stand up with the child only in case of proper connection, adjusting and fastening each element of the Biowalker®. If the straps and belts will be fastened improperly (not according to user manual), the risk of falling, hurting and injury is very high!

**4.8.** Stand up gently. Do not move instant. We recommend to use handrail or handle for better balance.

- (1) Fit the device by pulling the harness straps.
- (2) If necessary, adjust the fit of the lap belt.
- (3) Move the belt cover so that it is in the most stressed areas. This will eliminate discomfort and pressure on the bony prominences.

Wait for a few minutes and check the reaction of the child.

**WARNING!** If you will see any worrying symptoms such as dizziness, worst contact or fainting, do not continue the upright position and stop using the device. Remove the Patient from the Biowalker® and contact your doctor or therapist. If any worrying symptoms occur, you can start therapy after few minutes.

**4.9.** Products is reasy to use (front and rear view).

Incorrect product adjustment can lead to such side effects as fainting, body deformities, joint overload, etc. If the abovementioned symptoms occur, stop using the product immediately, take the user out of the product and lay them down. If the undesirable symptoms persist, contact a physician immediately. If the symptoms have disappeared, inform the attending physician about this fact in order to make a decision about the further rehabilitation process.

Serious incidents involving this device should be reported both to manufacturer and to the competent authority in your country.

## 6. TECHNICAL INSPECTION

Technical inspections should be performed regularly. Before servicing, the product should be thoroughly cleaned to reveal any hidden damage.

Frequency	What should be checked?	What should be done?	Remarks
<b>Before each use.</b>	General condition of the product.	<b>Check:</b> <ul style="list-style-type: none"> <li>the entire product for any visible damage, cracks or mechanical defects,</li> <li>if the product complete,</li> <li>if all bolts, screws, knobs, nuts and other elements of the construction are complete and tightened.</li> </ul>	In case of any damage or defects, discontinue the use of the product and contact your Supplier / Distributor or the Manufacturer.
	Fastening of the construction elements.	<b>Check all the mechanisms of the product and its moving parts. Check in particular:</b> <ul style="list-style-type: none"> <li>if the seat is properly mounted (if applicable),</li> <li>if the wheels are properly mounted (if applicable),</li> <li>if the elements securing the user are properly mounted and if they work well (if applicable),</li> <li>if all moving parts are complete, work properly, do not jam,</li> <li>if the adjustment mechanisms of the following elements work properly (if applicable): headrest, backrest, seat, abduction block, armrests, footrests.</li> </ul>	In case of any damage or defects, discontinue the use of the product and contact your Supplier / Distributor or the Manufacturer.
	Upholstery elements.	<b>Check:</b> <ul style="list-style-type: none"> <li>if there is any damage, tears, rips, loose seams or deformations on the upholstery elements,</li> <li>if the stabilizing straps, vests or safety belts are not frayed or twisted and if the buckles work properly,</li> <li>if the stabilizing straps, vests or safety belts work correctly and do not unfasten under pressure or when pulled.</li> </ul>	In case of any damage or defects, discontinue the use of the product and contact your Supplier / Distributor or the Manufacturer.
<b>Once a week.</b>	Product cleaning.	If the frame and upholstery get dirty (dusty, muddy, etc.) during everyday use, wipe them with a damp cloth without detergents. Remove dirt (e.g. hair, food, remains, etc.) from the moving parts of the product.	The product should be cleaned when necessary, but at least once a week. Do not use any chlorine based cleaners or methylated spirit. Before using the product, check if the upholstery is dry.
<b>Once a month.</b>	Product construction connectors (welding, threading, soldering).	<b>Check:</b> <ul style="list-style-type: none"> <li>the condition of the product frame in the places of joining (welds, threads, soldering).</li> </ul>	In case of any damage or defects, discontinue the use of the product and contact your Supplier / Distributor or the Manufacturer.
<b>Once every 6 months.</b>	Construction moving parts.	<b>Check:</b> <ul style="list-style-type: none"> <li>if the moving parts of the construction have not been damaged. Maintain them using a proper cleaning agent.</li> </ul>	In case of any damage or defects, discontinue the use of the product and contact your Supplier / Distributor or the Manufacturer.
	Product labeling.	<b>Check:</b> <ul style="list-style-type: none"> <li>if the marking and labels are readable.</li> </ul>	In case of any damage or defects, discontinue the use of the product and contact your Supplier / Distributor or the Manufacturer.

**REH4MAT sp. z o.o.**  
ul. Zenitowa 5a  
35-301 Rzeszów  
POLAND

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