

UP stairlift USER manual



— STAIRLIFT —

V2.1 28-5-2019

Index

Preface	2
Technical support	2
Guarantee and liability	2
Safety instructions	4
Overview	6
Unfold the seat	7
Turn the carriage on	8
Fasten the seatbelt	9
Put your feet on the footrest	10
Use the joystick	11
Powered swivel	12
Turn the carriage off	13
Use the app to move the carriage	14
Stop the carriage at a charging contact	15
In case of emergency	16
Safety sensors	17
Technical specifications	18
Maintenance	18
Cleaning	18
Contact	19
Check form	19

Preface

Your stairlift is especially designed to transfer people up to a maximum of 125 kg positioned in the chair. The information provided in this manual is based on general specifications concerning the constructions, material properties and working methods known to us at the time of publication; alterations and improvements are therefore excluded. Read this manual and familiarize yourself with the operation of the stairlift before usage. The stairlift is constructed with using state-of-the-art technology and complies with the safety regulations. Misusage of the stairlift can still pose risks to yourself and third parties. Exclusively use the stairlift in technical correct condition for the purpose mentioned above taking into account this manual. Keep this manual in close to the stairlift.

Technical support

If problems occur that are not handled in this document contact your dealer. For quick handling it is important to keep the following by hand:

- This manual;
- Your name, address, postal zip code, town and telephone number;
- Lift number

Guarantee and liability

As the user of this stairlift, you are at all times fully responsible to follow the local safety regulations and guidelines. Your UP stairlift is covered by a manufacturer's guarantee of 12 months that covers the costs of replacement parts.

Excluded from guarantee: damage caused by normal wear and tear, damage caused by overloading, damage caused by misusage of the product and the replacement of batteries.

Under no circumstances the manufacturer is responsible for death, personal injury, damage to property or incidental unforeseen damages and costs resulting from the use of the stairlift.

The implementation of adjustments on the product is not permitted. The manufacturer is not in any case liable for damages resulting from such alterations. The manufacturer accepts no liability for damage or injury caused by non-observance or inadequate observance of the safety regulations and safety instructions in this manual nor for negligence during the installation, use, maintenance and repair of the stairlift.

Installation and servicing may only be carried out under the responsibility of dealers or installers authorized by the manufacturer, where locally applicable regulations must be observed.

Safety instructions

When using the stairlift always follow locally applicable safety regulations regarding the risk of electric shock and physical injury. Furthermore, make sure to follow the safety instructions listed below:

1. Do not use the stairlift in case of a fire.
In case there is a fire do not use the stairlift for evacuation or for firefighting.
2. Provide an unobstructed environment.
Before using the stairlift clear the track from potential obstacles. Do not use the stairlift when the stair is used by another person or when any objects lay in the path of the stairlift.
3. Keep uninstructed persons and pets away from the stairlift.
The stairlift can only be used by adults who have signed the instructed persons declaration. Keep children and pets away from the stairlift. Minors may only use the stairlift under supervision.
4. Use the stairlift only to transfer people:
Do not use the lift to transfer any goods, animals or anything else than people.
5. Do not overload the stairlift.
Do not transfer more than 125 kg with the stairlift. This would result in overloading.
6. Do not use the stairlift with more than one person.
Never use the stairlift with multiple people, even not when the weight of those people combined does not exceed the maximum load.
7. Ensure a safe position on the chair.
When using the stairlift always ensure a correct and stable position. Sit straight with your back against the back support. Place both arms on the armrests. Place your feet on the firmly on the footrest. Finally, make sure the safety belt is always secured and tightened during movement transfer with the stairlift. Never use the stairlift while standing.
8. Keep limbs and clothing away from moving parts:
Make sure that limbs or (loose) clothing cannot become trapped between moving parts, for example the carriage and the rails.
9. Only step-off at a predefined stopping station.
Do not get off the stairlift when it is not located at a predefined stopping station. In case of an error and you getting trapped in the middle of the stairs, stay seated and ask for assistance with the intercom.
10. Have sufficient visibility.
From the stopping stations the entire track should be visible, possibly by using mirrors. Make sure the area where the stairlift is located is well lit.
11. Check for damages.
Before using the stairlift check for external damage. Report unusual noises, such as squeaking and creaking and any other oddities to the supplier.
12. Do not use the stairlift when a part is defective. Try not to repair the stairlift yourself, let maintenance only be performed by competent mechanics.

13. Always pay attention. At any times concentrate on the operation of the stairlift. Pay attention on the surroundings and take suitable action when required.
14. Lock the stairlift after use:
After stepping of the stairlift apply the childproof safety lock and disable it through the app to prevent unauthorized use of the stairlift.
15. Consider environment influences on the stairlift:
Do not expose the stairlift to severe sunlight. Furthermore, do not place the stairlift in very humid locations.
16. Do not remove the warning stickers. The stickers are an important part of our safety measures and therefore they may not be covered or removed.
17. Do not stand on the footrest.
18. Do not sit on the armrest.
19. Do not use the lift outside.

Overview

- 1- Power button
- 2- Joystick
- 3- Intercom
- 4- Armrest
- 5- Footrest
- 6- Main switch




Step one



Put the seat and armrest down.

Step two

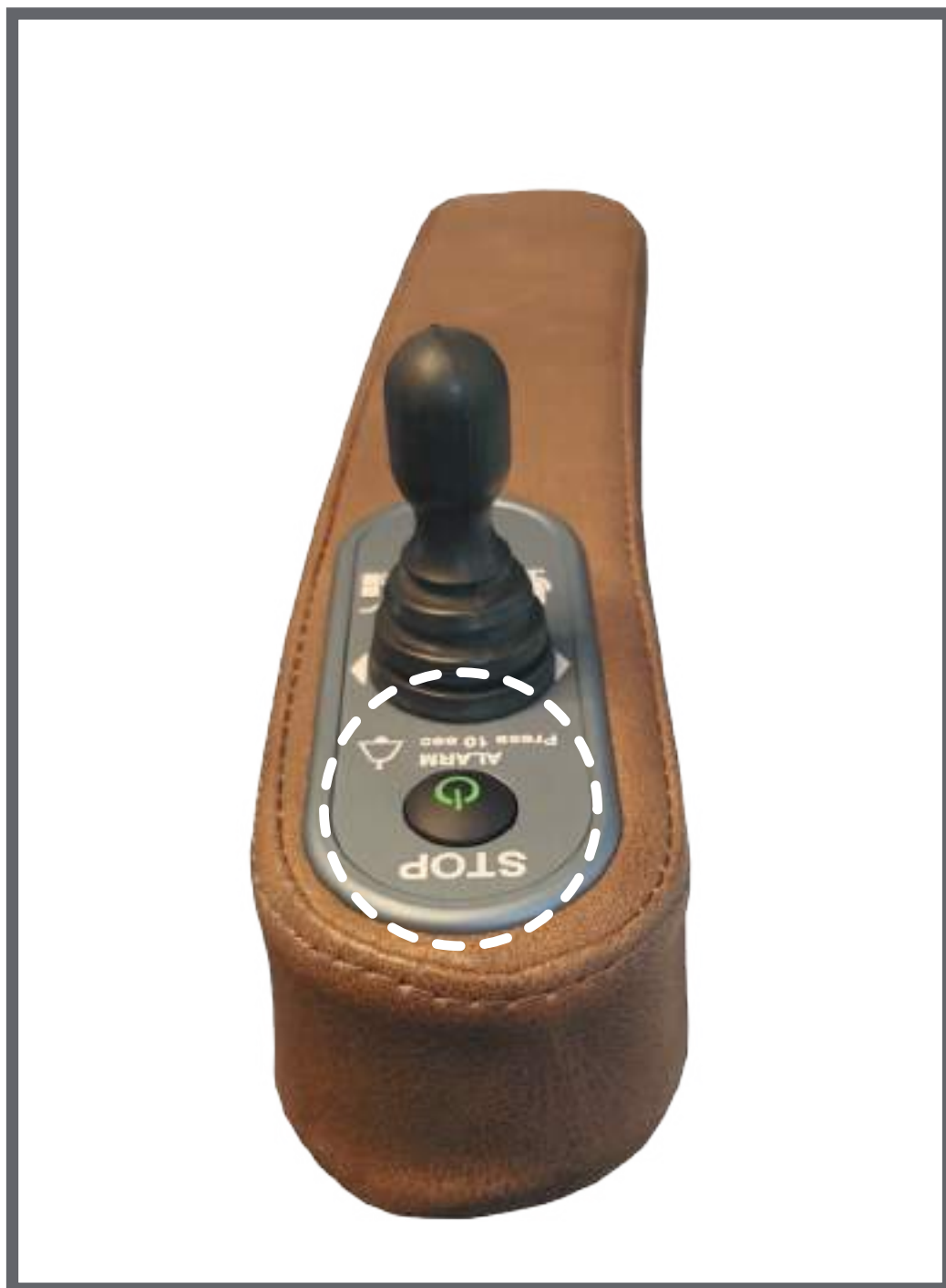


Push the power button. When you push the power button the light will go on. If the light flashes there is a problem with the stairlift.

Step three




Sit down and fasten the seatbelt.

Step four

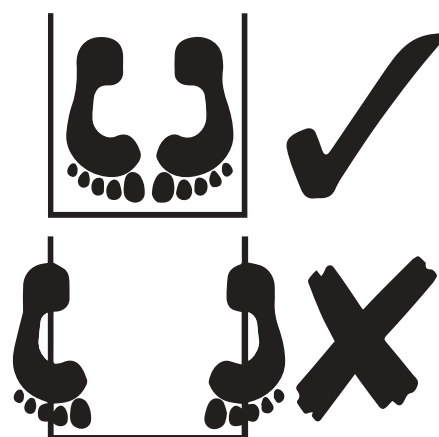


Make sure that your feet are entirely on the footrest.

Step five

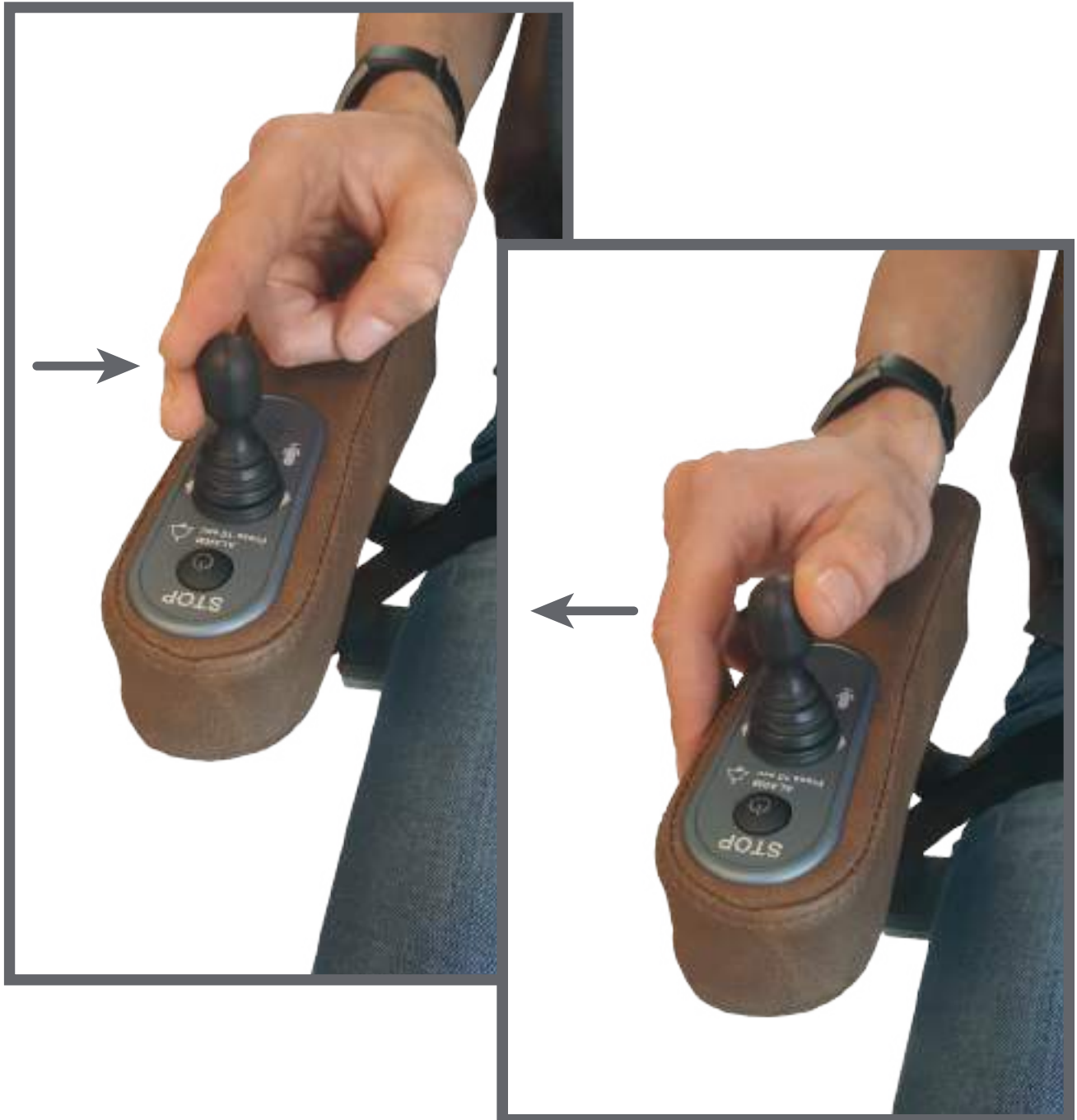

Before you can use the joystick you always have to press the on/off button.
Move the joystick to the desired drive direction.

Step six

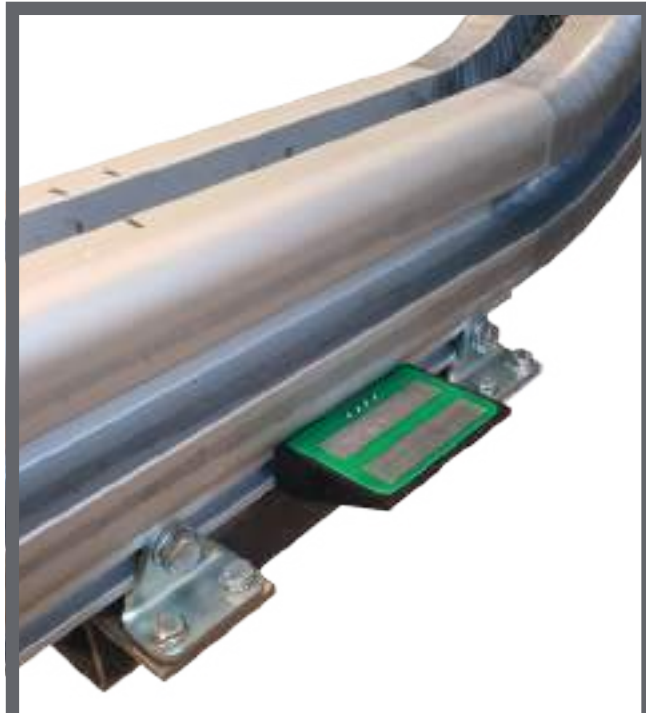

Powered Swivel


At the end of the rail or at the charging point, push the joystick one more time to turn the seat automatically.

Step seven




Release the seatbelt, place your feet on the ground and stand up.

Step eight

1. Install the remote app, that you got from the UP installer, on your phone.
2. Make sure that your bluetooth is on.
3. Select the UP-xxxxxx in your bluetooth settings.
4. When you select the UP you need to put in a password. The password is 1234.
5. Open the app on your phone.
6. Push the button "Connect".
7. Push the  button and scan the QR-code that is on your wall.
8. Now your phone is ready to use as a remote for your stairlift.

When you do not use the app for 30 seconds you need to re-scan the QR-code.

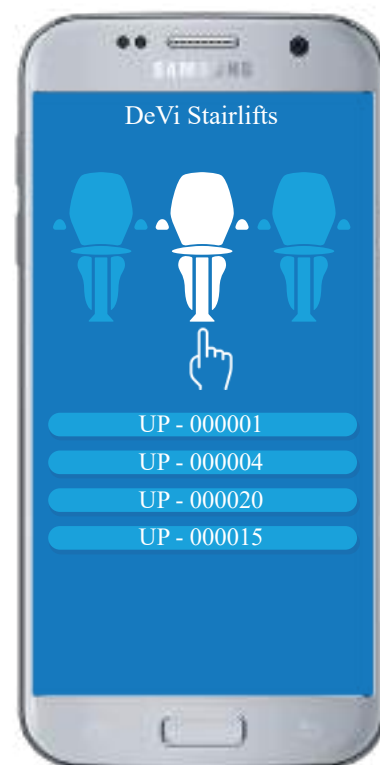

Step nine



The UP must always be parked on the charging contacts. If not you will hear a warning signal and the UP stairlift will not be charged.

In case of emergency



When you subscribe to our maintenance service, you just have to press the power button for 10 seconds. We will be there to help you 24/7.

Safety sensors



The UP stairlift has some safety sensors in the covers to prevent accidents. For example, when there is an obstruction on the staircase and the UP stairlift comes in contact with this obstruction, the UP stairlift will stop immediately. These safety sensors are in the footrest, the cover of the carriage and the cover of the wheels. See the arrows for the locations of the safety sensors.

Technical specifications

Motor output speed	0.15 meters per second
Method of drive	Rack and pinion
Power supply	24V DC (battery)
Maximum capacity	125Kg/275lbs
Track	Modular rail, Extruded aluminum, Casted aluminum
Mains supply	50-60Hz, 100-240V AC
This stairlift is designed to comply with:	
European safety for stairlift	EN 81-40:2008
Software and Hardware standard	IEC – 61508-2 and IEC – 61508-3
EMC standard	EN61000-6-2
Fire resistance	UL94

Maintenance

Regular maintenance, once a year, is needed to guarantee that your UP stairlift will keep working properly. This maintenance can only be carried out by DeV-Stairlifts authorised dealers/technicians. Never do repairs on the UP stairlift yourself. We therefore advise you to take out a maintenance contract to ensure that your stairlift will stay in optimal condition. Contact your dealer for more information about the maintenance contract.

Cleaning

Clean the UP stairlift on a regular basis. For cleaning avoid excessive use of water and caustic detergents. Clean the seat, back support, armrests and footrest exclusively with a damp cloth. Clean the rails regularly with a dry cloth. If there is dirt on the rails this can easily be removed by a cloth with lenient detergent. For the first 50 rides, polish can come off the lift due to the running in of the stairlift. Over time the pinion can leave traces on the rail.

Contact

For information about settings, maintenance or other matters that are not addressed in these instructions, you can contact your local dealer.

Dealer:

Name
Address
Tel

Check form

Name installer	
All control devices function correctly	
All electrical safety devices function correctly	
The alarm device when activated operates correctly	
The polarity of the charging stations	
The staircase is clear from any obstructive objects	
Undergo without failure a dynamic test to check the forces imposed by the maximum working load, 125Kg.	
	Signature of the customer:
Demonstrate the stairlift to the customer including instructions for the use of the controls, the positioning of the alarm signal and what to do in case of an accident or breakdown.	