# Sofao

joeletteandco.com

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*Ideal for adaptive recreation at pools (public or private) and beaches (seashores, lakeshores, etc.). Sofao is also a bathing chair, a rest chair, and a physical therapy device for healthcare professionals.*

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## Fold-out

It takes under 60 seconds to fold out **Sofao** with no tools needed.

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## Getting into the water

Getting in made easy!

Getting into pools is made easy using the 2 fitted handles (front and back), whether in pools equipped with a slipway, pools with step-down access, or even pools with no specialized access system.

The guide slips **Sofao** into the water by letting it roll along the pool wall. For the heaviest sit-swimmers, one guide at back and one guide in the pool can gently slip the bather in and ease them back out afterwards.

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Getting into the water is just as easy on any kind of beach (seaside, lakeside, swimming ponds), even those that do not offer specialized accessways, as **Sofao** Ø38 cm inflatable tyres let it roll across otherwise-unrideable roadways.

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## Floatability, total immersiveness

*The stand-out feature of **Sofao** is that it floats. The sit-swimmer can freely swim unassisted, without being held by a guide, thanks to the foldaway armrests and harnesses that hold the bather safe: it takes just small movements to move around in the water.*

*The 4 stabilizer floats are height-adjustable and can adaptively accommodate any-size swimmers while offering the same stable floatability.*

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## One size, multiple positions

***Sofao** is perfectly suited to even the most heavily mobility-challenged kids or adults, as it features a height-adjustable headrest, a 5-position reclining backrest, and 6-position back-end floaters.*

*Note too that there is a range of head-hugger options for optimal head support.*

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## Maintenance

***Sofao** does not require any particular maintenance: just a spray-rinse at the end of each day out on the water. Built in anodized aluminium/electropolished stainless steel and Batyline® fabric, Sofao is completely corrosionproof.*

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## Deckchair

 Once back out of the water, **Sofao** doubles up as a comfortable recliner to lie down and dry off in the sun. Adaptive relaxing for all!

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## *Out-and-about beachside, access to showers and changing rooms.*

 With **Sofao**, no need to go back-and-forth over and over!

The guide can lead the sit-swimmer from the car to the sea straight across the beach thanks to the fitted front pull-grip, rear handlebar, and two inflatable wheel with dual wheels on option for getting across very loose terrain.

The optional accessory Tire-Sofao is a third wheel fitted with a clip-on Sofao towbar to make the guide's job even easier.

The sit-swimmer can also be showered and changed on **Sofao**, either in changing rooms or beachside.

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## *Storage and handling*

 **Sofao** is lightweight with just 15 kilos, which makes it easy to handle. It is also fully foldable for an easy and space-efficient storage. It fits comfortably into a medium-size car boot.

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Sofao is a patented product and a registered trademark.